ITALIANO

MA CL200x-xx 02 A\_ITA\_11-2020

## **CARROZZINA IN ALLUMINIO ARDEA ONE**

---

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## INDICE

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### **INSTRUCTION MANUAL**

## INDEX

<b>1. CODES.....</b>	<b>PAG.3</b>
<b>2. INTRODUCTION.....</b>	<b>PAG.3</b>
<b>3. INTENDED USE.....</b>	<b>PAG.3</b>
<b>4. DECLARATION OF CONFORMITY .....</b>	<b>PAG.3</b>
<b>5. GENERAL WARNINGS .....</b>	<b>PAG.4</b>
<b>6. SYMBOLS .....</b>	<b>PAG.4</b>
<b>7. GENERAL DESCRIPTION.....</b>	<b>PAG.5</b>
7.1 View and parts list .....	pag. 5
<b>8. ASSEMBLY/ DISASSEMBLY.....</b>	<b>PAG.5</b>
8.1 Opening.....	pag. 5
8.2 Closing .....	pag. 5
8.3 Adjusting the wheelchair.....	pag. 6
8.3.1 Backrest .....	pag. 6
8.3.2 Rear wheels .....	pag. 6
8.3.3 Front wheels .....	pag. 6
8.3.4 Single footplate.....	pag. 7
8.3.5 Brakes.....	pag. 7
8.3.6 Sides .....	pag. 7
8.3.7 Adjusting the seat depth.....	pag. 8
8.3.8 Adjusting the elastic backrest .....	pag. 8
<b>9. BEFORE EVERY USE .....</b>	<b>PAG.8</b>
<b>10. WARNINGS ON USE.....</b>	<b>PAG.8</b>
<b>11. HOW TO USE.....</b>	<b>PAG.9</b>
11.1 Getting into the wheelchair.....	pag. 9
11.2 Getting out of the wheelchair .....	pag. 9
11.3 Stairs and height differences (advice on how best to deal with them).....	pag. 9
11.4 Use of the armrests (OPTIONAL CLA210) .....	pag. 10
11.5 Use of the large rear wheels .....	pag. 10
11.6 Use of the small wheels for narrow passageways (OPTIONAL CLA221) .....	pag. 10
11.7 Use of the brakes.....	pag. 10
11.8 Transporting the wheelchair.....	pag. 10
<b>12. MAINTENANCE .....</b>	<b>PAG.10</b>
<b>13. CLEANING AND DISINFECTION .....</b>	<b>PAG.10</b>
13.1 Cleaning the covers and padding .....	pag. 10
13.2 Cleaning the frame and the metal parts.....	pag. 10
13.3 Cleaning the wheels and brakes.....	pag. 11
13.4 Disinfection.....	pag. 11
<b>14. CONDITIONS OF DISPOSAL .....</b>	<b>PAG.11</b>
<b>15. TECHNICAL SPECIFICATIONS.....</b>	<b>PAG.11</b>
15.1 Dimensions and weights .....	pag. 11
15.2 Technical specifications .....	pag. 11
<b>16. OPTIONAL ACCESSORIES.....</b>	<b>PAG.12</b>
16.1 Armrests (CLA210).....	pag. 12
16.2 Separate footplates (CLA211) .....	pag. 12
16.3 Extended single footplate (CLA212) .....	pag. 12
16.4 Push handles (CLA240).....	pag. 12
16.5 Support for inclining the backrest (CLA235).....	pag. 12
16.6 Seat depth extension kit (CLA242-1 / CLA242-2).....	pag. 13
16.7 Front wheels (CLA215 - CLA216 - CLA217).....	pag. 13
16.8 Wheels for narrow passageways (CLA221) .....	pag. 14
16.9 Roll-over protection wheels (CLA220).....	pag. 14
16.10 Push rings with supergrip (CLA232) .....	pag. 14
16.11 Unpierceable rear wheels (CLA230).....	pag. 14
16.12 Twenty-four spoke rear tyres (CLA231).....	pag. 15
16.13 Headrest kit (CLA243).....	pag. 15
16.14 Backrest extension kit (CLA244).....	pag. 15
16.15 Radial brake kit (CLA241) .....	pag. 15
16.16 Foot straps kit (CLA245).....	pag. 16
16.17 Heel straps kit (CLA245).....	pag. 16
<b>17. WARRANTY .....</b>	<b>PAG.16</b>
<b>18. REPAIRS .....</b>	<b>PAG.16</b>
<b>19. REPLACEMENT PARTS.....</b>	<b>PAG.17</b>
<b>20. NON-LIABILITY CLAUSE .....</b>	<b>PAG.17</b>

CE | Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices**1. CODES**

<b>CL200x-38</b>	Atmos ultra lightweight folding wheelchair - 38 cm seat
<b>CL200x-40</b>	Atmos ultra lightweight folding wheelchair - 40 cm seat
<b>CL200x-42</b>	Atmos ultra lightweight folding wheelchair - 42 cm seat
<b>CL200x-44</b>	Atmos ultra lightweight folding wheelchair - 44 cm seat
<b>CL200x-46</b>	Atmos ultra lightweight folding wheelchair - 46 cm seat

**2. INTRODUCTION**

Thank you for choosing an **ARDEA ONE** wheelchair by Moretti S.p.A. This wheelchair is designed for people who have difficulty moving both inside and outside the home, while travelling or performing simple, everyday activities. This user manual contains some suggestions on how to use the wheelchair properly and safely. Please read this manual in full before you use the wheelchair. Should you have any queries, please contact your dealer for suitable advice and assistance.

**N.B. Check to ensure that no parts of the wheelchair have been damaged during shipment. Do not use the product if it is damaged and contact the retailer for further instructions.**

**3. INTENDED USE**

This wheelchair is designed for people with mobility problems and can be self-propelled.

**WARNING!**

- Do not use the product for any purpose other than that specified in this manual.
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual.
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features.

**4. DECLARATION OF CONFORMITY**

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of MANUAL WHEELCHAIRS - ATMOS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

- Please consult this manual carefully for correct use of the device.
- Always consult your doctor or therapist for correct use of the device.
- Keep the packaged product away from sources of heat, as the packaging is cardboard.
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts.
- Take extreme care in the presence of moving parts that could trap limbs and cause injury.
- Always take care when children are present.
- Close the wheelchair to lift it correctly, following the instructions in this manual, and grip it on backrest.
- Never lift the wheelchair gripping the removable parts, which could accidentally detach from it during lifting.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device


Conditions of disposal

## 7. GENERAL DESCRIPTION

### 7.1 View and parts list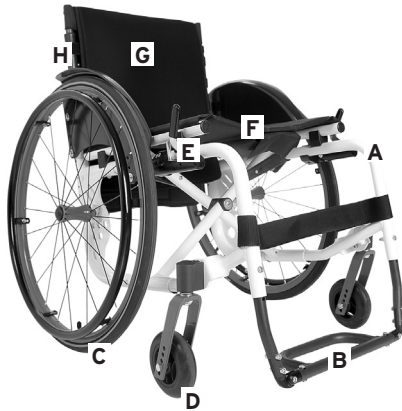

- A - Basic wheelchair frame
- B - Single tip-up footplate
- C - Large rear wheels (diameter 60 cm)
- D - Unpierceable front wheels (diameter 12 cm)
- E - Parking brakes
- F - Seat canvas
- G - Backrest padded cover
- H - Backrest elastic straps

## 8. ASSEMBLY/ DISASSEMBLY

The wheelchair is easy to fold and no tools are needed, but just a few manual operations.

### 8.1 Opening



**N.B.** Consult the safety warnings in this instruction manual.

- Grip the seat tubes and pull outwards, pushing downwards until the wheelchair is completely open.
- Lower the single footplate until it locks into position in the support. (FIG.1)

### 8.2 Closing

**N.B.** Consult the safety warnings in this instruction manual.

- Lift the single footplate out of its support.
- Lift the seat and bring the backrest tubes together. (FIG.2)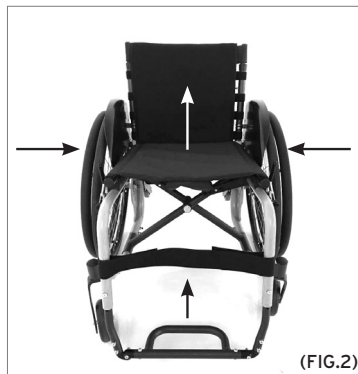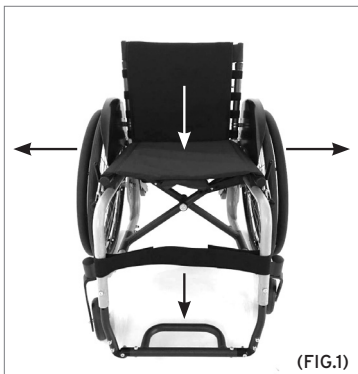

### 8.3 Adjusting the wheelchair


#### WARNING!

**WHEELCHAIR ADJUSTMENTS** - Adjustments are possible on the Atmos wheelchair. Adjustment of the backrest, front and rear wheels and seat should be performed with the supervision of a specialised operator, to ensure they are entirely suited to the wheelchair user's needs.


**N.B.** The product and optionals package does not include the tools and equipment needed for adjustments and/or assembly of original optionals. Always use quality tools to avoid damaging the structure and the materials.

#### 8.3.1 Backrest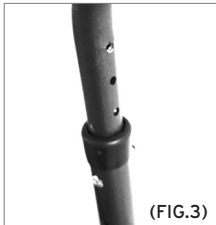

The height can be adjusted using the telescopic tubes on the backrest itself. Remove the padded cover of the backrest and the elastic strap protective cover. Loosen the elastic straps, remove the screws attaching the telescopic tubes and adjust them to the desired height. Tighten the screws and the elastic straps, then reposition the strap protection cover and the soft padding on the backrest. (FIG.3)

#### 8.3.2 Rear wheels

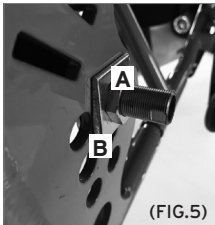
The height, depth and inclination can be adjusted. Remove the rear wheels by pressing the quick-release button at the centre of the wheels and removing the locking pins. (FIG.4)

Unscrew the wheel supports from the frame and tighten them in the holes at the desired height and/or depth (FIG.5-A). If you want to keep the initial camber at 4°, check that the adjustment plates remain in the initial position. If you want to take the camber to 0°, remove the plates (FIG.5-B). Reposition the washers and the bushing on the frame, reinsert the pins in the wheels and into the holes on the frame, ensuring that they are properly inserted.


#### WARNING!

Adjustment of the front wheels and brakes may also correspond with movement of the rear wheels.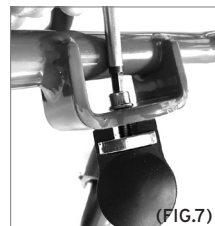

#### 8.3.3 Front wheels

The height and inclination can be adjusted. To adjust the height, use the holes on the front forks, by loosening the screws and tightening them again after repositioning the wheels at the desired height (FIG.6).

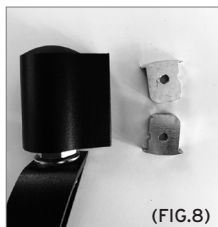(FIG.8)

(FIG.9)

(FIG.10)


(FIG.11)

To adjust the inclination, remove the fork from the frame (FIG.7), then remove the plates inside the structure of the frame (an upper one and a lower one) and reinsert them again in one of the four possible positions (FIG.8-9-10-11). Reinsert the screws and tighten them completely. Repeat the operation, with the same measurements, on the other front wheel.

**WARNING!**

Adjustment of the rear wheels does not necessarily correspond with movement of the front wheels.

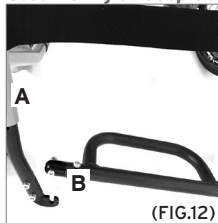
**8.3.4 Single footplate**

(FIG.12)

The height and inclination can be adjusted. To change the height, loosen the through bolts on the frame. Reinsert the through bolts in the holes at the desired height and tighten them completely (FIG.12-A). To adjust the inclination, loosen the through bolts on the side of the footplate (FIG.12-B), adjust the inclination to the desired position and tighten the through bolts completely.

**8.3.5 Brakes**

Loosen the screws on the frame to adjust both depth and height.

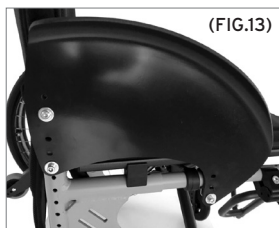**8.3.6 Sides**

(FIG.13)

Loosen the rear screws on the frame, loosen the front screw and reinsert and tighten them completely based on the wheel position. The sides must be positioned very close to the upper edges of the wheel, as they serve as cushioning when the wheelchair is being pushed upwards and when getting out of the wheelchair. (FIG.13)


### 8.3.7 Adjusting the seat depth

(FIG.14)

(FIG.15)

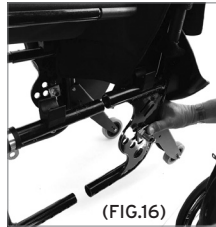(FIG.16)

Position the wheelchair on a comfortable surface (workbench).

Remove the screws on the frame, upper (FIG.14) and lower (FIG.15), and on the backrest (rear sides) (FIG.13) and free the telescopic tubes inside the structure (FIG.16).

Move the frame backwards as far as the new holes in the desired position (2 cm. steps), up to a maximum of 10 cm. Adjust the sides to the new seat measurement. Tighten the screws completely.

#### WARNING!

If you do not wish the distance of the seat from the backrest to be too much, it is recommended to mount OPTIONAL CLA242-1 or CLA242-2 Seat depth variation kit.

### 8.3.8 Adjusting the elastic backrest

(FIG.17)

The tension of the elastic backrest can be adjusted to facilitate containment of the trunk: Remove the padded cushion from the backrest, loosen the Velcro straps and position the wheelchair user in a manner that the backrest fits more snugly around their trunk. Close the straps and reposition the padded cushion on the backrest. (FIG.17)

## 9. BEFORE EVERY USE

- Always check for wear and tear of moving mechanical parts, in order to ensure that the wheelchair can be used in total safety for both people and property.
- Check that the wheelchair has been assembled correctly and particularly that the rear wheels have been inserted properly.
- Check that the seat is completely open and locked into the final position (if it is not, press down firmly on the seat towards the frame).
- Check correct functioning of the brake on the rear wheels.

## 10. WARNINGS ON USE

- The wheelchair is not designed for use as a seat in a motor vehicle.
- Always ensure that the brake control is properly regulated and functioning.
- Take care to avoid trapping fingers in certain parts, such as wheels or brakes, when using the wheelchair.
- Remember that the wheelchair surface temperature may increase when exposed to external heat sources, such as sunlight.

## 11. HOW TO USE

N.B. Consult the general warnings in this instruction manual

### WARNING!

- Getting into and out of a wheelchair on your own can be potentially risky: do so only if you are physically able and with care.
  - Do not lean on removable parts, but rather on the seat frame.
- DO NOT SHIFT YOUR WEIGHT ONTO THE FOOTPLATE WHEN GETTING INTO OR OUT OF THE WHEELCHAIR, AS IT COULD TIP OVER!!!**

### 11.1 Getting into the wheelchair

- Get as close as possible to the wheelchair, in a manner that it is alongside you
- Lock the brakes on the rear wheels. If the wheelchair has armrests, getting into the wheelchair is easier if you remove the one on the side where you want to get in
- Rest your weight on the frame structure using your arms and slowly lower yourself into the seat

### 11.2 Getting out of the wheelchair

- Get as close as possible to the side of another seat
- Lock the brakes on the rear wheels. If the wheelchair has armrests, getting out of the wheelchair is easier if you remove the one on the side where you want to get out
- Rest your arms on the frame of the seat into which you are moving and raise yourself slowly towards it

### 11.3 Stairs and height differences (advice on how best to deal with them)

### WARNING!

These techniques are only suitable for expert wheelchair users.  
Ask a carer to assist you in all other cases!

#### ASCENDING

Go up to the curb of the pavement or the step you want to ascend. Balance the wheelchair on the rear wheels to lift the front wheels just enough to rest them on the edge of the obstacle. Bend forwards from the waist and firmly push the handrail on the rear wheels forward. Continue pushing until the rear wheels have gone over the obstacle.

#### DESCENDING

Follow the procedure in reverse to descend. Position the rear side of the wheelchair in front of the obstacle, grip the handrail on the rear wheels firmly and slowly descend the step or the obstacle, then balance the wheelchair on the rear wheels in a manner to raise the front wheels slightly and carefully complete the descent.

### WARNING!

Do not use this technique when the difference in height is over 10 cm!

**11.4 Use of the armrests (OPTIONAL CLA210)**

The armrests can only be tip-up. This function is handy for getting into and out of the wheelchair. Rotate the armrest backwards to tip it.

**11.5 Use of the large rear wheels**

The 60 cm rear wheels are used when the wheelchair is self-propelled. Grip the handrail on the wheel and push it in the desired direction.

The rear wheels may be placed in different positions. Contact a specialised retailer for advice on the best and most correct position for you.

**11.6 Use of the small wheels for narrow passageways (OPTIONAL CLA221)**

This type of wheel serves exclusively for short movements of the wheelchair through particularly narrow passageways (such as the doors of a lift). To use these wheels, lift the wheelchair to extract the large rear wheels, push the wheelchair by resting it on the small wheels at the rear, then reinsert the large rear wheels on completing the movement.

**N.B.** The large rear wheels have a quick-release system. Press the button at the centre of the wheel to remove it. To reinsert it, simply align the wheel shaft with the hole on the hub and push inwards until the safety clip is correctly locked into position.

**11.7 Use of the brakes**

The large rear wheels have a manually engaged mechanical brake. Grip the lever and push it forwards to lock the wheels. Push the lever in the opposite direction to unlock the wheels.

**11.8 Transporting the wheelchair**

The wheelchair has been designed to be easily transported. Once the footplate and rear wheels have been removed, it can be closed and easily positioned in the car boot, for example. If this is not possible for reasons of space, place the wheelchair in front of the rear seat so that it is stable and cannot slip or slide. If possible, secure the wheelchair with the seatbelt.

**12. MAINTENANCE**

The ARDEA ONE range of products by Moretti S.p.A. are CE marked and carefully checked before being placed on the market. For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original spare parts and accessories for repairs. Before using it, carry out all the checks and controls suggested in the paragraph "BEFORE EVERY USE" of this instruction manual. ARDEA ONE wheelchairs only need a few simple and periodic maintenance operations. The periodic checks to carry out are:

Braking capacity	Monthly
Seat cleaning	As needed

**13. CLEANING AND DISINFECTION****13.1 Cleaning the covers and padding**

The covers and padding are made of nylon. Clean them with water and mild soap.

**13.2 Cleaning the frame and the metal parts**

Also clean with warm water and mild soap. Dry with a soft cloth to avoid scratching the paint on the frame.

**13.3 Cleaning the wheels and brakes**

Also clean with warm water and mild soap. Take care with drying them, as using the wheelchair with wet wheels could be extremely dangerous.

**13.4 Disinfection**

Use a common disinfectant cleaner if you need to disinfect the wheelchair.

**N.B** Never use acid, alkali products or solvents (acetone or thinner) to clean it.

**14. CONDITIONS OF DISPOSAL** 

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

**15. TECHNICAL SPECIFICATIONS**

**15.1 Dimensions and weights**

DIMENSIONS AND WEIGHTS												
Measurements in CM												User weight MAX
CL200*-38				38								
CL200*-40			Front 50÷55	40								
CL200*-42	85	40÷50		42	31÷41							
CL200*-44			Rear 44÷49	44								
CL200*-46				46								
							12.4 Kg	60	12	op-tional		120 Kg

\*Depends on the frame colour chosen:

- B - White
- N - Matt black
- R - Red
- P - Customised

Indicative measurements in cm.

**15.2 Technical specifications**

- Aluminium frame
- Max. user weight: 120 Kg
- Front wheels in PU diameter 12 cm
- 36-spoke large rear tyres diameter 60 cm
- Nylon cover
- Resistance to ignition: The materials used for the cover and padding have been tested using the methods specified in EN1021-1/2
- Reference standards: EN 12183
- Environmental temperature: -20 C° / + 40 C°
- Recommended storage: Store in a cool and dry place, away from direct heat sources, such as sunlight. (recommended temp. between +10 C° / + 40 C°)

## 16. OPTIONAL ACCESSORIES

**N.B.** The tools and equipment needed for adjustments and/or assembly of original optionals are not included. Always use quality tools to avoid damaging the structure and the materials.

### 16.1 Armrests (CLA210)

Remove the rear wheels and position the armrest at the desired height. Use an appropriate tool to open the hole on the frame (if not already open). Insert the through bolts provided with the kit and tighten them completely. Repeat the operation on the other armrest.

### 16.2 Separate footplates (CLA211)

Lift the single footplate, loosen the through bolts on the frame, remove the telescopic tubes of the single footplate and insert the support tubes of the footplates, one by one. Insert the through bolts and tighten completely. **N.B.** When lowering the separate footplates, make sure they are properly attached with the clasps on the edges of the footplates.

### 16.3 Extended single footplate (CLA212)

Lift the single footplate, loosen the through bolts on the frame, remove the telescopic tubes of the single footplate and insert the support tubes of the footplates, one by one. Insert the through bolts and tighten completely.

### 16.4 Push handles (CLA240)

Remove the padded cover of the backrest and the elastic strap protective cover. Remove the elastic straps. Remove the screws securing the standard telescopic tubes, replace them with those with push handles and adjust them to the desired height. Tighten the screws, reinsert the elastic straps and then reposition the strap protection cover and the soft padding on the backrest.

### 16.5 Support for inclining the backrest (CLA235)

Loosen the screws attaching the backrest to the frame, insert the support for adjusting inclination and attach it to the frame with the through bolts. Insert the backrest tubes into the support and secure them with the through bolts after adjusting the backrest to the desired inclination.


**16.6 Seat depth extension kit (CLA242-1 / CLA242-2)**

**N.B.** Position the wheelchair on a comfortable surface (workbench).

Position the wheelchair on a comfortable surface (workbench).

Remove the screws on the frame, upper (FIG.18) and lower (FIG.19), and on the backrest (rear sides) (FIG.13) and free the telescopic tubes inside the structure (FIG.20). Move the frame backwards as far as the new holes in the desired position (2 cm. steps), up to a maximum of 10 cm.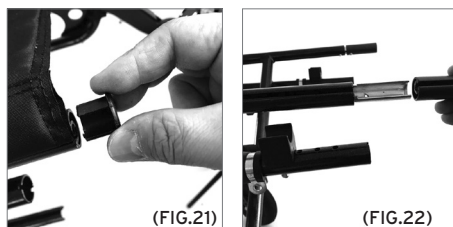

Remove the caps from the seat canvas (FIG.21) and remove it by bringing the sides of the wheelchair slightly closer to each other. Insert the supports of the depth variation kit (4/6 cm. or 8/10 cm.) into the holes on the rear of the seat frame (FIG.22). Secure the extension tubes using the specific screws. Reinsert all the frame screws and tighten them completely. Insert the canvas provided with the kit and slide it from the front of the seat (the storage pocket should be underneath and at the front). Replace the caps in the four holes. Check that the screws are tight with the wheelchair open.

**N.B.** Adjust the sides to the new seat measurement.

**16.7 Front wheels (CLA215 - CLA216 - CLA217)**

Loosen the screws, remove the wheels, insert the new wheels and tighten after repositioning at the desired height.

CLA215:  $\varnothing$  10 cm unpierceable , PAIR

CLA216:  $\varnothing$  15 cm unpierceable , PAIR

CLA217:  $\varnothing$  17.5 cm pneumatic, PAIR


**N.B.** Check inclination of the seat after installing the different wheels, compensating for any difference in height by adjusting the rear wheels.

#### 16.8 Wheels for narrow passageways (CLA221)

Remove the rear wheels, identify the height of the support of the small wheels for narrow passageways, open any pre-cut holes on the frame to insert the through bolts. Tighten the bolts completely. Reinsert the rear wheels and check that the small wheels are not touching the ground.

#### 16.9 Roll-over protection wheels (CLA220)

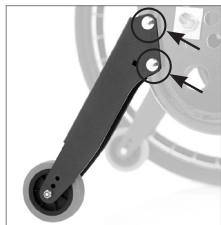Remove the rear wheels, identify the height of the support of the roll-over protection wheels, open any pre-cut holes on the frame to insert the through bolts. Tighten the upper bolts completely. Reinsert the rear wheels and incline the support of the small wheels to the desired position, insert the knob screw in the support of the frame and tighten it completely.

#### 16.10 Push rings with supergrip (CLA232)

Insert the push rings into the self-propelling ring present on the outer part of the wheels.

#### 16.11 Unpierceable rear wheels (CLA230)

Remove the rear wheels by pressing the quick-release button, remove the pins and reinsert them in the holes on the frame of the replacement wheel, checking that they are properly inserted.

**N.B.** If original spare parts are used, the wheel pins are compatible with all MORETTI S.p.A. models.

**16.12 Twenty-four spoke rear tyres (CLA231)**

Remove the rear wheels by pressing the quick-release button, remove the pins and reinsert them in the holes on the frame of the replacement wheel, checking that they are properly inserted.

N.B. If original spare parts are used, the wheel pins are compatible with all MORETTI S.p.A. models.

**16.13 Headrest kit (CLA243)**

Remove the padded cover of the backrest and the elastic strap protective cover. Remove the upper elastic strap and adjust the height of the backrest until the second upper hole on the backrest tube is visible. Reposition the upper elastic strap and fix it in position, together with the two headrest supports. Replace the elastic strap protective cover and the padded cover of the backrest in position. Insert the attachment bar of the headrest into the holes on the supports just mounted. Insert the headrest tube, adjust it to the desired height and tighten the lever.

**16.14 Backrest extension kit (CLA244)**

Remove the padded cover of the backrest and the elastic strap protective cover. Remove the upper elastic strap and adjust the height of the backrest until the second upper hole on the backrest tube is visible. Reposition the upper elastic strap and fix it in position, together with the two backrest extension supports. Replace the elastic strap protective cover and the padded cover of the backrest in position. Insert the extension into the holes on the supports just mounted.

**16.15 Radial brake kit (CLA241)**

Remove the standard parking brakes by loosening the clamp attached to the frame. Insert the radial parking brake support in the same position. Before tightening the screws, adjust the inclination and depth of the parking brake to the right position for good braking.

**16.16 Foot straps kit (CLA245)****With separate footplates (CLA211) or extended single footplate (CLA212)**

Insert the Velcro strips into the openings present and adjust them to the desired position.

**16.17 Heel straps kit (CLA245)****Only with separate footplates (CLA 211) or extended single footplate (CLA212)**

Insert the Velcro strips into the openings present and adjust them to the desired position.

**17. WARRANTY**

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual. Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

**18. REPAIRS**

**Repairs under warranty:** If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

**Repairing a product not covered by the warranty:** A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

**Non-defective products:** The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

#### **19. REPLACEMENT PARTS**

Moretti original spare parts are guaranteed for 6 (six) months from the day of delivery.

#### **20. NON-LIABILITY CLAUSE**

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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