**KYARA®**

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**Dongguan Aidisy Machinery & Electronic Equipment Co, Ltd**  
Part B, 3rd F, Block A, Wentang Industrial Park, Longhua Road, Zhouwu Dongheng  
District, Dongguan City, Guangdong Province, 523000, P.R. China

**MedPath GmbH****Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany****Importado y distribuido por:**

**Moretti S.p.a.** Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11  
[www.morettispa.com](http://www.morettispa.com) correo electrónico: [info@morettispa.com](mailto:info@morettispa.com)

<b>Introduction</b> .....	3
<b>Product Features</b> .....	3
<b>Intended Use</b> .....	4
<b>Safety</b> .....	4
<b>Classification and Explanation of Symbols</b> .....	6
<b>Package Contents</b> .....	7
<b>System Overview</b> .....	8
<b>Using the Battery</b> .....	9
<b>Cleaning and Disinfection</b> .....	12
<b>Troubleshooting</b> .....	14
<b>Technical Data</b> .....	15
<b>Disposal</b> .....	16
<b>Warranty</b> .....	17

## INTRODUCTION

Thank you for choosing Mesh Nebulizer. Our Nebulizer is a handheld device designed to aerosolize medication for respiratory therapy purpose, which considers the difference of your needs between traveling and indoor use.

This device can be operated on internal lithium battery. Due to the use of battery and compact size, you are able to continue your treatment while you are traveling. To help save the planet.

As this is a medical instrument, please read the instruction manual carefully before use.

## PRODUCT FEATURES

1. Microporous structure (MESH) and oscillator combined in atomization technology
2. USB charging
3. Compact and portable design
4. Handheld, easy to handle
5. Easy to use
6. Easy to clean and maintain

## INTENDED USE

Medical purpose and operating principle	The device adopts MESH technology, designed to atomize the solution in aerosols for respiratory therapy purposes.	
Intended user	Legally certified medical experts such as doctors, nurses, therapists or patients under the guidance of qualified medical experts. The user must also be able to understand the general operation of the device and the contents of this manual.	
Intended patient	This device is designed for patients of all ages, with the exception of patients who are unconscious and unable to breathe spontaneously or who are subject to pulmonary oedema.	
Recommended operation environment	This device is intended for use in medical environments, such as hospitals, clinics and doctor's surgeries as well as at home or outdoors.	
Durable period	The period is defined by considering a device that atomises 0.9% saline solution twice a day at room temperature (23°C), the duration may vary depending on the conditions environment and use.	
	Main unit	36 months
	Cup	12 months
Precautions for use	Warnings and precautions defined in this manual should be read carefully.	

## SAFETY

**READ ALL WARNINGS AND INSTRUCTIONS BEFORE USING THE DEVICE.**

Like any medical device, this device may become inoperative due to loss of power, dead batteries, or mechanical failure. It is therefore recommended to use a backup device. When using electronic devices, always follow basic safety precautions.

Pay particular attention to children when using electronic devices.

**WARNINGS**

- Only use the device for medicines prescribed by a doctor.
- The aerosol is intended for respiratory therapy only, any other application is improper and dangerous. The manufacturer assumes no liability for damage caused by improper or incorrect use.
- Do not share the device with others. It is intended for personal use only. In the case of sharing with others, there is a risk of spreading infectious diseases.
- Clean all aerosol parts before and after each use and after a long period of non-use.
- Adult supervision is required when the device is used by children or persons requiring special assistance.
- Never use this appliance if any of its parts are damaged or not working properly.
- Ensure that the appliance has been properly cleaned before use to avoid possible contamination.
- Do not plug in and unplug the power cord with wet hands.
- Do not drop medication on the main unit or USB port, if this happens, wipe the part dry immediately. Using the unit while wet can cause malfunction or injury.

**PRECAUTIONS**

- Do not try to clean the membrane with foreign objects, otherwise it could be damaged.
- Do not drop the unit, otherwise it could be damaged.
- Keep the unit away from sunlight, excessive heat and cold to avoid damage to the battery.
- Do not attempt to open, repair or modify this unit.
- Do not drop liquids on the main unit or the power supply unit, if this happens, wipe them up immediately.
- Follow local disposal and recycling instructions.
- If the USB cable gives problems inserting, there may be a compatibility problem. Make sure you are using the correct cable and that it is correctly positioned.

## DISPOSAL CONDITIONS

When disposing of the device, never use regular municipal solid waste disposal systems. Instead, it is recommended to dispose of the device at local recycling centers for the proper recycling of the materials used.

*Warnings for Proper Disposal of the Product  
in accordance with European Directive 2012/19/EU* 

At the end of its useful life, the product must not be disposed of with municipal waste. It can be handed over to the appropriate recycling centers provided by local authorities or to retailers who offer this service. Separately disposing of the product helps avoid potential negative consequences for the environment and health due to improper disposal and allows the recovery of the materials it is made of, leading to significant savings in energy and resources. To emphasize the obligation of separately disposing of medical devices, the product features the crossed-out wheeie bin symbol.

*Waste Battery Treatment (Directive 2006/66/EC)* 

The symbol on the product indicates that the batteries should not be treated as normal household waste. By ensuring that the batteries are disposed of properly, you help prevent potential negative consequences for the environment and health that could otherwise result from improper disposal. Recycling the materials helps conserve natural resources. Dispose of used batteries at the designated collection points for recycling. For more detailed information regarding the disposal of used batteries or the product, you can contact your local municipality, waste disposal service, or the store where you purchased the device.

## Warranty

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

## Warranty for rechargeable batteries (if provided)

The original batteries and the spare batteries are covered by a (ninety) 90-days warranty as far as their performance is concerned and by a (six) 6-months warranty as far as manufacturing faults are concerned or as required by law. Warranty on fully charged batteries left in an unused state for more than (three) 3 consecutive months is automatically void. Warranty on fully discharged batteries left in an unused state for more than (three) 3 consecutive days is automatically void.

**PACKAGE CONTENTS**
**Check before use**

The contents of the package are described below. Check that all parts are intact. Replace any damaged parts before use. In case of missing parts, malfunction or damage, contact your dealer.

**Main unit**

**Medicine cup**




**User manual**

**Adult and child  
mask**

**Mouth piece**

**USB cable**

## SYSTEM OVERVIEW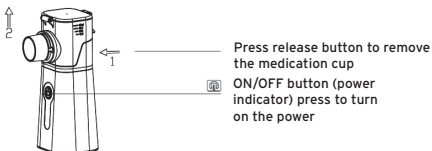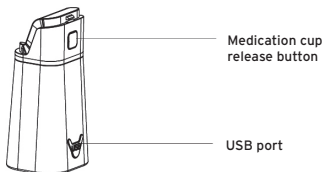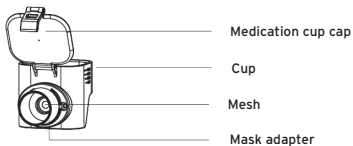

## HOW TO OPERATE WITH BATTERY


### About the battery:

The device has an internal lithium battery that cannot be replaced, do not use the device if the battery has been replaced. The device is rechargeable via a USB port, like most common mobile phones and electronic devices. It is possible to recharge the battery via the USB cable provided.

**Note:** The USB power supply is an optional accessory. Use only power supplies that comply with IEC60601-1:2012 standards, OUTPUT: 5V MAX 2A.

**Recharge battery with wall power supply:**

1. Open the cover to release the USB-C port.
2. Gently insert the type-C connector into the type-C port.
3. Gently insert the USB connector into the power supply.
4. Plug the power supply into the wall outlet.

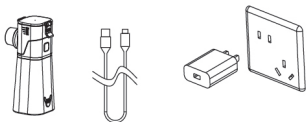**About the battery:**

If the device is not connected to a power source, the ON/OFF button has three possible states:

- No light: OFF
- Blue light: In use
- Orange light: Low battery

When the device is connected to a power source, there are two possible states:

- Flashing orange light: Charging
- Flashing blue light: Charging complete

Note: Rechargeable batteries have a limited lifespan. The lifespan and number of charge cycles vary depending on usage.

**Clean all parts of your aerosol before and after each use and after a long period of non-use.**

**1. Inserting the cup**

Step 1: Take the main unit and the ampoule.

Step 2: Insert the cup into the main unit.

Step 3: Ensure that the cup is correctly inserted into the main unit.


## 2. Insert the medicine

Make sure the power LED is off before adding the medicine.

- Open the ampoule.
- Fill the ampoule with the medicine. DO NOT EXCEED THE INDICATED LIMIT (10ml).
- Close the ampoule.

## 3. Attach new mask or mouthpiece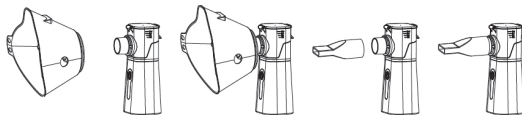

The device is now ready for use.

Refer to the next section for inhalation.

Warning: for type, dose and administration regimen of the medication, follow the instructions of your doctor or pharmacist.

### 1. TILT THE UNIT SLIGHTLY,

In this position, the vibrating membrane is immersed in the medication and the atomisation will be initiated.

Once the membrane is in contact with the medication the unit can be used at any angle.

Note: In some positions (e.g. vertical), atomisation may stop for a short time, in which case tilt the unit again to re-immers the membrane in the medication

**2. Position the mask to cover the nose and mouth or the mouthpiece.**

**3. Press the ON/OFF button to start/stop the atomisation. (The unit may switch off automatically when the medication is finished).**

Maintain calm and relaxed breathing during treatment. Breathe slowly and deeply to allow the medication to better penetrate the airways. Hold your breath slightly and then release it gently while removing the mask from your face. Do not breathe too quickly. Take a break if necessary.

**4. Always switch off the device after treatment.**

**5. Clean the unit after each use.**

1. Tilt the device slightly. When the solution has been fully atomised make sure that the residue is in contact with the membrane and atomise it completely.
2. If high-density solutions have been used, the atomisation may be lower.
3. If an excess of solution accumulates on the membrane, atomisation may stop. In this case, switch off the device and remove the excess solution with gauze or a lint-free cloth.
4. Do not drop solution on the device or USB port, if used in these conditions the device may cause problems or injury.

#### **CLEANING AND DISINFECTION**

If the device is not properly cleaned and disinfected as often as indicated, microorganisms may remain inside the unit and pose a risk of infection.

##### **1. After each use:**

Clean the cup with distilled water after each spray.

1. Remove residual solution from the cup.
2. Place distilled water in the cup.
3. Start the device to nebulise the inserted distilled water for about 2-3 minutes.
4. Disconnect the adapter.
5. Remove the cup.
6. Rinse the cup with distilled water and dry it.
7. Remove excess water and allow all parts to air dry completely on a clean, dry cloth.

8. Use a lint-free gauze or cloth to remove stains on the main unit if necessary.
9. Make sure that all parts are dry before storing or re-using them.

## 2. AFTER 9 USES

Clean the ampoule with 60% white food grade vinegar after about 9 uses.

1. Remove any residual solution from the cup.
2. Place distilled water in the ampoule.
3. Remove the inserted distilled water.
4. Insert 3 ml of approx. 60 % white food-grade vinegar.
5. Start the device and nebulize the inserted vinegar to clean the membrane, for 5-10 minutes.
6. Disconnect the adapter and remove the ampoule from the main unit.
7. Rinse and dry the ampoule with distilled water.
8. Remove water residue and allow all parts to dry on a clean cloth in a clean environment.
9. Use a lint-free gauze or cloth to remove stains from the main unit.
10. Make sure the cup and other parts are completely dry before storing or re-using them.

1. Always keep the battery dry.
2. Do not touch the membrane with fingers, cotton swabs or other objects.
3. Do not clean parts in the dishwasher.
4. Do not use the microwave to dry the parts.
5. Do not wash the cup and membrane with running water or other liquids.

## Daily disinfection

It is important to disinfect the cup daily.

1. Disinfection using boiling water
  - Rinse the cup with distilled water.
  - Bring distilled water to a boil.

## NOTE: DO NOT DIRECTLY HEAT THE CUP

- Carefully immerse the ampoule in boiling water for 15 minutes. Carefully remove the ampoule from the water and wipe off any excess liquid.
- Ensure that parts that are still hot and left to dry on a clean, dry cloth are away from children.
- Ensure that all parts are dry before storing or re-using the unit.

## 2. Disinfection using alcohol

- Rinse the ampoule with distilled water.
- Immerse the ampoule in 75% ethyl alcohol for 1 minute.

- Rinse the ampoule with distilled water again, remove excess liquid and ensure that the parts air dry on a clean, dry cloth.
- Ensure that all parts are dry before storing or re-using them.

### 3. Cleaning

Cleaning and disinfecting accessories (including ampoule, mouthpiece, etc.).

After using the accessories:

- Clean the accessories with distilled water.
- Completely immerse the accessories in approx. 75 % alcohol.
- Leave for about 30 minutes.
- Wash and rinse the accessories with lukewarm distilled water (approx. 50°C).
- Remove water residue and allow to air dry or with a clean, soft cloth.
- Ensure that all parts are completely dry, then store.

1. Alcohol is highly flammable. Do not use alcohol near heat sources.

**The following accessories can be disinfected in boiling water or alcohol.**

**The parts shown below CANNOT be disinfected with boiling water or alcohol.**

Do not rinse or immerse the main unit in liquid substances.

### Transport and storage

Close the mouthpiece cover and store the device and ampoule in a clean, dry environment.

1. Do not leave or transport the device with liquid residue in the ampoule.
2. Do not expose the device to direct light, or to extremely humid, hot or cold environments.
3. Keep away from heat, electromagnetic fields and out of the reach of children.

## TROUBLESHOOTING

If problems occur during use, check the table below.

Problem	Possible cause	Action
Low atomization	Batteries discharged (orange light)	Recharge batteries
	Dirt residue on electrodes	Use alcohol to clean electrodes
	Mesh holes are clogged or dirty	Refer to the disinfection procedures explained in the appropriate section of this manual
	Mesh is broken	Replace the cup
The device does not turn on and does not atomise	Batteries completely discharged	Recharge batteries
	Poor connection between the adapter and the main unit	Check and reconnect the USB to the main unit
The device turns on but does not atomise (power indicator is lit)	The ampoule is not inserted correctly	Refer to ampoule installation procedure in this manual.
	Membrane pores are stained or clogged	Refer to disinfection procedures explained in the appropriate section of this manual
	Mesh is broken	Replace the cup
Orange power button	Low battery	Recharge battery

1. If the device does not atomize normally after following the directions above, contact your dealer.
2. Do not use the device if any of the parts does not function correctly or is damaged.

## TECHNICAL DATA

Model	LTK200 (MESH-600C)
Product	Portable aerosol
Method of operation	Active vibrating mesh technology
Power supply	Power supply Internal power supply, lithium battery only
Power consumption	Approx 2.0W
Mode of operation	Auto power off after 20 minutes
	Press the button once to switch on the device and start normal spraying.
	Press the button again to start slow spraying.
	Slow spraying is about 80 % slower than normal spraying.
	Press again to switch off the device.

Light indicators	Blue light: in use.
	Orange light: low battery.
	Blinking orange light: charging battery.
	Flashing blue light: charging completed.
Vibrating frequency	Approx. 107 kHz $\pm$ 10%
Cup capacity	Max. 10ml
Nebulization rate	$\geq$ 0.25 ml/min
Particle size	$\geq$ 65% range (0-5 $\mu$ m)
MMAD	3.50 $\mu$ m $\pm$ 25%
Battery charging voltage and current	DC 5.0 V, max. 2A
Battery charging time	Approx 180 min
Battery Non-stop usage time	Approx 120 min
Lithium battery cycle life	The life cycle is not less than 300 times
Durable period	The period is defined by considering a unit that atomize 0.9% saline solution twice a day at room temperature (23°C), the duration may vary depending on environmental and usage conditions.
	Main unit: 36 months
	Cup: 12 months
Noise	$\leq$ 50 dBA (DIN EN13544-1)
Accessories	1xChildren's mask, 1xAdult's mask, 1xBoard, 1xUSB cable
Operating temperature and humidity	10 ~ 40°C, 30 ~ 85 % RH, 800~1060hPa
Transport and storage conditions	- 20 ~ 60°C, 20 ~ 75 % RH, 800~1060 hPa
Dimensions	Approx L: 50 x W: 60 x H: 110 mm
Weight	Approx. 105 g

## DISPOSAL CONDITIONS

When disposing of the device, never use regular municipal solid waste disposal systems. Instead, it is recommended to dispose of the device at local recycling centers for the proper recycling of the materials used.

### *Warnings for Proper Disposal of the Product in accordance with European Directive 2012/19/EU.*

At the end of its useful life, the product must not be disposed of with municipal waste. It can be handed over to the appropriate recycling centers provided by local authorities or to retailers who offer this service. Separately disposing of the product helps avoid potential negative consequences for the environment and health due to improper disposal and allows the recovery of the materials it is made of, leading to significant savings in energy and resources. To emphasize the obligation of separately disposing of medical devices, the product features the crossed-out wheeilee bin symbol.

### *Waste Battery Treatment (Directive 2006/66/EC)*

The symbol on the product indicates that the batteries should not be treated as normal household waste. By ensuring that the batteries are disposed of properly, you help prevent potential negative consequences for the environment and health that could otherwise result from improper disposal. Recycling the materials helps conserve natural resources. Dispose of used batteries at the designated collection points for recycling. For more detailed information regarding the disposal of used batteries or the product, you can contact your local municipality, waste disposal service, or the store where you purchased the device.

## Warranty

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### **Warranty for rechargeable batteries (if provided)**

The original batteries and the spare batteries are covered by a (ninety) 90-days warranty as far as their performance is concerned and by a (six) 6-months warranty as far as manufacturing faults are concerned or as required by law. Warranty on fully charged batteries left in an unused state for more than (three) 3 consecutive months is automatically void. Warranty on fully discharged batteries left in an unused state for more than (three) 3 consecutive days is automatically void.

**Repair***Warranty repairs*

In the event that a product Moretti is to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty.

Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

*Repair not covered by warranty*

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

*Non-defective device*

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

**Spare parts**

For spare parts only refer to MORETTI master catalogue.

*Disclaimer*

Beyond the specifications of this warranty and within the limits of law, Moretti makes no other express or implied representations, warranties or conditions with respect to merchantability, fitness for particular purposes, non-infringement and non-interference. Moretti does not warrant that the use of the Moretti product will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law is limited to the warranty period in accordance with the limits of the law. Some states or countries do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with consumer products. In such states and countries, some exclusions or limitations of this warranty may not apply to you. This warranty is subject to change without notice.

## WARRANTY CERTIFICATE

Product \_\_\_\_\_

Purchased on (date) \_\_\_\_\_

Retailer \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Sold to \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

**Dongguan Aidisy Machinery & Electronic Equipment Co, Ltd**

Part B, 3rd F, Block A, Wentang Industrial Park, Longhua Road, Zhouwu Dongheng District, Dongguan City, Guangdong Province, 523000, P.R. China

**MedPath GmbH****Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany****Imported and distributed by:**Moretti S.p.a. Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11  
www.morettispa.com email: info@morettispa.com*\* Please consult our website for the latest available version of the user manual*

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**MedPath GmbH****Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany****Dováža a distribuuje:**