**KYARA®**

**BILANCE PESAPERSONE DA CASA**

---

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## INDICE

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 **MORETTI S.P.A.**

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

**www.morettispa.com email: info@morettispa.com****MADE IN P.R.C.**

**KYARA®**

**HOME PERSONAL SCALES**  

---

**INSTRUCTIONS MANUAL**

## INDICE

<b>1. CODES.....</b>	<b>.....</b>	<b>PAG.3</b>
<b>2. INTRODUCTION.....</b>	<b>.....</b>	<b>PAG.3</b>
<b>3. INTENDED USE.....</b>	<b>.....</b>	<b>PAG.3</b>
<b>4. GENERAL WARNINGS .....</b>	<b>.....</b>	<b>PAG.3</b>
<b>5. SYMBOLS USED .....</b>	<b>.....</b>	<b>PAG.3</b>
<b>6. GENERAL DESCRIPTION .....</b>	<b>.....</b>	<b>PAG.4</b>
<b>7. BEFORE EACH USE .....</b>	<b>.....</b>	<b>PAG.4</b>
7.1 Calibration.....	.....	pag. 4
<b>8. HOE TO USE.....</b>	<b>.....</b>	<b>PAG.4</b>
<b>9. MAINTENANCE.....</b>	<b>.....</b>	<b>PAG.4</b>
<b>10. CLEANING AND DISINFECTION .....</b>	<b>.....</b>	<b>PAG.4</b>
10.1 Cleaning.....	.....	pag. 4
10.2 Disinfection .....	.....	pag. 5
<b>11. GENERAL DISPOSAL CONDITIONS .....</b>	<b>.....</b>	<b>PAG.5</b>
<b>12. TECHNICAL INFORMATION .....</b>	<b>.....</b>	<b>PAG.5</b>
<b>13. WARRANTY .....</b>	<b>.....</b>	<b>PAG.5</b>
<b>14. REPAIRING .....</b>	<b>.....</b>	<b>PAG.5</b>
14.1 Warranty repair .....	.....	pag. 5
14.2 Repair not cover from warranty.....	.....	pag. 6
14.3 Non-defective device.....	.....	pag. 6
<b>15. SPARE PARTS .....</b>	<b>.....</b>	<b>PAG.6</b>
<b>16. EXEMPT CLAUSES .....</b>	<b>.....</b>	<b>PAG.6</b>

## 1. CODES

- LTK650** Classic white private scale  
**LTK652** Black square private scale  
**LTK654** Black quadrant private scale

## 2. INTRODUCTION

Thank you for choosing a KYARA by Moretti bathroom scale. The product has been designed and manufactured to meet all your requirements for practical, correct and safe use. This user manual contains small suggestions for the correct use of your chosen device and valuable advice for your safety. We recommend that you read this entire manual carefully before using the scale in question. If you have any doubts, please contact your dealer, who will be able to help and advise you correctly.

**NOTE** Check that all parts of the product have not been damaged during shipment.  
In case of damage, do not use the product and contact your dealer for further instructions

### WARNING!

- It is forbidden to use the following device for purposes other than those defined in this manual
- Moretti S.p.A. declines any responsibility for damage resulting from improper use of the device or use other than that indicated in this manual.
- The manufacturer reserves the right to make changes to the device and to the following manual without prior notice in order to improve their characteristics.

## 3. INTENDED USE

This scale is intended for recording the body weight of persons, with the only limitation being the maximum capacity. (see section on technical characteristics).

## 4. GENERAL WARNINGS

- For a correct use of the device refer carefully to this manual.
- Keep the packaged product away from any source of heat as the packaging is made of cardboard.
- The useful life of the device is determined by the wear of non-repairable and/or replaceable parts.
- Always pay close attention to the presence of children.

## 5. SYMBOLS USED

Product code

CE mark

Manufacturer

Batch Lot


Read the instruction manual


Conditions of disposal

## 6. GENERAL DESCRIPTION


LTK650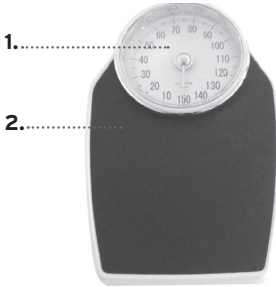

LTK652

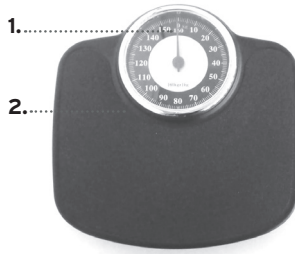LTK654

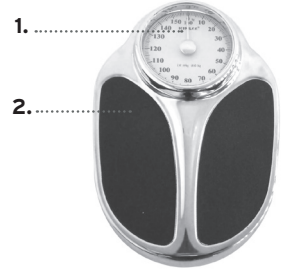(Images are illustrative purposes only)

1. Quadrant
2. Footboard
3. Regulation caster

## 7. BEFORE EACH USE

- Always check the state of wear and tear of the parts to ensure safe use of the device.
- Make sure that the pointer of the balance is correctly positioned on 0.
- Make sure that the balance is placed on a flat, rigid and stable surface.

### 7.1 Calibration

Before use, the balance must be calibrated by turning the adjusting wheel on the opposite side of the dial until the pointer is precisely positioned on 0.

## 8. HOE TO USE

**NOTE:** Refer to the general and operating instructions in this instruction manual.

- Ensure that the scale is placed on a flat, rigid and stable surface.
- Step on the platform with one foot at a time. If you place the scale too abruptly, the correct functioning of the measuring mechanism may be impaired.
- Try not to put your weight off balance and remain as still as possible.

## 9. MAINTENANCE

The products of the KYARA by Moretti line are carefully checked when they are put on the market and bear the CE mark.

In case of repair, only original spare parts and accessories must be used.

- It is extremely important that the device is checked the first time it is used.
- Be careful when moving the device, even small impacts can impair the measuring mechanism.

## 10. CLEANING AND DISINFECTION

### 10.1 Cleaning

Use only a damp, non-abrasive cloth and neutral detergents to clean the balance.

**WARNING!**

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, oxides, organic solvents or alkaline detergents, as these substances may damage the plastic. The manufacturer will not be responsible for damage caused by the use of materials that may deteriorate the surface of the product or by chemical corrosives used during cleaning.

**10.2 Disinfection**

If necessary, disinfect the product using a mild disinfectant detergent or using a commercial, chlorine-free disinfectant. The manufacturer will not be responsible for damage caused by the use of materials that may deteriorate the surface of the product or by chemical corrosives used during cleaning.

**WARNING!**

Sanitising products are corrosive. Avoid contact with the eyes and skin. In case of accidental contact, wash the affected area thoroughly with warm water and consult a doctor immediately.

**11. GENERAL DISPOSAL CONDITIONS** 

When disposing of the device, never use normal solid urban waste disposal systems. It is recommended that the device be disposed of through the common municipal ecological islands for the planned recycling operations of the materials used.

**12. TECHNICAL INFORMATION**

Model	Dimensions	Division	Max weight
LTK650	 L:28,5 cm  W: 42,5 cm  H: 8,5 cm	1 Kg	150 Kg
LTK652	 L:36 cm  W: 36,5 cm  H: 6,5 cm		160 Kg
LTK654	 L:45,8 cm  W: 34 cm  H:10,2 cm		

**13. WARRANTY**

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use. Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs , faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

**14. REPAIRING**
**14.1 Warranty repair**

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

#### **14.2 Repair not cover from warranty**

A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

#### **14.3 Non-defective device**

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

### **15. SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

### **16. EXEMPT CLAUSES**

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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