ITALIANO

MA ST711/x 02 A_ITA_10-2020

CUSCINO IN MEMORY FOAM E GEL

MANUALE D'ISTRUZIONI

INDICE

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REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 on medical devices

1. CODES

ST711/1	Memory Foam and fluid gel cushion 40x40x7h cm
ST711/2	Memory Foam and fluid gel cushion 43x43x7h cm
ST711/3	Memory Foam and fluid gel cushion 46x40x7h cm
ST711/4	Memory Foam and fluid gel cushion 50x40x7h cm
ST711/5	Memory Foam and fluid gel cushion 50x46x7h cm

2. INTRODUCTION

Thank you for choosing a cushion in the LEVITAS range by Moretti. Moretti devices have been designed and manufactured to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this entire manual carefully before you use the mattress. Should you have any queries, please contact your dealer for suitable advice and assistance.

3. INTENDED USE

The LEVITAS cushions by Moretti are intended to prevent the bedsores that can occur in any situation requiring lengthy hospitalisation and/or being laid up at home.

WARNING!

- Do not use the product for any purpose other than that specified in this manual.
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual.
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features.

4. EU DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products manufactured and sold by MORETTI SpA in the CUSHIONS product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES.

To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATIONS.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

5. WARNINGS

5.1 General warnings

- Please consult this manual carefully for correct use of the device
- Always consult your doctor or therapist for correct use of the device
- Use by one user only is recommended
- Use and store in the horizontal position
- Always sanitise the device before any reuse
- The devices are extremely sensitive to air and light.
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts
- Store and use the device away from open flames and sources of heat
- Always take care when children are present

5.2 Warnings on use

- Avoid exposure to sunlight and sources of heat.
- Do not use the cushion if it is damaged.
- For reasons of hygiene, each cushion must be used by a single patient only and must be cleaned regularly.
- Keep the cushion away from sharp objects.
- Before sitting down, ensure that no objects have been left under the cushion.
- The cushion must be used on its own. Do not use it with other heating or freezing devices.
- Always use the cushion with its anti-slip cover to avoid accidents. Please do not sit on the cushion without its cover.
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

6. SYMBOLS USED

Product code

Unique device identification

CE marking

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Disposal conditions

Hand wash at a maximum temperature of 30°C

Do not bleach

Do not iron

Do not tumble dry

Do not dry clean

7. GENERAL DESCRIPTION

ST711/x

Memory Foam cushion with a density of 620g, shaped on the front to make the seat more comfortable and to support thighs and buttocks. The cushion has a gel layer on the seat part that contributes to relieving pressure on the back and maintaining a slightly lower and more comfortable level. Cover in PU, removable with a zip on three sides and with an anti-slip lower layer.

8. HOW TO USE

Always check for wear and tear of parts, in order to ensure that the device can be used in total safety for both people and property. Position the device on the seat, taking care to ensure that the part with grooves is facing forwards.

9. CLEANING SILICONE HOLLOW FIBRE DEVICES

- DO NOT USE STAIN REMOVERS CONTAINING SOLVENTS.
- DO NOT USE BLEACH.
- DO NOT USE CHEMICAL OR ABRASIVE PRODUCTS.

Hand wash at a maximum temperature of 30°C

Do not bleach

Do not iron

Do not tumble dry

Do not dry clean

10. MAINTENANCE

LEVITAS devices manufactured by Moretti are CE marked and carefully checked before being commercialised.

For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original replacement parts and accessories for repairs.

11. DISPOSAL CONDITIONS

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

12. TECHNICAL SPECIFICATIONS

Dimensions and weight					
Product codes	ST711/1	ST711/2	ST711/3	ST711/4	ST711/5
 L:	400 mm	430 mm	460 mm	500 mm	500 mm
 D:	400 mm	430 mm	400 mm	400 mm	460 mm
 H:	70 mm				

- External cover in PU, removable with zips on three sides.
- Memory foam density: 620g

13. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations.

The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual.

Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual.

Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

14. REPAIRS

14.1 Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

14.2 Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

14.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

15. SPARE PARTS

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

16. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

 **LEVITAS®**
WARRANTY CERTIFICATE**Product** _____**Date of purchase** _____**Reseller** _____**Street** _____ **Town** _____**Sold to** _____**Street** _____ **Town** _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) - ITALY - Tel. +39 055 96 21 11

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