MOPEDIA®

MA RS975xx 02 B_ITA_10 2020

MANIGLIE DA BAGNO A VENTOSA

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ITALIANO

INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RS975-36 Suction grab bar - 36 cm

RS975-50 Suction grab bar - 50 cm

2. INTRODUCTION

Thank you for purchasing a MOPEDIA suction grab by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. INTENDED USE

The grab bar by Moretti is intended to help people with movement difficulties.

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SAFETY BATH HANDLES complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Keep the packed product away from heat sources
- The suction cup grab bar is not designed to hold or support your full body weight.
- This grab bar is designed to assist your natural motion, as when stepping into or out of a shower.
- Do not attach to moving surfaces.
- Maximum User Weight : 55 kg (120 lbs) short grab bar
- SERVICE LIFE- the product use limit is defined by the wear of the parts
- DO NOT allow children to play on or operate the bath chair
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

7. GENERAL DESCRIPTION

The suction grab bar is designed for use only on smooth and non-porous surfaces; tile, glass, porcelain or fiberglass. Not for use on wall board, wood, painted or papered surfaces. Usable in all environments where it is not permitted to drill the walls or outside the home.

8. BEFORE EACH USE

- Always check the attachment and condition of suction cup thanks to the indicator on the handle
- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly of the device

9. WARNINGS FOR USE

- You can not sit on the grab bar;
- If the grab bar is used in unfavorable conditions (eg. Suckers damaged or worn), the holding may disappear suddenly;
- Under water the grab bar holding decreases;
- The holding of the suction cups decreases in the presence of high temperatures and high altitudes;
- Keep the suckers away from direct sunlight, lose elasticity. It should not attach the handles to the outside surface illuminated type windows;
- If the handle remains attached in the same place for a prolonged period of time, to the removal, the underlying surface may remain blotchy.

10. ASSEMBLING

10.1 Assembling for short grab bar

The short grab bar is supplied already assembled.

10.2 Grab bar positioning

1. Thoroughly clean and the surface with alcohol wipe and allow to dry and check the condition of the suction cups. Do not use if damaged, cracked or split.
2. Raise both levers; place the grab bar so that both suction cups lie firmly and completely on a smooth surface.
3. Hold one end and press on one lever with your palm, You should feel resistance as the suction cup tightens.
4. Check that the handle adhered well to the surface, otherwise reposition.

10.3 Removal

1. Lift each of the two levers.
2. If suction remains, gently lift one edge with a finger.
3. Do not use a sharp instrument which may damage the suction cup.
4. Always to keep the levers flip up until next use.

10.4 Patented suction indicator

The indicator seal (patented system), consists in an alerter of the actual adhesion of the suction cup to the support thus ensuring maximum grip and safety during use.

“GREEN” means the correct attaching

Otherwise, it will turn into the “RED” request to check the mounting surface and re-attach again

Nota: Do not place over joints or grout lines.

11. TECHNICAL FEATURES

Each grab bar is made of ABS with suction cups made of PVC, they are mobile grab bars and compared to the fixed traditional ones, have several advantages:

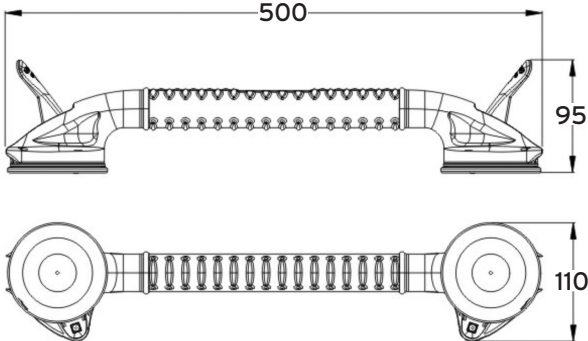
- Mounting without drilling holes;
- Horizontal, vertical and angled positioning
- They can be moved at any time and can follow the evolution of the disease.
- Quick and easy assembly and disassembly
- Available in different sizes:
 RS975-36 suction grab bar - 36 cm
 RS975-50 suction grab bar - 50 cm

11.1 Material

Handle ABS
 Suction PVC
 Spring and rivet steel

11.2 Dimension

<p>DIMENSION RS975-36</p>	
<p> MAXIMUM USER WEIGHT</p>	<p>55 KG (120 LBS) (LATERAL & VERTICAL) - 40 KG (90 LBS) (ANGLED)</p>

<p>DIMENSION RS975-50</p>	
<p> MAX LOADING WEIGHT</p>	<p>55 KG (120 LBS) (LATERAL & VERTICAL) - 40 KG (90 LBS) (ANGLED)</p>

12. CLEANING AND DISINFECTION

12.1 Cleaning

Use a damp cloth and mild soap. Then dry thoroughly before use.

12.2 Disinfection

If you need to disinfect the device use a common disinfectant cleaner.

Note: Never use acids, alkalis or solvents such as acetone.

13. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose.

The bath chair in the appropriate disposal areas for recycling.

14. SPARE PARTS AND ACCESSORIES

For spare parts only refer to MORETTI master catalogue.

15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

15.1 Repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

15.2 Spare parts

For spare parts only refer to MORETTI master catalogue.

15.3 Clause esonerative

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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