MOPEDIA®

MA RP75x 05 A_ITA_09-2021

DEAMBULATORI ALTI

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ITALIANO

INDICE

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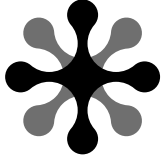
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of
5 April 2017 concerning medical devices

1. CODES

- RP755S** Painted steel demountable shoulder walker
RP755F Painted steel demountable shoulder walker with brakes
RP756S Painted steel foldable shoulder walker
RP754S Painted steel foldable shoulder walker, legs with adjustable height
RP756A Painted steel foldable walker with antibrachial support
RP754A Painted steel foldable walker with antibrachial support, legs with adjustable height
RP755T Painted steel demountable walker with antibrachial table
RP756T Painted steel foldable walker with antibrachial table
RP758S Painted steel foldable shoulder walker, legs with adjustable height
RP756M Painted steel foldable shoulder walker model mini
RP756AM Painted steel foldable walker with antibrachial support model mini

2. INTRODUCTION

Thank you for choosing a walker by MOPEDIA range, manufactured by MORETTI S.p.A. design and quality are a guarantee of comfort, safety and reliability. MOPEDIA walkers are designed and built to meet all your demands for a practical, correct use. This user manual provides you some suggestions as to how correctly use the walkers you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the walkers. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The MOPEDIA walkers by MORETTI S.p.A. are devices to be used with two arms and are intended for people with movement difficulties.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserve the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of WALKERS - MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.

3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual, checking with a particular attention at the warnings before use;
- For proper use of the device consult your doctor or therapist;
- Keep the packaged product away from any source of heat because the packaging is made of cardboard;
- The useful life of the accessory is determined by wear of parts which cannot be repaired and/or replaced;
- Be careful when children are nearby;
- Do not exceed the maximum payload.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code		Batch Lot
	Unique Device Identification		Medical Device...
	Read the instruction manual		CE mark
	Conditions of disposal		Manufacturer

7. GENERAL DESCRIPTION

The folding or demountable, shoulder or antibrachial walkers MOPEDIA, are made to support people with movement difficulties. Constructed with chromed and painted steel, with 4 castors 100mm, 2 with brakes, adjustable height, adjustable support in width and depth *(See model). The adjustments for height and width allow the use of the product by persons of small, medium or big size.

The walker can be used with a padded seat *(RP757 Optional). Available with underarm crutches or antibrachial support.

These devices facilitate the movement of people with limited static equilibrium and/or dynamic, increasing the amplitude to the supporting base. They provide support in situations of weakness and/or paralysis of the lower limbs. They help to download on the arms part of the person weight in case it is required a limitation of the load on one or both lower limbs.

7.1 To adjust height (RP754S-RP754A- RP758S)

To adjust the height, press the push button and slide the cane to the desired height. Be sure the push button is fully protruded through the desired hole when finished. Repeat described procedure for each leg, at the end, check all four legs have the same height. Maximum endurable weight 130kg.

7.2 Shoulder walkers
Demountable frame

RP755S

RP755F

Foldable frame

RP756S

RP754S

RP758S

RP756M

7.3 Antibrachial walkers

RP756A

RP754A

RP756AM

7.4 Antibrachial table walkers Demountable frame

RP755T

Foldable frame

RP756T

8. ASSEMBLY INSTRUCTIONS

8.1 Shoulder and antibrachial walkers


1. Start assembling your walker with the pole of the front frame which has to be inserted in the tubes of the lateral frame and fixed with two brass buttons.
2. *Foldable - Simply , open the frame. **RP756S - RP754S - RP754A - RP756A - RP756M - RP756AM**
*Demountable - Use the two horizontal poles to install in the back frame to strengthen the whole walker. Fix the poles with the four screw-buttons. **RP755S - RP755F**
3. Adjust the rests to the patient's desired height and fix them with the two lock stems.
4. It is easy to mount and dismount the seat which is hung by means of hooks on the poles of the lateral frame *(RP757 optional)

RP757

8.2 Walkers - RP758S

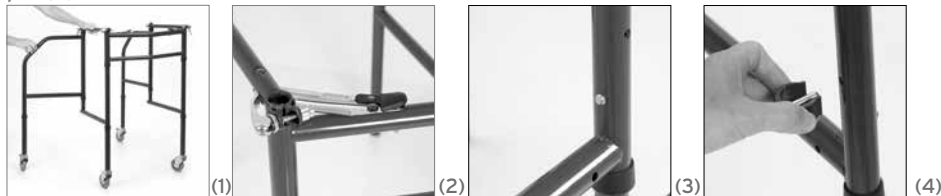1. Open the frame as in pic. (1.) Pay attention that safety buttons correctly block the sides of the walker, as in pic. (2 and 3.) Then, insert safety clips in the holes in the rear part of the walker as in pic. (4).


2. Connect legs with wheels to the frame (legs with brakes must be on the back). Safety buttons have to click when they are blocked.

NOTE Legs have to be all at the same height. Pic. (5)

3. Place the handles in the tubes and block them with lever. Pic. (6)

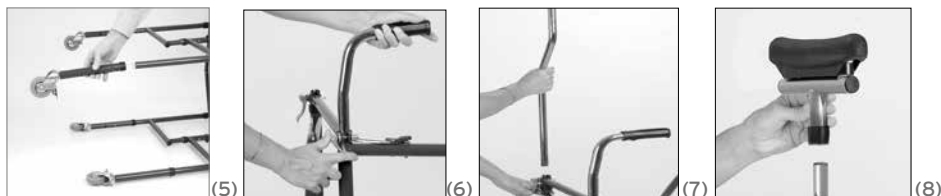NOTE If levers can not tighten enough the tube, tighten the screw nut.


4. Insert shoulder support tubes from the longer side, in the holes pic. (7) and block them with levers.

5. Insert the shoulder supports in the tubes. Pic. (8)

6. Adjust the height of the walker following the indication of the patient.
The seat can easily be installed in the walker thanks to 4 hooks.
***(RP757 optional)**


How to fold the walker:

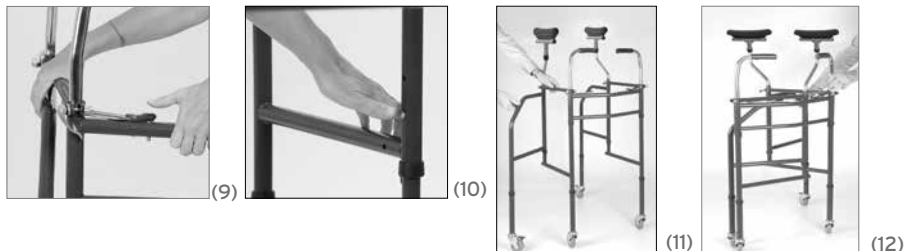1. Remove the seat.
2. Remove safety clips
3. Press the safety buttons in the front and in the rear part as in pic. (9-10) and fold the four parts inward. Pic. (11-12)

8.3 Antibrachial table walkers RP755T - RP756T

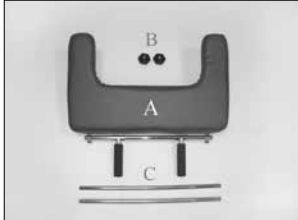1. Start assembling your walker with the pole of the back frame which has to be inserted in the tubes of the lateral frame and fixed with two brass buttons.

2. *Foldable - Simply , open the frame. **RP756T**

*Demountable - Use the two horizontal poles to install in the back frame to strengthen the whole walker. Fix the poles with the four screw-buttons. **RP755T**

3. To assembly the padded armrest, insert the poles (ref.C) into the support tubes, once they are at the same height block them with the levers on the walker.

4. Place the padded table (ref.A) and adjust the height, in order the user can rest his arms forming a 90° angle.

5. Adjust the handles at the desired opening and block them with the screw-buttons supplied with the padded table.

6. Assure that the screws are duly tighten.

7. The padded table is now correctly assembled and ready to use.

8. The seat can be easily assembled or removed thanks to the hooks on the poles of the lateral frame. *(**RP757 optional**)

9. BEFORE USE

WARNING!

- Always check the good condition of the product.
- Pay attention on the presence of electrical cables in way, wet surfaces, rugs, toys or debris, furniture, little steps and gaps in floors. Do not use the walker on stairs or escalators.
- **USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES.**
- Avoid to leave the walker in contact with water for long periods and control its sturdiness periodically.
- Keep it away from children to avoid any possible risks.

10. WARNINGS FOR USE

This device provides greater stability, support and safety while moving. In any case, it cannot be used by people with a functionality of the hand and / or arm visibly weak and even by people with balance problems. The walker can be used both outside and inside. We recommend using only on surfaces on which all four legs touch the ground. It should not be used on rocky terrain or on inclined surfaces because the walker can become unstable. The device is not designed for the transport of persons or other material.

- Use extra caution on wet and slippery surfaces.
- Do not use with patient who exceed the weight indicated in the product label and in the technical features chapter of the present manual.
- Avoid to leave the device in contact with water for long periods of time.
- Periodically check its sturdiness, the stability and the rubber tips are in good conditions. Replace them once worn out.
- In case of doubts about use, adjustment or maintenance, contact your dealer.
- Keep it away from children to avoid any possible risks.

11. HOW TO USE

The walker is a personal product, so you may want to adjust the height of the walker by specialized personnel.

Note: Refer to the safety instructions in this manual. Use the walker as aid device for walking. To do this use the two handles and push forward the aid thus obtaining a support convenient and reliable for walking.

12. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

- It is extremely important to check the walkers before each use. More specifically, please check that all screws and bolts are well tightened.
- Please, check the walkers can move freely with the wheels unlocked by the brakes.
- Check the rear wheel brakes function correctly.

13. CLEANING AND DISINFECTION

13.1 Cleaning

To clean the aid and the structure use a damp cloth with solution of alcohol or neutral soap (in equal parts) mixed with water.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacturer cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

13.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

14. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

15. SPARE PARTS ACCESSORIES


For spare parts and accessories, please always refer to the MORETTI master catalogue.

15.1 Accessories


RV7115

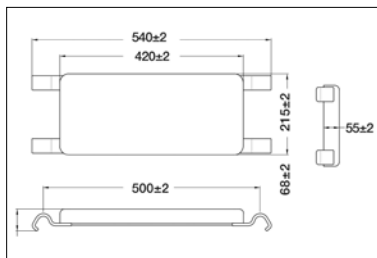Boards - PAIRS

RP757

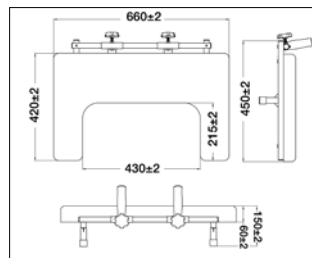Padded seat

RP759/1

Padded table

16. TECHNICAL FEATURES

16.1 Technical specifications

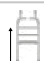
- Chrome plated and powder coated steel frame
- Castors with brake and block rotational Ø 10 cm (see models)
- Ø 15,5 cm (only RP755F rear castors)
- Maximum user's weight 130 kg

16.2 Dimension

Shoulder walkers - Demountable frame

								Usable space	
RP755S	77 cm	48 cm	81,5 cm	97,5 cm	117-151 cm	55 cm	30,5-67 cm	inside 46 cm	130 kg
RP755F	81,5 cm				119-152 cm		30,5-70cm	inside 47 cm	

Shoulder walkers - Foldable frame

								Usable space	
RP756S	79 cm	48 cm	81,5 cm	97,5 cm	117-152 cm	55 cm	30,5-67 cm	inside 46 cm	130 kg
RP754S		47-61,5 cm	82-95,5 cm	97-110,5 cm	113-127 cm				
		-						Usable space	
RP756M	79 cm	-	71,5 cm	85 cm	102,5-132,5 cm	64 cm	30,5-36,5 cm	inside 46 cm	130 kg
								Usable space	
RP758S	64 cm	47-61,5 cm	76-89,5 cm	80,5-94,5 cm	112-140 cm	48,5 cm	30,5-66,5 cm	inside 46 cm	130 kg

Antibrachial walkers Foldable frame

							Usable space	
RP754A	79 cm	47-61,5 cm	82-95,5 cm	97-110,5 cm	109-159 cm	63 cm	inside 46 cm	130 kg
RP756A		48 cm	81,5 cm	97,5 cm	109-146 cm	48,5 cm		
RP756AM		-	71,5 cm	85 cm	102,5-132,5 cm	63 cm		

Antibrachial table walkers

							Usable space 
RP755T Demountable	77 cm	48 cm	81,5 cm	97,5 cm	104-133 cm	66 cm	inside 46 cm table 42 cm 130 kg
							Usable space 
RP756T Foldable	79 cm	48 cm	81,5 cm	97,5 cm	101-135 cm	66 cm	inside 46 cm table 42 cm 130 kg

Note: in some drawings the padded seat RP757 is also shown for technical purposes only. Please note that the RP757 padded seat is sold separately as accessory.

17. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual.

The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

18. REPAIRING

18.1 Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the customer, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

18.2 Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti customer service. Labour and delivery costs for an out-of-warranty reparation will be fully at the customer's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the customer receives the repaired product.

18.3 Non defective device

The customer is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the customer and all the return delivery costs will be at his expense.

19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

20. EXEMPT CLAUSES

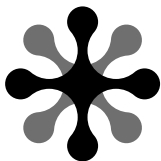
Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exoneration or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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