ITALIANO

MA CPA25x 01 A\_ITA\_03 2021

**KIT MONOGUIDA  
PER CARROZZINE A SGANCIO RAPIDO**

---

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## INDICE

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ENGLISH

MA CPA25x 01A\_ENG\_03 2021

**ONE-HAND DRIVE KIT  
FOR QUICK RELEASE WHEELCHAIRS**

---

**INSTRUCTION MANUAL**

## INDEX

<b>1. CODES</b> .....	<b>PAG.3</b>
<b>2. INTRODUCTION</b> .....	<b>PAG.3</b>
<b>3. INTENDED USE</b> .....	<b>PAG.3</b>
<b>4. GENERAL WARNING</b> .....	<b>PAG.3</b>
<b>5. SYMBOLS</b> .....	<b>PAG.4</b>
<b>6. GENERAL DESCRIPTION</b> .....	<b>PAG.4</b>
<b>7. ASSEMBLING/DISASSEMBLING</b> .....	<b>PAG.4</b>
7.1 Assembling CPA250-X.....	pag.4
7.2 Assembling CPA251-X.....	pag.6
7.3 Disassembling .....	pag.8
<b>8. BEFORE USE</b> .....	<b>PAG.8</b>
<b>9. HOW TO USE</b> .....	<b>PAG.8</b>
<b>10. MAINTENANCE</b> .....	<b>PAG.8</b>
<b>11. CLEANING AND DISINFECTION</b> .....	<b>PAG.8</b>
11.1 Cleaning.....	pag.8
11.2 Disinfection .....	pag.9
<b>12. CONDITION OF DISPOSAL</b> .....	<b>PAG.9</b>
<b>13. SPARE PARTS / ACCESSORIES</b> .....	<b>PAG.9</b>
<b>14. WARRANTY</b> .....	<b>PAG.9</b>
<b>15. REPAIRING</b> .....	<b>PAG.9</b>
15.1 Warranty repair .....	pag.9
15.2 Repair not covered by warranty.....	pag.9
15.3 Non-defective device.....	pag.9
<b>16. SPARE PARTS</b> .....	<b>PAG.10</b>
<b>17. EXEMPT CLAUSES</b> .....	<b>PAG.10</b>

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND  
OF THE COUNCIL of 5 April 2017 concerning medical devices

## 1. CODES

CPA250-1 ONE-HAND DRIVE KIT for CP102/CP103R/CP110/CP200 sizes from 38 to 43 cm;

CPA250-2 ONE-HAND DRIVE KIT for CP102/CP103R/CP110/CP200 sizes from 45 to 50 cm;

CPA251-1 ONE-HAND DRIVE KIT for CP770/CP78X/CP790 sizes from 40 to 43 cm;

CPA251-2 ONE-HAND DRIVE KIT for CP770/CP78X/CP790 sizes from 46 to 50 cm;

## 2. INTRODUCTION

Thank you for purchasing an ARDEA accessory by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advices and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

**N.B. Check that all parts of the product have not been damaged during shipment. In case of damage, do not use the product and contact your dealer for further instructions;**

## 3. INTENDED USE

The one-hand drive kit is an accessory available for ARDEA wheelchairs, intended for users with a reduced functionality to an upper limb.

### WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

## 4. GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual and the wheelchair manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur;
- Keep the packed kit away from heat sources;
- SERVICE LIFE- the wheelchair use limit is defined by the wear of the parts;
- ALWAYS keep hands and fingers clear of moving parts to avoid injury;
- DO NOT allow children to play on or operate the kit;

### WARNING!

Danger of tipping and/or deformation of the supports;

**5. SYMBOLS**

Product code

CE mark

Manufacturer

Read the instruction manual

ENGLISH

**6. GENERAL DESCRIPTION****CPA250-1/CPA250-2:**

- Pair of wheels d. 60 cm (one with double handrail);
- Telescopic bar (CPA250-1 26/36 cm, CPA250-2 34/45 cm);

**CPA251-1/CPA251-2:**

- Pair of wheels d. 60 cm (one with double handrail);
- Telescopic bar (CPA251-1 of 26/36 cm, CPA251-2 of 34/45 cm);
- Kit nr. 2 bearings + nr. 2 spacers;

**7. ASSEMBLING****7.1 Assembling CPA250-X**

1. Remove both quick release wheels that are installed on your wheelchair by pressing the central button on the wheel hub, and then pull the wheel towards you to remove it (fig. 1).

(fig. 1)

“Images for purposal only”

2. Insert one of the two supplied wheels by pressing the central button on the hub. Once the wheel axle is inserted all the way in, release the button (making sure that the ball at the top of the wheel axle has passed the wheel seat (fig. 2). Brake the wheel using the parking brake (Fig. 3).

(fig. 2)

(fig. 3)

“Images for purposal only”

3. Insert one of the two sides of the supplied telescopic bar around the previously mounted wheel axle (fig. 4).

(fig. 4)

“Images for purposal only”

4. Install the other wheel supplied, following the procedure in section 2.
5. Compress the telescopic bar and insert the free end around the axle of the second wheel mounted on the wheelchair (fig. 5).

(fig. 5)

"Images for purposal only"

6. Make sure again that the balls of the wheel axles have passed the wheel seats. Try to remove the wheels (without, of course, pressing the buttons on the wheel hubs).

### 7.2 Assembling CPA251-X

1. Remove both quick release wheels that are installed on your wheelchair by pressing the central button on the wheel hub, and then pull the wheel towards you to remove it (fig. 1).

(fig. 1)

"Images for purposal only"

2. Remove the hub supports (using nr.1 hexagonal spanner size 19, and nr.1 hexagonal spanner size 24 (fig. 6). Hexagonal spanners are not supplied).

(fig. 6)

"Images for purposal only"

3. Fit the kit of nr.2 bearings + nr.4 spacers (fig.7) in place of the hub holder supports (using nr.2 hexagonal spanners size 32, not supplied).

(fig. 7)

"Images for purposal only"

4. Insert one of the two wheels supplied by pressing the central button on the hub. Once the wheel axle is inserted all the way in, release the button (making sure that the ball at the top of the wheel axle has passed the wheel seat (fig. 2). Brake the wheel using the parking brake (Fig. 3).

(fig. 2)

(fig. 3)

"Images for purposal only"

5. Insert one of the two sides of the supplied telescopic bar around the previously mounted wheel axle (Fig. 4).

(fig. 4)

"Images for purposal only"

(fig. 5)

“Images for purposal only”

6. Mount the other wheel supplied, following the procedure in section 4.

7. Compress the telescopic bar and insert the free end around the axle of the second wheel mounted on the wheelchair (Fig. 5).

8. Make sure again that the balls of the wheel axles have passed the wheel seats. Try to remove the wheels (without, of course, pressing the buttons on the wheel hubs).

**N.B. Please refer to the safety warnings in this instruction manual.**

### 7.3 Disassembling

- Follow the previous instruction in reversed order

## 8. BEFORE USE

- Always check the state of wear of the mechanical parts involved in order to guarantee a safer use of the device.
- Check the correct assembly, in particular ensure the wheels are duly fixed to the hub supports, and the connection bar is duly locked.

## 9. HOW TO USE

1. Use both handrims to move the wheelchair straight on.
2. Use the external or the internal handrim to turn left or right.

## 10. MAINTENANCE

The ARDEA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

In case of repair, use only original spare parts and accessories.

## 11. CLEANING AND DISINFECTION

### 11.1 Cleaning

Use a damp cloth and mild soap. Then dry thoroughly before use.

**WARNING!**

Do not use abrasive substances, alkalis, acids, alcohol, chlorine-based detergents, disinfectants and acetone, because these disinfectants and acetone, as these will cause abrasion of plastic parts and rusting of metal surfaces.

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**11.2 Disinfection**

if you need to disinfect the device use a common disinfectant cleaner.

**N.B. Never use acids, alkalis or solvents such as acetone.**

**12. GENERAL CONDITIONS OF DISPOSAL**

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycling.

**13. SPARE PARTS/ACCESSORIES**

For spare parts and accessories refer to main catalogue.

**14. WARRANTY**

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

**15. REPAIRING****15.1 Warranty repair**

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

**15.2 Repair not covered by warranty**

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

**15.3 Non-defective device**

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

**16. SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

**17. EXEMPT CLAUSES**

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

ENGLISH

**WARRANTY CERTIFICATE**

**Product** \_\_\_\_\_

**Purchased on (date)** \_\_\_\_\_

**Retailer** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**Sold to** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

**MADE IN ITALY**

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11  
Fax. +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)