ITALIANO

MA CR500-X\_CR503-X\_CR505-X\_CR507-X\_CR509-X 01A\_ITA\_10-2021

**RAMPE PER DISABILI**

---

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## INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 concerning medical devices

## 1. CODES

- CR500-1** Telescopic ramps 152 cm - PAIR
- CR500-2** Telescopic ramps 213 cm - PAIR
- CR503-1** Foldable ramp 61 cm
- CR503-2** Foldable ramp 213 cm
- CR505-1** Tri-folding ramp 152 cm
- CR505-2** Tri-folding ramp 183 cm
- CR505-3** Tri-folding ramp 213 cm
- CR507-1** Fixed ramps 61 cm - PAIR
- CR507-2** Fixed ramps 183 cm - PAIR
- CR509-1** Platform ramp 61 cm
- CR509-2** Platform ramp 91 cm
- CR509-3** Platform ramp 122 cm

## 2. INTRODUCTION

Thank you for purchasing an ARDEA ramp by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

**Note:** check all parts for shipping damage and test before using. In case of damage, do not use the device. Contact your dealer for further instruction.

### WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without notice.

## 3. INTENDED USE

This ramp is intended for the removal of architectural barriers for persons with mobility difficulties and must be used with the assistance of an operator.

## 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of RAMPS - ARDEA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before using this equipment otherwise injury or damage may occur.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the ramps.
- Keep the packed ramps away from heat sources.
- The ramps use limit is defined by the wear of the parts.
- **ALWAYS** keep hands and fingers clear of moving parts to avoid injury.
- **DO NOT** allow children to play on or operate the ramps.
- Close the ramps before lifting it.
- **DO NOT** attempt to lift the ramps by any removable parts.
- The user and/or the patient will have to report any serious accident that have occurred
- Related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

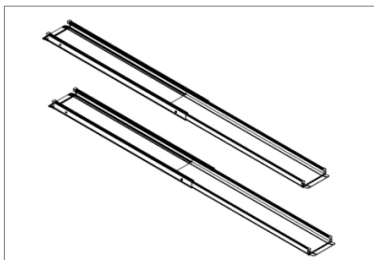
Conditions of disposal

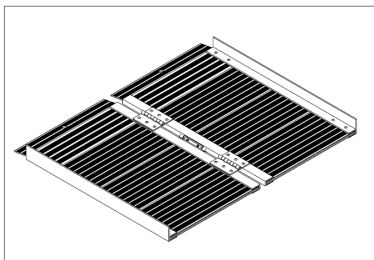
Maximum allowed weight

## 7. GENERAL DESCRIPTION

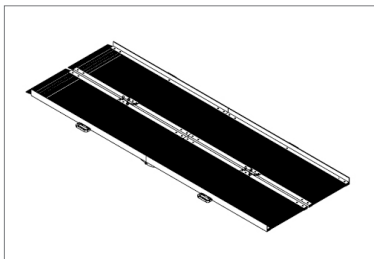
### 7.1 Parts view

**CR500-X:**

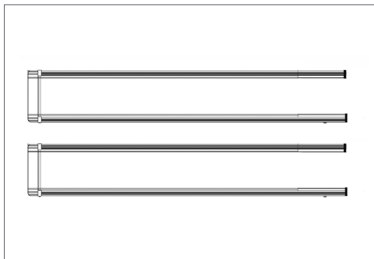
Telescopic ramp supplied in pairs available in 2 sizes (152 and 213 cm max. length)

**CR503-X:**

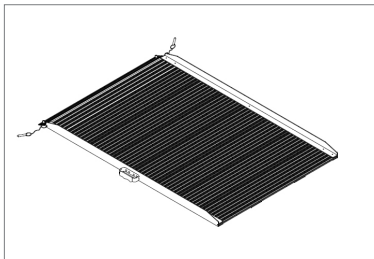
Foldable ramp in suitcase available in 2 sizes (61 and 213 cm max. length)

**CR505-X:**

Tri-folding ramp in suitcase available in 3 sizes (152, 183 and 213 cm max. length)

**CR507-X:**

Fixed ramp supplied in pairs available in 2 sizes (61 and 183 cm max. length)

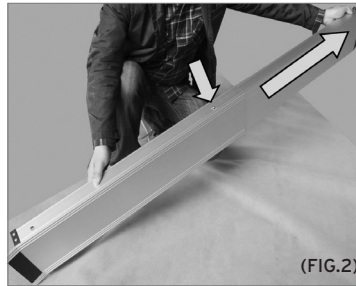
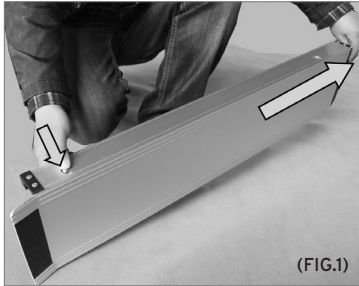
**CR509-X:**

Single platform ramp available in 3 sizes (61, 91 and 122 cm max. length)

## 8. ASSEMBLING/DISASSEMBLING

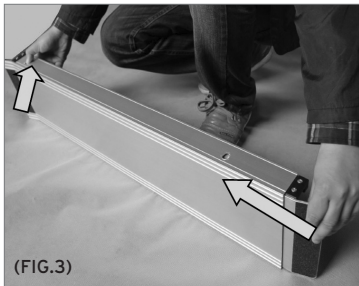
### 8.1 Opening model CR500-X

Press the button with one hand and with the other hand pull out the inner section of the footrest (fig.1) until the pressed button clicks into place (fig.2).

### 8.2 Closing model CR500-X

Press the button again with one hand and with the other hand press the inner section of the footrest towards the other hand until the situation shown in fig.3 is reached.

Make sure that the pin is correctly locked into the hole before storing/removing the footrest.

### 8.3 CR503-X | CR505-X folding models

Open the footrests before use, ensuring that they are stable and fully touching both surfaces they will be connecting. Fold them up once they have been used, so that they can be handled by the handle in the centre of the lower hinge.

## 9. BEFORE USE

- Always check the state of wear of the mechanical parts involved in order to ensure that the device can be used in complete safety for persons and property.
- Check the correct assembly of the device, paying particular attention to the correct locking of the adjustment.

## 10. USING WARNINGS

### WARNING!

THE OBSTACLE-RAMP LENGTH RATIO MUST NEVER BE LESS THAN 1:5 (e.g. for 30 cm obstacle, the ramp must be  $\geq 150$  cm)

- Use only the ramp for boarding/alighting wheelchairs.
- The ramp should be stepped on and off at low speed and without running.
- The ramp should be arranged in such a way that the curvature created at the end of the ramp is in full contact with the support area.
- The ramp must be placed on a stable base;

- The ramp should be used to overcome an obstacle up to 5 times shorter than the length of the ramp (e.g. to overcome an obstacle of 30 cm requires a ramp of at least 150 cm in length), when used by electric wheelchairs or wheelchairs guided by an assistant.  
If the disabled person has to drive up and/or down one of the ramps in this manual by himself, the obstacle-to-length ratio should not be less than 1:7 (e.g. a 30 cm high obstacle requires a ramp at least 210 cm long).
- Folding ramps should be opened away from the body.
- Care should be taken to avoid trapping fingers in the parting hinges (folding models CR503-X | CR505-X) and the open/close buttons (telescopic model CR500-X).

**WARNING!**

- On gravel, uneven ground or in wet conditions there is a risk that the rubber bases will not friction. Always adapt the conduct to the conditions of use.
- Additional loads in excess of the stated capacity may compromise the correct operation.

**11. HOW TO USE**

**Note:** make reference to “GENERAL WARNINGS” chapter.

1. Ensure proper tightening of the ramp locking pins when opened.
2. Always make sure that the non-slip bases at the ends of the ramp are properly positioned and fully adhered to the support surface.

**12. MAINTENANCE**

The ARDEA devices by Moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair use only original spare parts. Before using follow the “BEFORE USE” chapter.

Moretti ARDEA ramps require few elementary and periodic maintenance, below we list the checks to be carried out periodically:

- MONTHLY: braking capacity of rubber bases and abrasive strips.
- AS NEEDED: cleaning of the product.

**13. CLEANING AND DISINFECTION****13.1 Frame and metallic parts**

Clean with warm water and mild soap. Dry with a soft cloth in order to not scratch the painting. Clean the rubber bases and abrasive strips with warm water and mild soap. Pay attention in drying the rubber bases and abrasive strips, it's dangerous use the ramp with those parts wet.

**13.2 Disinfection**

if you need to disinfect the device use a common disinfectant cleaner.

**WARNING!**

Do not use solvent and abrasive product.

**14. CONDITIONS OF DISPOSAL**

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the ramp in the appropriate disposal areas for recycle.

**15. TECHNICAL FEATURES****15.1 Dimensions\* and weight**

CR500-1

L: 83.5÷152 cm

P: 19 cm

5,75 kg

|         |  |   |   |
|---------|--|---|---|
| CR500-2 |  L: 120÷213 cm  |  P: 19 cm    |  8 kg    |
| CR503-1 |  L: 61 cm       |  P: 36÷73 cm |  5 kg    |
| CR503-2 |  L: 213 cm      |  P: 36÷73 cm |  17,5 kg |
| CR505-1 |  L: 77÷152 cm   |  P: 36÷73 cm |  13,6 kg |
| CR505-2 |  L: 91,5÷183 cm |  P: 36÷73 cm |  16 kg   |
| CR505-3 |  L: 107÷213 cm  |  P: 36÷73 cm |  19,5 kg |
| CR507-1 |  L: 61 cm       |  P: 18,9 cm  |  1,33 kg |
| CR507-2 |  L: 183 cm      |  P: 18,9 cm  |  6,5 kg  |
| CR509-1 |  L: 61 cm       |  P: 73 cm    |  4,8 kg  |
| CR509-2 |  L: 91 cm       |  P: 73 cm    |  7 kg    |
| CR509-3 |  L: 122 cm      |  P: 73 cm    |  10 kg   |

\*The dimensions may change  $\pm 0,5$  cm

### 15.2 Technical specification

Two-section telescopic ramp:

- Security locks;
- Anti-slip surface;
- Easy to transport and stock;
- Maximum allowed weight 272Kg;
- Anti-rust anodised aluminium frame.

Folding ramps:

- Anti-rust anodised aluminium frame;
- Maximum allowed weight 272 Kg;
- Inox steel hinges;
- Rounded anti-slip milled surface;
- 2 security locks - one for each side.

### 16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

**17. REPAIRING****17.1 Warranty repair**

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

**17.2 Repair not cover by warranty**

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

**17.3 Non defective device**

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

**18. SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 months from the receiving date. For spare parts only refer to MORETTI master catalogue.

**19. EXEMPT CLAUSES**

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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