MOPEDIA®

ITALIANO

MA IM1xx 02 B_ITA_03 2021

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ENGLISH

MA IM1xx 02 B_ENG_03 2021

ELECTRICAL PATIENT HOIST HARNESSSES

INSTRUCTION MANUAL

CONTENTS

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. EU DECLARATION OF CONFORMITY	PAG.3
5. GENERAL PRECAUTIONS.....	PAG.4
6. SYMBOLS USED.....	PAG.4
7. STANDARD HARNESSES WITHOUT HEADREST IN MESH FABRIC	PAG.5
7.1 - General description	pag. 5
7.2 - Technical specifications	pag. 5
8. STANDARD HARNESS WITHOUT HEADREST IN CANVAS FABRIC	PAG.6
8.1 - General description	pag. 6
8.2 - Technical specifications	pag. 6
9. STANDARD HARNESSES WITH HEADREST IN MESH FABRIC	PAG.7
9.1 - General description.....	pag. 7
9.2 - Technical specifications.....	pag. 7
10. STANDARD HARNESSES WITH HEADREST IN CANVAS FABRIC	PAG.8
10.1 - General description	pag. 8
10.2 - Technical specifications	pag. 8
11. CONTAINMENT HARNESSES WITH SLATS IN CANVAS FABRIC	PAG.9
11.1 - General description	pag. 9
11.2 - Technical specifications	pag. 9
12. HARNESSES FOR BATHROOM AND SERVICES IN MESH FABRIC	PAG.10
12.1 - General description	pag. 10
12.2 - Technical specifications	pag. 10
13. HARNESSES FOR BATHROOM AND SERVICES WITH HEADREST IN MESH FABRIC	PAG.11
13.1 - General description	pag. 11
13.2 - Technical specifications	pag. 11
14. AMPUTEE HARNESS IN CANVAS FABRIC	PAG.12
14.1 - General description	pag. 12
14.2 - Technical specifications	pag. 12
15. HOW TO USE.....	PAG.13
16. WARNINGS ON USE.....	PAG.13
17. CLEANING	PAG.13
18. MAINTENANCE.....	PAG.13
19. DISPOSAL CONDITIONS	PAG.14
20. WARRANTY	PAG.14
21. REPAIRS	PAG.14
22. SPARE PARTS.....	PAG.14
23. NON-LIABILITY CLAUSE.....	PAG.14

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 on medical devices

1. CODES

- IM100 Standard harness without headrest in mesh fabric
- IM101 Standard harness without headrest in canvas fabric
- IM110 Standard harness with headrest in mesh fabric
- IM111 Standard harness with headrest in canvas fabric
- IM120 Containment harness with slats in canvas fabric
- IM140 Harness for bathroom and services in mesh fabric
- IM141 Harness for bathroom and services with headrest in mesh fabric
- IM150 Amputee harness in canvas fabric

2. INTRODUCTION

Thank you for choosing a harness in the MOPEDIA range by Moretti. Moretti harnesses have been designed and manufactured to satisfy all your needs, for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this entire manual carefully before you use the harness. Should you have any queries, please contact your dealer for suitable advice and assistance.

3. INTENDED USE

The MOPEDIA harnesses by Moretti S.p.A. are intended for the **movement of patients with mobility impairments**.

WARNING

- Do not use the product for any ' purpose other than that specified in this manual.
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual.
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features.

4. EU DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products made and sold by MORETTI SpA in the PATIENT HOIST HARNESSSES product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES. To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

5. GENERAL PRECAUTIONS

- Please consult this manual carefully for correct use of the device.
- Always consult your doctor or therapist for correct use of the device.
- Always sanitise the product before any reuse.
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts.
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

WARNING

Always consult the user manual of the patient hoist on which the harness is installed and strictly follow the instructions it contains.

WARNING

Always consult the user manual of the patient hoist on which the harness is installed and strictly follow the instructions it contains.

6. SYMBOLS USED

Product code

Unique device identification

CE marking

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Disposal conditions

Max Weight

Maximum washing temperature 60°C; normal process

Do not bleach

Do not iron


Do not tumble dry

7. STANDARD HARNESSES WITHOUT HEADREST IN MESH FABRIC

IM100 (available sizes S-M-L-XL)



The **IM100** harness has standard features and supports the entire body, with the exception of the head. It is suitable for the most common lifting situations. It is manufactured in a 100% polyester mesh fabric called Strongtex, which is reliable in any situation, including contact with water, as well as being tear-proof and breathable. A sturdy twill fabric belt (100% polyester) is stitched around the edge of the harness and forms a loop at each end with three different attachment points to ensure it remains symmetrical during lifting. The belt is continuous and stitched around the entire edge of the harness to make it stronger. The stitching is in highly resistant polyester thread.

7.1 - General description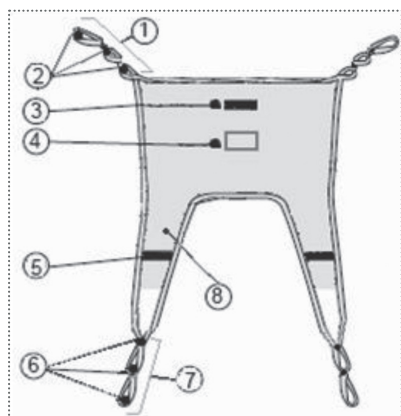

1. Upper loop
2. Positioning points
3. Handle
4. Product label with indication of size
5. Handle
6. Leg positioning points
7. Leg loop
8. Leg support

7.2 - Technical specifications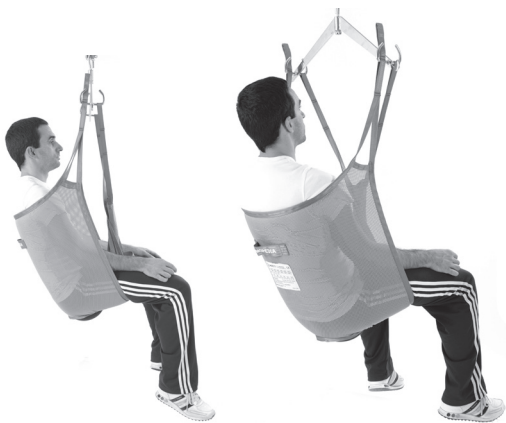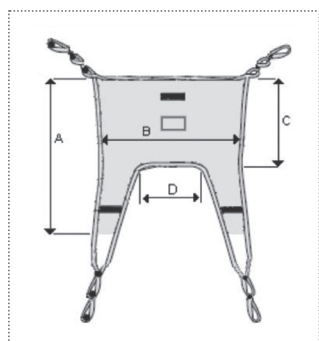


CODE	A	B	C	D	MAX. CAPACITY
IM100S	106 cm	79 cm	54 cm	20 cm	250 kg
IM100M	109 cm	86 cm	55 cm	27 cm	250 kg
IM100L	109 cm	96 cm	55 cm	37 cm	250 kg
IM100XL	111 cm	106 cm	55 cm	44 cm	250 kg

8. STANDARD HARNESS WITHOUT HEADREST IN CANVAS FABRIC

IM101 (available sizes S-M-L-XL)


The IM101 harness has standard features and supports the entire body, with the exception of the head. It is suitable for the most common lifting situations. It is manufactured in a 100% polyester fabric called Slipstex, which is highly resistant, slip-proof, tear-proof and breathable. A sturdy twill fabric belt (100% polyester) is stitched around the edge of the harness and forms a loop at each end with three different attachment points to ensure it remains symmetrical during lifting. The belt is continuous and stitched around the entire edge of the harness to make it stronger. The stitching is in highly resistant polyester thread.

8.1 - General description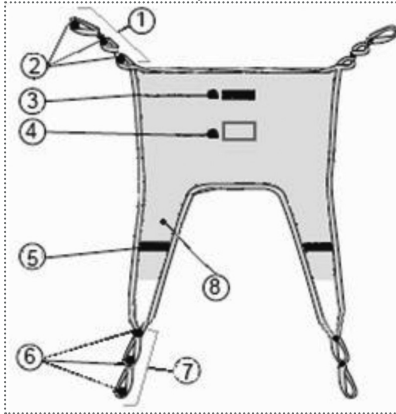

1. Upper loop
2. Positioning points
3. Handle
4. Product label with indication of size
5. Handle
6. Leg positioning points
7. Leg loop
8. Leg support

8.2 - Technical specifications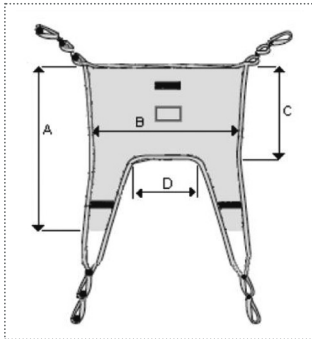


CODE	A	B	C	D	MAX. CAPACITY
IM101S	106 cm	79 cm	54 cm	20 cm	250 kg
IM101M	109 cm	86 cm	55 cm	27 cm	250 kg
IM101L	109 cm	96 cm	55 cm	37 cm	250 kg
IM101XL	111 cm	106 cm	55 cm	44 cm	250 kg

9. STANDARD HARNESSES WITH HEADREST IN MESH FABRIC

IM110 (available sizes S-M-L-XL)



The IM110 harness has standard features and supports the entire body, including the head. It is suitable for the most common lifting situations. It is manufactured in a 100% polyester mesh fabric called Strongtex, which is reliable in any situation, including contact with water, as well as being tear-proof and breathable. A sturdy twill fabric belt (100% polyester) is stitched around the edge of the harness and forms a loop at each end with three different attachment points to ensure it remains symmetrical during lifting. The belt is continuous and stitched around the entire edge of the harness to make it stronger. The stitching is in highly resistant polyester thread.

9.1 - General description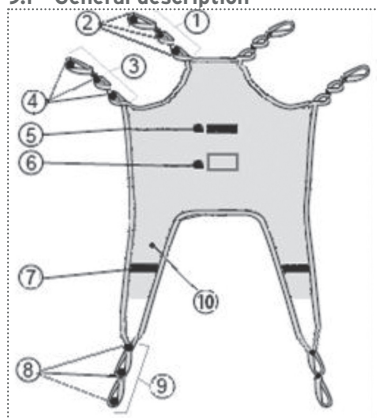

- 1. Head support loop
- 2. Positioning points
- 3. Shoulder loop
- 4. Positioning points
- 5. Handle
- 6. Product label with indication of size
- 7. Handle
- 8. Leg positioning points
- 9. Leg loop
- 10. Leg support

9.2 - Technical specifications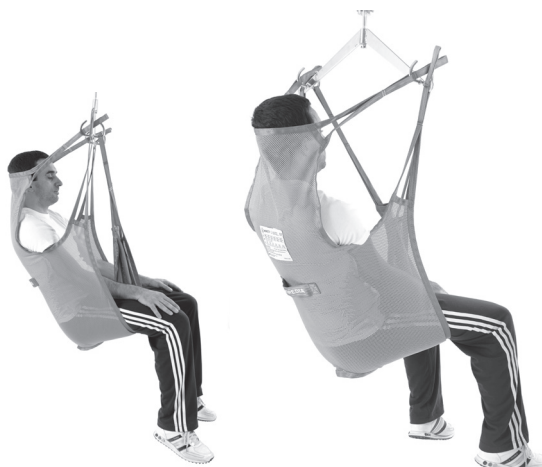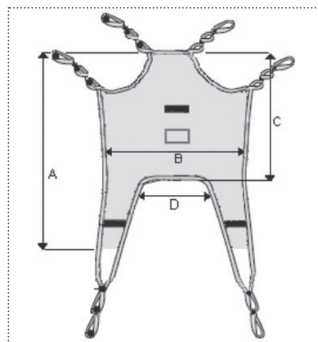


CODE	A	B	C	D	MAX. CAPACITY
IM110S	140 cm	77 cm	85 cm	20 cm	250 kg
IM110M	144 cm	84 cm	91 cm	27 cm	250 kg
IM110L	144 cm	94 cm	91 cm	37 cm	250 kg
IM110XL	144 cm	104 cm	91 cm	44 cm	250 kg

10. STANDARD HARNESSES WITH HEADREST IN CANVAS FABRIC

IM111 (available sizes S-M-L-XL)


The IM111 harness has standard features and supports the entire body, with the exception of the head. It is suitable for the most common lifting situations. It is manufactured in a 100% polyester fabric called Slipstex, which is highly resistant, slip-proof, tear-proof and breathable. A sturdy twill fabric belt (100% polyester) is stitched around the edge of the harness and forms a loop at each end with three different attachment points to ensure it remains symmetrical during lifting. The belt is continuous and stitched around the entire edge of the harness to make it stronger. The stitching is in highly resistant polyester thread.

10.1 - General description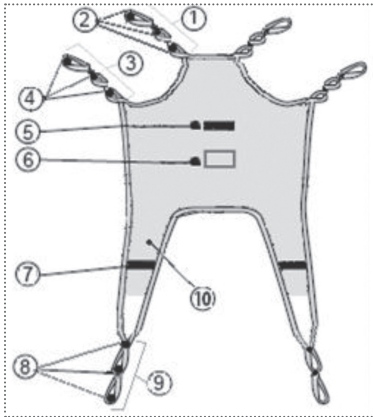

1. Head support loop
2. Positioning points
3. Shoulder loop
4. Positioning points
5. Handle
6. Product label with indication of size
7. Handle
8. Leg positioning points
9. Leg loop
10. Leg support

10.2 - Technical specifications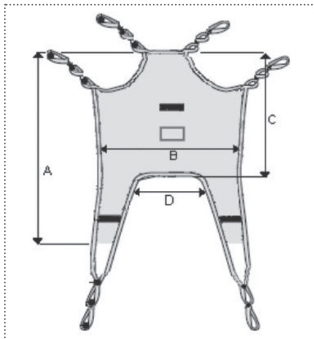


CODE	A	B	C	D	MAX. CAPACITY
IM111S	140 cm	77 cm	85 cm	20 cm	250 kg
IM111M	144 cm	84 cm	91 cm	27 cm	250 kg
IM111L	144 cm	94 cm	91 cm	37 cm	250 kg
IM111XL	144 cm	104 cm	91 cm	44 cm	250 kg

11. CONTAINMENT HARNESSES WITH SLATS IN CANVAS FABRIC

IM120 (available sizes M-L-XL)



The **IM120** harness has standard features and supports the entire body, including the head. It has polycarbonate supports for the head and lower back area. It is suitable for the most common lifting situations. It is manufactured in a 100% polyester fabric called Slipstex, which is highly resistant, slip-proof, tear-proof and breathable. A sturdy twill fabric belt (100% polyester) is stitched around the edge of the harness and forms a loop at each end with three different attachment points to ensure it remains symmetrical during lifting. The belt is continuous and stitched around the entire edge of the harness to make it stronger. The stitching is in highly resistant polyester thread.

11.1 - General description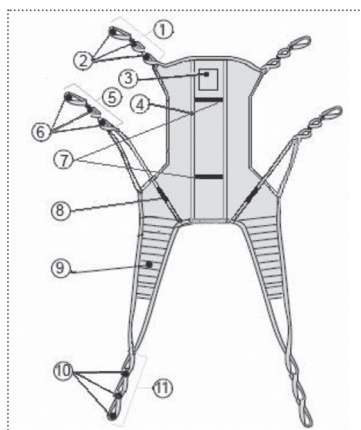

- 1. Shoulder loop
- 2. Positioning points
- 3. Product label with indications of size
- 4. Belt with removable inserts
- 5. Mid-belt loop
- 6. Positioning points
- 7. Handles
- 8. Handles
- 9. Padded leg support
- 10. Leg positioning points
- 11. Leg loop

11.2 - Technical specifications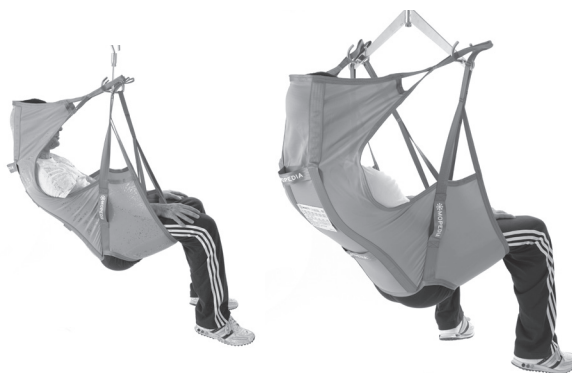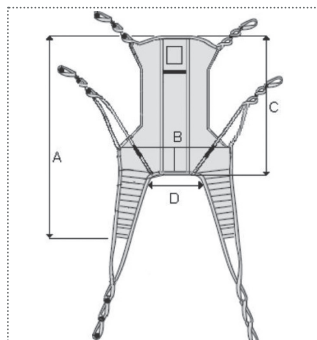


CODE	A	B	C	D	MAX. CAPACITY
IM120M	137 cm	100 cm	91 cm	40 cm	250 kg
IM120L	137 cm	110 cm	91 cm	50 cm	250 kg
IM120XL	145 cm	120 cm	91 cm	56 cm	250 kg

12. HARNESSES FOR BATHROOM AND SERVICES IN MESH FABRIC

IM140 (available sizes S-M-L-XL)


The IM140 harness is used for lifting, transfer and personal hygiene operations. It is manufactured in a 100% polyester mesh fabric called Strongtex, which is reliable in any situation, including contact with water, as well as being tear-proof and breathable. A sturdy twill fabric belt (100% polyester) is stitched around the edge of the harness and forms a loop with various attachment points to ensure it remains symmetrical during lifting. The belt is continuous and stitched around the entire edge of the harness to make it stronger. The stitching is in highly resistant polyester thread. Soft padding provides comfortable, underarm support.

12.1 - General description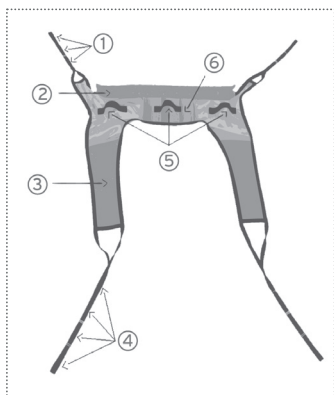

1. Upper loop with positioning points
2. Underarm padding
3. Padded leg support
4. Lower loop with positioning points
5. Handles
6. Padding

12.2 - Technical specifications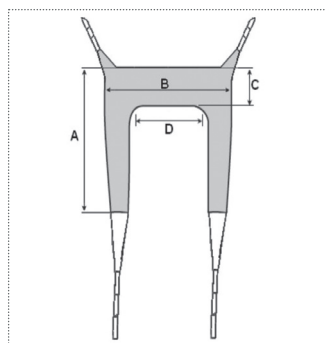


CODE	A	B	C	D	MAX. CAPACITY
IM140S	83 cm	87 cm	22 cm	42 cm	250 kg
IM140M	87 cm	95 cm	25 cm	50 cm	250 kg
IM140L	90 cm	103 cm	29 cm	58 cm	250 kg
IM140XL	95 cm	112 cm	32 cm	66 cm	250 kg

13. HARNESSES FOR BATHROOM AND SERVICES WITH HEADREST IN MESH FABRIC

IM141 (available sizes S-M-L-XL)


The IM141 harness with head support is used for lifting, transfer and personal hygiene operations. It is manufactured in a 100% polyester mesh fabric called Strongtex, which is reliable in any situation, including contact with water, as well as being tear-proof and breathable. A sturdy twill fabric belt (100% polyester) is stitched around the edge of the harness and forms a loop with various attachment points to ensure it remains symmetrical during lifting. The belt is continuous and stitched around the entire edge of the harness to make it stronger. The stitching is in highly resistant polyester thread. Soft padding provides comfortable, underarm support.

13.1 - General description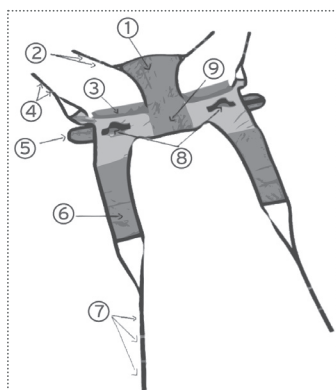

1. Head support
2. Head support loop with positioning points
3. Underarm padding
4. Head support loop with positioning points
5. Belt with Velcro fastening
6. Padded leg support
7. Leg support loop with positioning points
8. Handles
9. Padding

13.2 - Technical specifications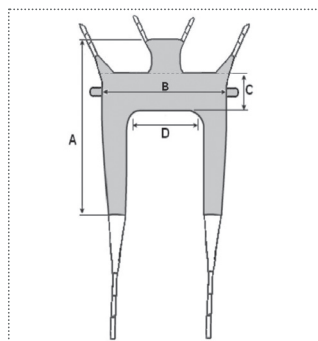


CODE	A	B	C	D	MAX. CAPACITY
IM141S	123 cm	88 cm	22 cm	42 cm	250 kg
IM141M	130 cm	96 cm	25 cm	50 cm	250 kg
IM141L	135 cm	104 cm	29 cm	58 cm	250 kg
IM141XL	140 cm	110 cm	32 cm	66 cm	250 kg

14. AMPUTEE HARNESS IN CANVAS FABRIC

IM150 (available sizes M-L)


The IM150 harness is used for lifting patients who have had one or both of their legs amputated. It is important for the user to have good neck control. The harness is manufactured in a 100% polyester fabric called Slipstex, which is highly resistant, slip-proof, tear-proof and breathable. A sturdy twill fabric belt (100% polyester) is stitched around the edge of the harness and forms a loop with various attachment points to ensure it remains symmetrical during lifting. The belt is continuous and stitched around the entire edge of the harness to make it stronger. There are two guide rings to ensure correct positioning of the leg supports. The stitching is in highly resistant polyester thread. Soft padding provides comfortable support for both the back and thighs.

14.1 - General description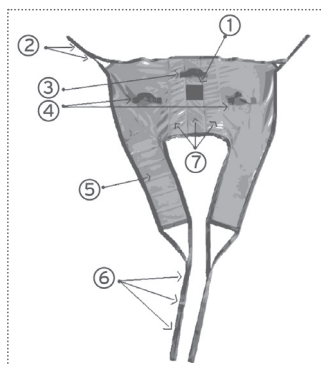

1. Label
2. Upper loop with positioning points
3. Handles
4. Guide rings
5. Padded leg support
6. Leg support loop with positioning points
7. Padded central support

14.2 - Technical specifications

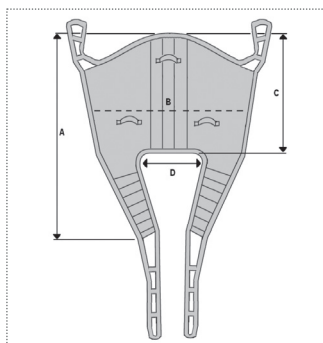CODE	A	B	C	D	MAX. CAPACITY
IM150M	110 cm	106 cm	60 cm	32 cm	250 kg
IM150L	120 cm	114 cm	64 cm	40 cm	250 kg

15. HOW TO USE

The harnesses in the MOPEDIA range by Moretti S.p.A. are suitable for use with both fixed and mobile lifters.

WARNING

Always consult the user manual of the patient hoist on which the harness is installed and strictly follow the instructions it contains.

WARNING

If harnesses with a different maximum load to the one envisaged for the hoist are used, always use the lowest maximum load

WARNING

Carry out a risk assessment to ensure that the body support unit used for the patient is of the correct dimensions, type and shape.

16. WARNINGS ON USE

- Before use, ensure that the device is not damaged: the fabric, belt, stitching and suspension loops should all be in perfect condition.
- Inspect periodically to check for signs of damage, wear and tear or potential sagging, at least every six months, particularly when it has to be cleaned more frequently than usual.
- Check that the harness loops are properly attached to the hoist.
- Store and use away from open flames and sources of heat.
- Never exceed the maximum load.

17. CLEANING

Wash in water at a maximum temperature of 60° with mild soaps or detergents. For models with polycarbonate slats, remove the slats before washing.

Maximum washing temperature 60°C; normal process

Do not bleach

Do not iron

Do not tumble dry

18. MAINTENANCE

The devices in the MOPEDIA range by Moretti have been carefully checked and CE marked when placed on sale. For patient and doctor safety, have the product checked at least every six months by the manufacturer or by an authorised laboratory to verify its suitability for use.

19. DISPOSAL CONDITIONS

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

20. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject to the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual.

Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual.

Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual.

Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

21. REPAIRS

- Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

- Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

- Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

22. SPARE PARTS

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

23. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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