KYARA®

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KYARA®

Compact compressor nebulizer
INSTRUCTION MANUAL

INDEX

1. Introduction	2
2. Product Identification	2
3. Important Safety Measures	3
4. Operation of the Compressor Nebulizer	3
5. Cleaning	4
6. Specifications	5
7. Explanation of Symbols	6
8. Disposal Conditions	6
9. Warranty	7


1. INTRODUCTION

Thank you for purchasing our compressor nebulizer. The device is a compact medical device designed to effectively deliver doctor-prescribed medication along the bronchial cavities. With proper care and use, the device will provide many years of effective treatment.

This device is designed to treat asthma, allergies and other respiratory problems. It releases a stream of air through the clean aerosol tube, once inside this stream converts the medicine present into a cloud that can be easily inhaled.

The nebulizer must be used under the supervision of a licensed physician or respiratory therapist. The user is encouraged to read the contents of this manual carefully to learn how to operate the device. Any use of the product other than its intended use is always discouraged.

2. PRODUCT IDENTIFICATION

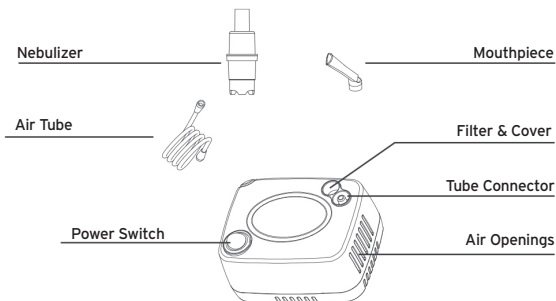3. IMPORTANT SAFETY NOTES

Note: Read the operating instructions carefully before starting treatment.

The following precautions are necessary when using an electronic device:

Warning: Failure to read and observe the warnings may result in personal injury or damage to equipment.

Device Warnings:

1. To avoid electric shock: Keep the unit away from water.
Do not immerse the power cord or the unit in liquids. Do not use the unit while bathing.
If the unit falls into water, do not attempt to pick it up unplug it immediately.
2. Never operate the device if it has damaged parts (including the power cord), if it has been dropped or immersed in water. Promptly send the device to a service center for inspection and repair.
3. The unit must not be used in the presence of flammable gases, oxygen, or aerosol products.
4. Unplug the unit from the power outlet before cleaning, filling, or after each use.
5. Do not use any accessories other than those recommended by the manufacturer.

Operating Warnings:

1. Connect the product to a power outlet suitable for the model in use.
2. Do not operate the product without supervision.
3. Never operate the device if the cord or plug is damaged, if it has been dropped or immersed in water, or if it is not functioning properly. Return the device to a service center for repair.
4. If any malfunctions occur, stop using the unit immediately until it has been inspected and repaired.
5. Always unplug the product immediately after use.
6. Never block the air openings of the main unit and do not place it in a location where the air openings could be obstructed.

Storage Precautions:

1. Do not store the unit in direct sunlight, at high temperatures, or in humid environments.
2. Keep the unit out of the reach of children.
3. Always keep the unit unplugged when not in use.

Cleaning Precautions:

1. Do not immerse the unit in water. This could damage the unit.
2. Unplug the unit from the power outlet before cleaning.
3. Clean all necessary parts after each use as indicated in this guide.

4. OPERATION

Note: Before first use, the nebulizer must be thoroughly cleaned.

1. Place the compressor nebulizer on a flat, stable surface.
Ensure that the controls are easily accessible while seated.
2. Remove all accessories from the carrying bag.

Important: Before first use, the nebulizer must be thoroughly cleaned by following the "Cleaning Procedures" outlined in the manual.

3. Gently twist the top part of the nebulizer counterclockwise to disassemble it.
4. Fill the bottom part of the nebulizer with the medication prescribed by your doctor.
Make sure the cone is inserted into the bottom section.
5. Gently twist the top part clockwise to reassemble the nebulizer.
Ensure the two sections are securely fitted together.
6. Connect one end of the air tube to the base of the nebulizer.
7. Connect the other end of the air tube to the air tube connector located on the front of the compressor.
8. Attach the mouthpiece or mask of your choice to the top of the nebulizer.
9. Plug the power cord into a suitable electrical outlet. Ensure that the power switch is in the "OFF" position at this stage.
10. Press the power switch to begin the prescribed treatment.

**Important:**

The compressor motor is equipped with thermal protection that will shut off the unit before it overheats.

If the thermal protection shuts down the unit, please:

- a. Turn off the unit.
 - b. Unplug the unit from the power outlet.
 - c. Wait 30 minutes for the motor to cool down before starting another treatment.
 - d. Make sure the air vents are not blocked.
11. At the end of the treatment, turn off the unit and unplug it from the power outlet.

5. CLEANING

It is recommended to thoroughly rinse the nebulizer, mouthpiece, and mask with warm water after each use, and clean them with mild detergent after the final treatment of the day. If your physician or respiratory therapist provides different cleaning instructions, always follow their guidance.

Rinsing (After Each Treatment)

1. Disconnect the air tube, nebulizer chamber, mouthpiece, and mask.
2. Gently twist the nebulizer chamber to open it.
3. Rinse the chamber, mouthpiece, and mask thoroughly with water.
4. Dry all components using a clean, soft towel or allow them to air dry completely.
5. Reassemble the nebulizer chamber only when it is completely dry, and store all components in a dry, sealed container.

Disinfection

Follow the steps below to disinfect the nebulizer, unless otherwise instructed by your physician. It is recommended to disinfect the device after the final treatment of the day.

1. Prepare a solution consisting of one part white vinegar and three parts distilled water.
Ensure that the volume of the solution is sufficient to fully submerge the nebulizer chamber,

mouthpiece, and mask.

2. Complete rinsing steps 1-3.
3. Wash the nebulizer chamber, mouthpiece, and mask using warm water and a mild detergent. Then rinse thoroughly with warm tap water.
4. Submerge all components in the vinegar-water solution for 30 minutes.
5. Complete rinsing steps 3-5.

Compressor Cleaning

1. Wipe the exterior of the compressor daily using a damp cloth.
2. Do not use abrasive cleaners or bar soap, as these may damage the surface finish.

Filter Replacement

1. Do not use cotton or alternative materials. Do not wash or clean the filter.
2. Replace the filter every 30 days or sooner if it becomes gray.
3. Replacement Procedure:
 - A. Remove the filter cover.
 - B. Replace the used filter with a new one.
 - C. Reattach the filter cover.

6. TECHNICAL DATA

Model	LTK210 (S600ME)
Power	AC 220V 60Hz, 180VA
Operating Pressure Range	Approx. 7.0 ~ 13.0 Psi/48.0 ~ 89.0 Kpa
Flow rate (L/min)	Approx. 3.0 ~ 6.0 lpm
Nebulization rate	≥ 0.20 ml/min
Respirable fraction and particle size	Approx. (72.68±0.39)% range 0.5-5 µm
MMAD	2.93 ± 0.03 µm
Cup capacity	6 ml
Noise	≤ 65 dBA (acc.to DIN EN 13544-1 section 26)
Service Life	1000 h
Residual volume	0.8 g
Operating mode	30 mins ON, 30 mins OFF
Operating temperature and humidity	10~40°C, 30 ~ 85 % RH, 800~1060hPa
Storage and transport	-20~70°C, 20 ~ 75 % RH, 800~1060hPa
Safety	Type BF ClassII
Dimension	Approx. L: 125 x W 94 x H: 75 mm
Weight	Approx. 0.73 kg
Standard accessories	Adult mask, child mask, nebulizer chamber, mouthpiece, air tube, filters

7. SYMBOL CLASSIFICATION AND EXPLANATION

	Class II equipment as per IEC 60601-1
	Applied part BF as per IEC 60601-1
IP21	IP21 protection
	Check user manual
	Manufacturer
	Date of manufacture
SN	Serial number
LOT	Batch number
	Disposal of Electrical & Electronic Equipment(WEEE): Do not treat the appliance as domestic waste
	CE marking: complies with the requirements of the Medical Device Directive 93/42/EEC
EC REP	Authorised European Representative

8. DISPOSAL CONDITIONS

When disposing of the device, never use regular municipal solid waste disposal systems. Instead, it is recommended to dispose of the device at local recycling centers for the proper recycling of the materials used.

Warnings for Proper Disposal of the Product in accordance with European Directive 2012/19/EU

At the end of its useful life, the product must not be disposed of with municipal waste. It can be handed over to the appropriate recycling centers provided by local authorities or to retailers who offer this service. Separately disposing of the product helps avoid potential negative consequences for the environment and health due to improper disposal and allows the recovery of the materials it is made of, leading to significant savings in energy and resources. To emphasize the obligation of separately disposing of medical devices, the product features the crossed-out wheeled bin symbol.

9. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly

followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

Repair

Warranty repairs

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

Spare parts

For spare parts only refer to MORETTI master catalogue.

Disclaimer

Beyond the specifications of this warranty and within the limits of law, Moretti makes no other express or implied representations, warranties or conditions with respect to merchantability, fitness for particular purposes, non-infringement and non-interference. Moretti does not warrant that the use of the Moretti product will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law is limited to the warranty period in accordance with the limits of the law. Some states or countries do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with consumer products. In such states and countries, some exclusions or limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE****Product** _____**Purchased on (date)** _____**Retailer** _____**Address** _____ **Town/city** _____**Sold to** _____**Address** _____ **Town/city** _____

Dongguan Aidisy Machinery & Electronic Equipment Co, Ltd
Part B, 3rd F, Block A, Wentang Industrial Park, Longhua Road. Zhouwu Dongheng
District, Dongguan City, Guangdong Province, 523000, P.R. China

**MedPath GmbH****Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany****Imported and distributed by:**

Moretti S.p.a. Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11
www.morettispa.com email: info@morettispa.com