SKEMA[®]FORM

MA MI425-MI426-MI427-MI432-MI433-MI435-MI436-MI437 04 A_ITA_02 2021

PIANTANA PER IPODERMOCLISI

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ITALIANO

INDICE

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Via _____ Località _____

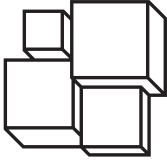
Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN ITALY**

Via Bruxelles, 3 - Melegnano
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com

SKEMA[®]FORM

MA MI425-MI426-MI427-MI432-MI433-MI435-MI436-MI437 04 A_ENG_02 2021

I.V. POLES

INSTRUCTION MANUAL

INDEX

1. CODES		PAG.3
2. INTRODUCTION		PAG.3
3. INTENDED USE		PAG.3
4. DECLARATION OF CONFORMITY		PAG.3
5. GENERAL WARNINGS		PAG.4
6. SYMBOLS		PAG.4
7. GENERAL DESCRIPTION		PAG.5
7.1 Parts view and description Aluminium I.V. Poles MI432, MI433	pag.5	
7.2 Parts view and description stainless steel I.V. poles MI435, MI436	pag.5	
7.3 Parts view and description stainless steel I.V. MI437	pag.5	
7.4 Parts view and description of Stainless Steel I.V. Pole with chromed metal base MI425	pag.6	
7.5 Parts view and description of Stainless Steel I.V. Pole with Stainless Steel tubular base MI426	pag.6	
7.6 Parts view and description of Stainless Steel I.V. Pole with Stainless Steel plate base MI427	pag.6	
8. ASSEMBLING		PAG.7
8.1 Assembling instructions - MI432, MI433, MI435, MI436	pag.7	
8.2 Assembling instructions MI437, MI425	pag.7	
8.3 Assembling Instructions MI426	pag.8	
8.4 Assembling Instructions MI427	pag.8	
9. HOW TO USE		PAG.9
10. CLEANING AND DISINFECTION		PAG.9
10.1 Cleaning.....	pag.9	
10.2 Disinfection	pag.9	
11. CONDITIONS OF DISPOSAL		PAG.9
12. ACCESSORIES		PAG.9
13. TECHNICAL FEATURES		PAG.9
13.1 Dimensions and weight.....	pag.9	
14. WARRANTY		PAG.10
15. REPAIRING		PAG.10
15.1 Warranty repair	pag.10	
15.2 Repair not covered by warranty.....	pag.10	
15.3 Non defective device	pag.11	
16. SPARE PARTS		PAG.11
17. EXEMPT CLAUSES		PAG.11

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

- MI425** Stainless steel I.V. Poles with 4 stainless steel Hooks, and chromed metal base, with castors
- MI426** Stainless steel I.V. Poles with 4 stainless steel Hooks, and tubular stainless steel base, with castors
- MI427** Stainless steel I.V. Poles with 4 stainless steel Hooks, and stainless steel plate base, with castors
- MI432** Aluminium I.V. Poles with 2 Plastic Hooks and castors
- MI433** Aluminium I.V. Poles with 4 Plastic Hooks and castors
- MI435** Stainless steel I.V. Poles with 2 Plastic Hooks and castors
- MI436** Stainless steel I.V. Poles with 2 Plastic Hooks and castors
- MI437** Stainless steel I.V. Poles with 4 stainless steel Hooks and castors

2. INTRODUCTION

Thank you for choosing a I.V. Poles from the SKEMAFORM by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. I.V. Poles from the SKEMAFORM by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the item you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the I.V. Poles. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The I.V. Poles for hypodermoclysis SKEMAFORM is a support device medical and health activities during administration to the patient of intravenous fluids.

WARNING!

- Do not use the product for a purpose not indicated in this manual;
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product;
- The manufacturer reserve the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.a. declares under its sole responsibility that the product made and traded by Moretti S.p.a. and belonging to the group of I.V. STAND complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.a. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- For proper use of the device refer to this manual;
- Keep the packaged product away from any source of heat because the packaging is made of cardboard;
- The useful life of the accessory is determined by wear of parts cannot be repaired and/or replaced;
- Be careful when children are nearby.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

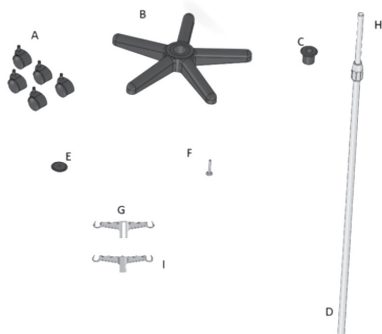
Medical Device

Conditions of disposal

7. GENERAL DESCRIPTION


7.1 Parts view and description Aluminium I.V. Poles MI432, MI433

Parts list:

- A. Base casters (5pcs)
- B. Base (1pc)
- C. Pole connection (1pc)
- D. Extendible pole, lower part (1pc)
- E. Plastic washer (1pc)
- F. Screw with washer for pole (1pc)
- G. Plastic hook female (1pc) *(Only MI433)
- H. Extendible pole, upper part (1pc)
- I. Plastic hook male (1pc)

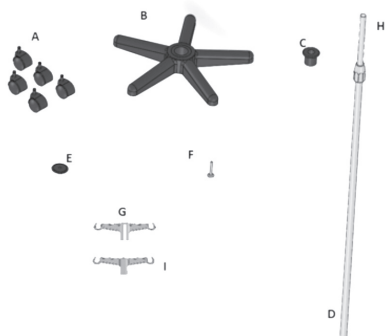
7.2 Parts view and description stainless steel I.V. poles MI435, MI436

Parts list:

- A. Base casters (5pcs)
- B. Base (1pc)
- C. Pole connection (1pc)
- D. Extendible pole, lower part (1pc)
- E. Plastic washer (1pc)
- F. Screw with washer for pole (1pc)
- G. Plastic hook female (1pc) *(Only MI436)
- H. Extendible pole, upper part (1pc)
- I. Plastic hook male (1pc)

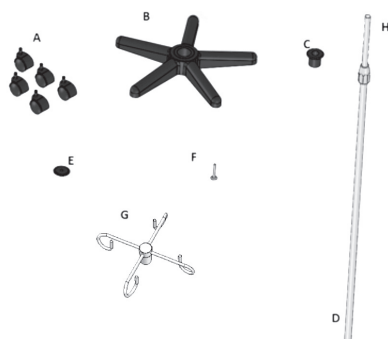
7.3 Parts view and description stainless steel I.V. MI437

Elenco parti:

- A. Base casters (5pcs)
- B. Base (1pc)
- C. Pole connection (1pc)
- D. Extendible pole, lower part (1pc)
- E. Plastic washer (1pc)
- F. Screw with washer for pole (1pc)
- G. Stainless steel hook (1pc)
- H. Extendible pole, upper part (1pc)

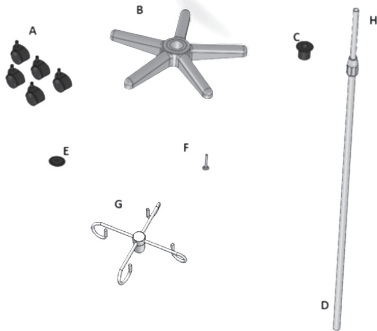
7.4 Parts view and description of Stainless Steel I.V. Pole with chromed metal base MI425

Parts List:

- A. Base castors (5pcs)
- B. Chromed Metal Base (1pc)
- C. Pole connection (1pc)
- D. Extendible pole, lower part (1pc)
- E. Plastic washer (1pc)
- F. Screw with washer for pole (1pc)
- G. Stainless steel hook (1pc)
- H. Extendible pole, upper part (1pc)

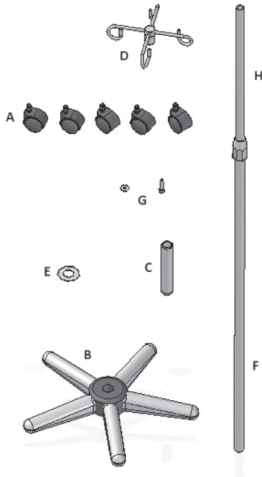
7.5 Parts view and description of Stainless Steel I.V. Pole with Stainless Steel tubular base MI426

Parts List:

- A. Base castors (5pcs)
- B. Stainless Steel tubular Base with weldings cover(1pc)
- C. Stainless Steel Pole connection (1pc)
- D. Stainless steel hook (1pc)
- E. Stainless Steel Ring (1pc)
- F. Extendible pole, lower part (1pc)
- G. Screw with washer for pole (1pc)
- H. Extendible pole, upper part (1pc)

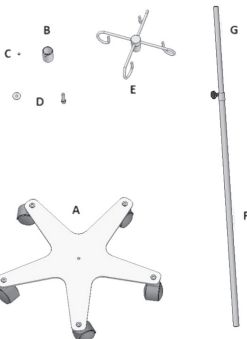7.6 Parts view and description of Stainless Steel I.V. Pole with Stainless Steel plate base MI427

Parts List:

- A. Stainless Steel plate Base with castors and fixing bushings + spacers for each castor (1pc)
- B. Stainless Steel Pole connection (1pc)
- C. Grab Screw for Pole (1 pc)
- D. Screw with washer for pole (1pc)
- E. Stainless steel hook (1pc)
- F. Extendible pole, lower part (1pc)
- G. Extendible pole, upper part (1pc)

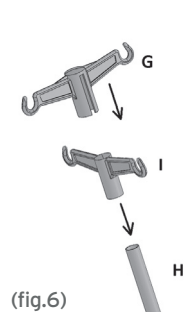
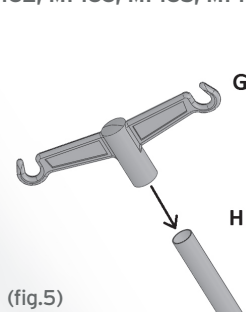
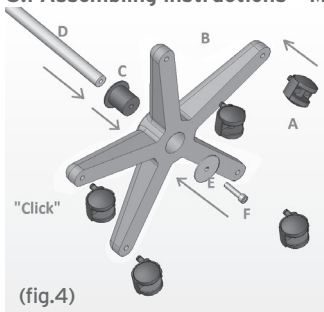
8. ASSEMBLING

ATTENTION!




During the assembling pay attention to avoid any injury to persons and/or damage to objects.

8.1 Assembling instructions - MI432, MI433, MI435, MI436

1. Insert the casters (rif.A) in the lower part of the base (rif.B) until you hear a "click". (Fig.4)
2. Insert the pole connection (rif.C) on the upper hole of the base. (Fig.4)
3. Insert the pole (rif.D) in the connection (rif.C). (Fig.4)
4. Block the pole using the screw (rif.F) and its washer (rif.E). (Fig.4)
5. Tighten the screw. (Fig.4)
6. Insert the female hook (rif.G) (Only for MI433- MI436 (Fig.5) in the male hook (rif.I) and both in upper hole of the pole (rif.H) with a light pressure. (Fig.6)
7. Now the I.V. pole is ready to use.

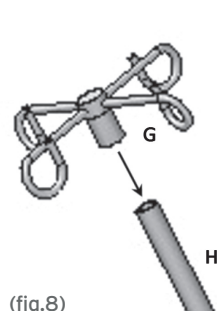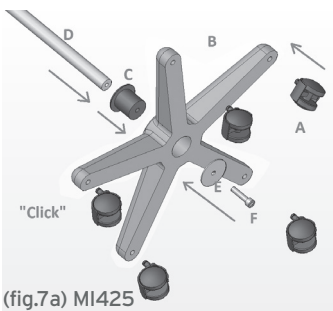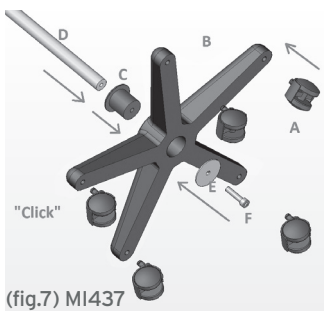
8.2 Assembling instructions MI437, MI425


1. Insert the casters (A) in the lower part of the base (B) until you hear a "click"
2. Insert the pole connection (C) on the upper hole of the base
3. Insert the pole (D) in the connection (C)
4. Block the pole using the screw (F) and its washer (E)
5. Tighten the screw
6. Insert the hook (G) into upper hole of the pole (H) with a light pressure
7. Now the I.V. pole is ready to use

Images only illustration purposes

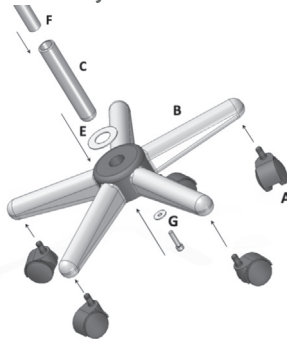8.3 Assembling Instructions MI426


(fig.9)

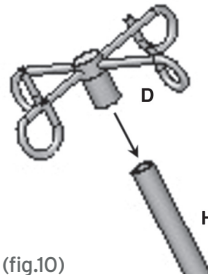
(fig.10)

1. Insert the castors (A) in the lower part of the base (B) until you hear a “click”
2. Lay down the stainless steel ring (E) on the weldings cover of the base. Insert the stainless steel connection adapter (C) on the central hole of the base, through the ring
3. Insert the pole (F) in the connection adapter (C)
4. Block the pole using the screw and its washer (G)
5. Tighten the screw
6. Insert the hook (D) into upper hole of the pole (H) with a light pressure
7. Now the I.V. pole is ready to use

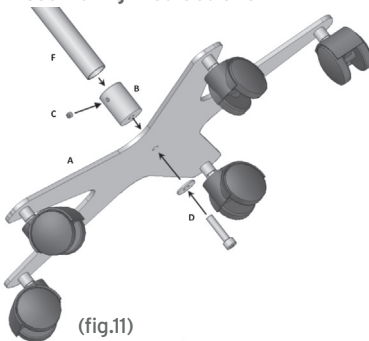8.4 Assembling Instructions MI427

(fig.11)

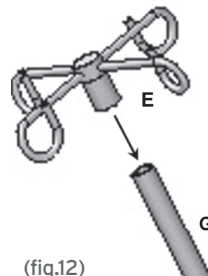
(fig.12)

1. Put the connection adapter (B) on the upper part of the base (A)
2. Insert the pole (F) in the connection adapter (B)
3. Lock the pole using the screw and its washer (D)
4. Tighten the screw
5. Tighten the grab screw (C) on the pole
6. Screw the handwheel onto the rod height locking bushing
7. Insert the hook (E) into upper hole of the pole (G) with a light pressure
8. Now the I.V. pole is ready to use

Images only illustration purposes

9. HOW TO USE

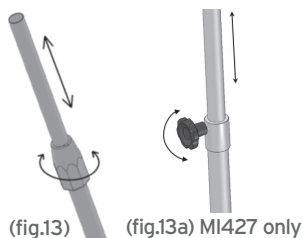To adjust the height of the I.V. Pole turn the knob counterclockwise, fix the pole to the right height and turn the knob clockwise to block it.

WARNINGS!

- Check the pole before every use. Make sure it hasn't any damage or signs of wear that could compromise its working. If it should be damaged, do not use the pole.
- Make sure the position of the pole is comfortable and safe for the patient, pay attention to the castors movement.
- Do not overload the hooks, check the maximum weight on the technical features.

10. CLEANING AND DISINFECTION

10.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

WARNINGS!

Do not use abrasive substances, acids, alkaline, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

10.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

11. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

12. ACCESSORIES

- MI420 Weighting Disc 1,750 Kg for IV stands
- MIA421 Instrumentation support for Moretti IV stands
- MIA421-1 Universal Instrumentation support for IV stands
- MIA422-1 2 antistatic castors Kit for standard base (MI432/MI433/MI435/MI436/MI437/MI425/MI426)
- MIA422-2 2 antistatic castors Kit for plate base (MI427)
- MIA423-1 2 castors with brake Kit for standard base (MI432/MI433/MI435/MI436/MI437/MI425/MI426)
- MIA423-2 2 castors with brake Kit for plate base (MI427)

Use only accessories listed in the Moretti general catalogue. Moretti declines any responsibility for any accident caused by misuse.

13. TECHNICAL FEATURES

- Anodized aluminium I.V. Pole with polypropylene base and ABS hooks (MI432-MI433)
- Stainless Steel I.V. Pole with polypropylene base and ABS hooks (MI435-MI436)
- Stainless Steel I.V. Pole with polypropylene base and stainless steel hooks (MI437)
- Stainless Steel I.V. Pole with chromed aluminium base and stainless steel hooks (MI425)
- Stainless Steel I.V. Pole with stainless steel base and stainless steel hooks (MI426-MI427)
- 5 swivel castors Ø 50 mm
- I.V. Pole height adjustment by knob

13.1 Dimensions and weight

Codes	Height (min/max)	Base dimension (mm)	Maximun load capacity per hook	Maximun payload*
MI425	1350mm/2400mm	500mm	5kg	20kg
MI426	1350mm/2400mm	540mm	5kg	20kg
MI427	1350mm/2400mm	500mm	10kg	40kg
MI432	1350mm/2400mm	500mm	5kg	10kg
MI433	1350mm/2400mm	500mm	5kg	20kg
MI435	1350mm/2400mm	500mm	5kg	10kg
MI436	1350mm/2400mm	500mm	5kg	20kg
MI437	1350mm/2400mm	500mm	5kg	20kg

* The maximum allowed payload has to be intended with stationary I.V. Pole (no movement) and with an equal load distribution on the hooks.

14. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

15. REPAIRING

15.1 Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

15.2 Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

15.3 Non defective device

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

16. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

17. EXEMPT CLAUSES

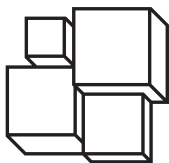
Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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