**SKEMA<sup>®</sup>FORM**

MA MOC330X - MOV330X - MOV333X - MOV340X 02 A\_ITA\_08 2020

**LETTINO DA VISITA MEDICA E FISIOTERAPIA**  

---

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ITALIANO

## INDICE

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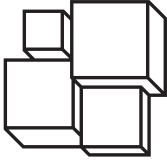
**SKEMA<sup>®</sup>FORM**

MA MOC330X - MOV330X - MOV333X - MOV340X 02\_A\_ENG\_08 2020

**EXAMINATION AND PHYSIOTHERAPY COUCH**

---

**INSTRUCTION MANUAL**

ENGLISH

## INDICE

<b>1. CODES</b> .....	<b>PAG.3</b>
<b>2. INTRODUCTION</b> .....	<b>PAG.3</b>
<b>3. INTENDED USE</b> .....	<b>PAG.3</b>
<b>4. DECLARATION OF CONFORMITY</b> .....	<b>PAG.3</b>
<b>5. GENERAL WARNINGS</b> .....	<b>PAG.4</b>
<b>6. SYMBOLS</b> .....	<b>PAG.4</b>
<b>7. GENERAL DESCRIPTIONS</b> .....	<b>PAG.5</b>
7.1 Drawing and description of the parts of MOC330-MOV330 .....	pag. 5
7.1.1 Drawing and description accessory kit of MOC330-MOV330 .....	pag. 5
7.2 Drawing and description of the parts of MOV333 .....	pag. 6
7.2.1 Drawing and description accessory kit of MOV333 .....	pag. 6
7.3 Drawing and description for the parts of MOV340 .....	pag. 7
7.3.1 Drawing and description accessory kit of MOV333 .....	pag. 7
<b>8. ASSEMBLY</b> .....	<b>PAG.8</b>
8.1 Assembling the couch MOC330-MOV330 .....	pag. 8
8.2 Assembling the couch MOV3338 .....	pag. 9
8.3 Assembling the couch MOV340.....	pag. 10
8.4 Assembling the paper holder (OPTIONAL) .....	pag. 11
<b>9. BEFORE EACH USE</b> .....	<b>PAG.12</b>
<b>10. PRELIMINARY CHECKS</b> .....	<b>PAG.12</b>
<b>11. USING THE COUCH</b> .....	<b>PAG.12</b>
<b>12. MAINTENANCE</b> .....	<b>PAG.13</b>
<b>13. CLEANING AND DISINFECTION</b> .....	<b>PAG.13</b>
<b>14. CONDITIONS OF DISPOSAL</b> .....	<b>PAG.13</b>
14.1 General disposal conditions.....	pag. 13
14.2 Warnings for the proper disposal of the product according to the European Directive 2012/19/UE: .....	pag. 13
<b>15. SPARE PARTS AND ACCESSORIES</b> .....	<b>PAG.13</b>
<b>16. TECHNICAL SPECIFICATIONS</b> .....	<b>PAG.14</b>
16.1 Dimension and weight of couches MOC330-MOV330 .....	pag. 14
16.1.1 Technical specifications of couches MOC330-MOV330 .....	pag. 14
16.2 Dimension and weight of couch MOV333 .....	pag. 15
16.2.1 Technical specifications of couch MOV333 .....	pag. 15
16.3 Dimension and weight of couch MOV340.....	pag. 16
16.3.1 Technical specifications of couch MOV340 .....	pag. 16
<b>17. WARRANTY</b> .....	<b>PAG.17</b>
<b>18. REPARATIONS</b> .....	<b>PAG.17</b>
18.1 Reparations under warranty .....	pag. 17
18.2 Reparation of an out-of-warranty product .....	pag. 17
18.3 Non-defective products .....	pag. 17
<b>19. SPARE PARTS</b> .....	<b>PAG.17</b>
<b>20. EXONERATION CLAUSES</b> .....	<b>PAG.17</b>

## CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

### 1. CODES

<b>MOC330X</b>	Examination couch LYBRA - Chromed
<b>MOV330X</b>	Examination couch LYBRA - Painted
<b>MOV333X</b>	Examination couch RYGEL - Painted
<b>MOV340X</b>	Examination and physiotherapy couch RYGEL - Painted

X: Indicates the upholstery colour.

### 2. INTRODUCTION

Thank you for choosing an examination couch from the SKEMA product range by MORETTI S.p.A..

Moretti examination couches have been designed and realized in order to guarantee comfort, safety and reliability. This user manual provides for some suggestions as to how correctly use the product you have chosen and offers a lots of valuable advice for your safety.

Please read carefully through the manual before using the examination couch. Should you have any queries, please contact your retailer for advice and assistance.

### 3. INTENDED USE

Moretti examination couches are intended for physiotherapy diagnosis and short treatments.

#### **WARNING!**

- It is strictly forbidden to use this device for different purposes from those ones specified here in this user manual.
- Moretti S.p.A. declines any responsibility for damages caused by an improper use of the device or by a different purpose from that one specified here in this manual.
- The manufacturer reserves the right to modify the product and this user manual without prior notice, in order to improve the characteristics of the device.

### 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of EXAMINATION COUCHES - SKEMA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

For all matters concerning a proper use of the product, please refer to this user manual:

- Keep the packing away from any heat source because it is a cardboard box
- The service life of the device is determined by the wear of its non-repairable and/or spare parts and in any case it is no more than 10 years
- Please, pay attention to moving parts that may trap limbs or cause personal injuries
- Always pay attention to children
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

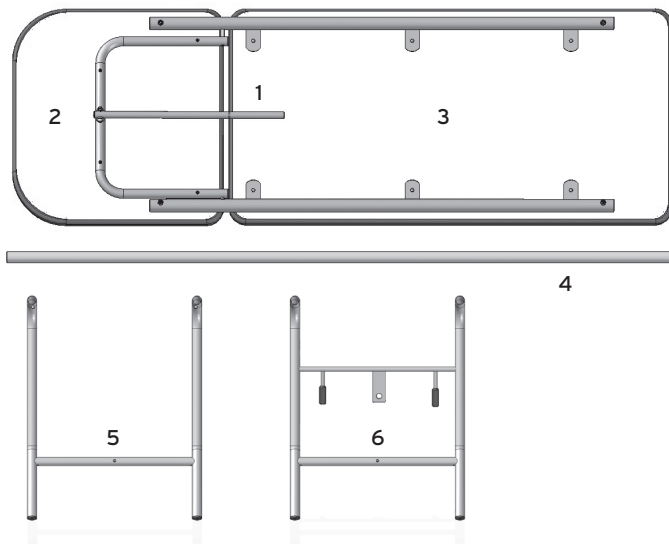
## 6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal
	Attention, should read the instructions
	Production dates
	Waste Electrical & Electronic Equipment (WEEE)
<b>IP54</b> 	Protection against dust and water splashes
	B Type

**7. GENERAL DESCRIPTIONS**

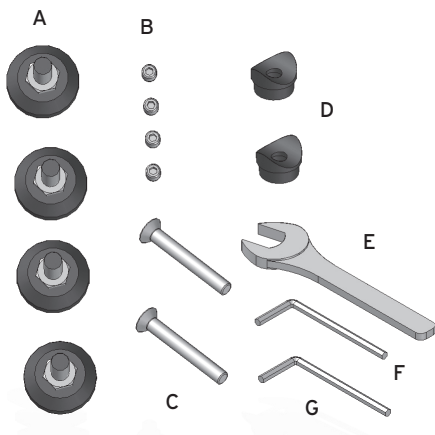
**7.1 Drawing and description of the parts of MOC330-MOV330**

**Parts list**

- 1. Frame
- 2. Padded backrest
- 3. Padded couch
- 4. Longitudinal tube for connecting the legs
- 5. Leg (feet)
- 6. Leg (head)

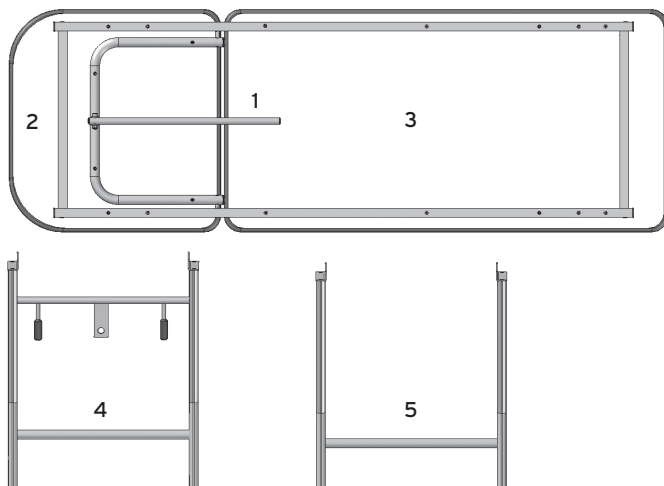
**7.1.1 Drawing and description accessory kit of MOC330-MOV330**

**Parts list**

- A. N° 4 feet with adjustment bolt
- B. N° 4 grub screws to secure the legs
- C. N° 2 countersunk screws M 8x60 for the connecting tube for legs
- D. N° 2 countersunk joints for the connecting tube for legs
- E. N° 1 wrench for foot adjustment bolt
- F. N° 1 14mm Hex key to secure the legs
- G. N° 1 15mm Hex key for the connecting tube for legs

## 7.2 Drawing and description of the parts of MOV333

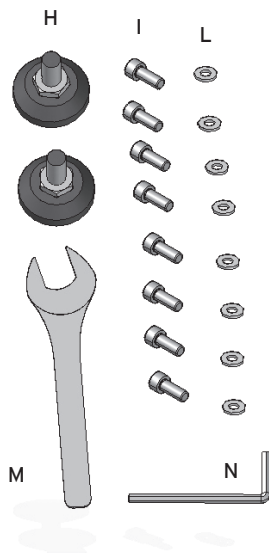
ENGLISH

### Parts list

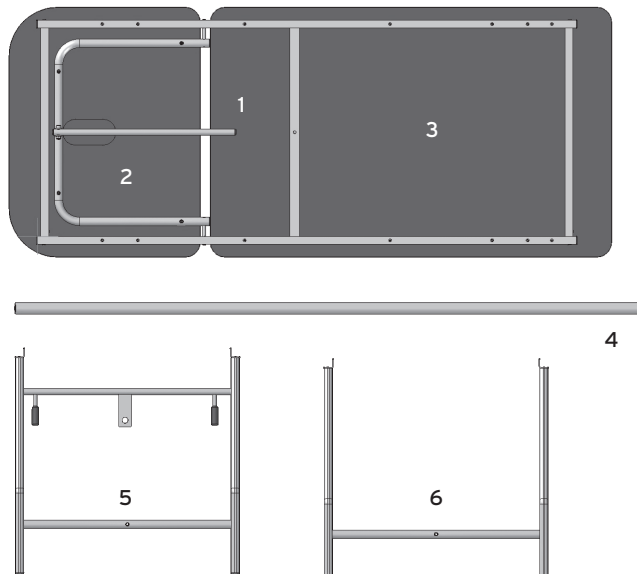
1. Frame
2. Padded backrest
3. Padded couch
4. Legs (head)
5. Legs (feet)

### 7.2.1 Drawing and description accessory kit of MOV333

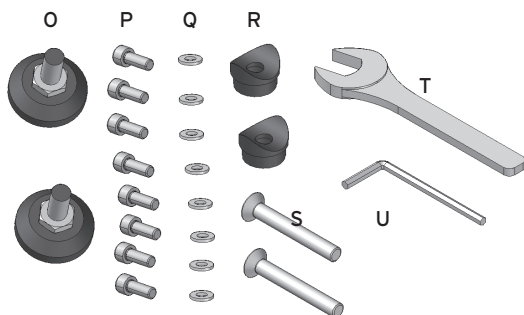
### Parts list

- H. N° 2 feet with adjustment bolt
- I. N° 8 round head screws M 6x16 to secure the legs
- L. N° 8 washer, diameter 6 for leg blocking screws
- M. N° 117mm wrench for foot adjustment bolt
- N. N° 1 5mm Hex key to secure the legs

**7.3 Drawing and description for the parts of MOV340**

**Parts list**

1. Frame
2. Padded backrest
3. Padded couch
4. Tube for connecting the legs
5. Legs (head)
6. Legs (feet)

**7.3.1 Drawing and description accessory kit of MOV333**

**Parts list**

- O. N° 2 feet with adjustment bolt
- P. N° 8 round head screws M6x16 to secure the legs
- Q. N° 8 washers for leg blocking screws
- R. N° 2 countersunk joints for the connecting tube for legs
- S. N° 2 countersunk screws M 8x60 for the connecting tube for legs
- T. N° 117mm Hex key to adjust feet
- U. N° 15mm Hex key to secure the legs

## 8. ASSEMBLY

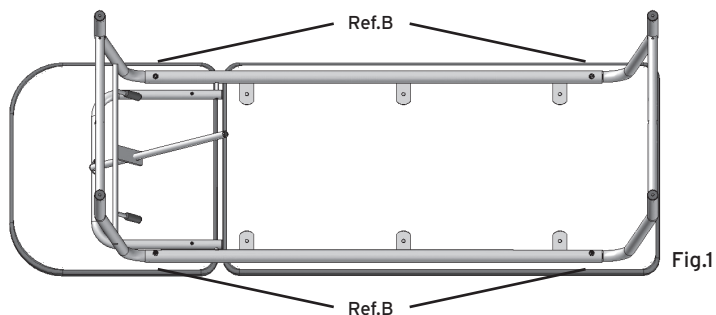
### CAUTION!

Perform these operations with the help of another person. Take precautions to prevent injuries and/or damages when moving the couch.

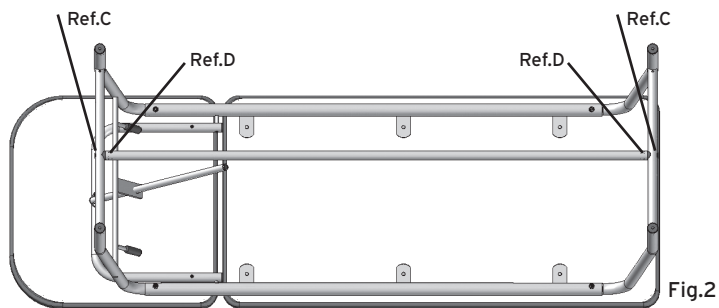
### 8.1 Assembling the couch MOC330-MOV330

1. Open the packing box and check that the content corresponds to the requested model and the parts have not been damaged by the transportation. Otherwise, please contact immediately your reseller.

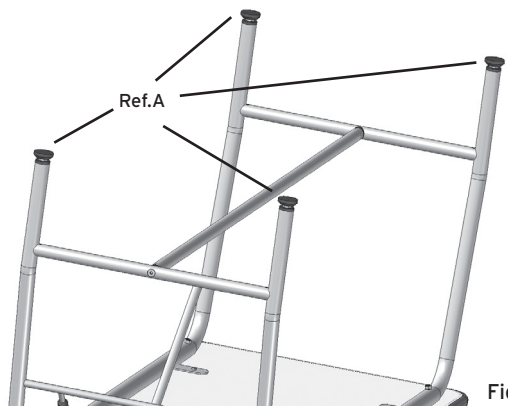
2. Assembling the legs

3. Assembling the tube that connects the legs

4. Foot with adjustment bolt

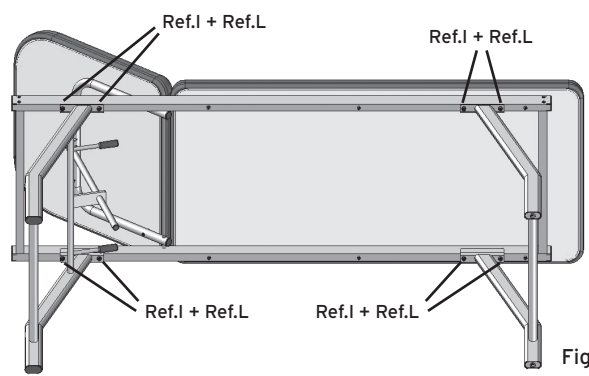
**Fig.3**

**N.B.** Adjust the feet with the provided wrench, so that the couch can be made stable on a flat surface.

**YOUR COUCH IS CORRECTLY ASSEMBLED AND READY TO USE**

**8.2 Assembling the couch MOV3338**

1. Open the packing box and check that the content corresponds to the requested model and the parts have not been damaged by the transportation. Otherwise, please contact immediately your reseller.
2. Assembling the legs

**Fig.4**

### 3. Assembling the feet with bolt

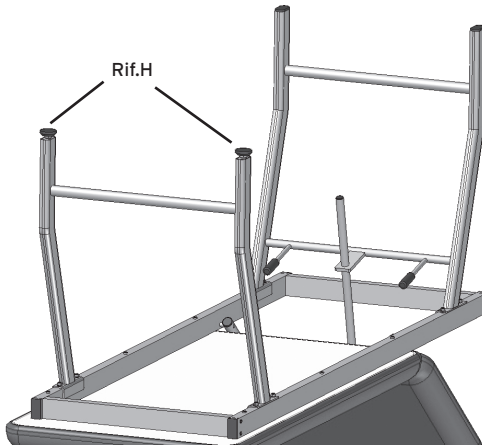
Fig.5

**YOUR COUCH IS CORRECTLY ASSEMBLED AND READY TO USE**

### 8.3 Assembling the couch MOV340

1. Open the packing box and check that the content corresponds to the requested model and the parts have not been damaged by the transportation. Otherwise, please contact immediately your reseller.
2. Assembling the legs

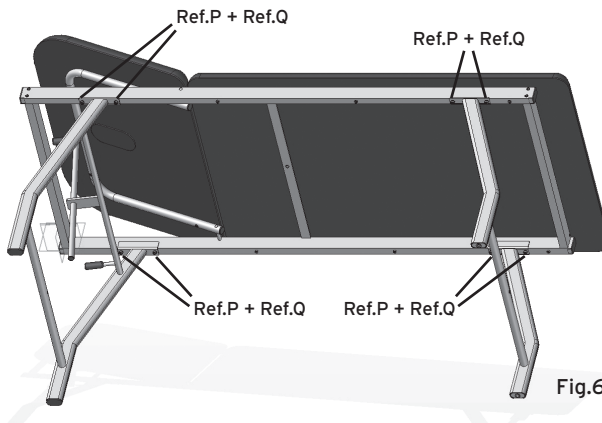
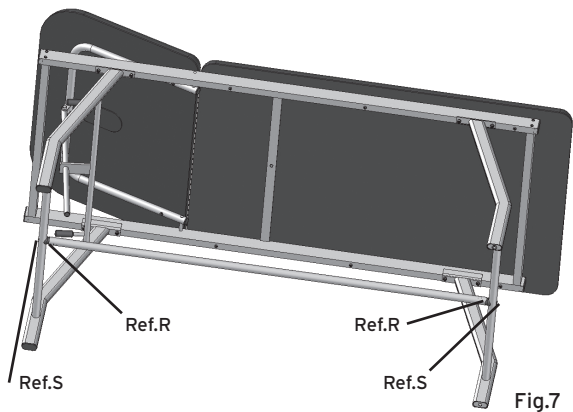
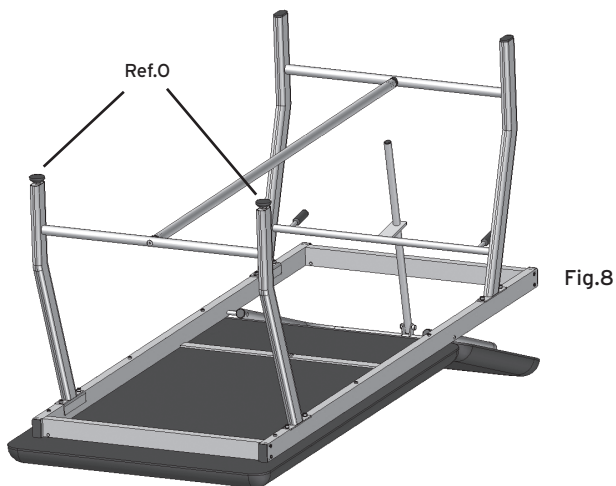
Fig.6

3. Assembling the tube that connects the legs

4. Foot with bolt

**YOUR COUCH IS CORRECTLY ASSEMBLED AND READY TO USE**

**8.4 Assembling the paper holder (OPTIONAL)**

N.B. It can be assembled on the head side or on the foot side.

Fig.9

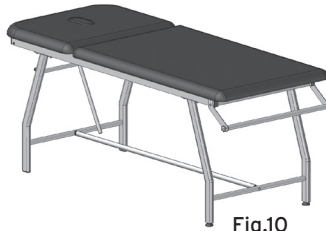
Fig.10

## YOUR KIT IS CORRECTLY ASSEMBLED AND READY TO USE

### 9. BEFORE EACH USE

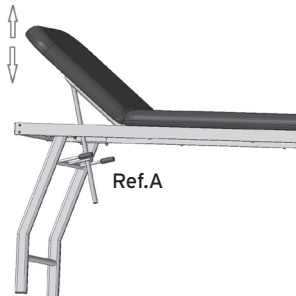
- Always check the mechanical parts for wear so that it can be guaranteed a totally safe use of the device for people and things
- Check the correct assembly of the device. Pay attention to the screws and blocking bolts of the backrest and of the mechanical joints in general.

### 10. PRELIMINARY CHECKS

- Before any adjustment of the couch, make sure it is in a stable position on its own legs.

### 11. USING THE COUCH

You may adjust the backrest inclination, acting on the handles (Ref. A of the picture). In order to secure it, support the backrest with one hand and activate one of the handles with the other hand, lifting it up. Block the backrest at the desired inclination and then release the blocking handle.

Ref.A

#### CAUTION:

Before placing a patient on the couch, always check the correct safety blocking system of the backrest.

## 12. MAINTENANCE

The devices from the SKEMA product range by MORETTI S.p.A. are carefully checked and provided with the EC label before being launched on the market.

As far as patient's and doctor's safety is concerned, you are recommended to have the product suitability checked at least once a year.

For periodic inspection we mean the following parts:

- The couch frame;
- Check all the welding joints. In case of reparation, only use original spare parts and accessories.

## 13. CLEANING AND DISINFECTION

### 13.1 Cleaning

Only use a damp cloth and neutral soap. Then dry well before use.

Do not use water warmer than 30° C.

### 13.2 Disinfection

If necessary, disinfect the device using a common cleaning disinfectant product.

**N.B.** In no way use acid, alkaline or solvent products as like acetone or thinner.

## 14. CONDITIONS OF DISPOSAL

### 14.1 General disposal conditions

As far as the product disposal is concerned, never use the common urban solid waste disposal systems. The product must be disposed of by the municipal sorted waste collection centres, designated for recycling the used materials.

### 14.2 Warnings for the proper disposal of the product according to the European Directive 2012/19/UE:

At the end of its service life, the product must not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted waste collection centre or to the resellers, in case they provide for this service. Sorted waste disposal reduces the environmental and health consequences of an improper disposal. Furthermore, the materials which make up the product may be recycled, thus resulting in energy and resource savings. In order to highlight the compulsory sorted disposal of the electro-medical equipment, a barred bin symbol is displayed on the product itself.

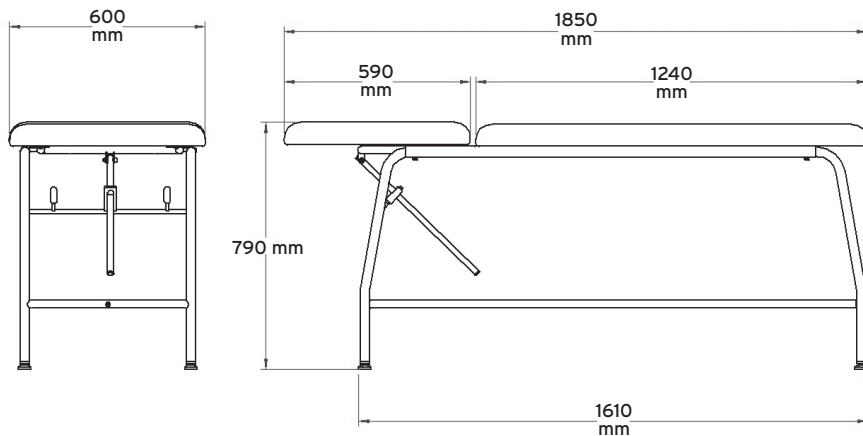
## 15. SPARE PARTS AND ACCESSORIES

<b>MIA370</b>	Paper holder only for the models MOC330-MOV330
<b>MIA385</b>	Paper holder only for the model MOV333
<b>MIA376</b>	Paper holder only for the model MOV340

## 16. TECHNICAL SPECIFICATIONS

### 16.1 Dimension and weight of couches MOC330-MOV330

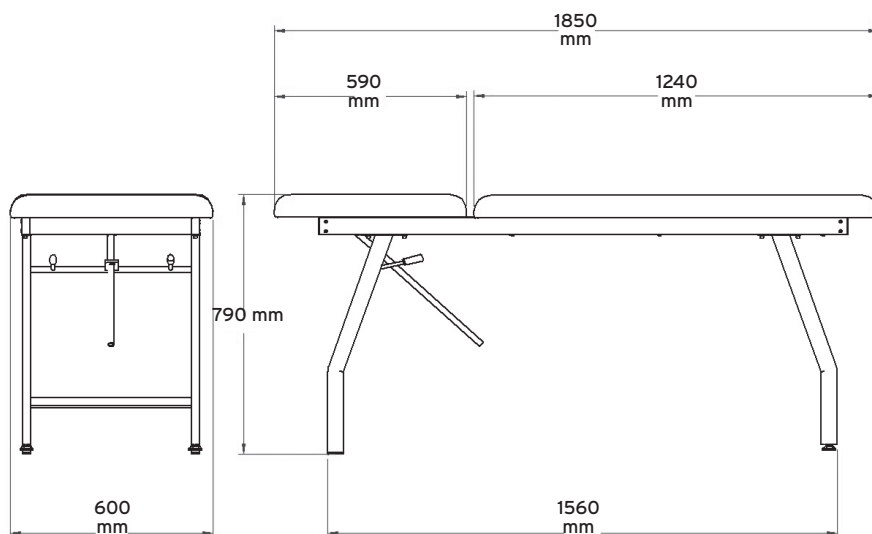
ENGLISH

 Packing dimension	 L: 1850 mm	 P: 600 mm	 H: 790 mm
 Packing weight	36 kg		
 Couch weight	33 kg		
 Max. load	180 kg		

#### 16.1.1 Technical specifications of couches MOC330-MOV330

Steel tubular structure diam. 35 mm chromed for MOC codes, painted for MOV codes. Tubular steel legs structure diam. 30 mm. Upholstery in polyurethane density 25, 5 cm high. Upholstery in washable and self-extinguishing imitation leather 1IM

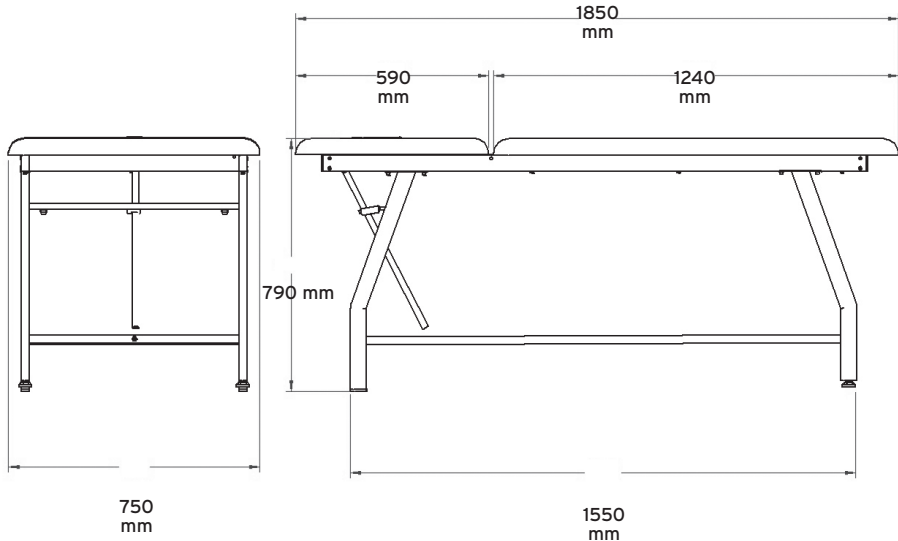
**16.2 Dimension and weight of couch MOV333**

 Packing dimension	 L: 1850 mm	 P: 600 mm	 H: 790 mm
 Packing weight	39 kg		
 Couch weight	36 kg		
 Max. load	180 kg		

**16.2.1 Technical specifications of couch MOV333**

Perimeter frame in painted rectangular tubular 40x25 mm. Couch legs frame in oval tubular steel, 50x25 mm. Padded in polyurethane, density 25 high 5 cm. Upholstery in washable and self-extinguishing synthetic leather, 11M.

## 16.3 Dimension and weight of couch MOV340

 Packing dimension	 L: 1850 mm	 P: 750 mm	 H: 790 mm
 Packing weight	43 kg		
 Couch weight	40 kg		
 Max. load	180 kg		

## 16.3.1 Technical specifications of couch MOV340

Perimeter frame in painted rectangular tubular 40x25 mm. Couch legs frame in oval tubular steel, 50x25 mm. Padded in polyurethane, density 25 high 5 cm. Upholstery in washable and self-extinguishing synthetic leather, 11M.

## **17. WARRANTY**

Every Moretti product is guaranteed for 2 (two) years from the purchase date, except the restrictions specified below.

The warranty shall not apply in case of improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual.

Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual.

The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

## **18. REPARATIONS**

### **18.1 Reparations under warranty**

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the repair is under warranty. A reparation or replacement does not renew nor extend the warranty.

### **18.2 Reparation of an out-of-warranty product**

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti Client Service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

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## **19. SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

## **20. EXONERATION CLAUSES**

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless.

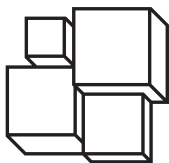
The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitations of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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