MOPEDIA®

MA MP090_MP091_MP091R_MP092_MP093_MP095_MP097 04 A_ITA_03 2022

LETTI DEGENZA

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Serie
ibisco

ITALIANO

INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

- MPO90** - Single-crank hospital bed (IBISCO 1)
MPO91 - Double-crank hospital bed (IBISCO 2)
MPO91R - Double-crank hospital bed with castors (IBISCO 2)
MPO92 - Single-crank hospital bed - Demountable (IBISCO 1s)
MPO93 - Double-crank hospital bed - Demountable (IBISCO 2s)
MPO95 - Double-crank hospital bed (IBISCO 2e)
MPO97 - Electric hospital bed (IBISCO 3)

2. INTRODUCTION

Thank you for choosing a MOPEDIA hospital bed by Moretti S.p.A. MOPEDIA hospital beds manufactured by Moretti S.p.A. are designed to satisfy all your requirements for a practical, correct and safe use. This manual provides some useful suggestions to correctly use the device you have selected and offers valuable advices for your safety. We recommend carefully reading the entire manual before using the hospital bed you have purchased. Should you have any queries, please contact your retailer for proper advice or assistance.

NOTE: Check all parts of the product have not been damaged during shipment. In case of damage do not use the product and contact your dealer for further instructions

3. INTENDED USE

MOPEDIA hospital beds are designed for people that have been hospitalized for health reasons in institutes, hospitals and/or health assistance facilities or within a family context.

CAUTION!

- Do not use the following device for purposes other than those described in this manual.
- Moretti S.p.A. disclaims any liability for damage resulting from improper use of the device or use other than that described in this manual.
- We reserve the right to make changes to the device and this manual without notice in order to improve the characteristics.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of BEDS IBISCO complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

4.1 Standards and guidelines reference

All safety tests required by UNI CEI 60601-2 -52 were executed with a foam mattress.
Density 30 kg / m³, size L: (3x) 63 tot.195 P: 85 H: 12.5 cm.

5. GENERAL WARNINGS

ATTENTION!

• Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts cannot be repaired and / or replaced.
- Be careful when children are nearby.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

Warning

Date of production

Disposal according to ec directive 12/2012

Maximum permissible weight

Portata max

Isolation class


Protection from solid bodies greater than 12 mm

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7. GENERAL DESCRIPTION


7.1 View and List of Components MP090 - MP091 - MP091R - MP095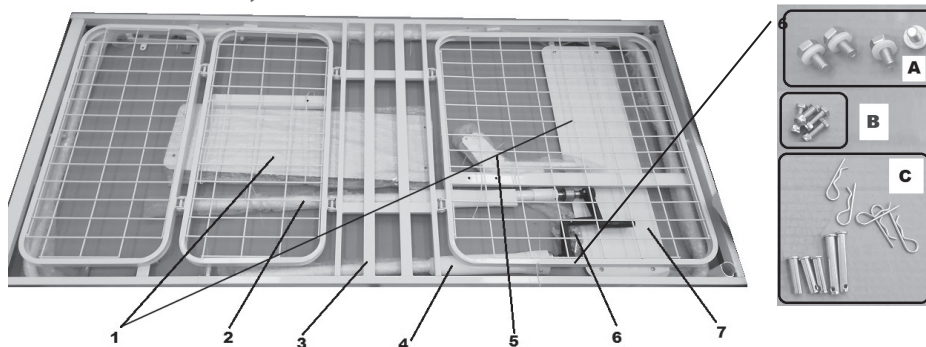

(Fig.1) (Picture with illustrative purpose only)

1. Headboard/Footboard (with castors only MP091R) (2pcs)
2. Head lifting crank (1pc)
3. Bed frame (1pc)
4. Foot lifting crank (1pc)
5. Brackets for Crank handle (2pcs)
6. Screws kit
 - A. Screws to fix the crank to the bed frame (4pcs)
 - B. Screws to fix the bed net to the headboard/footboard (4pcs)
 - C. Pins + split pins for fixing the brackets to the bed frame and the crank tubes (5pcs)
7. Headboard / Footboard (2pcs)


7.2 View and List of Components MP092 - MP093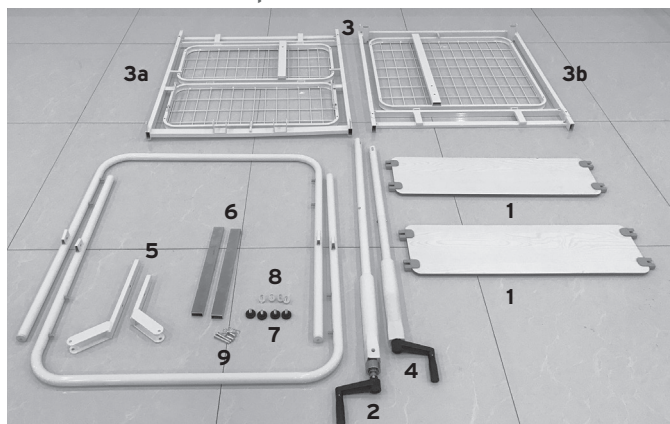

(Fig.2) (Picture with illustrative purpose only)

1. Panel board (2pcs)
2. Head lifting crank (1pc)
3. Bed frame (1pc)
 - 3a. Foot part - 3b. Head part
4. Foot lifting crank (1pz)
5. Brackets for Crank handle (2pz)
6. Fixing brackets
7. Fixing knobs
8. Screws to fix the crank to the bed frame
9. Pins + split pins for fixing the brackets to the bed frame and the crank tubes (5pcs)

7.3 View and list of components MP097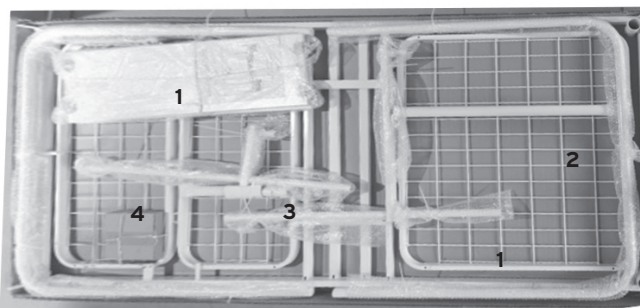

(Fig.3) (Picture with illustrative purpose only)


1. Panels (2 pcs)
2. Bed net (1 pcs)
3. Control unit/motor assembly bracket kit
4. Package containing the electrical parts

8. ASSEMBLING/DISASSEMBLING

8.1 Assembling MP090 - MP091 - MP091R - MP092 - MP093


NOTE: Illustrations for the installation and adjustments refers to the hospital bed 2 cranks. Also assembling and adjusting operations are valid for the single crank bed hospital.

- 1) Remove the product from its packaging and check that contents comply with the illustrated component description 7.1 for MP090, MP091, and 7.2 for MP092, MP093;
- 2) Insert the brackets into the bed frame and secure them with the black thumb screws (only for models MP092-MP093) (Fig.4)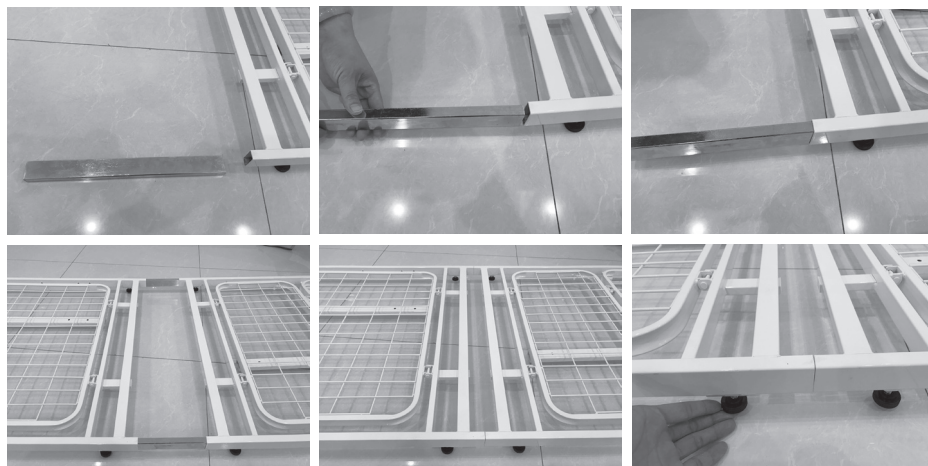


(Fig.4) (Picture with illustrative purpose only)

3) Screw the screws "A" to fix the crank mechanism on the mattress frame. (Fig.5)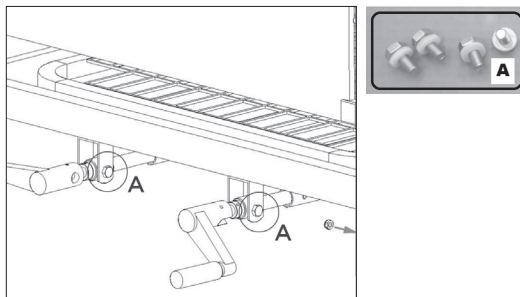

(Fig.5) (Picture with illustrative purpose only)

4) Insert the bracket into the mattress frame and fix them by pins and split pin, "C" (Fig.6-7)

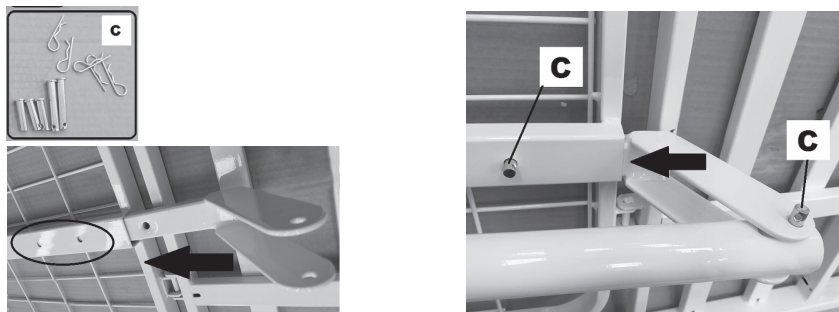
(Fig.6) (Picture with illustrative purpose only) (Fig.7) (Picture with illustrative purpose only)

5) Assembly the board panel on the headboard/footboard (Fig.8)


(Fig.8) (Picture with illustrative purpose only)

- 6) Hold the head frame and the foot frame while directly inserting the base frame bayonet hooks into the relative slots (Fig.9). (head frame and foot frame are equal).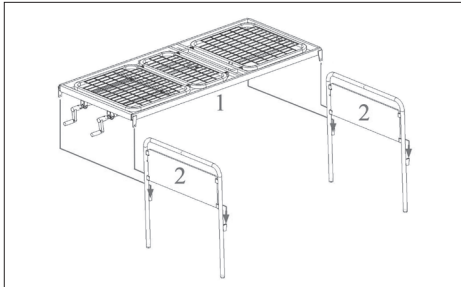


(Fig.9) (Picture with illustrative purpose only)

- 7) Using the screws and nuts "B"(4pcs) supplied fix the headboard and the footboard (Fig.10)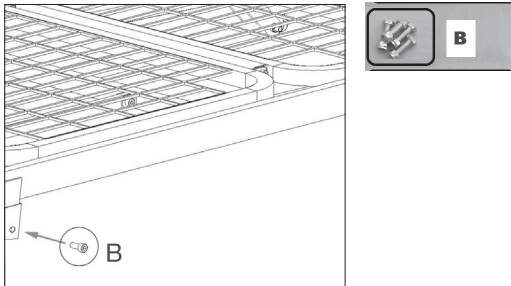


(Fig.10) (Picture with illustrative purpose only)

- 8) Your bed is fully assembled and ready to use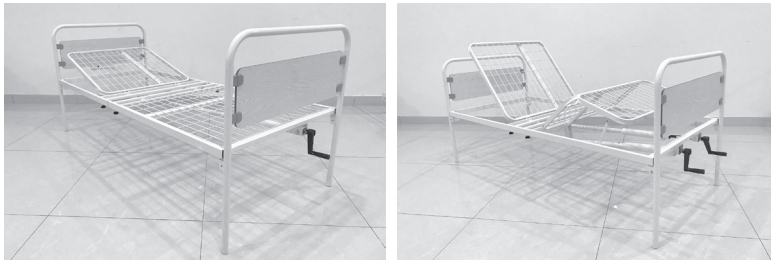


8.2 Assembling MPO95

- 1) Remove the product from its packaging and check that contents comply with the illustrated component description 7.1;
- 2) Insert the iron sheets, placed at the vertexes of the frame, in the appropriate seats according to the desired height, as in Fig.11;
- 3) Assemble then the headboard and footboard panels as in Fig.12;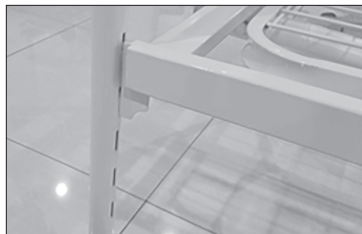


(Fig.11) (Picture with illustrative purpose only)

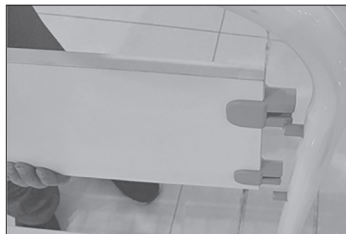(Fig.12) (Picture with illustrative purpose only)

- 4) Assemble the adjustment handles of the sections (left legs section the short one, right back section the long one), using the exagonal screws M10*20+washer 10.5+plastic gasket 10.5*4, as in fig.13 and 14;

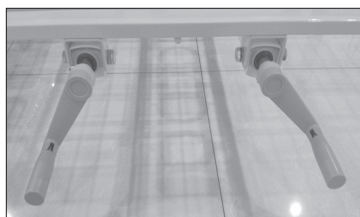(Fig.13) (Picture with illustrative purpose only)

(Fig.14) (Picture with illustrative purpose only)


(Fig.15) (Picture with illustrative purpose only)

(Fig.16) ((Picture with illustrative purpose only)

- 5) Insert the rectangular tubes inside the same seats and fasten them with the 10.5 + clamp pins (fig.15). Pay attention that the 45° inclined sheets, placed at the end, match with the relative hooking point placed on the circular tubes connected to the adjustment handles;
- 6) Now lock the circular tube into its seat using the 10.5 pins + the relative clamps. (Fig.16);
- 7) Your bed is ready to be used;

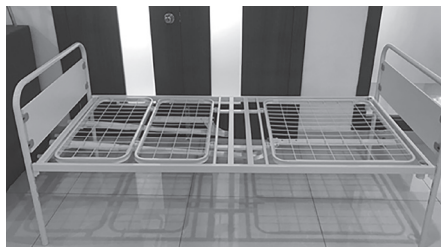8.3 Assembling MPO97


- 1) Remove the product from its packaging and check that contents comply with the illustrated component description 7.3;
- 2) Insert the iron sheets placed at the vertices of the frame in the appropriate seats as in fig.17;
- 3) Tighten the two parts just joined with the exagonal screws M6x25 + washer d. 6.5 and nut M6 as in Fig.18;
- 4) Then assemble the headboard and footboard panels as in Fig.19;

(Fig.17)

(Fig.18)

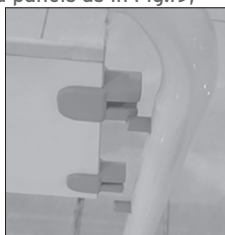(Fig.19)

(Picture with illustrative purpose only)

- 5) Block the housing bases of the drive shaft using the M8x15 screws + washer d.8.5 supplied as in fig.20;
- 6) Ensure that the accommodation bases are correctly positioned, all with the slanted side facing the legs of the bed as in fig.20a


(Fig.20)


(Fig.20a)

(Picture with illustrative purpose only)

- 7) Then assemble the rotating shaft, on which the actuator will then be housed, inside the newly prepared bases (fig.21);
- 8) Now place the actuator in the normal operating position (fig.22);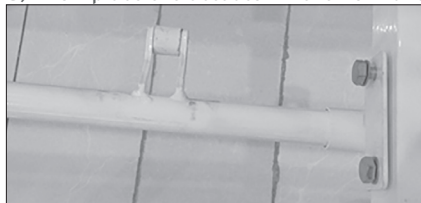

(Fig.21) (Picture with illustrative purpose only)

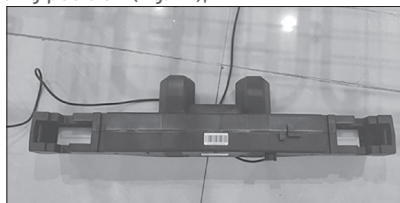
(Fig.22) (Picture with illustrative purpose only)

9) Then insert both movement arms (short and long) inside their respective holes as in fig. 23 and 24;


(Fig.23) (Picture with illustrative purpose only)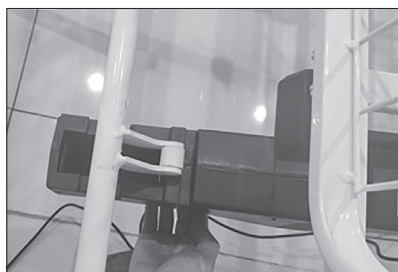


(Fig.24) (Picture with illustrative purpose only)

10) Place now the respective covers of the seats of the arms just inserted as in fig.25;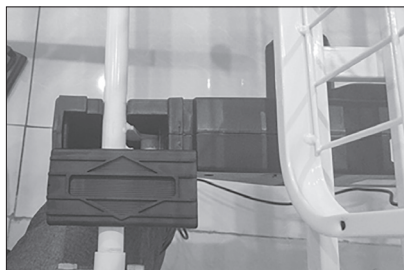

(Fig.25) (Picture with illustrative purpose only)

11) Your bed is ready to be used.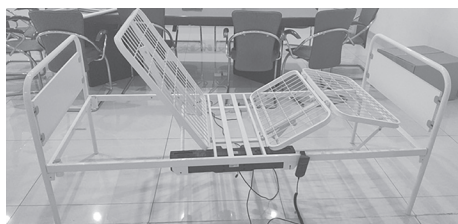

8.4 Disassembling MP090 - MP091 - MP091R - MP092 - MP093 - MP095 - MP097

- Follow the procedure described above in reverse.

9. BEFORE TO USE

- Always check the status of the mechanical parts in order to guarantee they are used in total safety for persons and environment.
- Check the proper assembly of the device with attention to the correct insertion and locking screws and nuts of the headrest and footrest.
- Check the correct raising/lowering of foot rest frame and head frame.

10. PRECAUTIONS FOR USE


- Do not exceed the maximum load capacity for any reason.
- When lowering the head or foot section, make sure that nobody inserts hands, legs or other body parts under the bed base.
- Make sure that the floor surface supporting the bed is uniformly level.

- Before any operation of cleaning, maintenance, or simply move the bed, disconnect the plug of the mains voltage.
- Do not use multiple plugs or extension cables.
- Do not connect to the bed, some electrical devices not due by the normal operation.


11. HOW TO USE

11.1 Raising and lowering the head section (for all models)

- Pull the crank outwards
- Turn the handle clockwise to raise the head section (Fig. 26), and anti-clockwise to lower it (Fig. 27)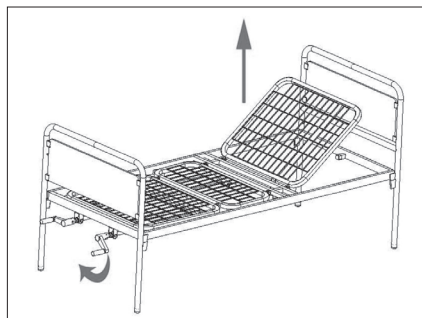

(Fig.26) (Picture with illustrative purpose only)

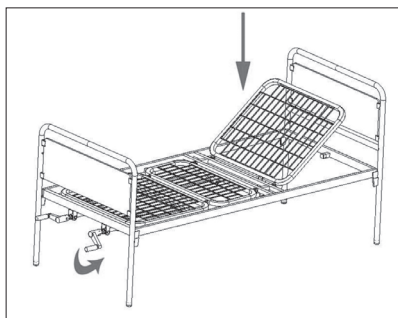(Fig.27) (Picture with illustrative purpose only)

- Push the crank inwards after adjusting the bed to avoid accidental knocks.


ATTENTION!


As the head of the bed nears the end of its range of motion (both when raising or lowering), the crank will become harder to turn. Do not turn the crank further as this may damage mechanical components!

11.2 Raising and lowering the foot section (only for MP092-MP093)

- Pull the crank outwards
- Turn the handle clockwise to raise the foot section (Fig.28), and anti-clockwise to lower it (Fig. 29)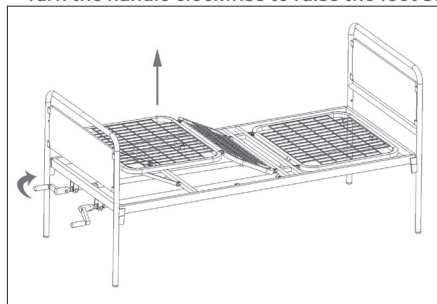

(Fig.28) (Picture with illustrative purpose only) (Fig.29) (Picture with illustrative purpose only)

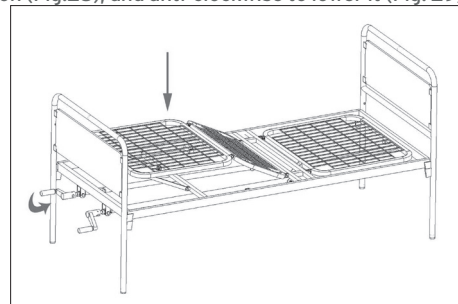- Push the crank inwards after adjusting the bed to avoid accidental knocks

N.B. As the foot of the bed nears the end of its range of motion (both when raising or lowering), the crank will become harder to turn. Do not turn the crank further as this may damage mechanical components!

11.3 Adjusting the leg section MP097

To raise the leg section of the bed, press the corresponding button on the remote control connected to the bed.

11.4 Mattress selection

Always use mattresses whose characteristics comply with the relevant standard requirements, its dimensions are not larger or particularly smaller than those of the bed. All mattresses of the Levitas by MORETTI range are suitable for IBISCO beds.

NOTE: The use of non-compatible mattresses can be dangerous. Consult the instructions for use.

12. MAINTENANCE

MOPEDIA by Moretti devices at the moment placing on the market are carefully checked and carry the CE mark. For the safety of the patient and the doctor are recommended to check by the manufacturer or by an approved laboratory, at least every two years, the suitability for use of your product. In case of repair should be used only original parts and accessories.

- It is extremely important that the bed is checked at the first use. In particular, please check that all screws and nuts are tight.
- For first use check very carefully all the bed, in particular, check all screws and nuts are tight.
- Bed's mechanical parts are able to move correctly (headrest / legsrest, cranks, etc ...).
- All the mechanical parts must be checked monthly and lubricated with a drop of oil in order to promote smoothness and resistance to wear.

13. CLEANING AND SANITISATION

13.1 Cleaning

To clean the product, only use a cloth soaked in water and dry with a clean, dry cloth or with lukewarm water and mild soap. a clean, dry cloth or with lukewarm water and mild soap.

ATTENTION!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants or acetone. Disinfectants and acetone, as these will cause abrasion of the plastic parts and rusting of the abrasion of plastic parts and rusting of metal surfaces.

The manufacturer shall not be liable for damage caused by the use of materials that may deteriorate the The manufacturer shall not be liable for damage caused by the use of materials that may deteriorate the surface of the product or by chemical corrosives used during cleaning.

13.2 Disinfection

If necessary, disinfect the product using a mild disinfecting detergent.

14. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.



14.1 Warnings for the correct disposal of the product in accordance with European Directive 2012/19/EU:

At the end of its useful life, the product must not be disposed of with municipal waste. It can be delivered to the special separate waste collection centres set up by the municipal authorities, or to retailers who provide this service. Disposing of the product separately avoids the possible negative consequences for the environment and health deriving from its improper disposal and allows the materials from which it is made to be recovered in order to obtain significant savings in energy and resources. To underline the obligation to dispose of electro-medical equipment separately, the product bears the following markings the crossed-out wheeled bin mark on the product.

15. TECHNICAL FEATURES



15.1 Dimension

MP090 - MP092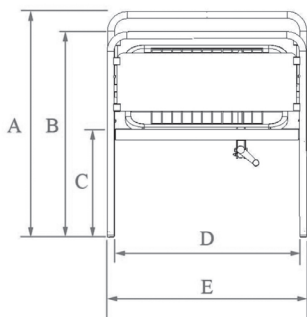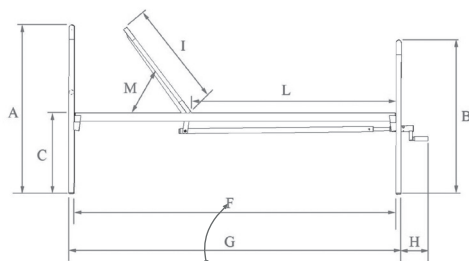



- A - Maximum height of the head frame 93 cm
- B - Maximum height of the foot frame 93 cm
- C - Height of base frame from the ground 46 cm
- D - Base frame Width 89 cm
- E - Max bed width 90 cm
- F - Max base frame length 198 cm
- G - Max bed length 204 cm
- H - Over hang handle extracted 4 cm
- I - Head frame length 74 cm
- L - Base fixed frame Length 117 cm
- M - Maximum angle headrest 0° - $80^{\circ} \pm 5^{\circ}$

MP091 - MP091R - MP093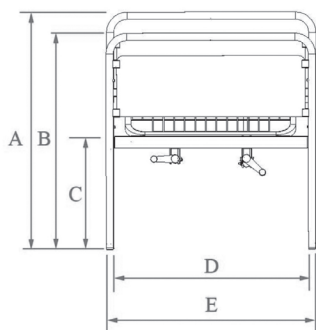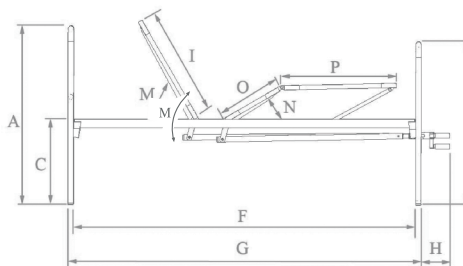

- A - Maximum height of the head frame 93 cm
- B - Maximum height of the foot frame 93 cm
- C - Height of base frame from the ground 46 cm
- D - Base frame Width 89 cm
- E - Max bed width 90 cm
- F - Max base frame length 198 cm
- G - Max bed length 204 cm
- H - Over hang handle extracted 4 cm
- I - Head frame length 74 cm
- M - Maximum angle headrest 0° - $80^{\circ} \pm 5^{\circ}$
- N - Maximum angle leg rest 0° - $40^{\circ} \pm 5^{\circ}$
- O - Femoral support length 39 cm
- P - Foot frame length 39 cm

MP095 - MP097

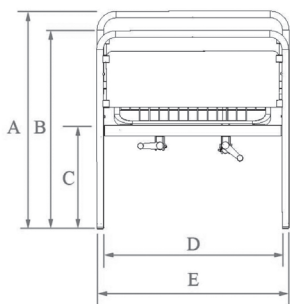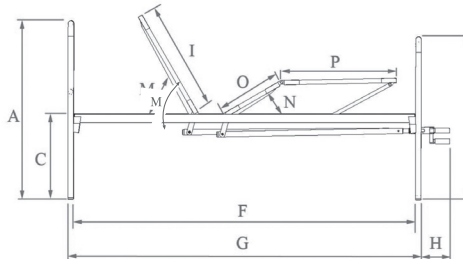- A - Maximum height of the head frame 93 cm
- B - Maximum height of the foot frame 93 cm
- C - Height of base frame from the ground 46 cm
- D - Base frame Width 89 cm
- E - Max bed width 90 cm
- F - Max base frame length 198 cm
- G - Max bed length 204 cm
- H - Over hang handle extracted 4 cm
- I - Head frame length 74 cm
- M - Maximum angle headrest 0° - $80^{\circ} \pm 5^{\circ}$
- N - Maximum angle leg rest 0° - $40^{\circ} \pm 5^{\circ}$
- O - Femoral support length 39 cm
- P - Foot frame length 39 cm

15.2 Technical specification

MP090-MP092

- Base frame: powder coated rectangular 40x20 mm tubular steel structure
- Bed Net: electro-welded steel mesh (4 mm thick wire)
- Head frame/Foot frame: powder coated round tubular steel structure with 30 mm diameter
- Panels: made of laminate with natural shade
- Backrest inclination: from 0° to 80° max $\pm 5^{\circ}$
- Head frame weight: roughly 4 Kg
- Foot frame weight: roughly 4 Kg
- Base frame weight: roughly 29 Kg
- Tot weight: roughly 37 Kg
- Max. load capacity: 135 Kg

MP091-MP091R-MP093-MP095

- Base frame: powder coated rectangular 40x20 mm tubular steel structure.
- Bed Net: electro-welded steel mesh (4 mm thick wire).
- Head frame/Foot frame: powder coated round tubular steel structure with 30 mm diameter
- Panels: made of laminate with natural shade.
- Backrest inclination: from 0° to 80° max $\pm 5^{\circ}$
- Footrest inclination: from 0° to 40° max $\pm 5^{\circ}$
- Head frame weight: roughly 4 Kg
- Foot frame weight: roughly 4 Kg
- Base frame weight: roughly 31 Kg
- Tot weight: roughly 39 Kg
- Max. load capacity: 135 Kg

MP097

- Base Frame: Powder coated rectangular 40x20 mm tubular steel structure
- Bed Net: Electro-welded steel mesh (4mm thick wire)
- Head frame/Foot frame: powder coated round tubular steel structure with 30 mm diameter
- Panels: In light coloured melamine faced MDF

Sections adjustment: **Backrest from 0° to 80° max**
Footrest from 0° to 40° max

Headframe weight: **roughly 4 Kg**

Footboard weight: **roughly 4 Kg**

Base frame weight: **roughly 31 kg**

Net weight: **roughly 39 kg**

Max load capacity: **135 kg**

16. TROUBLESHOOTING

16.1 Symptoms

No commands working

PROBABLE CAUSES

1. The electrical plug is not connected to the mains voltage socket
2. The remote control is not connected to the control unit
3. Possible fault in the remote control or control unit

SOLUTION

1. Connect the power plug
2. Check and, if necessary, connect the remote control to the control unit
3. Contact the service centre for a more precise diagnosis of the problem

16.2 Symptoms

The electric head and/or leg lifts do not work.

PROBABLE CAUSES

1. The headrest actuator is not connected to the control unit.
2. The actuator or remote control or control unit may be defective.

SOLUTION

1. Check the actuator and connect it to the control unit if necessary.
2. Contact the service centre for a more precise diagnosis of the problem

17. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17.1 Repairing

-Warranty repair-

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

-Repair not cover by warranty-

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

-Non defective device-

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

18. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

19. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

**WARRANTY CERTIFICATE**

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

MORETTI S.P.A.
Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com