MOPEDIA®

MA P0300x_P0310G-XL_P0301x_P0305x_P0310x_P0315R_P0320x_P0325x 02 B_ITA_03 2021

POLTRONE ELEVABILI

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ITALIANO

INDICE

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CE | Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

P0300M Ninfea Armchair without Roller System - Fabric RITZ Dark brown
P0300B Ninfea Armchair without Roller System - Fabric GORDON Blue
P0300G Ninfea Armchair without Roller System - Fabric GORDON Grey
P0300G-XL Ninfea Armchair without Roller System - Fabric GORDON Grey
P0300P Ninfea Armchair without Roller System - Fabric GORDON Pistachio
P0300R Ninfea Armchair without Roller System - Fabric GORDON Red
P0300S Ninfea Armchair without Roller System - Fabric GORDON Sand
P0300T Ninfea Armchair without Roller System - Fabric GORDON Brick red

P0301B Ninfea Armchair with Roller System - Fabric GORDON Blue
P0301R Ninfea Armchair with Roller System - Fabric GORDON Red
P0301S Ninfea Armchair with Roller System - Fabric GORDON Sand

P0305G Dafne Armchair without Roller System - Fabric CABREO Grey
P0305S Dafne Armchair without Roller System - Fabric CABREO Sand

P0310B Felce Armchair without Roller System - Fabric SIRMA Blue
P0310C Felce Armchair without Roller System - Fabric - SIRMA Cord

P0315R Anemone Armchair without Roller System - Fabric ALBIS Red

P0320S Timo Armchair with four wheels - Fabric GORDON Sand
P0320B Timo Armchair with four wheels - Fabric GORDON Blue

P0325G Timo Class Armchair with four wheels - Fabric GINOLFO Grey
P0325T Timo Class Armchair with four wheels - Fabric GINOLFO Dove grey

2. INTRODUCTION

Thank you for choosing a Lifting Armchair in the MOPEDIA range by Moretti. Moretti lifting armchairs have been designed and constructed to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this manual in full before you use the armchair you have purchased. Should you have any queries, please contact your dealer for suitable advice and assistance.

NOTE: Check to ensure that no parts of the product have been damaged during shipment. Do not use the product if it is damaged and contact the retailer for further instructions.

WARNING!

Do not use the product for any purpose other than that specified in this manual. Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual. The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features.

3. INTENDED USE

MOPEDIA lifting armchairs are designed and manufactured for the comfort and rest of adults at home. Only use this product for the use for which it has been specifically designed. Any improper use could be a source of danger.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of MEDICAL ELEVATING ARMCHAIRS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

WARNING!

Take extreme care in the presence of moving parts that could trap limbs and cause injury.

- Please consult this manual carefully for correct use of the device.
- Always consult your doctor or therapist for correct use of the device.
- Keep the packaged product away from sources of heat, as the packaging is cardboard.
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts.
- Always take care when children are present.
- The manufacturer and the retailer decline any responsibility for damage caused to people, animals or things as a result of work carried out by unauthorised personnel and failure to comply with this instruction manual.

WARNING!

Pay maximum care and attention to the rules of conduct and safety listed below, in order to avoid accidents.


- Do not use the armchair in damp environments.
- Do not smoke, use naked flames, incandescent objects and equipment that may cause dangerous sparks, do not store combustible materials on and/or near the product: **fire risk.**
- Position the armchairs on flat and firm floor surfaces.
- Use original accessories and/or components.
- Immediately disconnect the electric cable from the mains in the case of malfunctioning and check for any leakage current.
- Do not use electrical extension cables and/or moveable sockets.
- The armchairs are not suitable for use by several people at the same time.
- Do not pull on the electrical cable to remove it from the socket.

- Do not remove or insert the plug with wet hands.
- Do not place your hands inside the mechanical parts when the armchair is open.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

-  Product code
-  Unique Device Identification
-  CE mark
-  Manufacturer
-  Batch Lot
-  Read the instruction manual
-  Medical Device
-  Conditions of disposal
-  Attention, should read the instructions
-  Production dates
-  Waste Electrical & Electronic Equipment (WEEE)

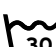
IP54 Protection against dust and water splashes

-  B Type
-  Classe di isolamento
-  Maximum washing temperature 30°C; very mild process
-  Do not bleach
-  Do not iron
-  Do not tumble dry

7. GENERAL DESCRIPTION

Lifting armchairs, designed for rest and relaxation. The backrest and seat have padding with differing densities, designed specifically to maintain correct posture. Painted steel structure, stain-proof and water-repellent technical fabric. The armchairs have two independent motors that allow the backrest and the footrest to be inclined separately, thus allowing several positions to be obtained, including the best one for reading or watching television, relaxing and/or digestion. The armchairs also have a stand-up function, thanks to the motor that allows the person to be lifted. With this lifting device, the seated person is accompanied to an almost fully erect position, thus making it easier for them to stand up. The various backrest and footrest positions are adjusted using a practical and easy-to-use remote control with four buttons.

7.1 View and parts list

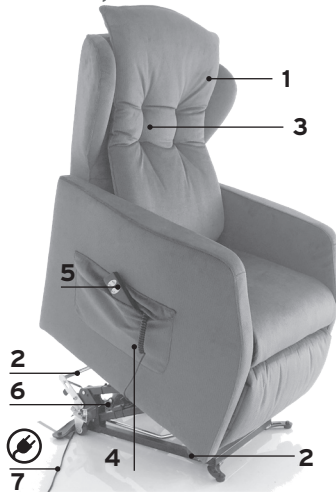1. Armchair
2. Front wheels or roller system
3. Reclining backrest
4. Storage pocket
5. Remote control
6. Two motors for independent movements
7. Electrical cable

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8. ASSEMBLY

8.1 Assembly

After removing the packaging, check that all the armchair components are intact and that there are no visibly damaged parts. In the case of damage, do not use the product and contact your retailer. For connection to the mains electricity, connect the cable with plug on the armchair to the socket. Do not rest any object on the power cable or step on it.

NB: Check that the voltage of the socket is identical to the one shown on the motor label.

NOTE: Even minor electrical work should be performed by professionally qualified people.

WARNING!

- The manufacturer may not be held liable for any damage caused by failure to comply with these indications and this could also invalidate the warranty.
- When using the commands, it must be possible to see the surrounding area clearly, in order to prevent people, animals or things being hit by the moving parts.
- The armchair must be assembled on a horizontal surface. Avoid uneven and fragile surfaces.
- Before use, check that the electrical cables do not interfere with movement of the sections, are not in the way and do not intersect with the power supply cables of other equipment or devices.
- Before use, check that no one inserts their hands, parts of the body or any other object between the moving sections during the procedure, to avoid crushing.

Position the armchair in an area with plenty of space and away from sources of heat (fireplaces, heaters, halogen lamps, etc.) and on a flat surface. Ensure that movement of the footrest and the backrest is not obstructed in any way. The armchair must be a minimum distance of 30 cm from any obstacle (wall, table, etc.). The armchair must be assembled and disassembled by qualified technical operators to avoid mechanical and/or electrical damage to the product. The armchair is supplied already assembled and ready for use, so the user simply has to connect it to the mains electricity.

9. WARNINGS ON USE

- Insert the electrical cable directly in the socket. Do not use an extension cable.
- Before sitting in the armchair, check that the remote control and the relative wire are not in the way.
- Keep hands and feet away from connection points, such as the armchair and footrest lifting mechanisms.
- Leave the armchair in the upright position and with the footrest closed when not in use.
- If the armchair does not function or malfunctions, stop using it immediately, switch it off, remove the plug and contact an authorised retailer or Assistance Centre to have it repaired.
- Do not use the armchair if there are any tears in the leather or fabric covering that cause the internal mechanisms to be exposed.
- Do not allow children to use or play on the armchair.
- Do not use the switches or the remote control and/or unplug the armchair with wet hands.
- Do not replace the plug.
- If the reclining backrest or footrest is used, ensure that there are no objects that could be damaged and/or people who could be injured nearby.
- Avoid sitting down heavily on the mechanism.
- Do not sit, climb or jump on the backrest, the armrests and the footrest.
- Do not place your hands or feet underneath the armchair or in the spaces between the backrest and the seat or the armrest.
- Do not use mobile or cordless telephones near the armchair.
- Do not move the armchair when there is someone sitting in it or objects are placed in it.
- Do not exceed the maximum permitted weight under any circumstances.
- If the armchair is used by children or by non-self-sufficient people, the assistance and presence of another person is necessary.
- Take care over small components that could be swallowed.
- Do not rest any objects on the power cable or step on it.
- It is recommended to use original accessories.
- Never sit on the footrest when the armchair is in the relaxation position (extended).
- Do not position the armchair near sources of heat, in damp environments or on uneven floors.

- The various commands should be used when someone is sitting in the armchair.
- Position the armchair at a minimum distance of 60 cm from the wall.
- Unplug the armchair if you need to move it or have maintenance operations performed.
- Do not place heavy objects on the armchair.
- Do not position the armchair near heaters or other sources of heat.

**WARNING!**

Risk of short-circuit or fire, electric shocks, injuries and accidents.

- Insert the plug completely in the socket.
- Make sure the remote control is placed in the specific pocket when the armchair is not in use.
- Never pull on the cable to unplug the armchair and always grip the plug directly and close to the socket. Do not use if you are wearing wet clothes.
- Do not use the armchair when it is wet.
- Do not use at a different voltage to the one indicated on the label and do not insert the plug in multiple sockets.
- Do not use in an environment with a high percentage of humidity, such as the bathroom or outdoors.
- Do not use if the electrical cable, the plug or the socket are damaged.
- Do not damage, break, alter, bend, stretch, twist or wind the electrical cable during use.
- Do not damage the plug with pins, water or dirt of any kind.
- Do not position the power supply cable close to sources of heat or hot surfaces.

10. HOW TO USE

Note: Consult the general precautions in this instruction manual

**WARNING!**

When using the commands, it must be possible to see the surrounding area clearly, in order to prevent people, animals or things being hit by the moving parts.

**WARNING!**

Do not lean on the armchair or apply downward force when the backrest is reclined. Risk of tipping over and injuries.

**WARNING!**


Do not use the roller system with someone seated in the armchair.

**WARNING!**


Never use the armchair when the wheels are unlocked and free to rotate.

**WARNING!**

The armchair is not designed to move the user around. Only move the armchair when there is no one sitting in it.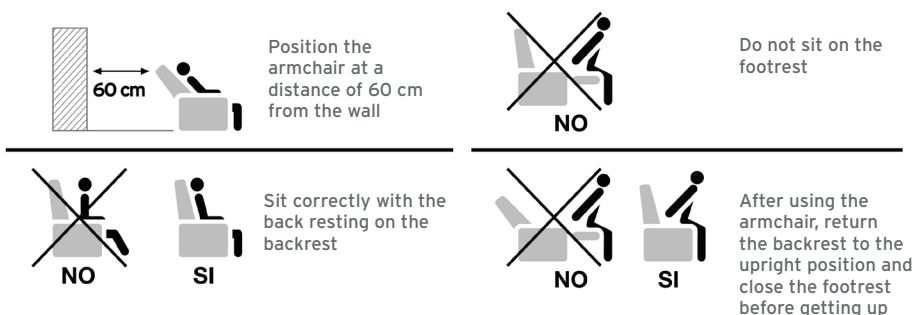

ENGLISH

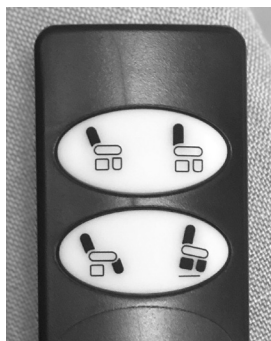TOP LEFT-HAND BUTTON: moves the armchair from the upright position to the lowered backrest position

BOTTOM LEFT-HAND BUTTON: starting from the stand-up position, moves the armchair to the upright position; starting from the upright position, extracts the footrest.

TOP RIGHT-HAND BUTTON: returns the armchair from the lowered backrest position to the upright position.

BOTTOM RIGHT-HAND BUTTON: moves the armchair from the upright position to the stand-up position; starting from the extracted footrest position, returns the footrest into its closed position.

10.1 Roller System

Fig.1

The roller system is installed as a standard feature on the PO301x armchairs. It is available as an accessory on the PO300x, PO305x, PO310x and PO315x models. Push the bar downwards with your foot to engage the roller system (Fig.1). Pull the bar back upwards with your foot to disengage the roller system.

WARNING!

Do not use the roller system with someone seated in the armchair! Do not go up steps over 1 cm!

11. MAINTENANCE

The products in the MOPEDIA range by Moretti have been carefully checked and CE marked when placed on sale. For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original spare parts and accessories for repairs.

It is extremely important to check the device before using it for the first time. In particular, please check that:

- All the screws and bolts are properly tightened.
- The mechanical parts of the device can move correctly (backrest, leg rest, armrests, wheels, etc.).
- Check that the wheel brakes are working properly *(if present).
- Check the mechanical parts (hinges and pins) once a month and lubricate them with a drop of oil to ensure they slide easily and to avoid wear.
- Check that the power cable is undamaged.
- Check efficiency of the electric motors periodically.
- Ensure that the electromechanical parts are unobstructed and functioning properly.
- Check that the metal movement parts of the footrest are not obstructed by any objects.

12. CLEANING

WARNING!

The cover of the armchairs in this manual is not removable.

Disconnect the plug from the electricity main before cleaning.

Clean often the armchair with the vacuum cleaner. In case of stubborn dirt, use neutral detergents with a soft brush.

Maximum washing temperature 30°C; very mild process

Do not bleach

Do not iron

Do not tumble dry

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfects and acetone, as these substances will scratch the plastic parts and cause the metal surfaces to rust.

The manufacture will not be responsible for damage caused by use of materials that could damage the surface of the product or by use of corrosive chemical products during cleaning.

WARNING!

Sanitising products are corrosive. Avoid contact with the eyes and skin. Wear protective clothing, gloves, protective goggles and safety shoes. In the case of accidental contact, wash the part concerned with plenty of warm water and seek medical advice immediately.

13. CARE OF THE ELECTRONIC SYSTEM

- Verify that no armchair cable is cut or damaged.
- In case of any damage contact your dealer
- Keep electronic circuits away from humidity and high temperatures

14. CONDITIONS OF DISPOSAL

14.1 General conditions of disposal

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

14.2 Instructions for correct disposal in accordance with European directive 2012/19/UE:

At the end of its working life, the product must not be disposed of together with normal urban waste. It must be delivered to municipal separated waste collection facilities, or to appropriate dealers that provide this service. Separated waste disposal helps to reduce possible negative effects on the environment and health deriving from improper disposal and allows for recycling the materials comprising the product, which translates into significant energy and resource savings. The product bears the barred bin symbol to underline the obligation of disposing of electro-medical equipment.

15. DECLARATION OF ELECTROMAGNETIC COMPATIBILITY

The LIFTING ARMCHAIR is designed for use in the electromagnetic conditions specified in the following tables. The user of the LIFTING ARMCHAIR must ensure that it is used in the conditions specified.

Table 1

MANUFACTURER'S GUIDE AND DECLARATION		
THE LIFTING ARMCHAIR IS INTENDED TO FUNCTION IN THE FOLLOWING ELECTROMAGNETIC CONDITIONS. THE CLIENT OR USER MUST ENSURE THAT IT IS USED IN THESE CONDITIONS.		
EMISSION TESTS	CONFORMITY	ELECTROMAGNETIC ENVIRONMENT - GUIDE
RF EMISSIONS CISPR 11	GROUP 1	THE LIFTING ARMCHAIR USES RF ENERGY ONLY FOR INTERNAL FUNCTIONING AND FOR CHARGING THE BATTERY. ITS RF EMISSIONS ARE THEREFORE VERY LOW AND IN ALL LIKELIHOOD DO NOT INTERFERE WITH NEARBY ELECTRONIC EQUIPMENT.
RF EMISSIONS CISPR 11	CLASS B	THE LIFTING ARMCHAIR IS INTENDED TO BE USED IN ALL BUILDINGS, INCLUDING THE HOME AND BUILDINGS CONNECTED DIRECTLY TO THE LOW VOLTAGE MAINS ELECTRICITY NETWORK FOR RESIDENTIAL APPLICATIONS.
HARMONIC EMISSIONS EN 61000-3-2	CONFORMING	
VOLTAGE FLUTTER/FLICKER EN 61000-3-3	CONFORMING	

Table 2

MANUFACTURER'S GUIDE AND DECLARATION			
THE LIFTING ARMCHAIR IS INTENDED TO FUNCTION IN THE FOLLOWING ELECTROMAGNETIC CONDITIONS. THE CLIENT OR USER MUST ENSURE THAT IT IS USED IN THESE CONDITIONS.			
IMMUNITY TESTS	TEST LEVEL IEC 60601	CONFORMITY LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDE
ELECTROSTATIC DISCHARGE (ESD) EN 61000-4-2	+ 4 KV CONTACT + 8 KV AIR	+ 4 KV CONTACT + 8 KV AIR	INFORMATION FOR USE OF THE DEVICE. THE FLOOR MUST BE WOOD, CONCRETE OR CERAMIC. IF THE FLOOR IS COVERED WITH A SYNTHETIC MATERIAL, THE RELATIVE HUMIDITY MUST BE AT LEAST 30%.
FAST TRANSIENTS AND BURSTS EN61000-4-4	1KV - 5KHZ INDIVIDUAL COUPLING, 2000V - 5KHZ PARALLEL COUPLING	1KV - 5KHZ INDIVIDUAL COUPLING, 2000V - 5KHZ PARALLEL COUPLING	THE QUALITY OF THE MAINS VOLTAGE MUST BE THAT OF A TYPICAL BUSINESS ENVIRONMENT OR HOSPITAL WHEN CHARGING THE BATTERY
IMPULSE VOLTAGES EN 61000-4-5	2KV PARALLEL COUPLING; 1KV INDIVIDUAL COUPLING	2KV PARALLEL COUPLING; 1KV INDIVIDUAL COUPLING	THE QUALITY OF THE MAINS VOLTAGE MUST BE THAT OF A TYPICAL BUSINESS ENVIRONMENT OR HOSPITAL WHEN CHARGING THE BATTERY
VOLTAGE DROP OUTS, BRIEF INTERRUPTIONS AND VARIATIONS OF VOLTAGE ON THE POWER SUPPLY LINES. EN 61000-4-11	0V/2000MS; 0V/500MS; 0V/50MS; 0V/20MS; 0V/500MS; 0.75V/2000MS; 0.75V/500MS; 0V/50MS;	0V/2000MS; 0V/500MS; 0V/50MS; 0V/20MS; 0V/500MS; 0.75V/2000MS; 0.75V/500MS; 0V/50MS;	THE QUALITY OF THE MAINS VOLTAGE MUST BE THAT OF A TYPICAL BUSINESS ENVIRONMENT OR HOSPITAL WHEN CHARGING THE BATTERY.
CONDUCTED RF EMISSIONS EN55011	0.15 + 0.5 MHZ 79 DBDV 0.5 + 30 MHZ 73 DBDV	0.15 + 0.5 MHZ 79 DBDV 0.5 + 30 MHZ 73 DBDV	THE QUALITY OF THE MAINS VOLTAGE MUST BE THAT OF A TYPICAL BUSINESS ENVIRONMENT OR HOSPITAL WHEN CHARGING THE BATTERY.
RADIATED RF EMISSIONS EN55011	CLASS B 30 - 230 MHZ, 40 (3 M) 230 - 1000 MHZ, 47 (3 M) CLASS A 30 - 230 MHZ, 50 (3 M) 230 - 1000 MHZ, 57 (3 M)	CLASS B 30 - 230 MHZ, 40 (3 M) 230 - 1000 MHZ, 47 (3 M) CLASS A 30 - 230 MHZ, 50 (3 M) 230 - 1000 MHZ, 57 (3 M)	THE QUALITY OF THE MAINS VOLTAGE MUST BE THAT OF A TYPICAL BUSINESS ENVIRONMENT OR HOSPITAL WHEN CHARGING THE BATTERY.

Table 3

MANUFACTURER'S GUIDE AND DECLARATION			
THE LIFTING ARMCHAIR IS INTENDED TO FUNCTION IN THE FOLLOWING ELECTROMAGNETIC CONDITIONS. THE CLIENT OR USER MUST ENSURE THAT IT IS USED IN THESE CONDITIONS.			
IMMUNITY TESTS	TEST LEVEL IEC 60601	CONFORMITY LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDE
CONDUCTED DISTURBANCES INDUCED BY RF FIELDS 61000-4-6	3 V EFF. FROM 150 KHZ TO 80 MHZ	3 V EFF. FROM 150 KHZ TO 80MHZ	<p>PORTABLE AND MOBILE RF COMMUNICATIONS DEVICES SHOULD NOT BE USED NEAR ANY PART OF THE LIFTING ARMCHAIR, INCLUDING THE CABLES, UNLESS THE RECOMMENDED SEPARATION DISTANCES ARE OBSERVED, CALCULATED BY THE EQUATION APPLICABLE TO THE TRANSMITTER FREQUENCY.</p> <p>RECOMMENDED DISTANCES $D = 1.2\sqrt{P}$ FROM 80 MHZ TO 800 MHZ $D = 2.3\sqrt{P}$ FROM 800 MHZ TO 2.5 GHZ</p> <p>WHERE "P" INDICATES THE MAXIMUM RATED OUTPUT OF THE TRANSMITTER IN WATT (W) ACCORDING TO THE TRANSMITTER MANUFACTURER AND "D" IS THE RECOMMENDED SEPARATION DISTANCE IN METRES (M). THE INTENSITY OF THE FIELD GENERATED BY FIXED RF TRANSMITTERS, AS DETERMINED IN AN ON-SITE ELECTROMAGNETIC INSPECTION, MIGHT BE BELOW THE LEVEL OF CONFORMITY IN EACH FREQUENCY BAND. INTERFERENCE MAY BE EXPERIENCED IN THE PROXIMITY OF EQUIPMENT MARKED WITH THIS SYMBOL:</p> 
RF DISTURBANCES EN 61000-4-3	3 V/M FROM 80 MHZ TO 2.5 GHZ	3 V/M FROM 80 MHZ TO 2.5 GHZ	
<p>NOTE 1: AT 80 MHZ AND 800 MHZ, APPLY THE SEPARATION DISTANCE FOR THE HIGHER FREQUENCY BAND. NOTE 2: THESE GUIDELINES MAY NOT APPLY IN ALL SITUATIONS. ELECTROMAGNETIC PROPAGATION IS AFFECTED BY ABSORPTION AND REFLECTION FROM STRUCTURES, OBJECTS AND PEOPLE.</p>			
<p>^A THE INTENSITY OF THE FIELD FOR FIXED TRANSMITTERS, SUCH AS BASE STATIONS FOR RADIO-TELEPHONES (CELLULARS AND CORDLESS) AND TERRESTRIAL MOBILE RADIOS, AMATEUR RADIO EQUIPMENT, AM AND FM RADIO TRANSMITTERS AND TV TRANSMITTERS MAY NOT BE KNOWN PRECISELY IN THEORY. TO ESTABLISH AN ELECTROMAGNETIC ENVIRONMENT CAUSED BY FIXED RF TRANSMITTERS, ONE MUST RUN AN ON-SITE ELECTROMAGNETIC INSPECTION. IF THE FIELD INTENSITY MEASURED IN THE PLACE WHERE THE LIFTING ARMCHAIR IS USED EXCEEDS THE APPLICABLE RF LEVEL OF CONFORMITY INDICATED ABOVE, NORMAL FUNCTIONING OF THE LIFTING ARMCHAIR MUST BE MONITORED. IF ABNORMALITIES ARE OBSERVED, ADDITIONAL MEASURES MAY BE NECESSARY, SUCH AS RE-ORIENTATION OR REPOSITIONING OF THE LIFTING ARMCHAIR.</p> <p>^B THE FIELD INTENSITY OVER THE FREQUENCY RANGE FROM 150 KHZ TO 80 MHZ SHOULD BE LESS THAN 3 V/M.</p>			

ENGLISH

Table 4

RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RADIO COMMUNICATIONS EQUIPMENT AND THE LIFTING ARMCHAIR			
THE LIFTING ARMCHAIR IS INTENDED TO OPERATE IN AN ELECTROMAGNETIC ENVIRONMENT IN WHICH RADIATED RF DISTURBANCES ARE UNDER CONTROL. THE CUSTOMER OR USER OF THE LIFTING ARMCHAIR CAN CONTRIBUTE TO PREVENTING EM INTERFERENCE BY ENSURING A MINIMUM DISTANCE BETWEEN MOBILE AND PORTABLE RF COMMUNICATIONS DEVICES (TRANSMITTERS) AND THE LIFTING ARMCHAIR BATTERY CHARGER DURING CHARGING AND THE ENTIRE LIFTING ARMCHAIR DURING ITS OPERATION AS RECOMMENDED BELOW, IN RELATION TO THE MAXIMUM OUTPUT POWER OF THE RF COMMUNICATIONS DEVICES			
TESTS	DURING USE OF THE LIFTING ARMCHAIR		
	FROM 150 KHZ TO 80 MHZ D = 1.2VP	FROM 80 KHZ TO 800 MHZ D = 1.2VP	FROM 800 KHZ TO 2.5 GHZ D = 1.2VP
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

FOR TRANSMITTERS WHOSE MAXIMUM RATED OUTPUT POWER IS NOT LISTED ABOVE, THE RECOMMENDED SEPARATION DISTANCE D, IN METRES (M), CAN BE CALCULATED USING THE EQUATION APPLICABLE TO THE TRANSMITTER FREQUENCY, WHERE P IS THE MAXIMUM RATED OUTPUT POWER OF THE TRANSMITTER, IN WATT (W), AS GIVEN BY THE TRANSMITTER'S MANUFACTURER.
 NOTE 1: AT 80 MHZ AND 800 MHZ, APPLY THE SEPARATION DISTANCE FOR THE HIGHER FREQUENCY BAND.
 NOTE 2: THESE GUIDELINES MAY NOT APPLY IN ALL SITUATIONS. ELECTROMAGNETIC PROPAGATION IS AFFECTED BY ABSORPTION AND REFLECTION FROM STRUCTURES, OBJECTS AND PEOPLE.

ENGLISH

16. TECHNICAL SPECIFICATIONS

16.1 Dimensions and weights*

Dimensions and weight											
Codes											
PO300x	53	46	48	77	85	61	100	160	150	130	52
PO300G-XL	64	46	48	88	85	61	100	160	150	160	55
PO301x	53	46	48	77	85	61	100	160	150	130	52
PO305x	53	46	48	77	85	61	110	165	150	130	55
PO310x	53	47	49	66	85	61	110	165	150	130	55
PO315x	52	48	48	75	85	61	110	165	150	130	55
PO320x	52	47	49	72	85	61	110	170	150	130	58
PO325x	52	47	49	72	85	61	110	175	150	130	58

*The dimensions are indicated in cm and may differ by ± 1cm

16.2 Technical specifications

PO300x - PO300G-XL - PO301x (NINFEA)

- Powder coated steel and wooden frame;
- Water-repellent and stain-proof fabric;
- Armchair cover
- Non-deformable polyurethane foam padding at different densities (backrest 18-22 Kg/m³, footrest 18 Kg/m³, seat 35 Kg/m³, armrests 40 Kg/m³);
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Stand-up function;
- 2 independent motors;
- Four-button remote control with cable inside the pocket;
- 2 small front wheels for movement over short distances (only on PO300x - PO300G-XL);
- Roller system (standard feature only on PO301x) four wheels, Ø 50 mm;
- 230V - 110W;
- Operating temperature -20°C / + 60°C;
- Recommended care conditions: keep in a cool and dry place, away from direct sources of heat, such as sunlight (recommended temperature between -10°C and + 40°C);
- In the case of a lengthy period of non-use, disconnect the power supply and protect the parts that could be damaged by dust deposits.

PO305x (DAFNE)

- Powder coated steel and wooden frame;
- Water-repellent and stain-proof fabric;
- Armchair cover
- Non-deformable polyurethane foam padding at different densities (backrest 18-22 Kg/m³, footrest 18 Kg/m³, seat 35 Kg/m³, armrests 40 Kg/m³);
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Stand-up function;
- 2 independent motors;
- Four-button remote control with cable inside the pocket;
- 2 small front wheels for movements over short distances;
- 230V - 110W;
- Operating temperature -20°C / + 60°C;
- Recommended care conditions: keep in a cool and dry place, away from direct sources of heat, such as sunlight (recommended temperature between -10°C and + 40°C);
- In the case of a lengthy period of non-use, disconnect the power supply and protect the parts that could be damaged by dust deposits.

PO310x (FELCE)

- Powder coated steel and wooden frame;
- Water-repellent and stain-proof fabric;
- Non-deformable polyurethane foam padding at different densities (backrest 18-22 Kg/m³, footrest 18 Kg/m³, seat 35 Kg/m³, armrests 35 Kg/m³);
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Stand-up function;
- 2 independent motors;

- Four-button remote control with cable inside the pocket;
- 2 small front wheels for movements over short distances;
- 230V - 110W;
- Operating temperature -20°C / + 60°C;
- Recommended care conditions: keep in a cool and dry place, away from direct sources of heat, such as sunlight (recommended temperature between -10°C and + 40°C);
- In the case of a lengthy period of non-use, disconnect the power supply and protect the parts that could be damaged by dust deposits.

P0315x (ANEMONE)

- Powder coated steel and wooden frame;
- Water-repellent and stain-proof fabric;
- Non-deformable polyurethane foam padding at different densities (footrest 18 Kg/m³, seat 35 Kg/m³, armrests 35 Kg/m³);
- Backrest in silicon wadding;
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Stand-up function;
- 2 independent motors;
- Four-button remote control with cable inside the pocket;
- 2 small front wheels for movements over short distances;
- 230V - 110W;
- Operating temperature -20°C / + 60°C;
- Recommended care conditions: keep in a cool and dry place, away from direct sources of heat, such as sunlight (recommended temperature between -10°C and + 40°C);
- In the case of a lengthy period of non-use, disconnect the power supply and protect the parts that could be damaged by dust deposits.

P0320x (TIMO)

- Powder coated steel and wooden frame;
- Water-repellent and stain-proof fabric;
- Non-deformable polyurethane foam padding at different densities (backrest 18-22 Kg/m³, footrest 18 Kg/m³, seat 35 Kg/m³, armrests 35 Kg/m³);
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Extractable armrests;
- Extractable ear flaps;
- Four wheels, Ø 50 mm, of which two with brake;
- Push bar;
- Stand-up function;
- 2 independent motors;
- Four-button remote control with cable inside the pocket;
- 230V - 110W
- Operating temperature -20°C / + 60°C;
- Recommended care conditions: keep in a cool and dry place, away from direct sources of heat, such as sunlight (recommended temperature between -10°C and + 40°C);
- In the case of a lengthy period of non-use, disconnect the power supply and protect the parts that could be damaged by dust deposits.

P0325x

- Powder coated steel and wooden frame
- Water-repellent and stain-proof fabric
- Non-deformable polyurethane foam padding at different densities (backrest 18-22 Kg/m³, footrest 18 Kg/m³, seat 35 Kg/m³, armrests 35 Kg/m³);
- Seat with additional padding in 3 cm memory foam;
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Extractable armrests;
- Extractable ear flaps;
- Four wheels, Ø 50 mm, of which two with brake;
- Push bar;
- Stand-up function;
- Vertical lift position;
- Bed position function;
- 2 independent motors;
- Four-button remote control with cable inside the pocket;
- 230V - 110W
- Operating temperature -20°C / + 60°C;
- Recommended care conditions: keep in a cool and dry place, away from direct sources of heat, such as sunlight (recommended temperature between -10°C and + 40°C);
- In the case of a lengthy period of non-use, disconnect the power supply and protect the parts that could be damaged by dust deposits.

16.3 Fabric specifications

All the armchair fabrics are:

ANTI-MITE

STAIN-PROOF

ANTI-MICROBIAL
AND ANTI-BACTERIAL

BREATHABLE

17. ACCESSORIES

- Roller system (ONLY FOR P0300x, P0300G-XL, P0305x, P0310x, P0315x)
- Armchair covers kit (ONLY FOR P0310x, P0315x, P0320x, P0325x)
- Small table (ONLY FOR P0320x E P0325x)
- Portable battery kit

Note: The list of accessories is revised continually. We recommend contacting retailers to find out about any new features and availability and to obtain more information.

17.1 Instructions for installation of the Roller System

WARNING!

Installation operations must be performed by qualified personnel.

WARNING!

All responsibility is declined for malfunctions or damage caused by incorrect installation and placing in operation by unqualified personnel.

Follow the steps below with care to install the roller system accessory (POA3xx):

- Place the armchair in operation and put it in the lift position;
- Disconnect the plug;
- Detach the two front wheels using a standard 10 mm wrench;
- Tip the armchair forwards;

Fig.2

Fig.3

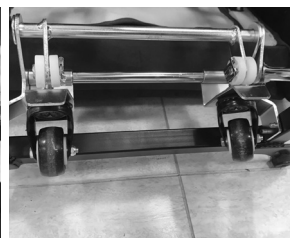
Fig.4

- Insert the roller system (supplied already partially assembled) into the steel base of the armchair (fig. 2);
- Line up the four holes on the front of the armchair frame base with those present on the roller system supports and lock them in place with the four screws and nuts provided (Figs. 3 and 4);

Fig.5

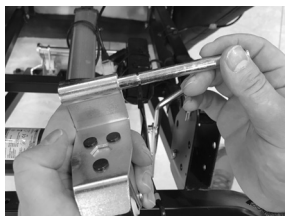
Fig.6

Fig.7

Fig.8

- Take the components shown in Fig.5;
 - Insert the long pin on the wheel support (Fig. 6);
 - Insert the pin and wheel in the rear right-hand plate of the frame (Figs. 7 and 8);
 - Insert the stop on the pin on the outer side of the frame;
 - Insert the washer on the pin from the inner side of the frame;
- Assemble the roller system rod on the pin previously installed (Fig. 9) and insert the stop on the inner side as well;

Fig.9

Fig.10

Fig.11

- Take the components shown in Fig. 10;
- Insert the short pin on the wheel support;
- Insert the pin and wheel in the rear left-hand plate of the frame (Fig. 11);
- Insert the washer on the pin from the inner side of the frame;
- Insert the stops on both ends of the pin;

Fig.12

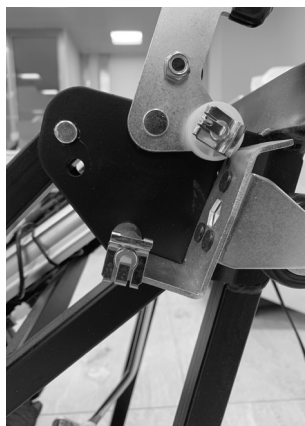
Fig.13

- Take the steel bar with the two remaining pins and place it close to the frame plates, turning it so that the black rubber tips face downwards;
- Insert the right-hand curved rod of the roller system inside the hole on the bar (Fig.12);
- Insert the pins to block the bar on the armchair frame plates;
- Insert the stops on both pins (Fig.13);
- Tip the armchair back into its normal position;
- Connect the armchair to the mains current and return it to the upright position.

18. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject to the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual. Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

Any improper use of the armchair not envisaged in this manual will invalidate the manufacturer's responsibility and warranty in the case of accidents or damage.

19. REPAIRS

19.1 Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

19.2 Repairing a product not covered by the warranty

Product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

19.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

20. SPARE PARTS

Moretti original spare parts are guaranteed for 6 (six) months from the day of delivery. Consult the Moretti catalogue only for the list of spare parts.

21. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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