**MOPEDIA®**

MA P0400x\_P0400G-XL\_P0401x\_P0403x\_P0405x\_P0410x\_P03415R\_P0420x\_P0425x\_P0430x 01 A\_ITA\_01 2024

**POLTRONE ELEVABILI**  

---

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## INDICE

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REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 withcerning medical devices

**1. CODES**

<b>PO400B</b>	Ninfea armchair without Roller System - Fabric FRANZIA CORTA Light Blue
<b>PO400G-XL</b>	Ninfea armchair without Roller System - Fabric FRANZIA CORTA Grey
<b>PO400R</b>	Ninfea armchair without Roller System - Fabric FRANZIA CORTA Coral red
<b>PO400S</b>	Ninfea armchair without Roller System - Fabric FRANZIA CORTA Sand
<b>PO401B</b>	Ninfea armchair with Roller System - Fabric FRANZIA CORTA Blue
<b>PO401R</b>	Ninfea armchair with Roller System - Fabric FRANZIA CORTA Red
<b>PO401S</b>	Ninfea armchair with Roller System - Fabric FRANZIA CORTA Sand
<b>PO401M</b>	Ninfea armchair with Roller System - Fabric FRANZIA CORTA Terracotta
<b>PO401T</b>	Ninfea armchair with Roller System - Fabric FRANZIA CORTA Dove grey
<b>PO401R</b>	Ninfea armchair with Roller System - Fabric FRANZIA CORTA Coral red
<b>PO403G</b>	Chloe armchair with Roller System - Fabric INCANTO Grey Perla
<b>PO403R</b>	Chloe armchair with Roller System - Fabric INCANTO Antique pink
<b>PO405G</b>	Dafne armchair without Roller System - Fabric INCANTO Grey
<b>PO405S</b>	Dafne armchair without Roller System - Fabric INCANTO Sand
<b>PO405R</b>	Dafne armchair with Roller System - Fabric INCANTO Antique pink
<b>PO405T</b>	Dafne armchair with Roller System - Fabric INCANTO Dove grey
<b>PO410B</b>	Agave armchair with Roller System - Fabric LUX Blue
<b>PO410R</b>	Agave armchair with Roller System - Fabric LUX Red
<b>PO410S</b>	Agave armchair with Roller System - Fabric LUX Sand
<b>PO420S</b>	Myrtho armchair with 4 wheels - Fabric LUX Sand
<b>PO420B</b>	Myrtho armchair with 4 wheels - Fabric LUX Blue
<b>PO420G</b>	Myrtho armchair with 4 wheels - Fabric LUX Grey
<b>PO420R</b>	Myrtho armchair with 4 wheels - Fabric LUX Red
<b>PO425G</b>	Myrtho Class armchair with 4 wheels - Fabric LUX Grey
<b>PO425B</b>	Myrtho Class armchair with 4 wheels - Fabric LUX Blue
<b>PO425R</b>	Myrtho Class armchair with 4 wheels - Fabric LUX Red
<b>PO425S</b>	Myrtho Class armchair with 4 wheels - Fabric LUX Sand
<b>PO430G</b>	Clyzia armchair Zero Gravity - Fabric INCANTO Grey
<b>PO430R</b>	Clyzia armchair Zero Gravity - Fabric INCANTO Antique pink
<b>PO430T</b>	Clyzia armchair Zero Gravity - Fabric INCANTO Dove grey

**2. INTRODUCTION**

Thank you for choosing a Lifting Armchair in the MOPEDIA range by Moretti. Moretti lifting armchairs have been designed and built to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this manual in full before you use the armchair you have purchased. Should you have any queries, please contact your dealer for suitable advice and assistance.

**NOTE: Check to ensure that no parts of the product have been damaged during shipment. Do not use the product if it is damaged and contact the retailer for further instructions.**

**WARNING!**

Do not use the product for any purpose other than that specified in this manual. Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual. The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features.

### 3. INTENDED USE

MOPEDIA elevating chairs are intended for indoor patient sitting and resting.

### 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of MEDICAL ELEVATING ARMCHAIRS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

### 5. GENERAL WARNINGS

#### WARNING!

Take extreme care in the presence of moving parts that could trap limbs and cause injury.

- The useful life of the device is determined by the wear and tear of nonrepairable parts and not more than 10 years
- Please consult this manual carefully for correct use of the device.
- Always consult your doctor or therapist for correct use of the device.
- Keep the packaged product away from sources of heat, as the packaging is cardboard.
- Always take care when children are present.
- The manufacturer and the retailer decline any responsibility for damage caused to people, animals or things as a result of work carried out by unauthorised personnel and failure to comply with this instruction manual.

#### WARNING!

Pay maximum care and attention to the rules of conduct and safety listed below, in order to avoid accidents.

- Do not use the armchair in damp environments.
- Do not smoke, use naked flames, incandescent objects and equipment that may cause dangerous sparks, do not store combustible materials on and/or near the product: **fire risk**.
- Position the armchairs on flat and firm floor surfaces.
- Use original accessories and/or components.
- Immediately disconnect the electric cable from the mains in the case of malfunctioning and check for any leakage current.
- Do not use electrical extension cables and/or moveable sockets.
- The armchairs are not suitable for use by several people at the same time.
- Do not pull on the electrical cable to remove it from the socket.
- Do not remove or insert the plug with wet hands.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.


**6. SYMBOLS**

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Max load
	Conditions of disposal
	Attention, should read the instructions
	Production dates
	Waste Electrical & Electronic Equipment (WEEE)
<b>IP54</b>	Protection against dust and water splashes
	B Type
	Insulation class
	Washing 30°
	Do not bleach
	Dry cleaning
	Do not tumble dry
	Hand washing
	Do not iron
	Drying on a flat surface in the shade

## 7. GENERAL DESCRIPTION

Lifting armchairs, designed for rest and relaxation. The backrest and seat have padding with differing densities, designed specifically to maintain correct posture. Painted steel structure, Polyester fabric. The armchairs have two independent motors that allow the backrest and the footrest to be inclined separately, thus allowing several positions to be obtained, including the best one for reading or watching television, relaxing and/or digestion. The armchairs also have a stand-up function, thanks to the motor that allows the person to be lifted. With this lifting device, the seated person is accompanied to an almost fully erect position, thus making it easier for them to stand up. The various backrest and footrest positions are adjusted using a practical and easy-to-use remote control with four buttons.

### 7.1 View and parts list

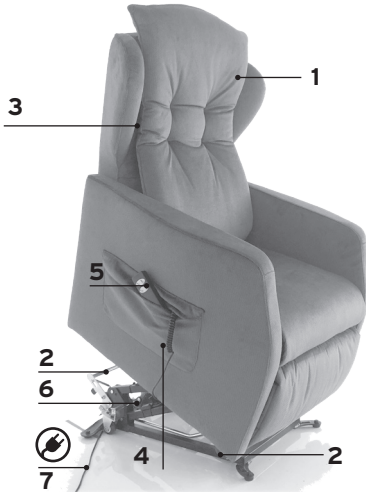1. Armchair
2. Front wheels or roller system
3. Reclining backrest
4. Storage pocket
5. Remote control
6. Two motors for independent movements
7. Electrical cable

Image only for illustrative purpose

## 8. ASSEMBLY

### 8.1 Assembly for every model except Chloe armchair PO403-xx

After removing the packaging, check that all the armchair components are intact and that there are no visibly damaged parts. In the case of damage, do not use the product and contact your retailer. For the connection to the mains electricity, connect the cable with plug on the armchair to the socket. Do not rest any object on the power cable or step on it.

### 8.2 Assembling of PO403-xx

The PO403 armchair is shipped with and backrest to be assembled (Fig.1)

1. Remove the support guard on which the armchair back will be housed (Fig.2)
2. Check that the support on which the backrest on the back side of the chair will be housed is facing outward (Fig. 3)
3. Insert the backrest by matching the 2 joints until the slide is complete. (Fig.4)
4. To disassemble the ears, it will be sufficient to simply slide them off the support rail. For the backrest, it is necessary to have a screwdriver to facilitate release from the interlocking support placed on the chair. (Fig.5)

Fig.1

Fig.2

Fig.3

Fig.4

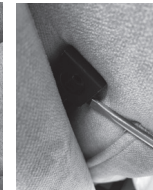
Fig. 5

**NB:** Check that the voltage of the socket is identical to the one shown on the motor label.

**NOTE:** Even minor electrical work should be performed by professionally qualified people.

**WARNING!**

- The manufacturer may not be held liable for any damage caused by failure to comply with these indications and this could also invalidate the warranty.
- When using the commands, it must be possible to see the surrounding area clearly, in order to prevent people, animals or things being hit by the moving parts.
- The armchair must be assembled on a horizontal surface. Avoid uneven and fragile surfaces.

- Place the chair in wide places, away from heat sources (fireplaces, stoves, halogen lamps, etc.).
- Make sure that the movement of the footboard and backrest is not hindered in any way.
- The armchair must have a minimum distance of 60 cm from any obstacle (wall, table, etc.)
- The armchair is supplied already assembled and ready to use, except for model PO403x (CHLOE); the user only needs to connect it to the power supply.

**9. WARNINGS ON USE**

- Connect the cord directly to the electrical outlet, do not replace it or use an extension cord.
- Do not rest objects, walk or place the power cord near hot surfaces or heat sources.
- Take care to properly plug the outlet into a single socket and unplug it, without pulling the cord, but from the plug when performing handling or maintenance.
- Do not handle cords with wet hands or clothing and make sure the voltage used is congruent with what is written on the label
- Do not damage the plug and/or power cord and do not use them if they are.
- Before sitting in the chair make sure the remote control and its wire are not in the way
- When not in use, leave the chair the footboard closed.
- Make sure to always place the chair on completely flat surfaces, both during use and not, to ensure stability and proper operation.
- Never sit on the footrest and generally do not sit violently on the chair. Avoid jumping or standing on the device.
- Make sure that the person sits by centering his or her weight on the padded cushion, regardless of whether the chair reclines or not.
- To move the chair, make sure it is not reclined and to raise the footboard by at least 2 inches.
- Do not allow children to misuse the device, and ensure that if they or dependent persons use it there is always adequate supervision
- The chair is intended for people of sound physical and mental constitution (not using alcohol or drugs, not suffering from mental illness or other diseases that cause hallucinations, imbalances or the like)
- Get hands and feet away from attachment points such as the armchair and footrest lifting mechanisms, between the gaps of the backrest or arm rest.

**WARNING!**

Hand crushing

- Do not use the device if any covers are torn resulting in leakage of internal equipment.
- Do not use cell phones or laptops near the chair.
- Beware of small parts that could be swallowed.
- It is recommended that original accessories be used.
- It is important that the remote control is used only with the person sitting in the chair.
- Do not use in very humid environments such as bathrooms or outdoors
- Do not expose the chair to direct sunlight (UV) for an extended period of time.
- The product upholstery is not designed to be fire retardant.
- Do not smoke on or around the armchair.

**WARNING!**

Risk of short-circuit or fire, electric shocks, injuries and accidents.

**10. HOW TO USE**

**Note:** consult the general precautions in this instruction manual

**WARNING!**

When using the commands, it must be possible to see the surrounding area clearly, in order to prevent people, animals or things being hit by the moving parts.

**WARNING!**


Never use the chair when the wheels or roller system are unlocked and free to rotate

**WARNING!**

The armchair is not designed to move the user around. Only move the armchair when there is no one sitting in it.

**WARNING!**

Proper operation of the armchair requires that it be used only with the person seated and in the correct position

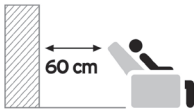Position the armchair at a distance of 60 cm from the wall


Do not sit on the footrest

Sit correctly with the back resting on the backrest

After using the armchair, return the backrest to the upright position and close the footrest before getting up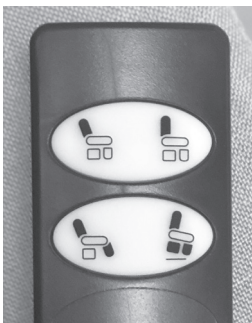

**TOP LEFT-HAND BUTTON:** moves the armchair from the upright position to the lowered backrest position

**BOTTOM LEFT-HAND BUTTON:** starting from the stand-up position, moves the armchair to the upright position; starting from the upright position, extracts the footrest.

**TOP RIGHT-HAND BUTTON:** returns the armchair from the lowered backrest position to the upright position.

**BOTTOM RIGHT-HAND BUTTON:** moves the armchair from the upright position to the stand-up position; starting from the extracted footrest position, returns the footrest into its closed position.

(images for illustrative purposes)

### 10.1 Roller System

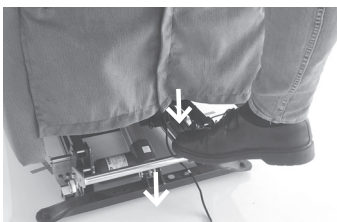
Fig.1

The roller system is installed as a standard feature on the PO401X, PO405X, PO410X armchairs. It is available as an accessory on the PO400x model. Push the bar downwards with your foot to engage the roller system (Fig.1). Pull the bar back upwards with your foot to disengage the roller system.

**WARNING!**

Do not use the roller system with someone seated in the armchair! Do not go up steps over 1 cm!

**11. MAINTENANCE**

The products in the MOPEDIA range by Moretti have been carefully checked and CE marked when placed on sale. For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original spare parts and accessories for repairs.

**WARNING!**

- Adjustment or repair work may only be carried out by a technician qualified.
- Before carrying out any work on maintenance or cleaning of the chair, disconnect the power supply.

It is extremely important to check the device before using it for the first time. In particular, please check that:

- All the screws and bolts are properly tightened.
- The mechanical parts of the device can move correctly (backrest, leg rest, armrests, wheels, etc.).
- Check that the wheel brakes are working properly \*(if present).
- Check the mechanical parts (hinges and pins) once a month and lubricate them with a drop of oil to ensure they slide easily and to avoid wear.
- Check that the chair frame does not make any squeaks or suspicious noises. If necessary, lubricate the pivot points on the frame.
- Check that the power cable is undamaged.
- Check efficiency of the electric motors periodically.
- Ensure that the electromechanical parts are unobstructed and functioning properly.
- Check that the metal movement parts of the footrest are not obstructed by any objects.

**12. TROUBLESHOOTING**

PROBLEM	CAUSE	RESOLUTION
THE CHAIR DOES NOT PERFORM THE REQUIRED MOVEMENTS DUE TO MECHANICAL PROBLEMS	INSUFFICIENT SPACE WITHIN THE RANGE OF CHAIR MOVEMENT	REMOVE OBJECTS THAT HINDER ITS MOVEMENT OR MOVE IT AWAY FROM THE WALL
	OBJECT LOCKED IN THE HANDLING MECHANISM	ATTEMPT TO REMOVE THE OBJECT WITHOUT GETTING HURT OR INJURY, IN CASE IT IS CONSIDERED HIGH-RISK BEHAVIOR CONTACT THE DEALER
REMOTE CONTROL PROBLEMS	PROBLEMS WITH REMOTE CONTROL WITH "CLICK" DURING BUTTON MASHING	THE PROBLEM CONCERNS THE ELECTRONIC PART THE REMOTE CONTROL WORKS NORMALLY
	PROBLEMS WITH REMOTE CONTROL WITHOUT "CLICK" DURING BUTTON MASHING	THE PROBLEM IS WITH THE REMOTE CONTROL AND WILL NEED TO BE REPLACED

HANDLING PROBLEMS DUE TO THE ELECTRICAL SYSTEM	ELECTRICAL PLUG NOT CONNECTED OR NOT CONNECTED PROPERLY	CONNECT THE PLUG TO THE ELECTRICAL OUTLET
	DAMAGED CABLE OR PLUG	CONTACT YOUR DEALER
	ELECTRICAL COMPONENTS NOT CONNECTED PROPERLY	CHECK THE CONNECTIONS OF ELECTRICAL COMPONENTS

### 13. CLEANING

#### WARNING!

The cover of the armchairs in this manual is not removable.

Disconnect the plug from the electricity main before cleaning.

Clean often the armchair with the vacuum cleaner. In case of stubborn dirt, use specific for fabrics with a soft clean mechanical components regularly to limit dust accumulation in the frame.

#### Washing symbols PO400x; PO401x (FRANCIA CORTA fabric 315gr/mq)

Washing 30°

Do not bleach

Dry cleaning

Do not dry by rotary drum

#### Washing symbols PO410x; PO420x; PO425x (LUX fabric 467gr/mq)

Hand washing

Do not bleach

Do not iron

Drying on a flat surface in the shade

Do not dry by rotary drum

Dry cleaning

#### Washing symbols PO403x; PO405x; PO430x (INCANTO fabric 400gr/mq)

Washing 30°

Do not bleach

Do not dry by rotary drum

Dry cleaning

**WARNING!**

Incanto and Lux are the only hydrorepellent fabric.

**WARNING!**

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfects and acetone, as these substances will scratch the plastic parts and cause the metal surfaces to rust.

The manufacture will not be responsible for damage caused by use of materials that could damage the surface of the product or by use of corrosive chemical products during cleaning.

**WARNING!**

Sanitising products are corrosive. Avoid contact with the eyes and skin. Wear protective clothing, gloves, protective goggles and safety shoes. In the case of accidental contact, wash the part concerned with plenty of warm water and seek medical advice immediately.

**13.1 Most common stain removals**

- Blot the stain as soon as possible with paper towels before acting with a solution of warm water and mild soap.
- Test the product on a small portion first to make sure it is not affected. Use a soft cloth moistened with the soap and water solution and wipe gently until completely removed, rinse with pure water and dry with a clean cloth.
- Do not use harsh detergents, stain removers or degreasers as it may damage the material.

**WARNING!**

Sanitising products are corrosive. Avoid contact with eyes and skin. In case of contact accidentally, wash the affected part thoroughly with warm water and consult a doctor immediately.

**14. CARE OF THE ELECTRONIC SYSTEM**

- Verify that no armchair cable is cut or damaged.
- In case of any damage in contact your dealer
- Keep electronic circuits away from humidity and high temperatures

**15. CONDITIONS OF DISPOSAL**

**15.1 General conditions of disposal**

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

**15.2 Instructions for correct disposal in accordance with European directive 2012/19/UE:**

At the end of its working life, the product must not be disposed of together with normal urban waste. It must be delivered to municipal separated waste collection facilities, or to appropriate dealers that provide this service. Separated waste disposal helps to reduce possible negative effects on the environment and health deriving from improper disposal and allows for recycling the materials comprising the product, which translates into significant energy and resource savings. The product bears the barred bin symbol to underline the obligation of disposing of electro-medical equipment.

## 16. DECLARATION OF ELECTROMAGNETIC COMPATIBILITY

The LIFTING ARMCHAIR is designed for use in the electromagnetic conditions specified in the following tables. The user of the LIFTING ARMCHAIR must ensure that it is used in the conditions specified.

Table 1

MANUFACTURER'S GUIDE AND DECLARATION		
THE LIFTING ARMCHAIR IS INTENDED TO FUNCTION IN THE FOLLOWING ELECTROMAGNETIC CONDITIONS. THE CLIENT OR USER MUST ENSURE THAT IT IS USED IN THESE CONDITIONS.		
EMISSION TESTS	COMPLIANCE	ELECTROMAGNETIC ENVIRONMENT - GUIDE
RF EMISSIONS CISPR 11	GROUP 1	THE LIFTING ARMCHAIR IS INTENDED TO BE USED IN ALL BUILDINGS, INCLUDING THE HOME AND BUILDINGS CONNECTED DIRECTLY TO THE LOW VOLTAGE MAINS ELECTRICITY NETWORK FOR RESIDENTIAL APPLICATIONS.
RF EMISSIONS CISPR 11	CLASS B	
HARMONIC EMISSIONS EN 61000-3-2	COMPLIANCE	
VOLTAGE FLUTTER/ FLICKER EN 61000-3-3	COMPLIANCE	

Table 2

MANUFACTURER'S GUIDE AND DECLARATION			
THE LIFTING ARMCHAIR IS INTENDED TO FUNCTION IN THE FOLLOWING ELECTROMAGNETIC CONDITIONS. THE CLIENT OR USER MUST ENSURE THAT IT IS USED IN THESE CONDITIONS.			
IMMUNITY TESTS	TEST LEVEL IEC 60601	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDE
ELECTROSTATIC DISCHARGE (ESD) EN 61000-4-2	± 4 KV IN CONTACT ± 8 KV AIR	± 4 KV IN CONTACT ± 8 KV AIR	INFORMATION FOR USE OF THE DEVICE. THE FLOOR MUST BE WOOD, CONCRETE OR CERAMIC. IF THE FLOOR IS COVERED WITH A SYNTHETIC MATERIAL, THE RELATIVE HUMIDITY MUST BE AT LEAST 30%.
FAST TRANSIENTS AND BURSTS EN61000-4-4	1KV - 5KHZ INDIVIDUAL COUPLING, 2000V - 5KHZ PARALLEL COUPLING	1KV - 5KHZ INDIVIDUAL COUPLING, 2000V - 5KHZ PARALLEL COUPLING	THE QUALITY OF THE MAINS VOLTAGE MUST BE THAT OF A TYPICAL BUSINESS ENVIRONMENT OR HOSPITAL WHEN CHARGING THE BATTERY
IMPULSE VOLTAGES EN 61000-4-5	2KV PARALLEL COUPLING; 1KV INDIVIDUAL COUPLING	2KV PARALLEL COUPLING; 1KV INDIVIDUAL COUPLING	THE QUALITY OF THE MAINS VOLTAGE MUST BE THAT OF A TYPICAL BUSINESS ENVIRONMENT OR HOSPITAL WHEN CHARGING THE BATTERY
VOLTAGE DROP OUTS, BRIEF INTERRUPTIONS AND VARIATIONS OF VOLTAGE ON THE POWER SUPPLY LINES. EN 61000-4-11	0V/2000MS; 0V/500MS; 0V/50MS; 0V/20MS; 0V/500MS; 0.75V/2000MS; 0.75V/500MS; 0V/50MS;	0V/2000MS; 0V/500MS; 0V/50MS; 0V/20MS; 0V/500MS; 0.75V/2000MS; 0.75V/500MS; 0V/50MS;	THE QUALITY OF THE MAINS VOLTAGE MUST BE THAT OF A TYPICAL BUSINESS ENVIRONMENT OR HOSPITAL WHEN CHARGING THE BATTERY.
CONDUCTED RF EMISSIONS EN55011	0.15 ÷ 0.5 MHZ 79 DBµV  0.5 ÷ 30 MHZ 73 DBµV	0.15 ÷ 0.5 MHZ 79 DBµV  0.5 ÷ 30 MHZ 73 DBµV	THE QUALITY OF THE MAINS VOLTAGE MUST BE THAT OF A TYPICAL BUSINESS ENVIRONMENT OR HOSPITAL WHEN CHARGING THE BATTERY.

RADIATED RF EMISSIONS EN55011	<p><b>CLASS B</b> 30 - 230 MHZ, 40 (3 M) 230 - 1000 MHZ, 47 (3 M)</p>	<p><b>CLASS B</b> 30 - 230 MHZ, 40 (3 M) 230 - 1000 MHZ, 47 (3 M)</p>	<p>THE QUALITY OF THE MAINS VOLTAGE MUST BE THAT OF A TYPICAL BUSINESS ENVIRONMENT OR HOSPITAL WHEN CHARGING THE BATTERY.</p>
	<p><b>CLASS A</b> 30 - 230 MHZ, 50 (3 M) 230 - 1000 MHZ, 57 (3 M)</p>	<p><b>CLASS A</b> 30 - 230 MHZ, 50 (3 M) 230 - 1000 MHZ, 57 (3 M)</p>	

Table 3

MANUFACTURER'S GUIDE AND DECLARATION			
<p>THE LIFTING ARMCHAIR IS INTENDED TO FUNCTION IN THE FOLLOWING ELECTROMAGNETIC CONDITIONS. THE CLIENT OR USER MUST ENSURE THAT IT IS USED IN THESE CONDITIONS.</p>			
IMMUNITY TESTS	TEST LEVEL IEC 60601	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDE
<p>CONDUCTED DISTURBANCES INDUCED BY RF FIELDS 61000-4-6</p>	<p>3 V EFF. FROM 150 KHZ TO 80 MHZ</p>	<p>3 V EFF. FROM 150 KHZ TO 80MHZ</p>	<p>PORTABLE AND MOBILE RF COMMUNICATIONS DEVICES SHOULD NOT BE USED NEAR ANY PART OF THE LIFTING ARMCHAIR, INCLUDING THE CABLES, UNLESS THE RECOMMENDED SEPARATION DISTANCES ARE OBSERVED, CALCULATED BY THE EQUATION APPLICABLE TO THE TRANSMITTER FREQUENCY.</p> <p>RECOMMENDED DISTANCES D = 1.2√P FROM 80 MHZ TO 800 MHZ D = 2.3√P FROM 800 MHZ TO 2.5 GHZ</p> <p>WHERE "P" INDICATES THE MAXIMUM RATED OUTPUT OF THE TRANSMITTER IN WATT (W) ACCORDING TO THE TRANSMITTER MANUFACTURER AND "D" IS THE RECOMMENDED SEPARATION DISTANCE IN METRES (M). THE INTENSITY OF THE FIELD GENERATED BY FIXED RF TRANSMITTERS, AS DETERMINED IN AN ON-SITE ELECTROMAGNETIC INSPECTION, MIGHT BE BELOW THE LEVEL OF COMPLIANCE IN EACH FREQUENCY BAND. INTERFERENCE MAY BE EXPERIENCED IN THE PROXIMITY OF EQUIPMENT MARKED WITH THIS SYMBOL:</p> 
<p>RF DISTURBANCES EN 6100-4-3</p>	<p>3 V/M FROM 80 MHZ TO 2.5 GHZ</p>	<p>3 V/M FROM 80 MHZ TO 2.5 GHZ</p>	
<p>NOTE 1: AT 80 MHZ AND 800 MHZ, APPLY THE SEPARATION DISTANCE FOR THE HIGHER FREQUENCY BAND. NOTE 2: THESE GUIDELINES MAY NOT APPLY IN ALL SITUATIONS. ELECTROMAGNETIC PROPAGATION IS AFFECTED BY ABSORPTION AND REFLECTION FROM STRUCTURES, OBJECTS AND PEOPLE.</p>			
<p>^ THE INTENSITY OF THE FIELD FOR FIXED TRANSMITTERS, SUCH AS BASE STATIONS FOR RADIO-TELEPHONES (CELLULARS AND CORDLESS) AND TERRESTRIAL MOBILE RADIOS, AMATEUR RADIO EQUIPMENT, AM AND FM RADIO TRANSMITTERS AND TV TRANSMITTERS MAY NOT BE KNOWN PRECISELY IN THEORY. TO ESTABLISH AN ELECTROMAGNETIC ENVIRONMENT CAUSED BY FIXED RF TRANSMITTERS, ONE MUST RUN AN ON-SITE ELECTROMAGNETIC INSPECTION. IF THE FIELD INTENSITY MEASURED IN THE PLACE WHERE THE LIFTING ARMCHAIR IS USED EXCEEDS THE APPLICABLE RF LEVEL OF WITHFORMITY INDICATED ABOVE, NORMAL FUNCTIONING OF THE LIFTING ARMCHAIR MUST BE MONITORED. IF ABNORMALITIES ARE OBSERVED, ADDITIONAL MEASURES MAY BE NECESSARY, SUCH AS RE-ORIENTATION OR REPOSITIONING OF THE LIFTING ARMCHAIR.</p>			
<p>® THE FIELD INTENSITY OVER THE FREQUENCY RANGE FROM 150 KHZ TO 80 MHZ SHOULD BE LESS THAN 3 V/M.</p>			

Table 4

**RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE  
RADIO COMMUNICATIONS EQUIPMENT AND THE LIFTING ARMCHAIR**

THE LIFTING ARMCHAIR IS INTENDED TO OPERATE IN AN ELECTROMAGNETIC ENVIRONMENT IN WHICH RADIATED RF DISTURBANCES ARE UNDER CONTROL. THE CUSTOMER OR USER OF THE LIFTING ARMCHAIR CAN CONTRIBUTE TO PREVENTING EM INTERFERENCE BY ENSURING A MINIMUM DISTANCE BETWEEN MOBILE AND PORTABLE RF COMMUNICATIONS DEVICES (TRANSMITTERS) AND THE LIFTING ARMCHAIR BATTERY CHARGER DURING CHARGING AND THE ENTIRE LIFTING ARMCHAIR DURING ITS OPERATION AS RECOMMENDED BELOW, IN RELATION TO THE MAXIMUM OUTPUT POWER OF THE RF COMMUNICATIONS DEVICES

TESTS	DURING USE OF THE LIFTING ARMCHAIR		
	FROM 150 KHZ TO 80 MHZ D = 1.2√P	FROM 80 KHZ TO 800 MHZ D = 1.2√P	FROM 800 KHZ TO 2.5 GHZ D = 1.2√P
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

FOR TRANSMITTERS WHOSE MAXIMUM RATED OUTPUT POWER IS NOT LISTED ABOVE, THE RECOMMENDED SEPARATION DISTANCE D, IN METRES (M), CAN BE CALCULATED USING THE EQUATION APPLICABLE TO THE TRANSMITTER FREQUENCY, WHERE P IS THE MAXIMUM RATED OUTPUT POWER OF THE TRANSMITTER, IN WATT (W), AS GIVEN BY THE TRANSMITTER'S MANUFACTURER.  
NOTE 1: AT 80 MHZ AND 800 MHZ, APPLY THE SEPARATION DISTANCE FOR THE HIGHER FREQUENCY BAND.  
NOTE 2: THESE GUIDELINES MAY NOT APPLY IN ALL SITUATIONS. ELECTROMAGNETIC PROPAGATION IS AFFECTED BY ABSORPTION AND REFLECTION FROM STRUCTURES, OBJECTS AND PEOPLE.

**16. TECHNICAL SPECIFICATIONS**
**16.1 Dimensions and weights\***
**Dimensions and weight**

Codes													
PO400x	52	48	50	77	87	61	107	160	150	-	130	52	
PO400G-XL	64	48	50	88	87	61	107	160	150	-	160	55	
PO401x	52	48	50	77	85	61	107	160	150	-	130	52	
PO403x	50	45	48	70	90	61	110	162	140	7	130	45	
PO405x	52	48	48	77	85	61	115	160	150	-	130	55	
PO410x	52	48	49	66	90	61	105	160	150	-	130	55	
PO420x	52	48	41	71	90	61	112	162	150	-	130	58	
PO425x	52	48	41	71	90	61	112	175	150	10	130	58	
PO430x	52	48	44	72	90	63	115	180	150	12	130	58	

\*Since this is an upholstered product, measurements may vary by a few cm

## 16.2 Technical specifications

### PO400x - PO400G-XL - PO401x (NINFEA FRANCIA CORTA fabric)

- Powder coated steel and wooden frame;
- Stain-proof fabric;
- Armchair and arm rest cover
- Non-deformable polyurethane foam padding at different densities (backrest 18-22 Kg/m<sup>3</sup>, footrest 18 Kg/m<sup>3</sup>, seat 35 Kg/m<sup>3</sup>, armrests 40 Kg/m<sup>3</sup>);
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Total relax function; Tv function; Stand-up function
- 2 independent motors;
- Four-button remote control with cable inside the pocket;
- 2 pins for movement;
- Roller system (standard feature only on PO401x) four wheels, Ø 50 mm;

### PO403x (CHLOE INCANTO fabric)

- Powder coated steel and wooden frame;
- Water-repellent and stain-proof fabric;
- Nondeformable polyurethane foam padding with differentiated density. (back 18-22 Kg/m<sup>3</sup>, footboard 18 Kg/m<sup>3</sup>, seat 35 Kg/m<sup>3</sup>, armrests 40 Kg/m<sup>3</sup>);
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Stand-up function;
- 2 independent motors;
- Four-button remote control with cable inside the pocket;
- Courtesy wheels in the rear, PVC feet in the front

### PO405x (DAFNE)

- Powder coated steel and wooden frame;
- Water-repellent and stain-proof fabric;
- Armchair and arm rest cover
- Nondeformable polyurethane foam padding with differentiated density. (back 18-22 Kg/m<sup>3</sup>, footboard 18 Kg/m<sup>3</sup>, seat 35 Kg/m<sup>3</sup>, armrests 40 Kg/m<sup>3</sup>);
- 6-7 cm In molded polyurethane covered with wadding flap pillow;
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Total Relax function; TV function; Stand up function
- 2 independent motors;
- Four-button remote control with cable inside the pocket;
- 2 frontal tips;
- Roller system 4 wheels, Ø 50mm;

### PO410x (AGAVE)

- Powder coated steel and wooden frame;
- Water-repellent and stain-proof fabric;
- Non-deformable polyurethane foam padding at different densities (backrest 18-22 Kg/m<sup>3</sup>, footrest 18 Kg/m<sup>3</sup>, seat 35 Kg/m<sup>3</sup>, armrests 35 Kg/m<sup>3</sup>);
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Total Relax function; TV function; Stand up function
- 2 independent motors;
- Four-button remote control with cable inside the pocket;
- 2 small frontal tips
- Roller system 4 wheels. Ø 50 mm

**PO420x (MYRTHO)**

- Powder coated steel and wooden frame;
- Water-repellent and stain-proof fabric;
- Nondeformable polyurethane foam padding with differentiated density. (back 18-22 Kg/m<sup>3</sup>, footboard 18 Kg/m<sup>3</sup>, seat 35 Kg/m<sup>3</sup>, armrests 40 Kg/m<sup>3</sup>); (head rest area, softer - trunk and lumbar area, stiffer);
- Polyurethane foam seat covered with 2 cm of viscoelastic;
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Extractable armrests;
- Extractable ear flaps;
- Four wheels, Ø 75 mm, of which two with brake;
- Push bar;
- Total relax function; TV function; Stand up function;
- 2 independent motors;
- Four-button remote control with cable inside the pocket;

**PO425x (MYRTHO CLASS)**

- Powder coated steel and wooden frame
- Water-repellent and stain-proof fabric
- Nondeformable polyurethane foam padding with differentiated density. (back 18-22 Kg/m<sup>3</sup>, footboard 18 Kg/m<sup>3</sup>, seat 35 Kg/m<sup>3</sup>, armrests 40 Kg/m<sup>3</sup>);
- Seat polyurethane foam covered with 2 cm of viscoelastic;
- Footrest adjustable to several positions;
- Backrest adjustable to several positions;
- Extractable armrests with possibility of inserting table;
- Extractable ear flaps;
- Backrest with removable and replaceable cushions
- Four wheels, Ø 75 mm, of which two with brake;
- Push bar;
- Total relax function; TV function; Stand up function; Vertical lift 10 cm; Bed function (180°)
- Four-button remote control with cable inside the pocket;

**PO430X (CLYZIA)**

- Powder coated steel and wooden frame
- Water-repellent and stain-proof fabric
- Nondeformable polyurethane foam padding with differentiated density. (back 18-22 Kg/m<sup>3</sup>, footboard 18 Kg/m<sup>3</sup>, seat 35 Kg/m<sup>3</sup>, armrests 40 Kg/m<sup>3</sup>);
- Double footboard that allows an extension of 11 cm
- Backrest adjustable to several positions;
- Total relax function; TV function; Stand up function; Vertical lift 12 cm; Bed function (180°); Zero gravity function;
- Removable armrests with possibility of inserting small table;
- Roller system four wheel, Ø 50 mm;
- 2 independent motors with Zero Gravity function;
- Four-button remote control with cable inside the pocket;

**16.3 general technic specification**

- 230V - 110W;
- Operating temperature -20°C / + 60°C;
- Recommended care conditions: keep in a cool and dry place, away from direct sources of heat, such as sunlight (recommended temperature between -10°C and + 40°C);

- In the case of a lengthy period of non-use, disconnect the power supply and protect the parts that could be damaged by dust deposits.

**16.4 Fabric specifications**

All the armchair fabrics are:

ANTI-MITE

STAIN-PROOF

ANTI-MICROBIAL  
AND ANTI-BACTERIAL

BREATHABLE

**17. ACCESSORIES**

- Roller system (ONLY FOR PO400x)
- Armchair covers kit (PO403x, PO420x, PO425x)
- Small table (ONLY FOR PO420x, PO430x)
- Portable battery kit

**Note:** The list of accessories is revised continually. We recommend in contacting retailers to find out about any new features and availability and to obtain more information.

**17.1 Instructions for the Roller System**

**WARNING!**

Installation operations must be performed by qualified personnel.

**WARNING!**

All responsibility is declined for malfunctions or damage caused by incorrect installation and placing in operation by unqualified personnel.

Follow the steps below with care to install the roller system accessory (POA3xx):

- Place the armchair in operation and put it in the lift position;
- Disconnect the plug;
- Disassemble, where present, the two front wheels using a standard 10 mm wrench;
- Tip the armchair forwards;

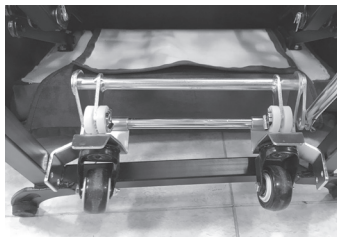
Fig.2

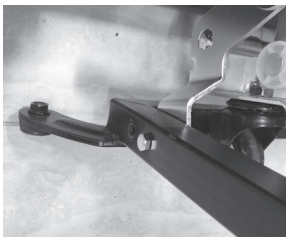
Fig.3

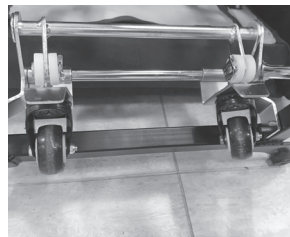
Fig.4

- Insert the roller system (supplied already partially assembled) into the steel base of the armchair (fig. 2);
- Line up the four holes on the front of the armchair frame base with those present on the roller system supports and lock them in place with the four screws and nuts provided (Figs. 3 and 4);

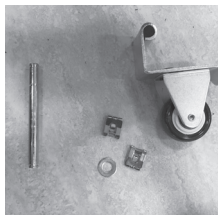
Fig.5

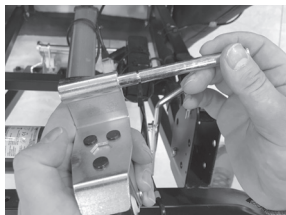
Fig.6

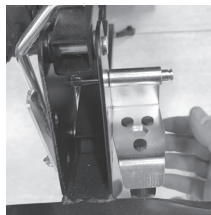
Fig.7

Fig.8

- Take the components shown in Fig.5;
- Insert the long pin on the wheel support (Fig. 6);
- Insert the pin and wheel in the rear right-hand plate of the frame (Figs. 7 and 8);
- Insert the stop on the pin on the outer side of the frame;
- Insert the washer on the pin from the inner side of the frame;
- Assemble the roller system rod on the pin previously installed (Fig. 9) and insert the stop on the inner side as well;

Fig.9

Fig.10

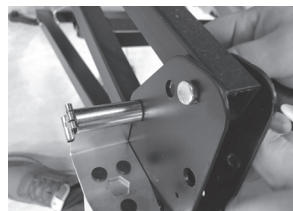
Fig.11

- Take the components shown in Fig. 10;
- Insert the short pin on the wheel support;
- Insert the pin and wheel in the rear left-hand plate of the frame (Fig. 11);
- Insert the washer on the pin from the inner side of the frame;
- Insert the stops on both ends of the pin;

Fig.12

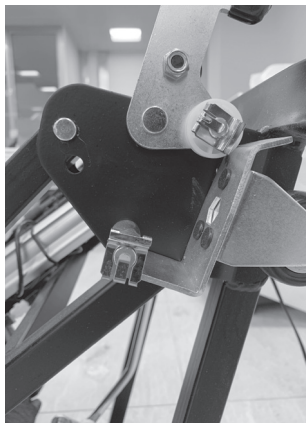
Fig.13

- Take the steel bar with the two remaining pins and place it close to the frame plates, turning it so that the black rubber tips face downwards;
- Insert the right-hand curved rod of the roller system inside the hole on the bar (Fig.12);
- Insert the pins to block the bar on the armchair frame plates;
- Insert the stops on both pins (Fig.13);
- Tip the armchair back into its normal position;
- Connect the armchair to the mains current and return it to the upright position.

## 18. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject to the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual. Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously withforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

Any improper use of the armchair not envisaged in this manual will invalidate the manufacturer's responsibility and warranty in the case of accidents or damage.

## 19. REPAIRS

### 19.1 Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

### 19.2 Repairing a product not covered by the warranty

Product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

### 19.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

## 20. SPARE PARTS

Moretti original spare parts are guaranteed for 6 (six) months from the day of delivery. Consult the Moretti catalogue only for the list of spare parts.

## 21. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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