MOPEDIA®

MA RC300-xx RC310-xx RC320-43 02 B_ITA_05 2021

SEDIE COMODE

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ITALIANO

INDICE

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CE Class I medical device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 on medical devices

1. CODES

| | |
|----------|---|
| RC300-43 | 4-tip height-adjustable commode chair - 43 cm |
| RC300-45 | 4-tip height-adjustable commode chair - 45 cm |
| RC300-48 | 4-tip height-adjustable commode chair - 48 cm |
| RC310-43 | 4-wheel commode chair - 43 cm |
| RC310-45 | 4-wheel commode chair - 45 cm |
| RC310-48 | 4-wheel commode chair - 48 cm |
| RC320-43 | 4-wheel folding commode chair - 43 cm |

2. INTRODUCTION

Thank you for choosing a MOPEDIA commode chair by MORETTI S.p.A., designed to assist people who have difficulties in walking, evacuation and mobility. The product satisfies the requirements of current European regulations on medical devices (2017/745). This user manual contains some suggestions on how to use the device properly and safely. Please read this entire manual carefully before you use the device. Should you have any queries, please contact your retailer for suitable advice and assistance.

NB Check to ensure that no parts of the device have been damaged during shipment. Do not use the product if it is damaged and contact the retailer for further instructions.

3. INTENDED USE

Commode seats are devices to assist in evacuation and mobility for people with walking difficulties.

WARNING!

- Do not use the product for any purpose other than that specified in this manual
- MORETTI S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual
- The manufacturer reserves the right to modify the device and this manual without prior notice, in order to improve its features

4. EU DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products made and sold by MORETTI SpA in the COMMODE CHAIRS - MOPEDIA product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES. To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATIONS.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

5. WARNINGS

5.1 General warnings

- DO NOT install or use the product before reading and understanding this instruction manual. If you are unable to understand the warnings, precautions and instructions, contact a healthcare operator, retailer or technical operator before attempting to install the product, in order to avoid damage or personal injury;
- Always consult your doctor or therapist for correct use of the device;
- Keep the packaged product away from sources of heat, as the packaging is cardboard;
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts;
- Always take care when children are present;
- Never lift the chair gripping the removable parts, which could accidentally detach from it during lifting.
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

WARNING!

Take extreme care in the presence of moving parts that could trap limbs and cause injury

5.2 Warnings on use

- Ensure that the tray has not been shifted or removed;
- Do not rest your weight on the footplates to get onto or off the chair;
- Do not lean out from the commode chair to pick up objects on furniture or other surfaces;
- Avoid sitting on the edge of the commode chair, as it could tip over.

6. SYMBOLS USED

Product code

Unique device identification

CE marking

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Disposal conditions

7. GENERAL DESCRIPTION

image for illustration purposes

RC300-xx

1. Backrest
2. Tip-up armrests
3. "U"-shaped seat
4. Pan
5. Height-adjustable legs

image for illustration purposes

RC310-xx

1. Backrest
2. Tip-up armrests
3. "U"-shaped seat
4. Pan
5. Castors
(rear ones with parking brake)
6. Footplates

image for illustration purposes

RC320-xx

1. Backrest
2. Tip-up armrests
3. "U"-shaped seat
4. Pan
5. Castors
(rear ones with parking brake)
6. Reposapiés extraíbles

8. ASSEMBLY

The MOPEDIA commode chair comes with a backrest and disassembled footplates (only for models with footplates). Open the frame by grasping the external tubes and lock it in place using the holed seat with the relative clips. For assembly of the backrest, it is sufficient to slide the two ends of the tubes into the specific housings on the back of the seat. At this point, attach the backrest using the clips provided. Only for models with footplates - insert the footplates on the outside of the commode chair, by inserting the upper part of the footplate onto the two pins. The footrests must always be on the inside of the commode chair. To secure the footplate, ensure that the locking lever is firmly secured on the inside of the commode seat.

8.1 Adjustment of the footplates (only for models with footplates)

It is possible to adjust the height of the footplates on the commode chair, in order to offer the greatest comfort to the patient. Use the buttons on the side of the footplate itself to adjust it. Press the button and slide the tube to the ideal height. Always check that the two footplates are at the same height at the end.

8.2 Disassembly

Follow the assembly procedure in reverse.

9. BEFORE EVERY USE

- Check correct assembly of the device.
- Always check for wear and tear of moving mechanical parts, in order to ensure that the device can be used in total safety for both people and property.
- Check correct functioning of the brakes on the rear wheels (only for models with wheels). Insert the brakes on both wheels, push the chair gently backwards and forwards: the wheels should not move.

10. WARNINGS ON USE

- The commode chair is only for internal use and it is not designed for use as a seat in a motor vehicle;
- Always and only use the rear push bar to push the chair and never push it using other parts;
- Take particular care if the commode seat is being used on a sloping surface;
- Avoid sudden movements if it is necessary to stop on a sloping surface;
- If two people are lifting the commode chair, avoid gripping it at points that could be dangerous to the patient, such as the footplates and armrests, and use only the frame and the rear grips;
- It is recommended to store the product away from sources of heat, as certain parts, such as the covering of the backrest and the seat, the push handles and the armrests, if exposed to sunlight, could reach temperatures that might damage the skin.

10.1 Brakes

Lock the chair wheels before getting in, getting out, leaning forward. The brakes are applied by pressing the lever on each wheel of the chair (Fig. 1).

(FIG. 1)

image for illustration purposes

10.2 Wheels

Check the wheels periodically and lubricate, if necessary.

11. HOW TO USE

Note: Consult the general warnings and user instructions in this manual

WARNING!

- Getting into and out of a commode chair on your own can be potentially risky: do so only if you are physically able and with care.
- Do not lean on removable parts, but rather on the seat frame.
DO NOT SHIFT YOUR WEIGHT ONTO THE FOOTRESTS WHEN GETTING INTO OR OUT OF THE COMMODOE CHAIR, AS IT COULD TIP OVER!!!

11.1 Getting into the commode chair

1. Get as close as possible to the chair, in a manner that it is positioned alongside you
2. Lock the brakes on the front wheels (only for the model with wheels); to make getting into the commode chair easier, raise the armrest on the side where these operations are being performed.
3. Rest your weight on the frame structure using your arms and slowly lower yourself into the seat

11.2 Getting out of the commode chair

1. Get as close as possible to the side of another seat
2. Lock the brakes on the front wheels (only for the model with wheels); to make getting out of the commode chair easier, remove the armrest on the side where these operations are being performed.
3. Rest your arms on the frame of the chair into which you are moving, raise yourself and slide slowly towards it

12. MAINTENANCE

MOPEDIA devices manufactured by Moretti S.p.A. are CE marked and carefully checked before being commercialised. For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original replacement parts and accessories for repairs. MOPEDIA commode chairs only need a few simple and periodic maintenance operations. The periodic checks to carry out are:

| Element | Frequency |
|--------------------------------------|-----------------|
| Braking capacity | Monthly |
| Cleaning the upholstery | As needed |
| General check by qualified personnel | Every two years |

WARNINGS!

Before using it, carry out all the checks and controls suggested in the paragraph "BEFORE EVERY USE" of this instruction manual.

13. CLEANING AND DISINFECTION

13.1 Cleaning the covers and padding

The covers and padding are made of PVC. Clean them with water and mild soap.

13.2 Cleaning the WC

Clean and disinfect the WC on a regular basis, using a normal bathroom disinfectant. Always clean at least before each use.

13.3 Cleaning the frame and the metal parts

Use water and neutral soap for the frame and metal parts as well, then dry with a soft cloth, to avoid scratching the paint of the frame.

13.4 Cleaning the wheels (only for models with wheels)

Use water and neutral soap to clean the wheels. Take extreme care in drying them, as using the commode chair with damp wheels could be extremely dangerous.

13.5 Disinfection

Use a common disinfectant cleaner if you need to disinfect the product.

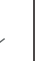
NB Never use acid, alkali or solvent products, such as acetone or thinner.

14. GENERAL CONDITIONS OF DISPOSAL

Never dispose of the device as normal domestic waste. Dispose of the device at a sorted waste collection centre for recycling.

15. TECHNICAL SPECIFICATIONS

15.1 Dimensions and weight

| Dimensions and weight | | | | | | | | | | | | | | |
|-----------------------|---|---|---|---|---|---|---|---|---|---|---|---|---|--|
| Measurements in cm |  |  |  |  |  |  |  |  |  |  |  |  |  |  |
| RC300-43 | - | 53 | 43 | 96÷103.5 | 75.5÷83 | 51÷59 | - | 43 | 45 | 43 | 11.5 | - | - | |
| RC300-45 | - | 53 | 43 | 96÷103.5 | 75.5÷83 | 51÷59 | - | 45 | 45 | 43 | 11.5 | - | - | |
| RC300-48 | - | 53 | 43 | 96÷103.5 | 75.5÷83 | 51÷59 | - | 48 | 45 | 43 | 11.5 | - | - | |
| RC310-43 | 81 | 62.5 | 43 | 97.5 | 77 | 52.5 | 30÷38 | 43 | 45 | 46 | 12.5 | 12.5 | 12.5 | |
| RC310-45 | 81 | 62.5 | 43 | 97.5 | 77 | 52.5 | 30÷38 | 45 | 45 | 46 | 12.5 | 12.5 | 12.5 | |
| RC310-48 | 81 | 62.5 | 43 | 97.5 | 77 | 52.5 | 30÷38 | 48 | 45 | 46 | 12.5 | 12.5 | 12.5 | |
| RC320-43 | 81 | 62.5 | 43 | 95.5 | 75 | 50.5 | 30÷38 | 43 | 45 | 46 | 12.5 | 12.5 | 12.5 | |

Max. capacity RC300-xx and RC320-xx: **120 Kg**

Max. capacity RC310-xx: **135 Kg**

15.2 Technical specifications

| | |
|---|---|
| FRAME | IN PAINTED STEEL |
| COVER | IN PVC |
| ARMREST PADDING | IN PU |
|  MAXIMUM CAPACITY RC300-XX and RC320-XX | 120 KG |
|  MAXIMUM CAPACITY RC310-XX | 135 KG |
| ENVIRONMENTAL TEMPERATURE | - 20 C° / + 40 C° |
| RECOMMENDED STORAGE | STORE IN A COOL AND DRY PLACE, AWAY FROM DIRECT HEAT SOURCES, SUCH AS SUNLIGHT. (RECOMMENDED TEMP. BETWEEN +10 C° / + 40 C°) |
| WHEELS | Ø 12 CM |

16. REPLACEMENT PARTS AND ACCESSORIES

Use only replacement parts and accessories listed in the Moretti general catalogue.

17. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual.

Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual.

Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual.

Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

18. REPAIRS
18.1 Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

18.2 Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

18.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

19. REPLACEMENT PARTS

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

20. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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