MOPEDIA®

MA RP18x 03 A_ITA_07 2020

COLLARI CERVICALI

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ITALIANO

INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices**1. CODES**

RP180S	Rigid cervical collar
RP180M	Rigid cervical collar
RP180L	Rigid cervical collar
RP180XL	Rigid cervical collar
RP181S	Rigid cervical collar with chin
RP181M	Rigid cervical collar with chin
RP181L	Rigid cervical collar with chin
RP182S-06	Soft cervical collar H6
RP182M-06	Soft cervical collar H6
RP182L-06	Soft cervical collar H6
RP182S-08	Soft cervical collar H8
RP182M-08	Soft cervical collar H8
RP182L-08	Soft cervical collar H8
RP182S-10	Soft cervical collar H10
RP182M-10	Soft cervical collar H10
RP182L-10	Soft cervical collar H10
RP183S	Semi-rigid cervical collar with internal support
RP183M	Semi-rigid cervical collar with internal support
RP183L	Semi-rigid cervical collar with internal support

2. INTRODUCTION

Thank you for purchasing a Mopedia CERVICAL COLLAR by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The cervical collars MOPEDIA by Moretti are intended for the treatment of stress, tension or stretching of the cervical muscles, and in any case for minor neck injury or for the relief of muscle.

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of CERVICAL COLLARS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- READ carefully the entire manual before using the device .
- Make sure that the product is delivered complete and keep it packed away from heat sources
- SERVICE LIFE- the device use limit is defined by the wear of the parts
- DO NOT allow children to play or operate the device
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

- For the correct choice of the type of collar and for its proper use, always consult your doctor or therapist.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

Wash by hand; Maximum temperature 40°C

Rinse very gently, do not wring

Do not bleach

Do not iron

Flat drying in the shade

Do not tumble dry


7. GENERAL DESCRIPTION

DESCRIPTION	SMALL	MEDIA	LARGE	X-LARGE
Rigid cervical collar	RP180S	RP180M	RP180L	RP180XL
Rigid cervical collar with chin	RP181S	RP181M	RP181L	
Soft cervical collar H6	RP182S-06	RP182M-06	RP182L-06	
Soft cervical collar H8	RP182S-08	RP182M-08	RP182L-08	
Soft cervical collar H10	RP182S-10	RP182M-10	RP182L-10	
Semi-rigid cervical collar with internal support	RP183S	RP183M	RP183L	

7.1 Rigid collar with and without chin

Rigid cervical collar Mopedia are indicated in the treatment of soft tissue damages, osteoarthritis and whiplash injuries. Made of polyethylene with vinyl-covered padding at the edges for maximum patient comfort and for easy cleaning. Fully height adjustable with Velcro system to ensure flexion and / or extension of placement. With holes for added breathability.


Without chin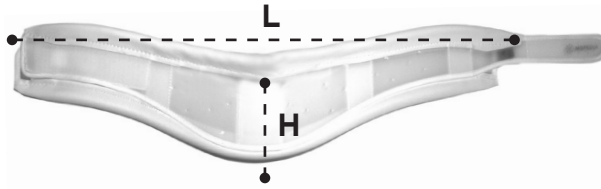

RP180S _ H: 9,5cm
L: 35+40cm
RP180M _ H: 10,5cm
L: 40+46cm
RP180L _ H: 12cm
L: 46+54cm
RP180XL _ H: 13cm
L: 54+60cm

With chin


The model with CHIN thanks to its contoured shape provides a correct stabilization of the cervical spine.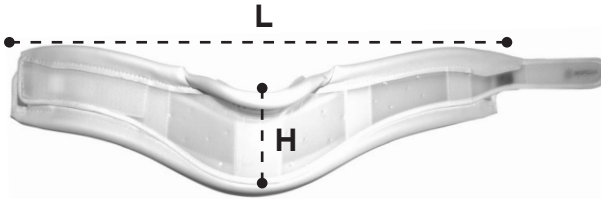

RP181S _ H: 11,5cm
L: 35+40cm
RP181M _ H: 12,5cm
L: 40+46cm
RP181L _ H: 13cm
L: 46+54cm

7.2 Soft collar

Soft cervical collars Mopedia are recommended for the relief of muscle tension and / or treatment of minor injuries of the neck including whiplash. Made of 100% FOAM, ideal to wear during the night, and lined with 100% polyamide mesh hypoallergenic, fully washable and adjustable in length thanks to the Velcro closure system to better meet the needs of the user.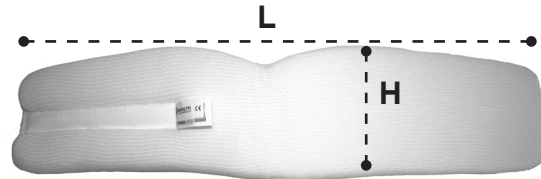

RP182S-06 _ H: 6cm L: 40cm
RP182S-08 _ H: 8cm L: 40cm
RP182S-10 _ H: 10cm L: 40cm
RP182M-06 _ H: 6cm L: 44cm
RP182M-08 _ H: 8cm L: 44cm
RP182M-10 _ H: 10cm L: 44cm
RP182L-06 _ H: 6cm L: 48cm
RP182L-08 _ H: 8cm L: 48cm
RP182L-10 _ H: 10cm L: 48cm


7.3 Semi-rigid collar internal and external support

Semi-rigid cervicals collar Mopedia allow you to get the right balance between stability and comfort. Indicated in the treatment of soft tissue injuries, for the treatment of minor injuries of the neck, in the case of osteoarthritis and whiplash, and to support the cervical spine in a neutral position.

Adjustable in length thanks to the Velcro closure system to better meet the needs of the user.

With internal support

Made of 100% FOAM with plastic inner rigid support and linen hypoallergenic 100% polyamide.

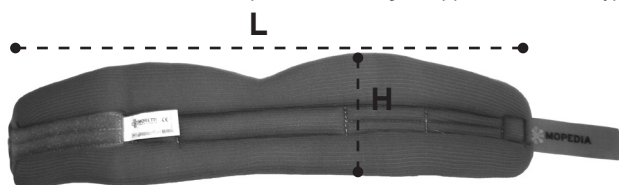RP183S _ H: 7cm L: 40cm
 RP183M _ H: 8cm L: 44cm
 RP183L _ H: 9cm L: 48cm

ENGLISH

8. BEFORE USE

- Check if the device is damaged in order to guarantee a safe use for people and things

9. USING WARNINGS

- For a correct use of the device, refer to this manual
- In the event of uncertainty regarding the adjustment or maintenance of the product, contact your dealer
- To avoid any risks to keep the device away from children.
- Do not use the device in the presence of deterioration or damage that may impair the effectiveness

10. HOW TO USE

Lay the collar between the chin and the breast in such a way as to obtain support for the head with the Velcro closure in the back. Adjust via the Velcro closing the collar itself, ensuring the immobility of the neck and the maximum patient comfort.

11. MAINTENANCE

The MopediA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

12. CLEANING

Wash by hand; Maximum temperature 40°C

Rinse very gently, do not wring

Do not bleach

Do not iron

Flat drying in the shade

Do not tumble dry

Note: Never use acids, alkalis or solvents such as acetone

13. CONDITION OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

14. TECHNICAL FEATURES

Model	Rigid	Rigid with chin	Soft	Semi-rigid with internal support	Semi-rigid with external support
Color	White	White	White	Beige	Beige
Material	Polyester and vinyl	Polyester and vinyl	Foam	Foam + vinyl internal support	Foam + vinyl external support
Closure	Adjustable with velcro	Adjustable with velcro	Adjustable with velcro	Adjustable with velcro	Adjustable with velcro
Ventilation	With holes	With holes	Hypoallergenic cotton stockinette	Hypoallergenic cotton stockinette	Hypoallergenic cotton stockinette
No Neoprene	X	X	X	X	X
No latex	X	X	X	X	X

15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use.

Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair , damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

13.1 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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