MOPEDIA®

MA RP200-210-215-220-225-240 02 B_ITA_03 2021

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ITALIANO

INDICE

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Via _____ Località _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

MOPEDIA®

MA RP200-210-215-220-225-240 02 B_ENG_03 2021

RESTRAINT SYSTEMS

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL PRECAUTIONS.....	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.5
7.1) RP200 - Chest strap.....	pag. 5
7.2)RP210 - Girdle chest belt	
RP215 - Butterfly-shaped girdle chest belt	pag. 5
7.3)RP220 - Full pelvic belt	
RP225 - Separate pelvic belt.....	pag. 5
7.4) RP240 - Wrist strap	pag. 6
8. BEFORE EVERY USE	PAG.6
9. WARNINGS ON USE	PAG.6
10. CLEANING AND DISINFECTION	PAG.6
10.1) Cleaning	pag. 6
10.2) Disinfection	pag. 7
11. GENERAL CONDITIONS OF DISPOSAL	PAG.7
12. TECHNICAL SPECIFICATIONS.....	PAG.7
13. WARRANTY	PAG.8
13.1) Repairs	pag. 8
13.2) Replacement parts	pag. 9
13.3) Non-Liability clause	pag. 9

 Class I medical deviceREGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 on medical devices

1. CODES

RP200	Chest strap
RP210	Girdle chest belt
RP215	Butterfly-shaped girdle chest belt
RP220	Full pelvic belt
RP225	Separate pelvic belt
RP240	Wrist strap

2. INTRODUCTION

Thank you for choosing an aid in the MOPEDIA range by Moretti. The KOALA restraint systems by Moretti have been designed and constructed to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this entire manual carefully before you use the device. Should you have any queries, please contact your dealer for suitable advice and assistance.

NOTE: Check to ensure that no parts of the product have been damaged during shipment. Do not use the product if it is damaged and contact the retailer for further instructions.

Do not use the product for any purpose other than that specified in this manual. Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual. The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features.

3. INTENDED USE

The KOALA restraint systems by Moretti are used to avoid slipping forwards and as a restraint for patients seated in a wheelchair or on a commode chair. These aids are particularly comfortable, thanks to the padded fabric used in their manufacture. They have a length-adjustable tape and clip closures that allow quick and correct positioning for any build and on all wheelchair, commode chair or high chair models, and also quick release in the case of an emergency.

4. DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products manufactured and sold by MORETTI SpA in the RESTRAINT SYSTEMS product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES.

To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

5. GENERAL PRECAUTIONS

- Please consult this manual for correct use of the device.
- Always consult your doctor or therapist for correct use of the device
- Check that the product delivered to you is undamaged and keep it packaged away from sources of heat.
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts
- Always take care when children are present
- The manufacturer and the retailer decline any responsibility for damage caused to people, animals or things as a result of work carried out by unauthorised personnel and failure to comply with this instruction manual.
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

6. SYMBOLS

	Product code
	Unique device identification
	CE marking
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Disposal conditions
	Wash by hand; Maximum temperature 30°C
	Do not bleach
	Do not iron
	Do not tumble dry

7. GENERAL DESCRIPTION

7.1) RP200 - Chest strap

Used to avoid slipping forwards and as a restraint for patients seated in a wheelchair or in a commode chair. The strap is particularly comfortable, thanks to the padded fabric used in its manufacture. It has a length-adjustable tape and clip closures that allow quick and correct positioning for any build and on all wheelchair, commode chair or high chair models, and also quick release in the case of an emergency.

How to use

1. Seat the patient in the commode chair or in the wheelchair
2. Wrap the strap around the chest
3. Pass the two belts behind the back
4. Close the buckle
5. Adjust the length of the belt according to need.

7.2)RP210 - Girdle chest belt

RP215 - Butterfly-shaped girdle chest belt

Used to avoid slipping forwards and as a chest restraint for patients seated in a wheelchair or in a commode chair. The belt is particularly comfortable, thanks to the padded fabric used in its manufacture. It has length-adjustable tapes, with clip couplings on an X-shaped support, clip closures that allow quick and correct positioning for any build and on all wheelchair, commode chair or high chair models, and also quick release in the case of an emergency.

How to use

1. Seat the patient in the commode chair or in the wheelchair
2. Position the girdle on the patient's chest
3. Position the two braces at the top in a manner that they pass across the back of the backrest
4. Also pass the two tapes at the bottom at the sides of the harness across the back of the backrest
5. Attach the four couplings to the X-shaped connector and lock them in place with the snap buckles
6. Adjust the length of the braces and belt to ensure a comfortable fit for the patient

7.3)RP220 - Full pelvic belt

RP225 - Separate pelvic belt

Used to avoid slipping forwards and as a restraint for patients seated in a wheelchair or in a commode chair who are unable to maintain a correct seated position. Keeps the thighs divided. The belt is shaped to guarantee comfortable dividing of the legs and restraint. Comfort is guaranteed by the padded fabric used in its manufacture. It has length-adjustable tapes and clip closures that allow quick and correct positioning for any build and on all wheelchair, commode chair or high chair models, and also quick release in the case of an emergency.

How to use RP220

1. Position the pelvic belt on the seat of the commode chair or wheelchair, positioning it with the **Koala** trademark facing downwards
2. Pass the two highest tapes on the back of the backrest, adjust the length to obtain a stable fit and secure the tapes with the snap buckle
3. Seat the patient with the pelvic belt already positioned on the seat
4. Pass one of the two straps from the inner thigh outwards to connect it with the smaller tape, adjust the length of the tape to ensure a comfortable fit for the patient and secure the strap with the snap buckle
5. Repeat the operation with the other leg

How to use RP225

1. Position the two pelvic belts on the seat of the commode chair or wheelchair, positioning them with the velvet type fabric facing downwards
2. Pass the two rear tapes across the back of the backrest
3. Seat the patient with the pelvic belts already positioned on the seat
4. Pass one of the two straps from the inner thigh outwards and pass the tape across the back of the commode chair or wheelchair
5. Repeat the operation with the other strap
6. Secure the straps with the snap buckles, crossing them
7. Adjust the length of the tapes to ensure a comfortable fit for the patient

7.4) RP240 - Wrist strap

Used for wrist restraint of patients seated in a wheelchair or on a commode chair. The strap is particularly comfortable, thanks to the padded fabric used in its manufacture. Reinforced around the entire perimeter with a strong safety tape with snap closure. The Velcro tape with adjustable closure helps attach it to the wheelchair.

How to use

1. Seat the patient in the commode chair or in the wheelchair
2. Position the wrist strap on the armrest of the commode chair or wheelchair
3. Secure the wrist strap on the armrest using the Velcro tape on the lower part of the strap
4. Place the patient's wrist on the strap
5. Secure the wrist by adjusting the Velcro closure to ensure a comfortable fit
6. Pass the safety tape over the strap, adjust the length and secure with the snap buckle

8. BEFORE EVERY USE

Always check the state of wear and tear of the parts, particularly the closure of the CLIPS, in order to ensure the device is used in total safety.

9. WARNINGS ON USE

RP200 - Do not use the chest strap on normal chairs, as their low weight would not prevent the person from getting up and dragging the chair with them

WARNING!

The Koala restraints in the MOPEDIA range are not substitutes for car safety belts in any circumstances. However, they can be used to position the patient on the car seat, but must always be used in combination with the car safety belts

10. CLEANING AND DISINFECTION

10.1) Cleaning

Wash by hand; Maximum temperature 30°C

Do not bleach

Do not iron

Do not tumble dry

10.2) Disinfection

Avoid the use of solvents that could damage the belts


Note: Never use acid, alkali or solvent products, such as acetone or thinner.

11. GENERAL CONDITIONS OF DISPOSAL 

Never dispose of the product as normal domestic waste. Dispose of the device at local sorted waste collection centres for recycling of the materials used.

12. TECHNICAL SPECIFICATIONS

- RP200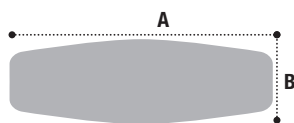


One size
A = 40 cm
B = 13 cm

Materials

- External fabric
- 100% polyester
- Tape reinforcements:
- 100% polyester
- Padding:
- 100% polyurethane

- RP210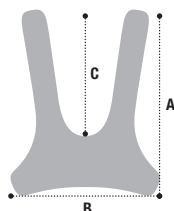


One size
Dimensions
A = 46cm
B = 30cm
C = 38 cm

Materials

- External fabric
- 100% polyester
- Tape reinforcements:
- 100% polyester
- Padding:
- 100% polyurethane

- RP215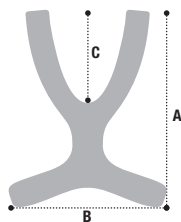

One size
Dimensions
A = 42cm
B = 30cm
C = 20cm

Materials

- External fabric
- 100% polyester
- Tape reinforcements:
- 100% polyester
- Padding:
- 100% polyurethane

- RP220


One size
Dimensions
A = 52cm
B = 30cm

Materials

- External fabric
- 100% polyester
- Tape reinforcements:
- 100% polyester
- Padding:
- 100% polyurethane

- RP225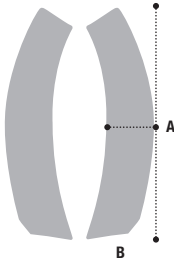

One size
Dimensions
A = 50cm
B = 11cm

Materials

External fabric
- 100% polyester
Tape reinforcements:
- 100% polyester
Padding:
- 100% polyurethane

- RP240

One size
A = 35cm
B = 8cm

Materials

External fabric
- 100% polyester
Tape reinforcements:
- 100% polyester
Padding:
- 100% polyurethane

13. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject to the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual. Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

13.1) Repairs

Repairs under warranty If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

Repairing a product not covered by the warranty Product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

Non-defective products The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

13.2) Replacement parts

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

13.3) Non-Liability clause

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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