MOPEDIA®

MA RP440 03 A_ITA_08 2020

RIALZO DUO PER BIDET E WATER

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ITALIANO

INDICE

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MOPEDIA®

MA RP440 03 A_ENG_08 2020

DUO RAISED TOILET SEAT FOR BIDET AND WC

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.4
8. BEFORE USE	PAG.5
9. HOW TO USE	PAG.5
10. CLEANING AND DISINFECTION	PAG.5
10.1 Cleaning	pag. 5
10.2 Disinfection	pag. 6
11. CONDITION OF DISPOSAL	PAG.6
12. TECHNICAL FEATURES.....	PAG.6
13. WARRANTY	PAG.6
14. REPAIR.....	PAG.6
14.1 Warranty repair	pag. 6
14.2 Repair not covered by warranty.....	pag. 6
14.3 Non-defective device.....	pag. 6
15. SPARE SPARTS.....	PAG.7
16. EXEMPT CLAUSES.....	PAG.7

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices**1. CODES**

RP440

Bidet and wc Duo raised toilet seat - h 10 cm

2. INTRODUCTION

Thank you for purchasing universal raised bidet and wc seat MOPEDIA by Moretti manufactured to support people with movement difficulty. Their design and their quality guarantee maximum comfort, liability and safety. MOPEDIA by Moretti universal raised bidet and wc seat are designed and realised to satisfy all your needs. This instruction manual contains some useful tips for a correct use of the device you have bought as well as some important advises for your safety. In case of doubts, please contact your retailer who will be able to help you correctly.

3. INTENDED USE

The MOPEDIA universal raised bidet and wc seat is a device to help people with movement difficulties using the bidet and wc seat.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of TOILET SEATS - MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

READ carefully the entire manual before using the device.


- Make sure that the product is delivered complete and keep it packed away from heat sources
- We recommend a single patient use.
- The device must be sanitized before any re-use.
- SERVICE LIFE the device use limit is defined by the wear of the parts that can't be repaired/ replaced. Service life limit won't be in any case greater than 10 years.
- DO NOT allow children to play or operate the device.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

7. GENERAL DESCRIPTION

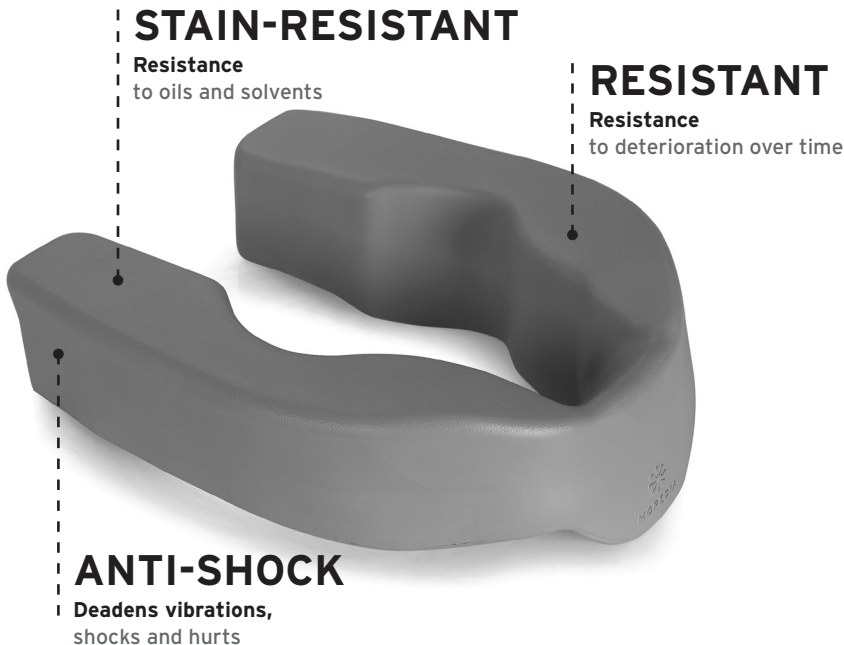Bidet and wc seat in EVA, plastic expanded material, closed cells foam.

WARNING!

The DUO raised bidet and WC seat is made of a newly developed material called E.V.A. (Ethylene Vinic Acetato) and allows to make particularly elastic and soft products that they are adaptable to different uses. The use of E.V.A. also allows the product finished being stain and waterproof. The production of the DUO raised seat takes place carrying the E.V.A. at a temperature of about 150°C for 10 minutes permitting polymerization and consequently product expansion. The cooling stabilizes the rise making it remain elastic and soft. Features productive of the DUO raised seat and the material used can lead to a surface that it is not perfectly smooth or to the presence of small indentations visible to the human eye but imperceptible in the use of the product. These formations are exclusively to be attributed to the characteristic of the material used and cannot be considered defects of the DUO raised seat. The presence of these formations does not negatively influence the nature stability and elasticity of the product nor diminish its effectiveness.

8. BEFORE USE

- Check that the shape of the bidet and/or wc seat is suitable for positioning;
- Check the correct positioning of the raised seat on the bidet and/or wc seat;
- Check the stability of the seat by checking that it does not slip from the bidet and/or wc seat;
- Make sure that the EVA seat has no damage which compromises its stability.

9. HOW TO USE

Bidet

Wc

- Place the raised seat on the bidet and/or wc seat as shown in the pictures;
- Do not exceed the maximum capacity indicated;
- Do not use the raised toilet seat as a support to sit and stand up on the toilet.

10. CLEANING AND DISINFECTION

10.1 Cleaning

To clean the raised toilet seat use only a damp cloth and neutral soap. Then dry well with a clean, dry cloth before use.

10.2 Disinfection

If it is necessary, use a common disinfectant detergent.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone.

11. CONDITION OF DISPOSAL

In case of disposal of the device don't use the common container for municipal waste. We recommend to dispose the device in the appropriate disposal areas for recycling.

12. TECHNICAL FEATURES

External dimensions:  L: 36,4 cm  P: 46,3 cm  H: 10 cm

Weight:  1,5 kg

13. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

14. REPAIR

14.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

14.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

14.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

15. SPARE SPARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

16. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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