**MOPEDIA®**

MA RP518 01 B\_ITA\_07-2022

**ROLLATOR IN ALLUMINIO  
SERIE ATOS**

---

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ITALIANO

## INDICE

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**CE** I Class Medical Device  
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

## 1. CODES

**RP518B** Alluminium rollator atos - Blue

**RP518R** Alluminium rollator atos - Red

## 2. INTRODUCTION

Thank you for purchasing the rollator, **MOPEDIA** by MORETTI S.p.A., realized to help people with movement difficulty inward and outward, during voyages or in everyday life. This instruction manual gives you suggestion for a correct use and for your safety. We recommend you to read carefully this instruction manual before using. In case of doubts, please contact your dealer.

**Note:** Check all parts to verify it won't be any shipping damage. In case of shipping damages, do not use. Contact the dealer for further instructions.

### WARNING!

Do not use the product for a purpose not indicated in this manual. Moretti Spa declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product. The manufacturer can modify the device or the instruction manual without prior notification.

## 3. INTENDED USE

This rollator fits for people with movement problems. Can be used as assistance to facilitate ambulation or even servo-assisted ambulatory. Anyway, it can't be used by people with evident malfunction of the hand or the arm, even by people with balance problems. The rollator can be used inward and outward. We recommend using on flat surface where the wheels can reach the land. It can't be used on rock ground or on slanted area as the rollator can be unstable. The device is not intended to carry people or things.

## 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of: **ROLLATORS - MOPEDIA** complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. WARNINGS

### 5.1 General warnings

- Do NOT install or use the device before reading and understanding this manual; If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional dealer or technical personnel before attempting to install this equipment to avoid injure or damage;
- The rollator should only be used by patients with a suitable motor capacity which ensures their use without risks;
- Advise with physician or therapist to determinate the correct adjustment and the correct use;
- Physician or therapist should contribute to height adjustment for the maximum support, for a correct brake activation and for a general assembly;
- The rollators are not intended to be used as self-propelled devices.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

### 5.2 Warnings for use

1. The pressure brake function is performed by applying weight to the handles or the seat.
2. Breaking this simple rules can cause personal damages.
3. The rollator can be used only on a hard, flat surface. Do not use the rollator to climb the stairs.
4. Leaning always on both dumbbells to maintain balance.
5. Pay particular attention to wet, icy or snowy.
6. Do not overload the rollator.
7. Do not use the rollator seat (where present) to carry persons or property, the seat is for the user's session.
8. All wheels must be in contact with the floor at any time during use to ensure the proper balancing of the rollator and maximum safety.
9. Ensure that you are on a flat, non-slippery surface before using the rollator for sitting.

**Note** Applying too much pressure on the handles or the seat will apply the brake even while walking.

### 5.3 Stability warnings

- The rollator can provide ambulatory assistance to an individual weighting up to 100kg including the weight of the baskets;
- The rollator basket has a weight limitation of 5kg, don't overload to avoid rollover and imbalance risks.

### 5.4 Assembling & folding warnings

- Verify that both handle support tubes are correctly inserted in the frame and secured with locking knobs locked into the holder with its main wing nuts and knobs without movement;
- Both handle tubes are securely locked in the main support part with wing nuts and knobs without any movements;
- Do not hanging anything on the rollator frame to avoid distortions of the tubes when the rollator is open and damage the stability. Injury may also be caused by distortions;
- Take care to entrapment of fingers and / or clothing during the opening and closing.

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

-  Manufacturer
-  Batch Lot
-  Read the instruction manual
-  Medical Device
-  Conditions of disposal
-  Maximun loading weight

## 7. GENERAL DESCRIPTION

## 8. BEFORE USE

Before using the device, always check:

1. That all parts of the rollator are locked in a secure manner;
2. Check that all screws are tight;
3. Make sure that the seat is properly locked (if that 'not press further with the power seat bottom frame);
4. Check that the locking of the seat is correctly inserted;
5. Check the correct operation of the push down brake.

## 9. ASSEMBLING INSTRUCTION

### 9.1 Before assembling the product

Unpack the product and check that no damage has occurred during shipping. In the event when the rollator is damaged, please contact your nearest dealer.

### 9.2 Open the rollator

Open the rollator pulling the front part of the frame by leveraging on the legs.

### 9.3 Pushing handles assembly

Place the handle tubes in the frame. Turn the locking knobs until they are locked.

### 9.4 Adjustment of the handles

Adjust the height of the handles using the side knobs. Unscrew the knob, place the handles to the desired height, turn the knob clockwise to tighten and fasten it again. Repeat for the other handle and

make sure that both handles are positioned at the same height.

**WARNING!**

Make sure that the handle is in line with the direction of movement of the rollator. If the handle does not stay straight, there is a risk that the rollator wobbling or tipping over, especially if you push it harder.

**9.5 Folding the rollator**

Make lever on the lateral bars and bring back the front of the rollator.

**9.6 Basket**

The shopping basket is hung to the rollator frame.

**WARNING!**

Use the basket for small items only. Do not overload the basket to prevent imbalances and the risk of overturning

**9.7 Seat**

Before sitting, place the rollator on a flat and solid surface and then lock the brakes. Make sure the brakes are securely locked and that the rollator is in a stable position.

**WARNING!**

Sit down distributing your weight on the seat. Do not sit by pushing only on one side.

**10. HOW TO USE**

- Start pushing when you are in an upright position with both hands on the handle;
- Make sure your hands are dry before using the rollator to avoid the risk of slipping;
- Walking maintaining an upright position with his eyes facing forward and not down;
- Do not push the rollator too far away from the body, this may cause falls and loads inadequate;
- Do not use the rollator to overcome steps higher than 20mm;
- When using outdoors, prefer the sidewalks properly prepared in accordance regulations for the removal of architectural barriers.

**11. BRAKES**

**11.1 To brake**

To brake the rollator shifting your weight on the handles while walking until the rollator or stops. To continue to move the rollator, release the pressure.

**12. CARE AND MAINTANANCE**

The devices MOPEDIA by Moretti SpA on the market are carefully checked and carry the CE mark. For the safety of the patient and the doctor is recommended to be checked by the manufacturer or by an approved laboratory, at least every 2 years, the suitability for use of your rollator. If repairs are to be used only original spare parts and accessories. The useful life of the product depends on the use, maintenance, storage and cleaning.

**13. CLEANING AND DISINFECTION**

**13.1 Cleaning**

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water. The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

### 13.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

**WARNING!**

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

### 14. WHEEL CARE

1. Clean the wheels with warm water and mild detergent. Wipe dry with a clean cloth.
2. Periodically lubricate the wheels.

### 15. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

### 16. TECHNICAL FEATURES

Series ATOS										
Product codes	RP518									
Seat	32x35 cm									
Frame	Aluminum									
Weight and dimensions										
Measures in CM										
RP518	61	80,1-90	56,27	61	46	20	8,7	15	15	100 kg

### 17. SPARE PARTS / ACCESSORIES

For spare parts and accessories please refer to the Moretti S.p.A. master catalogue.

### 18. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use.

Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### 19. REPAIRING

**Warranty repair:** If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its

unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

**Repair not cover from warranty:** A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

**Non-defective device:** The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

## 20. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months as from the receiving date.

## 21. EXEMPT CLAUSES

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be fixed by according to the law is limited by the warranty period, in the limits of law. Some states or countries don't allow to limit the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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