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Via _____ Località _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

MOPEDIA®

EN

MA RP540_RP545x_RP551 05 B*_EN_02 2025

Aluminum rollators
INSTRUCTION MANUAL

INDEX

EN	1. CODES.....	PAG.3
	2. INTRODUCTION	PAG.3
	3.INTENDED USE.....	PAG.3
	4. DECLARATION OF CONFORMITY	PAG.3
	5. WARNINGS	PAG.4
	5.1 General warnings	pag. 4
	5.2 Warnings for use.....	pag. 4
	5.3 Stability warnings	pag. 4
	5.4 Assembling & folding warnings	pag. 4
	6. SYMBOLS	PAG.5
	7. BEFORE USE	PAG.5
	8. ASSEMBLING INSTRUCTION	PAG.5
	8.1 Before assembling the product.....	pag. 5
	8.2 Assembling model RP540	pag. 5
	8.2.1 Folding the rollator	pag. 6
	8.2.2 Installation of the handles and adjustable handles	pag. 6
	8.2.3 Bag installation	pag. 6
	8.2.4 Seat	pag. 6
	8.2.5 Footrest	pag. 6
	8.3 Assembling model RP545	pag. 6
	8.3.1 Folding the rollator	pag. 7
	8.3.2 Installation of the handles and adjustable handles	pag. 7
	8.3.3 Bag installation	pag. 7
	8.3.4 Seat	pag. 7
	8.4 Assembling model RP551	pag. 7
	8.4.1 Fold the rollator	pag. 8
	8.4.2 Installation of the handles and adjustable handles.....	pag. 8
	8.4.3 Installation of the backrest.....	pag. 8
	8.4.4 Bag installation.....	pag. 8
	8.4.5 Seat	pag. 8
	9. HOW TO USE	PAG.9
	10. BRAKES	PAG.9
	10.1 To brake	pag. 9
	10.2 Lock the brake (handbrake)	pag. 9
	11. CARE AND MAINTANANCE.....	PAG.9
	12. CLEANING	PAG.9
	13. WHEEL CARE	PAG.9
	14. TECHNICAL FEATURES.....	PAG.10
	15. CONDITIONS OF DISPOSAL	PAG.10
	16. SPARE PARTS / ACCESSORIES	PAG.10
	17. WARRANTY.....	PAG.10
	17.1 Repairing.....	pag. 11
	17.2 Spare parts	pag. 11
	17.3 Exempt clauses	pag. 11

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RP540	4-wheel aluminum rollator with seat and footrest GAYA 2.0
RP545G	4-wheel aluminum rollator demountable with seat OCEANO 2.0 Gray
RP545A	4-wheel aluminum rollator demountable with seat OCEANO 2.0 Orange
RP545L	4-wheel aluminum rollator demountable with seat OCEANO 2.0 Lime
RP545R	4-wheel aluminum rollator demountable with seat OCEANO 2.0 Pink
RP545B	4-wheel aluminum rollator demountable with seat OCEANO 2.0 Blu
RP545V	4-wheel aluminum rollator demountable with seat OCEANO 2.0 Green
RP551	4-wheel aluminum rollator with seat and antibrachial support SATURNO

2. INTRODUCTION

Thank you for choosing a MOPEDIA range rollators by Moretti S.p.A. MOPEDIA range rollators manufactured by Moretti S.p.A. are designed to satisfy all your requirements for practical, correct and safe use. This manual provides some useful suggestions for correctly using the device you have selected and offers valuable advice for your safety. We recommend reading the entire manual carefully before using the rollator you have purchased. Should you have any queries, please contact your retailer for proper advice or assistance.

3. INTENDED USE

This rollator fit for people with movement problems. Can be used as assistance to facilitate ambulation or even servo-assisted ambulatory. Anyway, it can't be used by people with evident malfunction of the hand or the arm, even by people with balance problems. The rollator can be used inward and outward. We recommend using on flat surface where the wheels can reach the land. It can't be used on rock ground or on slanted area as the rollator can be unstable. The device is not intended to carry people or things.

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ROLLATORS - MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. WARNINGS

5.1 General warnings

- Do NOT install or use the device before reading and understanding this manual; If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional dealer or technical personnel before attempting to install this equipment to avoid injure or damage;
- The rollator should only be used by patients with a suitable motor capacity which ensures their use without risks;
- Advise with physician or therapist to determinate the correct adjustment and the correct use;
- Physician or therapist should contribute to height adjustment for the maximum support, for a correct brake activation and for a general assembly;
- The rollators are not intended to be used as self-propelled devices.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

5.2 Warnings for use

- Insure the brake cable to be correctly set;
- Verify the correct operation of the brake after every height adjustment to avoid a free flexion of the cable;
- Breaking this simple rules can cause personal damages;
- The rollator can be used only on a hard, flat surface. Do not use the rollator to climb the stairs
- Leaning always on both dumbbells to maintain balance;
- Pay particular attention to wet, icy or snowy;
- Do not overload the rollator;
- Do not use the rollator seat (where present) to carry persons or property, the seat is for the user's session with the parking brake engaged;
- All wheels must be in contact with the floor at any time during use to ensure the proper balancing of the rollator and maximum safety;
- The brake must be engaged before using the seat.

Note When the rollator is used in steady position, the handbrake must be locked

5.3 Stability warnings

- The rollator can provide ambulatory assistance to an individual weighing up to 136 kg, INCLUDING the loading bag;
- The rollator bag has a weight limitation of 5 kg.

5.4 Assembling & folding warnings

- Verify that both handle support tubes are correctly inserted in the frame and secured with locking knobs locked into the holder with its main wing nuts and knobs without movement;
- Both handle tubes are securely locked in the main support part with wing nuts and knobs without any movements;
- Do not hanging anything on the rollator frame to avoid distortions of the tubes when the rollator is open and damage the stability. Injury may also be caused by distortions;
- Take care to entrapment of fingers and / or clothing during the opening and closing.

6. SYMBOLS

-  REF Product code
-  UDI Unique Device Identification
-  CE CE mark
-  Manufacturer
-  LOT Batch Lot
-  Read the instruction manual
-  MD Medical Device
-  Conditions of disposal
-  Maximun loading weight

7. BEFORE USE

Before using the device, always check:

1. That all parts of the rollator are locked in a secure manner.
2. Check that all screws are tight.
3. Make sure that the seat is properly locked (if that 'not press further with the power seat bottom frame).
4. Check that the locking of the seat is correctly inserted.
5. Check the correct operation of the brake and the regular brake.

8. ASSEMBLING INSTRUCTION

8.1 Before assembling the product

Unpack the product and check that no damage has occurred during shipping. In the event when the rollator is damaged, please contact your nearest dealer.

8.2 Assembling model RP540

- The rollator is dispatched with footrest packed separately.
- The first time that the rollator will be mounted handles with the knobs will not be connected, and then insert the two tubes into the frame.

- Press the seat of the rollator up to arrest the frame tubes in position lock.
- Before use, always make sure that the rollator does not tend to close up under the effect of a small lateral force.
- Install the footrest on the rollator frame.

WARNING! Make sure that the knobs is located into the holes

8.2.1 Folding the rollator

Lift up the seat firmly pulling the straps in the middle of seat.

8.2.2 Installation of the handles and adjustable handles

Insert the handles into the frame. Simply depress the spring button and set to the desired height. Repeat the opposite side and make sure that both handles are secure and set at the same height before use. The rollator is equipped with height adjustable handles through the pins to meet all user needs.

WARNING!

Adjust the height of the rollator so that a user who is standing with his arms at his sides have the pulse height handles. This will ensure the positioning of the arms bent at about 20° -30° when using the rollator

8.2.3 Bag installation

To install the bag, hang the ears of the bag on the seat, stop the tape with Velcro to the structure

WARNING! Use the basket for small items only. Do not overload the bag to prevent imbalances and the risk of overturning


8.2.4 Seat

Before sitting , place the rollator on a flat and solid surface and then lock the brakes. Make sure the latch of the lock is secured properly and that the rollator is in a stable position.

WARNING! Questions to sit down with your weight on the seat assembly. Do not sit by pushing only on one side.

8.2.5 Footrest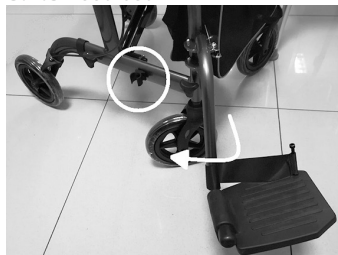

Insert the footrest into the pin's frame. If you don't need fix they on the frame using the clip, like in the picture.

8.3 Assembling model RP545

- Turn the wheel in the correct direction, unbuckle the belt which locks the rear wheels and lock the rear wheels and front with knobs supplied.
- The first time that the rollator will be mounted handles with the knobs will not be connected, and then insert the two tubes in the frame.

- Press the seat of the rollator up to arrest the frame tubes in position lock.

WARNING! Make sure that the knobs is located into the holes. Before use, always make sure that the rollator does not tend to close up under the influence a small lateral force. Be careful not to twist the brake cables when connecting the handlebar and wheels.

8.3.1 Folding the rollator

Lift up the seat firmly pulling the straps in the middle of seat

8.3.2 Installation of the handles and adjustable handles

Insert the handles into the frame. Simply depress the spring button and set to the desired height. Repeat the opposite side and make sure that both handles are secure and set at the same height before use. The rollator is equipped with height adjustable handles through the pins to meet all user needs.

WARNING! Adjust the height of the rollator so that a user who is standing with his arms at his sides have the pulse height handles. This will ensure the positioning of the arms bent at about 20 ° -30 ° when using the rollator.

8.3.3 Bag installation

To install the bag, hang the ears of the bag on the seat, stop the tape with Velcro to the structure.

WARNING! Maximum capacity for bag 5 kg. Use the bag for small items only. Do not overload the bag to prevent imbalances and the risk of overturning

8.3.4 Seat


Before sitting , place the rollator on a flat and solid surface and then lock the brakes. Make sure the latch of the lock is secured properly and that the rollator is in a stable position.


WARNING! Questions to sit down with your weight on the seat assembly. Do not sit by pushing on one side.

8.4 Assembling model RP551

- Release the safety lock, if it is inserted.
- The first time that the rollator will be mounted handles with the knobs will not be connected, and then to open the rollator insert the two tubes into the frame.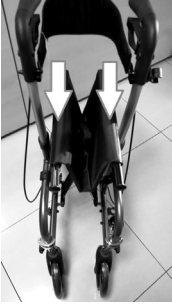

- Press the seat of the rollator up to arrest the frame tubes in position lock and then lock the safety clip.

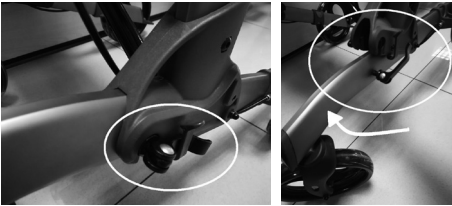- Rotate the wheels frame and fix them with the pins like in the picture.

WARNING! Make sure that the knobs are located into the holes.

Before use, always make sure that the rollator does not tend to close up under the influence of a small lateral force. Be careful not to twist the brake cables when connecting the handlebar and wheels.

8.4.1 Fold the rollator

- Lift up the seat firmly to close the rollator

8.4.2 Installation of the handles and adjustable handles

Insert the handles into the frame, and lock them in an appropriate manner. The rollator is equipped with height adjustable handles through the knobs to meet all user needs.

WARNING!

Adjust the height of the rollator so that a user who is standing with his arms at his sides has the pulse height handles. This will ensure the positioning of the arms bent at about 20° -30° when using the rollator.

8.4.3 Installation of the backrest

Insert the backrest into the guides on the plastic structure of the rollator

8.4.4 Bag installation

To install the bag, hang the ears of the bag on the seat, stop the tape with Velcro to the structure

WARNING! Maximum capacity for bag 5 kg. Use the bag for small items only. Do not overload the bag to prevent imbalances and the risk of overturning

8.4.5 Seat

Before sitting, place the rollator on a flat and solid surface and then lock the brakes.

Make sure the latch of the lock is secured properly and that the rollator is in a stable position.

WARNING! Questions to sit down with your weight on the seat assembly. Do not sit by pushing only on one side.

9. HOW TO USE

- Start pushing when you are in an upright position with both hands on the handle;
- Make sure your hands are dry before using the rollator to avoid the risk of slipping;
- Walking maintaining an upright position with his eyes facing forward and not down;
- Do not push the rollator too far away from the body, this may cause falls and loads inadequate;
- Do not use the rollator to overcome steps higher than 20mm;
- When using outdoors, prefer the sidewalks properly prepared in accordance regulations for the removal of architectural barriers;
- To overcome small obstacles to leverage the rear footrests to raise the front of the rollator (models RP540 - RP545x- RP551)

10. BRAKES

10.1 To brake

To brake the rollator pull the brake lever with your fingers upwards until the rollator slows down or stops. To continue to move the rollator, release the brake handles.

10.2 Lock the brake (handbrake)

Press down on the brake lever with the palm of your hand, when you hear a "click" wheels are locked. Lift the handle of the brake releasing it from the stop position.

WARNING! When the user is stationary, the brakes must be locked

11. CARE AND MAINTANANCE

The devices **MOPEDIA** by Moretti SpA on the market are carefully checked and carry the CE mark. For the safety of the patient and the doctor is recommended to be checked by the manufacturer or by an approved laboratory, at least every 2 years, the suitability for use of your rollator. If repairs are to be used only original spare parts and accessories.

The useful life of the product depends on the use, maintenance, storage and cleaning.

12. CLEANING

To clean parts, use only a damp cloth with a neutral detergent, not use cleaners containing solvents and metal brushes or abrasive or sharp objects, may damage the product.

13. WHEEL CARE

1. Clean the wheels with warm water and mild detergent. Wipe dry with a clean cloth.
2. Periodically lubricate the wheels.

14. TECHNICAL FEATURES

Series GAYA 2.0

Product codes	RP540
Seat	45x31 cm
Frame	Aluminum

Weight and dimensions*

										
RP540	75	85 97	55	63	52	28	9,6	20	20	136 kg

Series	OCEANO 2.0	SATURNO
Product codes	RP545	RP551
Seat	45x20 cm	44x24 cm
Frame	Aluminum	

Weight and dimensions*

										
RP545x	69	79 92	54	62	47	27	7,6	20	20	136 kg
RP551	70	115÷130 Antibrachial 100÷115	61	68	52	28	7,9	25		

* The dimensions are in cm and could be changed $\pm 0,5$ cm

15. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

16. SPARE PARTS / ACCESSORIES

For spare parts and accessories please refer to the Moretti S.p.A. master catalogue

17. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use.

Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

Repair not cover from warranty: A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

Non-defective device: The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

17.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months as from the receiving date.

17.3 Exempt clauses

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be fixed by according to the law is limited by the warranty period, in the limits of law. Some states or countries don't allow to limit the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user.

The present warranty may be modified without prior notification.

EN

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

** Please consult our website for the latest available version of the user manual*

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Para pedir repuestos y accesorios, ver el catálogo master Moretti S.p.A.