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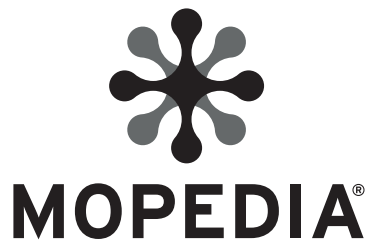
 **MORETTI S.P.A.**

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN CHINA**

MA RP688 01 A_EN_07-2024

STEEL ROLLATOR

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODE	PAG.3
2. INTRODUCTION	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. WARNINGS	PAG.4
5.1 General warnings	pag. 4
5.2 Warnings for use.....	pag. 4
5.3 Stability warnings	pag. 4
5.4 Assembling & folding warnings	pag. 4
6. SYMBOLS	PAG.5
7.....	GENE-
RAL DESCRIPTION.....	PAG.5
8. BEFORE USE	PAG.5
9. ASSEMBLING INSTRUCTION	PAG.6
9.1 Before assembling the product.....	pag. 6
9.2 Open the rollator	pag. 6
9.3 Folding the rollator.....	pag. 6
9.4 Adjustment of the handles.....	pag. 6
9.5 Rollator closing.....	pag. 6
9.6 Tray	pag. 6
9.7 Basket	pag. 6
9.8 The glove compartment.....	pag. 6
9.9 Seat	pag. 6
10. HOW TO USE.....	PAG.7
11. BRAKES	PAG.7
11.1 Brake use	pag. 7
11.2 To lock the wheels	pag. 7
11.3 To unlock the wheels	pag. 7
12. MAINTANANCE.....	PAG.7
13. CLEANING AND DISINFECTION	PAG.7
13.1 Cleaning.....	pag. 7
13.2 Disinfection.....	pag. 7
14. WHEEL CARE	PAG.8
15. CONDITIONS OF DISPOSAL	PAG.8
16. TECHNICAL FEATURES.....	PAG.8
17. WARRANTY	PAG.8
18. REPAIRING	PAG.9
18.1 Warranty repair	pag. 9
18.2 Repair not cover from warranty.....	pag. 9
18.3 Non-defective device.....	pag. 9
19. SPARE PARTS	PAG.9
20. EXEMPT CLAUSES	PAG.9

CE | Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODE

RP688V 4 Wheel steel rollator DYONE 2.0 - Green**RP688L** 4 Wheel steel rollator DYONE 2.0 - Lavander**RP688A** 4 Wheel steel rollator DYONE 2.0 - Light blue

2. INTRODUCTION

Thank you for purchasing the rollator, **MOPEDIA** by MORETTI S.p.A., realized to help people with movement difficulty inward and outward, during voyages or in everyday life. The production has been made according to the highest quality and safety standards in accordance with European Directive on Medical devices (2017/745). This instruction manual gives you suggestion for a correct use and for your safety. We recommend you to read carefully this instruction manual before using. In case of doubts, please contact your dealer.

Note: Check all parts to verify it won't be any shipping damage. In case of shipping damages, do not use. Contact the dealer for further instructions.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti Spa declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer can modify the device or the instruction manual without prior notification.

3. INTENDED USE

This rollator fit for people with movement problems. Can be used as assistance to facilitate ambulation or even servo-assisted ambulatory. Anyway, it can't be used by people with evident malfunction of the hand or the arm, even by people with balance problems. The rollator can be used inward and outward. We recommend using on flat surface where the wheels can reach the land. It can't be used on rock ground or on slanted area as the rollator can be unstable. The device is not intended to carry people or things.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ROLLATORS - MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. WARNINGS

5.1 General warnings

- Do NOT install or use the device before reading and understanding this manual; If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional dealer or technical personnel before attempting to install this equipment to avoid injure or damage;
- The rollator should only be used by patients with a suitable motor capacity which ensures their use without risks;
- Advise with physician or therapist to determinate the correct adjustment and the correct use;
- Physician or therapist should contribute to height adjustment for the maximum support, for a correct brake activation and for a general assembly;
- The rollators are not intended to be used as self-propelled devices.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

5.2 Warnings for use

1. Insure the brake cable to be correctly set;
2. Verify the correct operation of the brake after every height adjustment to avoid a free flexion of the cable;
3. Breaking this simple rules can cause personal damages;
4. The rollator can be used only on a hard, flat surface. Do not use the rollator to climb the stairs;
5. Leaning always on both dumbbells to maintain balance;
6. Pay particular attention to wet, icy or snowy;
7. Do not overload the rollator;
8. Do not use the rollator seat (where present) to carry persons or property, the seat is for the user's session with the parking brake engaged;
9. All wheels must be in contact with the floor at any time during use to ensure the proper balancing of the rollator and maximum safety;
10. The brake must be engaged before using the seat.

Note: When the rollator is used in steady position, the handbrake must be applied

5.3 Stability warnings

- The rollator can provide ambulatory assistance to an individual weighing up to 130 kg, INCLUDING the loading bag;
- The rollator bag has a weight limitation of 5 kg;
- The glove compartment has a weight limitation of 7 kg.

5.4 Assembling & folding warnings

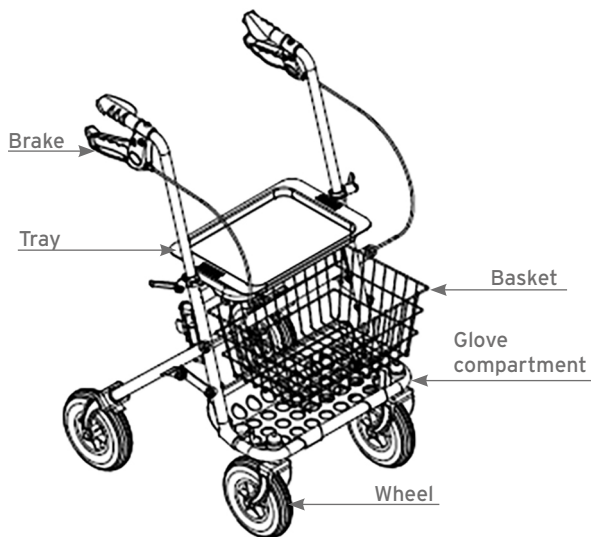
- Verify that the frame is secured with locking knobs locked into the holder with its main wing nuts and knobs without movement;
- Verify that both handle tubes are securely locked in the main support part with wing nuts and knobs without any movements;
- Do not hanging anything on the rollator frame to avoid distortions of the tubes when the rollator is open and damage the stability. Injury may also be caused by distortions;
- Take care to entrapment of fingers and / or clothing during the opening and closing.

6. SYMBOLS

-  REF Product code
-  UDI Unique Device Identification
-  CE CE mark
-  Manufacturer
-  LOT Batch Lot
-  Read the instruction manual
-  MD Medical Device
-  Conditions of disposal

7. GENERAL DESCRIPTION

8. BEFORE USE

Before using the device, always check:

- That all parts of the rollator are locked in a secure manner;
- That all screws are tight;
- That the locking of the seat is correctly inserted;
- The state of wear of the mechanical parts in play so as to ensure the use of the device in total safety for people and things

9. ASSEMBLING INSTRUCTION

9.1 Before assembling the product

Unpack the product and check that no damage has occurred during shipping. In the event when the rollator is damaged, please contact your nearest dealer.

9.2 Open the rollator

Open the rollator pulling the front part of the frame by leveraging on the rear legs. Place the handle tubes in the frame. Turn the locking knobs until they are locked.

9.3 Folding the rollator

Make lever on the lateral bars and bring back the front of the rollator.

9.4 Adjustment of the handles

Adjust the height of the handles using the side knobs. Unscrew the knob, place the handles to the desired height, turn the knob clockwise to tighten and fasten it again. Repeat for the other handle and make sure that both handles are positioned at the same height.

WARNING! Make sure that the handle is in line with the direction of movement of the rollator. If the handle does not stay straight, there is a risk that the rollator wobbling or tipping over, especially if you push it harder

9.5 Rollator closing

Close the rollator pulling the side bars and bringing back the front part of the rollator frame.

9.6 Tray

The tray serves only to support small objects.

WARNING! Do not place hot objects, sharp or heavy on the tray. To fix the tray to the frame, insert the support that are on the bottom of the tray in the corresponding slots on the seat.

9.7 Basket

The shopping basket is hung by two hooks to the rollator frame.

WARNING! Maximum capacity for bag 5 kg. Use the basket for small items only. Do not overload the basket to prevent imbalances and the risk of overturning.

9.8 The glove compartment

The lower glove compartment is fixed to the rollator frame by means of plastic hooks.

WARNING! Maximum capacity for the glove compartment is 7 kg. Use it for small items only. Do not overload the compartment to prevent imbalances and the risk of overturning.

9.9 Seat

Before sitting, place the rollator on a flat and solid surface and then lock the brakes. Make sure the brakes are securely inserted and that the rollator is in a stable position.

WARNING! Sit down distributing your weight on the seat. Do not sit by pushing only on one side.

10. HOW TO USE

- Start pushing when you are in an upright position with both hands on the handle;
- Make sure your hands are dry before using the rollator to avoid the risk of slipping;
- Walking maintaining an upright position with his eyes facing forward and not down;
- Do not push the rollator too far away from the body, this may cause falls and loads inadequate;
- Do not use the rollator to overcome steps higher than 20mm;
- When using outdoors, prefer the sidewalks properly prepared in accordance regulations for the removal of architectural barriers.

11. BRAKES

The good functioning of the brakes depends on the wear and the dirt accumulated on the wheels. Check the condition of the brakes and wheels before each use.

11.1 Brake use

To brake the rollator pull the brake lever with your fingers upwards until the rollator slows down or stops. To continue to move the rollator, release the brake handles.

11.2 To lock the wheels

Press down on the brake lever with the palm of your hand, when you hear a “click” wheels are locked.

WARNING! When the user is stationary, the brakes must be locked

11.3 To unlock the wheels

Pulling on both brake levers until they are unlocked

12. MAINTANANCE

The devices MOPEDIA by Moretti SpA on the market are carefully checked and carry the CE mark. For the safety of the patient and the doctor is recommended to be checked by the manufacturer or by an approved laboratory, at least every 2 years, the suitability for use of your rollator. If repairs are to be used only original spare parts and accessories.

The useful life of the product depends on the use, maintenance, storage and cleaning.

13. CLEANING AND DISINFECTION

13.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

13.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

WARNING! Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

14. WHEEL CARE

1. Clean the wheels with warm water and mild detergent. Wipe dry with a clean cloth.
2. Periodically lubricate the wheels.

15. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

16. TECHNICAL FEATURES

DYONE 2.0

Product code	RP688X
Seat	38x18x2 cm
Frame	Steel

Weight and dimensions

										
RP688V			59 seat							
RP688L	64,5	78		58	43,5	59x27x78	9	20	20	130
RP688A		95	61 above tray							

17. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use. Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

18. REPAIRING

18.1 Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

18.2 Repair not cover from warranty

A product without warranty can be given back after Moretti authorization. The labor and

shipping costs for a bath lift out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

18.3 Non-defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

20. EXEMPT CLAUSES

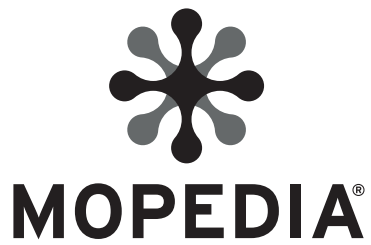
Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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