MOPEDIA®

MA RS810-RS815-RS817 02 B_ITA_07 2021

SEDILI DA BAGNO

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ITALIANO

INDICE

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CE I Class Medical Device
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RS810	Plastic bathtub seat with handle
RS815	Fodable bath bench
RS817	Bath stool

2. INTRODUCTION

Thank you for purchasing a MOPEDIA bath products by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

NOTE: Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. INTENDED USE

The bath chair by Moretti is intended for disable patient assistance in bath tub

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SHOWER - BATHROOM CHAIRS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Keep the packed bath chair away from heat sources
- SERVICE LIFE- the product use limit is defined by the wear of the parts
- ALWAYS keep hands and fingers clear of moving parts to avoid injury
- DO NOT allow children to play on or operate the bath chair
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

7. GENERAL DESCRIPTION

7.1 RS810

Plastic bathtub seat with anti-slip plastic handle and holes for the water. Adjustable width for better adaptability.

7.2 RS815

Folding bath chair with painted steel structure and plastic seat, folding to save space during storage. Non-slip rubber tips.

7.3 RS817

Round shape bath seat. With anodized aluminum frame.

8. ASSEMBLING


8.1 RS810

The product is supplied unassembled and it is easy to assemble without the use of any tools.


Is composed by:

- Seat (1 pc)
- Stoppers (4 pcs)
- Handle (1 pc)
- Screws

Screw the four large plastic screws on the underside of the seat. Tighten them well with a screwdriver or a small coin. To assemble the handle, position the seat with the stoppers facing downwards, introduce the handle into the holes at the ends of the seat. The handle should face the center of the seat. (Fig.1)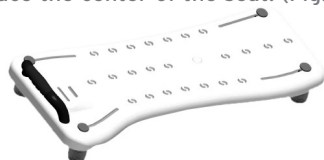

(Fig.1)

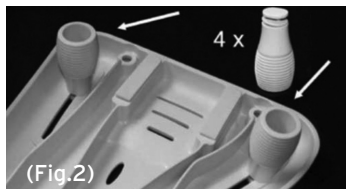(Fig.2)

Seat adjustment

(Only purpose picture)

- Turn the seat, loosen the 4 screws so that they can move
- Slide the 4 screws into the center of the seat centre so that they can easily adapt to the bathtub
- Turn to the seat again so you place in the tub with the handle facing the wall.
(The handle must always be on the side of the wall)
- Place the bath seat in the best position.
- Slide the cleats so that they touch the inside of the tub and tighten the thumbscrews
- Flip back the seat so that the cleats are facing upwards and adjust outward to about 2 mm on each cleat. Now tighten the brackets so that they do not move easily.
- Turn the hot seat from the right and press down gently in the bathroom (not force). If the seat slides along the tub, you may need to adjust the supports further outward. To do this, repeat steps from 2 to 7


8.2 RS815

The product is supplied assembled.

8.3 RS817

The product is supplied unassembled and it is easy to assemble. In the package you will find:

- Round seat with frame (1 pcs)
- Legs (4pcs)

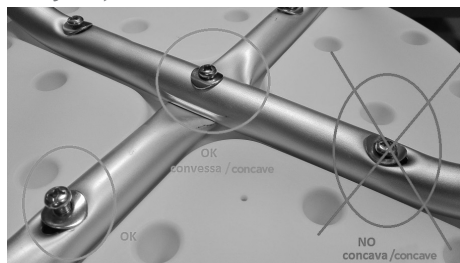9. BEFORE USE

- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly of the device

10. CAUTIONS FOR USE

The manufacturer and distributor are not responsible for any responsibility for the consequences of improper use of the product.

- Always help the patient with walking difficulties to position themselves on the seat
- Do not use with patients who exceed the maximum weight shown on the label placed on the product frame
- Avoid sudden movements during positioning
- Be careful on wet floors, on wet or slippery surfaces, it could make difficult the patient positioning
- Do not store the product outside. Periodically check the stability of the product and the correct fastening screws and the status of protections rubber
- In case of doubt for the use, the adjustment or the maintenance of the product, please contact your dealer

 **Do not exceed the maximum load capacity for any reason!**

11. HOW TO USE

11.1 RS810

- Place the seat in the tub and tighten the stoppers to make it stable
- Place the handle on the side opposite to the point of access to the tub
- If necessary help the patient to position on the seat
- Use the handle as an aid to move and position themselves correctly in the seat

11.2 RS815

Position the seat and open it on a smooth flat surface. Check the stability of the seat before use.

11.3 RS817

Height adjustment

Press the pin to adjust the seat by removing the pole and putting it into the holes to the desired height and the maneuverability of the patient. Return the stop in the hole to lock it. Check that all four legs are of the same height.

Place seat on a flat, level surface. Check the stability of the seat before use

12. CLEANING AND DISINFECTION

12.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water. Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

12.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

13. CONDITIONS OF DISPOSAL

In case of disposal don't use the  sorting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

14. TECHNICAL FEATURES

RS810

***DIMENSION:**  L: 41÷63 CM  W: 31,5 CM  H: 18 CM

 **WEIGHT:** 3,1 KG

 **MAX LOADING WEIGHT:** 160 KG

RS815

***DIMENSION:**  L: 49 CM  W: 38 CM  H: 47 CM

SITTING: 49X29 CM

 **WEIGHT:** 3,2 KG

 **MAX LOADING WEIGHT:** 130 KG

RS817

***DIMENSION:**  L: 35 CM  W: 35 CM  H: 36÷53,5 CM

SITTING: Ø 31 CM

 **WEIGHT:** 1,5 KG

 **MAX LOADING WEIGHT:** 110 KG

*** THE MEASUREMENTS REFER TO THE OVERALL DIMENSIONS**

15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

16. REPAIR

16.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

16.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

16.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

17. SPARE SPARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

18. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free.

The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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