**MOPEDIA®**

MA\_RS844-XX\_RS846-XX\_RS847-XX\_RS848\_RH840-XX\_03 A\_ITA\_04-2021

**SEDIE DA DOCCIA E WC**

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ITALIANO

## INDICE

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- Portata max 136 Kg (per RS844-XX, RS846-XX, RS847-XX, RS848); 200 kg (per RH840-XX)

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**CE** I Class Medical Device  
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

### 1. CODES

- RS844-XX** - Tilting shower chair with WC and 4 castors  
**RS846-XX** - Detachable shower chair with WC and 4 castors  
**RS847-XX** - Self-propelled shower chair  
**RS848** - Foldable shower chair with WC and 4 castors  
**RH840-XX** - Self-propelled bariatric wheel chair

XX = sizes 40/46/50 for RS844/RS846/RS847, 50/60 for RH840

### 2. INTRODUCTION

Thank you for choosing a MOPEDIA shower and WC chair by Moretti S.p.A. MOPEDIA shower and WC chair manufactured by Moretti S.p.A. are designed to satisfy all your requirements for a practical, correct and safe use. This manual provides some useful suggestions to correctly use the device you have selected and offers valuable advices for your safety. We recommend carefully reading the entire manual before using the hospital bed you have purchased. Should you have any queries, please contact your retailer for proper advice or assistance.

**NOTE: Check all parts of the product have not been damaged during shipment. In case of damage do not use the product and contact your dealer for further instructions.**

#### CAUTION!

- Do not use the following device for purposes other than those described in this manual.
- Moretti S.p.A. disclaims any liability for damage resulting from improper use of the device or use other than that described in this manual.
- We reserve the right to make changes to the device and this manual without notice in order to improve the characteristics.

### 3. INTENDED USE

The MOPEDIA shower and WC chairs are created for people with movement difficulty and can be used indoors as substitution of toilet or as aid under the shower.

### 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SHOWER CHAIRS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**NOTE: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.**

## 5. GENERAL WARNINGS

### WARNINGS!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts cannot be repaired and / or replaced.
- Be careful when children are nearby.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

## 6. SYMBOLS

	Product code		Batch Lot
	Unique Device Identification		Medical Device
	Read the instruction manual		CE mark
	Conditions of disposal		Manufacturer

## 7. GENERAL DESCRIPTION

### 7.1 View and list of Components

RS844-XX Tilting WC and shower chair with 4 wheels (Fig.1)

1. Frame - 1 pc
2. Backrest with armrests - 1 pc
3. Pivoting wheels without brake - 2 pcs
4. Pivoting wheels with brake - 2 pcs
5. Headrest - 1 pc
6. Footrests - 2 pcs
7. Seat - 1 pc
8. Seat cover - 1 pc
9. Bucket - 1 pc

Fig.1

Images are illustrative purposes only

RS846-XX Demountable WC and shower chair with 4 wheels (Fig.2)

1. Left frame - 1 pc
2. Right frame - 1 pc
3. Connecting pipe - 2 pcs
4. Backrest with armrests - 1 pc
5. Footrests - 2 pcs
6. Seat - 1 pc
7. Seat cover - 1 pc
8. Bucket - 1 pc

Fig.2

Images are illustrative purposes only

RS847-XX Self-propelled WC and shower chair with 4 wheels (Fig.3)

Fig.3

1. Left frame - 1 pc
2. Right frame - 1 pc
3. Connecting pipe - 2 pcs
4. Backrest with armrests - 1 pc
5. Footrests - 2 pcs
6. Seat - 1 pc
7. Seat cover - 1 pc
8. Bucket - 1 pc

RS848 Foldable WC and shower chair with 4 wheels (Fig.4)

Fig.4

1. Foldable frame - 1 pc
2. Backrest with armrests - 1 pc
3. Footrests - 2 pcs
4. Seat - 1 pc
5. Seat cover - 1 pc
6. Bucket - 1 pc

RH840-XX Self-propelled shower chair with WC (fig.5)

Fig.5

1. Detachable steel frame - 1 pc
2. PU seat - 1 pc
3. PU backrest - 1 pc
4. Tilting armrests - 2 pcs
5. Detachable and tilting footrest - 2 pcs


Images are illustrative purposes only

## 8. ASSEMBLING/DISASSEMBLING

### 8.1 Assembling RS844-XX

To complete the assembly carry out the following operations:

- Instal the 4 wheels on the frame using screws and washers provided. The wheels without brakes must be installed on the front side of the frame (fig.5);
- Insert the backrest inside the holes on the wheelchair frame and fix the backrest using the two fixing knobs (Fig. 6);
- Insert the headrest in the support already installed on the push handle and tighten the support (Fig.6);
- Place the seat on the wheelchair frame and press down to fix its position, then add the cover (Fig. 7);
- Insert the footrests from the side and rotate them to the front until fastening into the holder (Fig. 8);
- Insert the bucket from the rear side of the shower and WC chair by sliding it into the slots.

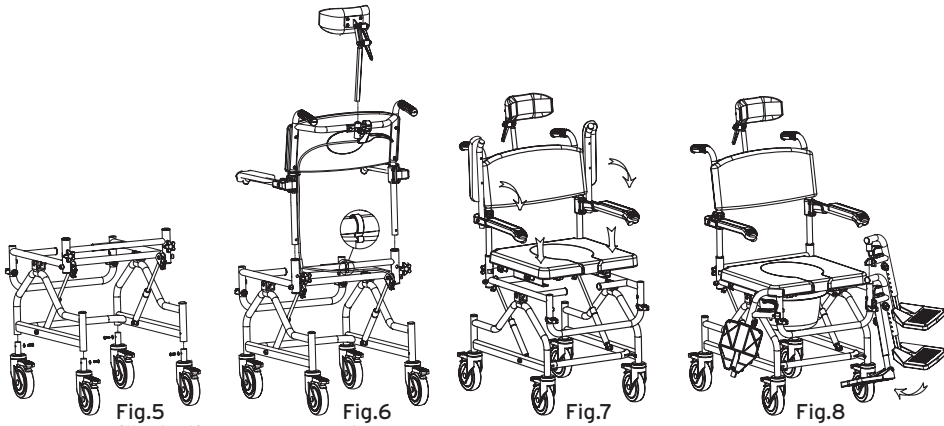


Images are illustrative purposes only



### 8.2 Assembling RS846-XX - RS847-XX

To complete the assembly carry out the following operations:

- Insert the fixing tubes on the left or right side frame, align the holes and insert the C-clips as shown in Fig.9;
- Connect the newly assembled part to the other side of the frame by using the same two fixing tubes and two additional C-clips as described in the previous point (Fig.10);
- Insert the backrest inside the holes on the wheelchair frame and fix the backrest using the two fixing knobs (Fig.11);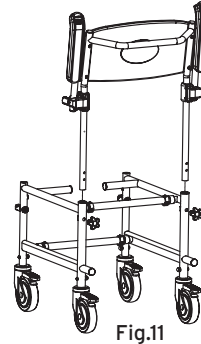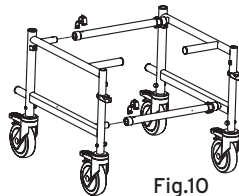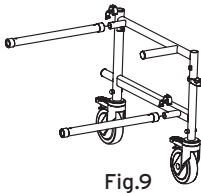

- Place the seat on the wheelchair frame and press down to fix its position, then add the cover (Fig.12);
- Insert the footrests from the side and rotate them to the front until fastening into the holder (Fig.13);
- Insert the bucket from the rear side of the shower and WC chair by sliding it into the slots (Fig.14).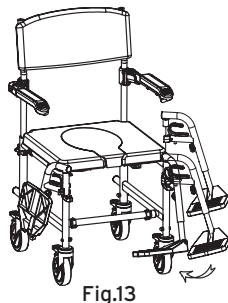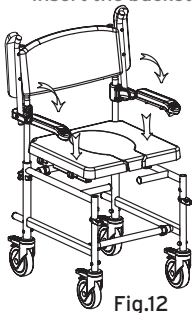

Images are illustrative purposes only

### 8.3 Assembling RS848

To complete the assembly carry out the following operations:

- Open the frame;
- Insert the backrest inside the holes on the wheelchair frame and fix the backrest using the two fixing knobs (Fig.15);
- Place the seat on the wheelchair frame and press down to fix its position, then add the cover (Fig.16);
- Insert the footrests from the side and rotate them to the front until fastening into the holder (Fig.17);
- Insert the bucket from the rear side of the shower chair by sliding it into the slots (Fig.18).

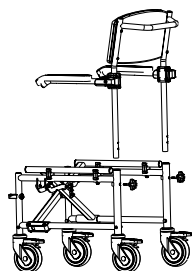
Fig.15

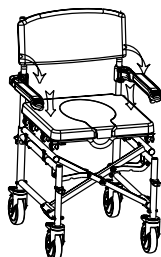
Fig.16

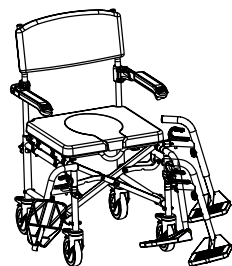
Fig.17

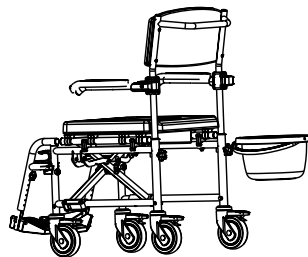
Fig.18

Images are illustrative purposes only

### 8.4 Assembling RH840-XX

The product will be delivered partially assembled and in order to complete the assembly no tools are needed. The shower chair is made of these parts:

- Frame left side;
- Frame right side;
- Backrest;
- Right/left footrest;
- Seat and seat cover;
- 4x C-clips with safety lock;
- 2x tightening nobs;
- 2x fixing pipes;

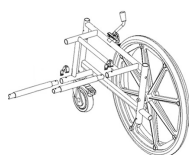
Fig.19

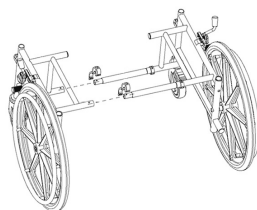
Fig.20

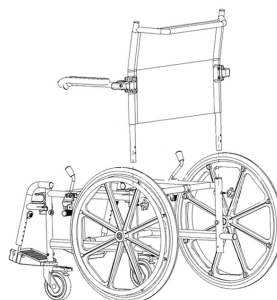
Fig.21

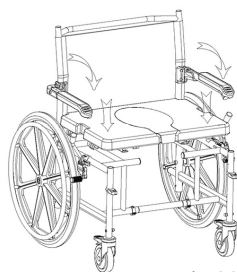
Fig.22

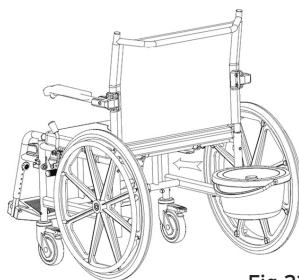
Fig.23

### TO COMPLETE THE ASSEMBLY FOLLOW THESE STEPS:

- Insert the fixing tubes on the left or right side of the frame, align the holes and insert the C-clips as shown in Fig.1;
- Connect the newly assembled part to the other side of the frame by means of the same two fixing tubes and two more C-clips as described in the previous point (Fig.2);
- Insert the backrest inside the holes on the wheelchair frame and fix the backrest by means of the two fixing knobs (Fig.3);
- Place the seat on the wheelchair frame and press down to fix its position, then add the cover (Fig.4);
- Add the footrests sideways, then rotate them forward until they lock;
- Insert the bucket from the back side of the shower chair by sliding it into the appropriate guides (Fig.5).

### 8.5 Disassembling

Follow the procedure described above in reverse.

### 9. BEFORE TO USE

- Always check the status of the mechanical parts in order to guarantee they are used in total safety for persons and environment.
- Check the proper assembly of the device with attention to the correct insertion and locking screws and nuts of the backrest and footrest.

### 10. PRECAUTIONS FOR USE

- The manufacturer and distributor are not responsible for any responsibility for the consequences of improper use of the product.
- Always help the patient with walking difficulties to position themselves on the seat.
- Do not use with patients who exceed the maximum weight shown on the label placed on the product frame.
- Avoid sudden movements during positioning.
- Be careful on wet floors, on wet or slippery surfaces, it could make difficult the patient positioning.
- Do not store the product outside. Periodically check the stability of the product and the correct fastening screws and the status of protections rubber.
- In case of doubt for the use, the adjustment or the maintenance of the product, please contact your dealer.
- Do not exceed the maximum load capacity for any reason.
- Assist always the patient during the handling operations.
- Don't skew the chair more than 10% without assistance.
- Block the wheels during patient transfer.
- Wheelchair has to be in a stable surface and wheels locked during user transfer.
- Before leaning or get up, be sure casters are in straight position and rear castors are blocked in order to avoid a turnover.
- Do not lean out of the backrest to avoid a turnover.
- Pay attention to your fingers during assembling, regulations or simply during the use.
- When using the self-propelled wheel, pay attention to your fingers.
- Keep the chair far from fire.
- Use always original accessories and spare parts.

**WARNING!** Do not exceed the maximum load capacity for any reason!

### 11. HOW TO USE

Use only the back handle to move the chair, never use other supports. Know your own capabilities and limitations in terms of strength and endurance before attempting to go up a ramp. Assistance is recommended when in slope higher than 10%. If you need to stop in a sloping surface, avoid abrupt movements. During the downhill, do not change suddenly the direction. Lift the chair up

if necessary, using the main frame and not armrests or footrests.

**WARNING!** The patient must not put pressure on footrests when go up and down from the chair.

### 11.1 RS844-XX

- Setting out to sit with your weight on the seat assembly. Do not sit by pushing on one side only;
- Leaning always on both arms to get up and keep your balance;
- In case of use of the chair by people with movement difficulties, help them always in the sitting or in getting up from the chair.

#### FOOTREST ADJUSTMENT

The footrests of shower and WC chair can be adjusted according to your needs:

- Unfasten the screw;
- Adjust the footrest height;
- Fasten the screw again. Check that both footrests are adjusted at the same height.

#### HEADREST ADJUSTMENT

The headrest can be easily adjusted in different positions. To make adjustments, unscrew the knobs on the headrest one at a time and adjust their position. Then tighten the knob firmly. Carry out the same operations also on the other knobs if necessary.

#### TILT ADJUSTMENT

To tilt the chair back, pull the lever installed on the push handle and push down.

To return the chair to its normal position, simply pull the lever installed on the push handle. The hydraulic pistons are designed to return to the starting position without the user applying any force.

### 11.2 RS846-XX - RS848

- Setting out to sit with your weight on the seat assembly. Do not sit by pushing on one side only;
- Leaning always on both arms to get up and keep your balance;
- In case of use of the chair by people with movement difficulties, help them always in the sitting or in getting up from the chair.

#### HEIGHT ADJUSTMENT

The backrest and the armrests can be adjusted together on two different heights. Unscrew and remove the knobs that lock the backrest to the seat and raise or lower the backrest as necessary. Finally insert and tighten the knobs again.

#### FOOTREST ADJUSTMENT

The footrests of shower and WC chair can be adjusted according to your needs:

- Unfasten the screw;
- Adjust the footrest height;
- Fasten the screw again. Check that both footrests are adjusted at the same height.

### 11.3 RS847-XX

Use only the back handle to move the chair, never use other supports. Know your own capabilities and limitations in terms of strength and endurance before attempting to go up a ramp. Assistance is recommended when in slope higher than 10%. If you need to stop in a sloping surface, avoid abrupt movements. During the downhill, do not change suddenly the direction. Lift the chair up if necessary, using the main frame and not armrests or footrests.

#### **WARNING!**

The patient must not put pressure on footrests when go up and down from the chair.

**BRAKES**

You can brake the chair pushing the lever on each wheel. If you need to adjust the brake lever, you can unscrew the nut, move the clamp at the required position and then tighten again.

**WC**

Use the toilet when the wheelchair is completely blocked. To use the toilet take out the removable part of the seat using the handle on the front part. Take out the toilet using the handle.

**HEIGHT ADJUSTMENT**

The backrest and the armrests can be adjusted together on two different heights. Unscrew and remove the knobs that lock the backrest to the seat and raise or lower the backrest as necessary. Finally insert and tighten the knobs again.

**FOOTREST ADJUSTMENT**

The footrests of shower and WC chair can be adjusted according to your needs:

- Unfasten the screw;
- Adjust the footrest height;
- Fasten the screw again. Check that both footrests are adjusted at the same height.

**11.4 RH840-XX**

To push the chair, always use only the rear handle, never push by forcing other parts. Before moving on an inclined surface, be sure of your abilities and limitations. Consult movement techniques with qualified personnel. However, assistance is recommended for surfaces with a slope greater than 10%. If you must stop on a sloped surface, avoid sudden movements. Do not use wheel brakes to slow down on a slope. Do not change direction abruptly when descending. If two people are lifting the Chair, avoid gripping it in places that are dangerous to the patient such as footrests and armrests, but use only the frame and rear handles.

**WARNING!**

The patient must not put pressure on footrests when go up and down from the chair.

**BRAKES**

the rear wheel brake lever may need adjustment. If this is the case, adjust the brakes by loosening the lock nut and sliding the clamp forward or backward as needed. When the adjustment is complete, secure the locking nut.

**WC**

Use the toilet with the chair fully braked. To use the WC, pull out the removable part of the seat and pull the bucket out from the back by the handle.

**HEIGHT ADJUSTMENT**

The backrest and armrests can be adjusted together to two different heights. Unscrew and remove the knobs that secure the backrest by one hole as needed. Then insert and screw the knobs back on.

**ADJUSTING THE FOOTRESTS:**

The chair footrests can be adjusted to the desired height:

- Loosen the adjustment knob;
- Adjust the footrest to the desired height;
- Tighten the knob securely. Always check that both footrests are at the same height.

## 12. MAINTENANCE

The MOPEDIA by MORETTI S.p.A. chairs need minimal maintenance and general care, but we suggest you to control regularly:

- Braking system: monthly.
- Cleaning parts (seat, backrest, armpad, wheels and so on) if necessary.
- Cleaning the upholstery of the seat as required.
- General check by qualified personnel every two years.

It is extremely important that the wheelchair is checked at the first use. In particular, please check that all screws and nuts are tight.

## 13. CLEANING AND SANITISATION

### 13.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

**WARNING!** Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

### 13.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

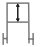
**WARNING!** Sanitising agents are corrosive. Avoid contact with the eyes and skin. Wear protective clothing, gloves, protective glasses and safety shoes. In case of accidental contact, rinse the affected part thoroughly using warm water and consult a doctor immediately.

## 14. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

## 15. TECHNICAL FEATURES

### 15.1 Dimensions and weight

Dimensions* and weight												
Measures in cm												
<b>RS844-40</b>		40										
<b>RS844-46</b>	98	46	102	78	54	40÷55	46	49	49	17 kg	12	12
<b>RS844-50</b>		50										
<b>RS846-40</b>		40										
<b>RS846-46</b>	85	46	96÷98	73÷75	50	40÷55	46	45÷48	49	14 kg	12	12
<b>RS846-50</b>		50										
<b>RS847-40</b>		40										
<b>RS847-46</b>	101	46	96÷98	73÷75	50	40÷55	46	45÷48	51	17 kg	60	12
<b>RS847-50</b>		50										
<b>RS848-46</b>	85	46	96÷98	73÷75	50	40÷55	46	45÷48	49	15 kg	12	12
<b>RH840-50</b>		50										
<b>RH840-60</b>	100	60	96,8	73÷75	50,5	40÷55	61	41	73,1	21 kg	60	12

\*The dimension could be change  $\pm 0.5\text{cm}$

### 15.2 Technical specifications

- Powder coated aluminum frame
- PU seat, armrest and back (for RS844-XX, RS846-XX, RS847-XX, RS848); schienale a rete (for RH840-XX)
- Folding armrests
- Removable WC bucket with cover
- Max User weight 136 Kg (for RS844-XX, RS846-XX, RS847-XX, RS848); 200 kg (for RH840-XX)

## 16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

## 17. REPAIRING

### Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

**Repair not cover by warranty**

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

**Non defective device**

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

**18. SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

**19. EXEMPT CLAUSES**

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

**WARRANTY CERTIFICATE****Product** \_\_\_\_\_**Purchased on (date)** \_\_\_\_\_**Retailer** \_\_\_\_\_**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_**Sold to** \_\_\_\_\_**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

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