**MOPEDIA®**

ITALIANO

MA RS900 02 B\_ITA\_03 2021

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RS900 Bath lift

## 2. INTRODUCTION

Thank you for purchasing the bath lift from the **MOPEDIA** line by MORETTI S.p.A. its design and qualities guarantee comfort, safety and reliability to the user. The bath lift by MORETTI S.p.A. has been designed to satisfy all your demands. This instruction manual contains advices and for your safety and for a correct use of the aid. We recommend reading thoroughly the instruction manual before using the bath lift. In case of doubts, contact your retailer.

## 3. INTENDED USE

The Moretti bath lift is intended for lowering and lifting a person in a bathtub.

### WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

## 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of BATH LIFTERS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

- Read carefully the entire manual before using the device
- Make sure that the product is delivered complete and keep it packed away from heat sources
- Service life the device use limit is defined by the wear of the parts
- Do not allow children to play or operate the device
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.
- For a correct use take reference to the following manual
- Person with severe physical handicaps or mentally disturbed persons should never be left unattended in the bath
- The seat may slip if the suction pads do not adhere properly.  
Clean the bathtub and the suction pad before use
- DO NOT move the bath lift once you have installed it in the bathtub
- Replace the suction pads whenever they are damaged or worn out
- DO NOT use the bath lift without putting on the soft mat
- Always sit in the middle of the bath lift and keep your weight centering in the seat while moving in or out the bathtub and bathing
- Only raise or lower the actuator when the backrest is upright and securely locked
- There is a risk of injury while the lifting scissors move. DO NOT reach between the lifting scissors during assembling it
- The bath lift must be inaccessible to children and animals
- Use only the battery charger that comes with your bath lift
- The battery charger must only be used outside the bathroom in a dry environment
- DO NOT connect the battery charger to the power sources until the battery pack has been correctly connected with it
- DO NOT place the battery pack near hot, sparking or burning objects
- Always disconnect the battery pack before transporting the bath lift
- Batteries contained in the battery pack are rechargeable and replaceable, but can only be replaced by qualified personnel. Do not try to open or replace them by yourself. The used batteries must be disposed of in the proper manner
- Repairs must only be undertaken by authorized specialists or the manufacturer
- The actuator is not designed to be used under water level. In order not to cause any danger or damage do not operate the bath lift while the actuator is submerged in the water
- DO NOT use a damaged or badly worn bath lift
- DO NOT overload the bath lift. Otherwise, serious injury may occur

### WARNING!

- Never use the bath lift as an aid to get in or out of anything except for the bathtub
- Never use as a support, a ladder, lifting platform or for similar purpose
- Any other use which this bath lift does not intended for may cause serious injury or damage to personal property

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

- LOT      Batch Lot
-       Read the instruction manual
- MD      Medical Device
-       Conditions of disposal
-       Attention, should read the instructions
-       Production dates
-       Waste Electrical & Electronic Equipment (WEEE)


**IP54**      Protection against dust and water splashes

      B Type

      Class II Equipment

**7. GENERAL DESCRIPTION**


The bath lift is composed by a unit base with sturdy collapsible frame enables the chair to be raised and lowered from 6.5 to 42 cm. The actuator is located on the back of the backrest. As the actuator stretching and retracting, the bath lift can be lifted up, lowered down or adjusted the slope of the backrest. The movements of this actuator are controlled by the handled controller attached with the backrest. The battery pack which has Ni-MH batteries inside, offer the power needed to drive the actuator. Due to its integrated power control, the bath lift only lowers if there is still sufficient power available to raise it.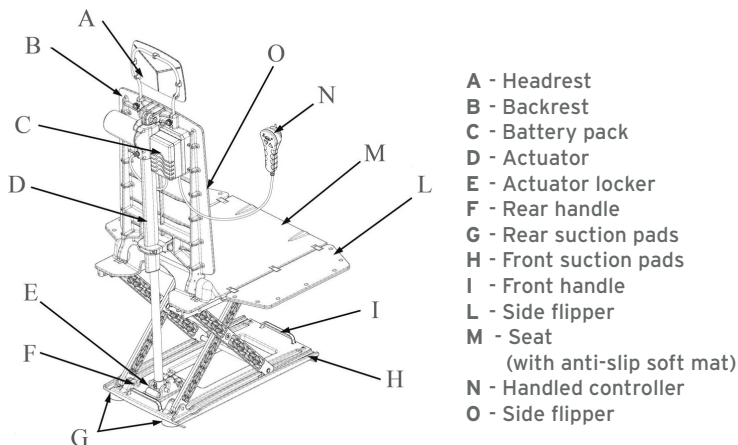

## 8. ASSEMBLING


Firstly proceed unpacking the box, please verify the following:

- Check for any obvious damage to the carton or its contents, If damage is evident, notify the carrier, or your local dealer
- Remove all loose packing from the carton
- Carefully remove all components from the carton
- Check the contents correspond to the following components: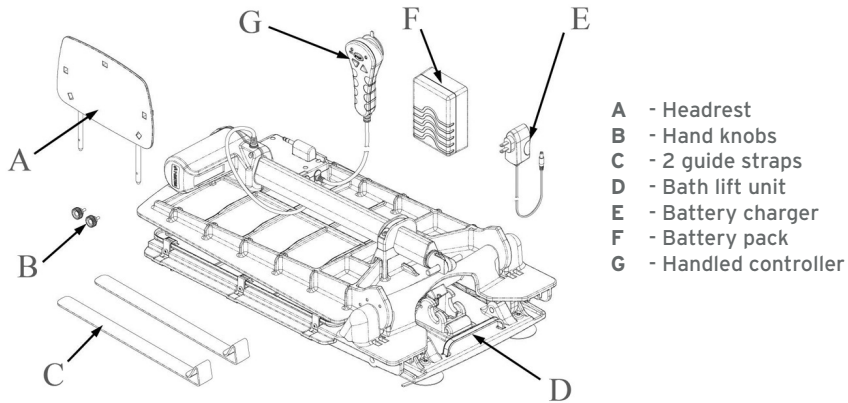

For ease handling and storage, the backrest may be separated from the bath lift. To assembly the backrest in the correct way follow the procedure:

1. Lift the backrest straight by using the actuator as handle
2. Assembly the backrest by aligning the openings on the backrest with the lugs of the hinge on the base unit. (fig.1)
3. Rotate the backrest to slide the actuator pin into the locker to fix the position (for this operation is requested the actuator to be completely close. In contrary case connect the battery on the backrest, press the button until the actuator is in its final position).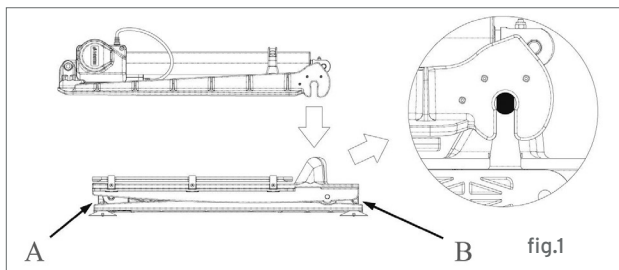

- A - Front handle  
B - Rear handle

**9. BEFORE EACH USE**

- Check the correct device assembly, in particular check if the backrest is duly locked with unit base and the headrest is duly fixed with the block
- Check the mechanical parts wear, as long as the safety for things and people is guaranteed.
- Check the suction pads condition and their adhesion with bathtub

**10. HOW TO USE**


**WARNING!**


- The battery has to be charged of at least 8 hours before its first use.
- We suggest to recharge them after each use

**10.1 Install the bath lift**


1. Always install the bath lift to the bathtub in two sections: backrest and unit base.
2. Fold the side flippers onto the seat
3. Fit your bath lift into the bathtub as shown in fig.2 (install the longitudinal direction of the bath lift parallel with the longitudinal direction of bathtub)
4. Fix the device by pressing the four powerful suction pads.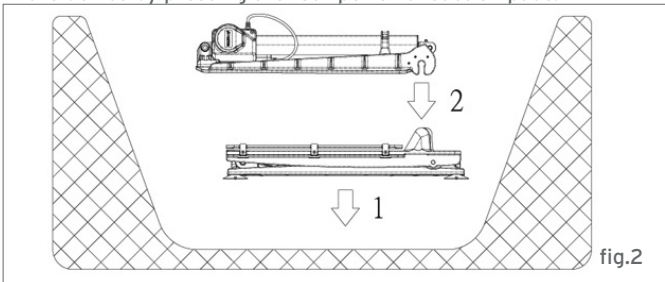


5. Before every use, make sure:
  - a. The bathtub is clean, and free from debris. Check to confirm effective adhesion of the suction pads. The suction pads will not adhere to non-slip or texture surfaces. Be sure to avoid these kind of surface
  - b. Allow sufficient space behind the bath lift for the backrest to recline. Ensure no obstacles will impede the bath lift, such as taps, overflows, and soap dishes
  - c. Fitting guide straps- where bathtub grasps bars are present. The bath lift can be damaged if guide straps are not fitted to the bathtub grasp bars correctly. (fig.3)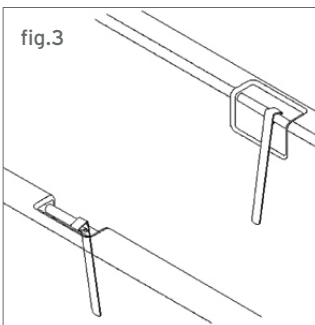

6. Follow the procedures described in section "ASSEMBLY" to reassemble the backrest
7. Make sure that the actuator is in its shortest position. Rotate the actuator to set the backrest to its final reclined position as shown in fig.4, and lock the roller shaft into the actuator locker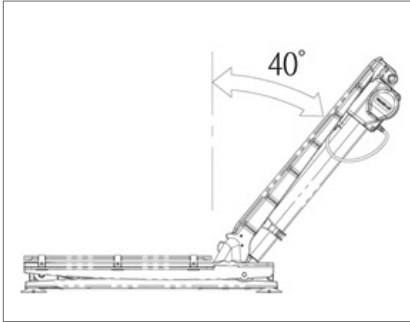

8. Align and slide the battery pack onto the battery seat on the backrest as shown in fig.5 to connect the battery pack. Lift the clip on the lower portion of the battery pack and pull it away from the battery seat to disconnect.

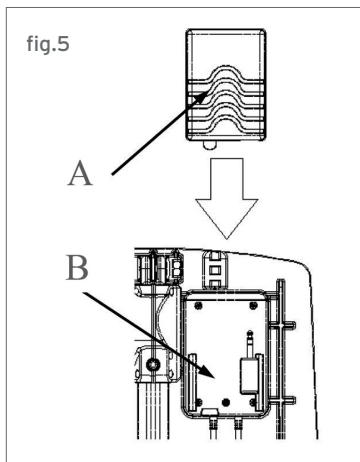A - Battery pack  
B - Battery pack compartment

**WARNING!**

- Do not connect the battery pack to the bath lift until it is assembled and is set to its reclined position in the bath
- The battery pack **MUST** be fully charged before use

## 10.2 Correct seating

### WARNING!


- Before using the bath lift consider means of requesting help if needed
- Seek advice from a qualified person such as an occupational therapist

1. Lift the bath lift until the seat reaches the bath edges
2. Facing outwards, sit on the bath lift side panel and then move to the seat area
3. Slowly lift your legs separately over into the bathtub, ease yourself into the central area of the seat
4. Face forwards and have your legs and feet extended. Always keep your hands away from the seat, folding side panels and mechanism, when the bath lift is operating.
5. Lower the bath lift in order to allow the partial immersion
6. To get out the bathtub follow the procedure inverse

**Note:** in case of purchasing of swivel base (RP901), fit it on the seat with slide flipper outward of the bathtub. Use the swivel unit as an aid to get in and out the bathtub

## 10.3 Use the handled controller

1. Make sure that the fully batteries are correctly installed
2. Press the DOWN button to lower and the UP button to raise the bath lift. If you wish to recline the backrest further, when the seat has reached its lower position continue to press the down button. Once the desired angle is reached, release the button.
3. To raise the backrest, press UP button. Continue pressing the UP button to raise the seat to its original position- level with the bathtub edges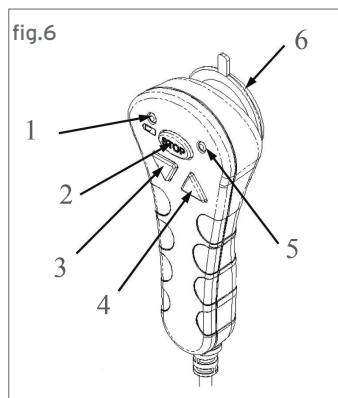

1. Low power indicator
2. Emergency - STOP button
3. "Down" button
4. "Up" button
5. STOP indicator
6. Suction pad

**Note:** The handled controller can be fitted to a smooth surface by using the suction pads underneath. To release, just pull the edge rim or side strap of the suction pads. The handled controller has a built in safety feature. If the battery do not has sufficient power to lower and then raise the bath lift, the unit will not descend. A red warning light will blink on the handled controller to indicate low battery power, wen the user press down or up button.

**EMERGENCY STOP**


If the handled controller buttons do not respond while the bath lift needs to be stopped, press the emergency stop button on the handled controller. A red led will be illuminated and the bath lift will stop immediately. To operate the bath lift, press this button again to release it

**10.4 Removing the bath lift from the bathtub**

1. Let the water leak out of the bathtub
2. Always remove the bath lift from the bathtub in two sections: backrest and unit base. Gently fold the backrest over until it touches the seat in order to unlock the actuator from its locker
3. Raise it again straight and you can lift it away
4. Fold the side flippers onto the seat
5. Lift the bath lift with the front handle and pull to release the front suction pads. Gently tilt the bath lift by the front handle  
**DO NOT RELEASE STRAP OF THIS HANDLE**
6. Lift the rear side of the bath lift by pulling the rear handle.
7. Then lift the bath lift away from the bathtub with two hands.

**10.5 Recharge the battery****WARNING!**

- When recharging always use a socket away from the bathroom and recharge in a dry environment
- The batteries are rechargeable and replaceable
- Disposal of battery pack: please conform to local regulations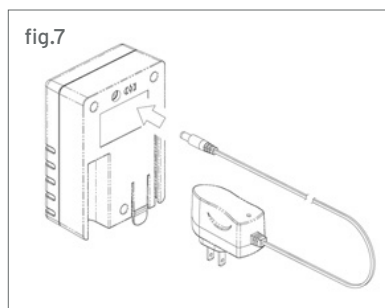

Follow the following procedures to recharge your battery pack:

1. Remove the battery pack from the bath lift
2. Plug the charger into the charging socket on the battery pack, fig.7
3. Plug the charger into a suitable power supply (220 VAC/50 Hz)

**Note:**

- For new battery, the first charge shall be no less than 8 hours
- Recharge the battery as soon as possible after each use. DO NOT overcharge it for more than 24 hours
- Disconnect the battery pack from the bath lift while not in use. Store it in a cool dry place
- Use only battery charger which comes with your bath lift
- To increase the battery life, recharge it at least once a month if you don't use the device for extended period of time

## 11. MAINTENANCE

All the Moretti devices are checked carefully and supplied with CE mark, once launched on the market. For patient and physician's safety we recommend to be checked by the manufacturer once a year, to certify the suitability. Make a check in particular for unit base, for the lift device and its hinges, controls, safety device, person support device, and each trial which should be useful. Every check should include trials with the maximum charge. In case of repair only original spare parts can be used.

## 12. CLEANING

- It's recommended to clean the bath lift after use (with warm water) in order to maintain its good hygienic condition, and to avoid accumulation, which can cause inconveniences.
- Do not flood the actuator
- Do not use acetic acid detergents
- The cover can be clean in washing machine at 60°
- Do not iron
- Do not use dryer

## 13. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

### 13.1 Disposal of batteries and electronic components

#### **Disposal instructions, as required by directive 2012/19/UE**

At the end of its service life, the product may not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted disposal centre or to the reseller, who provides this service. Sorted waste disposal reduces the environmental and health consequences of improper disposal and enables its materials to be recycled, this resulting in energy and resource savings. To highlight the requirement for sorted disposal of electromedical equipment, the product bears the barred bin symbol.

### 13.2 Treatment of exhausted batteries (Directive 2006/66/EC):



This symbol indicates that the batteries may not be treated like common household waste. Make sure the batteries are disposed of properly; this prevents the possible environmental and health consequences of incorrect disposal. Recycling helps to conserve natural resources. Deliver exhausted batteries to a proper recycling centre. For more detailed information regarding the disposal of exhausted batteries or the product itself, contact the municipality, local waste disposal service or the reseller from whom you purchased the device.

## 14. TROUBLE SHOOTING

SYMPTOM	PROBABLE CAUSE	SOLUTION
	Main loss power	
No motor sound/ the bath lift doesn't move	<ol style="list-style-type: none"> <li>1. The battery pack is not installed properly</li> <li>2. The battery is flat</li> <li>3. The contacts are dirty</li> <li>4. The emergency stop button has been pressed</li> <li>5. The cable is damaged</li> </ol>	<ol style="list-style-type: none"> <li>1. Re-install the battery pack</li> <li>2. Recharge the battery</li> <li>3. Clean the contacts</li> <li>4. Press the emergency stop button again to release it</li> <li>5. Call dealer to replace the cable</li> </ol>
Red light illuminated. The bath lift can be raised but not lowered	<ol style="list-style-type: none"> <li>1. Battery low. Low-voltage protection is activated</li> <li>2. Internal repairs required</li> </ol>	<ol style="list-style-type: none"> <li>1. Recharge the battery for at least 8 hours</li> <li>2. Call dealer for service</li> </ol>
The bath lift stops while being raised	<ol style="list-style-type: none"> <li>1. The load is excessive</li> <li>2. Internal repairs required</li> </ol>	<ol style="list-style-type: none"> <li>1. Check to see if the Max load capacity is exceeded</li> <li>2. Call dealer for service</li> </ol>
The battery cannot be recharged	<ol style="list-style-type: none"> <li>1. Internal repairs required</li> <li>2. The battery is faulty</li> </ol>	<ol style="list-style-type: none"> <li>1. Replace the battery</li> <li>2. Call the dealer for service</li> </ol>
The bath lift slips	<ol style="list-style-type: none"> <li>1. The suction pads are old or damaged</li> </ol>	<ol style="list-style-type: none"> <li>1. Replace the damaged suction pads</li> </ol>

## 15. TECHNICAL FEATURES

## 15.1 Dimension

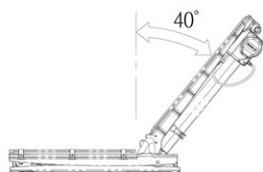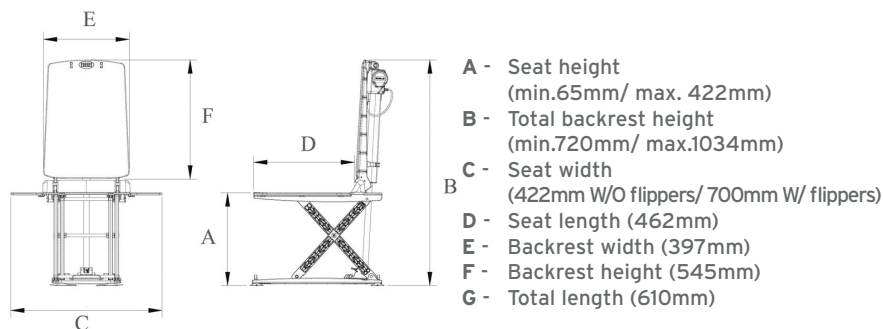**Note:** Max. Backrest tilt angle 40°/ mat thickness 3mm

## 15.2 Technical specifications

<b>Supported weight</b>	136kg
<b>Total weight (hand control included)</b>	14kg
<b>Seat weight</b>	8.5kg
<b>Backrest weight</b>	5kg
<b>Hand control weight</b>	0.5kg
<b>Weight package included</b>	17kg
<b>Battery life</b>	4.8A (12V DC)
<b>Battery</b>	Ni-MH/14.4/1300mAh
<b>Battery and control panel's fuse</b>	10A/250V
<b>Necessary force to use the hand controller</b>	5N
<b>AC/DC adapter (only use original adapter)</b>	Model: MR101 Input: 100-240 Vac 50-60 Hz (0.2A) Output: 17.6V DC 170 mA Hand controller: IP67 Actuator: IP66
<b>Environmental protection</b>	<45dB(A)
<b>Sound level</b>	Max 10%= one bath/15min.
<b>Active actuator cycle</b>	10°-40°C
<b>Environmental operative temperature</b>	860-106hpa (12.5-15.4psi)
<b>Hydrostatic pressure</b>	Dry (up to 95% non-condense)
<b>Storage</b>	Airy area -20°C-60°C
<b>Materials</b>	PVC and polyester: anti slide mat
<b>Rules</b>	IEC EN60601-1 EN ISO 10535

## 16. INSTRUCTION REGARDING EMC

1. The bath lift is in compliance with EN 60601-1-2
2. The bath lift should not be use adjacent to or stacked with other equipment
3. The bath lift is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies building used for domestic purpose
4. Mobile RF communications equipment (transmitters) can affected the bath lift. However, the user can help prevent electromagnetic interferences by maintaining distance as recommended in the technical description.

## TECHNICAL DESCRIPTION REGARDING ELECTROMAGNETIC COMPATIBILITY

### Guidance and manufacturer's declaration - electromagnetic emissions

The bath lift with the model with the model no.A903 is intended for use in the electromagnetic environment specified below. The customer or the user of the above mentioned product should assure that it is used in such environment.

EMISSION TEST	COMPLIANCE	ELECTROMAGNETIC ENVIRONMENT- GUIDANCE
RF emissions CISPR 11/EN 550011	Group 1	The product use RF Energy only for its internal function. Therefore ,its RF emissions are very low and not likely to cause any interference in nearby electronic equipment. The RS900 is suitable for use in all establishments and those directly connected to the public low-voltage power supply network that supplies building used for domestic purpose
RF emissions CISPR 11/EN 550011	Class B	
Harmonics emissions IEC61000-3-2/EN 61000-3-2	Class A	
Voltage fluctuation/ flicker emissions IEC 61000-3-2/ EN61000-3-3	Complies	

#### Guidance and manufacture's declaration - electromagnetic immunity

The bath lift RS900 is intended for use in the electromagnetic environment specified below. The customer or the user of the above mentioned product should assure that it is used in such environment.

Immunity test	IEC/EN 60601 TEST LEVEL	Compliance level	Electromagnetic environment - guidance
<b>Electrostatic discharge(ESD)</b> IEC 61000-4-2	+/-6KV contact +/-8KV air	+/-6KV contact	The RS900 is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that supplies building used for domestic purpose
<b>Electrical fast transient/burst</b> IEC61000-4-4	+/-2KV for power supply lines +/-1KV for input/output lines	+/-2KV for power supply lines	
<b>Surge IEC</b> 61000-4-5	+/-1KV for L-N +/-2KV for lines to earth	+/-1KV for L-N	
<b>Voltage dip and interruption</b> IEC 61000-4-11	<5% UT(>95% dip in UT) for 0.5 cycle 40%UT (60%dip in UT) for 5 cycles <5% UT (>95%dip in UT) for 5 cycles	<5% UT (>95% dip in UT) for 0.5 cycle <5% UT (>95%dip in UT) for 5 cycles	
<b>Power frequency magnetic field</b> IEC61000-4-8	3 A/m	N/A Not contain component sensitive to magnetic field	

**Note** - UT is the AC mains voltage prior to application of the test level.

**Guidance and manufacturer's declaration-electromagnetic immunity**

The bath lift RS900 is intended for use in the electromagnetic environment specified below. The customer or the user of the above mentioned product should assure that it is used in such environment.

Immunity test	IEC/EN 60601-1-2 Test level	Compliance level	Electromagnetic environment- guidance
<b>Conducted RF</b> <b>IEC 61000-4-6</b>	3 Vrms 150KHZ to 80Mhz	3Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the above models, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  Recommended separation distance: $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P}$ 80 MHz a 80 MHzù $d = 2.3 \sqrt{800 P}$ a 2.5 GHz
<b>Radiated RF</b> <b>IEC 61000-4-3</b>	3V/m 80Mhz to2.5Ghz	3V/m, 10V/m	Where P is the maximum output rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters. (m) Field strength from fixed RF transmitters, as determined by electromagnetic site survey, <b>(NOTE 3)</b> should be less than the compliance level in each frequency range. <b>(NOTE 4)</b> . Interference may occur in the vicinity of equipment marked symbol:

**Note 1** - at 800Mhz and 800Mhz, the higher frequency range applies

**Note 2** - these guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structure, objects and people.

**Note 3** - field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast cannot be predicted theoretically with accuracy.

To assess the electromagnetic environment due to fixed RF transmitter, an electromagnetic site survey should be considered. If the measured field strength in the location in which the above models are used exceeds the applicable RF compliance level above, the above models should be observed to verify normal operation. If abnormal performance is observed additional measures may be necessary such as re-orienting or relocating the above models.

**Note 4** - over the frequency range 150KHZ to 80MHZ, field strength should be less than 3V/m.

**Recommended separation distance between portable and mobile RF communications equipment and the bath lift with RS900 model.**

The bath lift RS900 is intended for use in electromagnetic environment in which the radiated RF disturbances are controlled. Mobile RF communications equipment (transmitters) can affect the bath lift. However, the customer or the user of the above mentioned product can help prevent electromagnetic interferences by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the bath lift as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 KHz to 80MHz d=1.2P	80MHz to 800MHz d=1.2P	800MHz to 2.5 GHz d=2.3 P
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitter at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**Note 1** - at 800Mhz and 800MHz, the higher frequency range applies

**Note 2** - these guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structure, objects and people.

## 17. SPARE PARTS/ACCESSORY

For spare parts only make reference to Moretti master catalogue

## 18. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use.

Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

### **18.1 Rechargeable battery warranty (if included)**

The original and rechargeable batteries are under a 90 days warranty referring to performances and a 6 month warranty referring to production defects or according to the law requests.

If charged batteries are not used for more than 3 months, the warranty lapses.

If law batteries are not used for more than 3 days, the warranty lapses.

### **18.2 Warranty repair**

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

### **18.3 Repair not cover from warranty**

A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

### **18.4 Non-defective device**

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

### **18.5 Spare parts**

The original Moretti spare parts are guaranteed for 6 months as from the receiving date.

## **19. EXEMPT CLAUSES**

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be fixed by according to the law is limited by the warranty period, in the limits of law. Some states or countries don't allow to limit the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification

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