MA ST327\_ST328 01A\_ITA\_09\_2022

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## CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

### 1. CODES

**ST327** - PVC water cushion with central hole

**ST328** - PVC water cushion

### 2. INTRODUCTION

Thank you for purchasing a **LEVITAS** water cushion by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly

**NOTE check all parts for shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions**

### 3. INTENDED USE

The water cushion have been designed for bed sore care

- The use of the following device for purposes other than those defined in this manual is prohibited.
- Moretti S.p.A. declines any responsibility for damages resulting from improper use of the manual.
- The manufacturer reserves the right to make changes to the device and to the following manual without prior notice in order to improve its features.

### 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of CUSHIONS FOR PATIENTS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

- For proper use of the device please refer to the following manual carefully
- Always consult your doctor or therapist for proper use of the device
- Keep the item away from heat sources
- The service life of the device is determined by the wear of parts that cannot be repaired and/or replaced
- DO NOT allow children to play on or operate the cushion
- In case of extended use, is suggested to empty the cushion
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

### WARNING!

Do not use the device with your skin in direct contact with its surface.

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device


Conditions of disposal

## 7. GENERAL DESCRIPTION

The water cushion is made of electrosoldered PVC, to guarantee maximum strength. It has interior bellows to avoid wave-like motion of water and a refilling tube.

**ST327**  
PVC water cushion with central hole

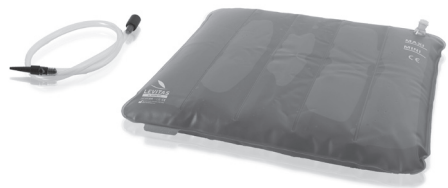**ST328**  
PVC water cushion

## 8. BEFORE USE

- Inspect the frame for any damage in order to guarantee a safe use for people and things
- Verify the filling valve is duly closed

## 9. HOW TO USE

1. Insert the refilling tube into the valve
2. Start pouring water with the help of a funnel
3. Stop when the desired quantity is reached
4. Close correctly the filling valve

## 10. MAINTENANCE

The **LEVITAS** devices by MORETTI S.p.A. are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories

## 11. CLEANING AND DISINFECTION

### - Cleaning

Use a damp cloth and mild soap. Then dry thoroughly before use

### - Disinfection

If you need to disinfect the device use a common disinfectant cleaner

**N.B. Never use acids, alkalis or solvents such as acetone**

## 12. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the cushion in the appropriate disposal areas for recycling.

## 13. TECHNICAL FEATURES

- |                          |   |
|--------------------------|---|
| - <b>Material</b>        | k80 PVC - non-toxic                     |
| - <b>Thickness</b>       | 4/10 for paramedical use                |
| - <b>Dimensions</b>      | 45x50 cm                                |
| - <b>Weight</b>          | <b>ST327</b> 400gr - <b>ST328</b> 450gr |
| - <b>Max user weight</b> | 90 kg                                   |

#### 14. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

#### 15. CLAUSOLE ESONERATIVE

Moretti does not offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

**WARRANTY CERTIFICATE**

ENGLISH

**Product** \_\_\_\_\_

**Purchased on (date)** \_\_\_\_\_

**Retailer** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**Sold to** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

**MADE IN ITALY**

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11  
Fax. +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)