MA ST625-725-728-729-732 02 A_10-2020

MATERASSI IN POLIURETANO ESPANSO

MANUALE D'ISTRUZIONE

INDICE

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Class I medical device

 REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT
 AND OF THE COUNCIL of 5 April 2017 on medical devices

1. CODES

ST625/x Mattress in polyurethane foam with a truncated pyramid-shaped surface

Mattresses certified in fire reaction class 1 IM

ST725/x Mattress in fire-retardant polyurethane foam with a truncated pyramid-shaped surface

ST728/x Mattress in fire-retardant composite polyurethane foam with a textured surface

ST729/x Mattress in fire-retardant composite polyurethane foam with an ovoid-shaped surface

ST732/x Mattress in fire-retardant composite polyurethane foam with a truncated pyramid-shaped surface

2. INTRODUCTION

Thank you for choosing an anti-decubitus device in the **LEVITAS** range by Moretti. Moretti anti-decubitus devices have been designed and manufactured to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this entire manual carefully before you use the mattress. Should you have any queries, please contact your dealer for suitable advice and assistance.

NOTE, CAUTION AND WARNING

NOTE: indicate several recommendations.

CAUTION: indicate the correct operating or maintenance procedures, in order to avoid damage to the device, the equipment or other items.

WARNING: draw attention to a potential hazard that requires correct procedures or practices to avoid injury.

3. INTENDED USE

The **LEVITAS** anti-decubitus mattresses by Moretti are intended to prevent the bedsores that can occur in any situation requiring lengthy hospitalisation and/or being laid up at home.

WARNING!

- Do not use the product for any purpose other than that specified in this manual
- The device must be installed by personnel qualified to perform general nursing procedures and who have received appropriate training in the prevention and treatment of bedsores
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual
- The manufacturer reserves the right to modify the device and this manual without prior notice, in order to improve its features

4. EU DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products made and sold by MORETTI SpA in the POLYURETHANE MATTRESSES product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES.

To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT INSTRUMENTS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATIONS.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

5. GENERAL PRECAUTIONS

- Please consult this manual carefully for correct use of the device
- Always consult your doctor or therapist for correct use of the device
- Use by one user only is recommended
- Use and store in the horizontal position
- Always sanitise the device before any reuse
- The devices are extremely sensitive to air and light
- Any fading of the colour does not alter the efficacy of the mattress
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts
- Store and use the device away from open flames and sources of heat
- Always take care when children are present
- Do not place the patient in direct contact with the mattress. Always use a full cover
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

6. SYMBOLS USED

	Product code
	Unique device identification
	CE marking
	Manufacturer
	Batch Lot
	Read the instruction manual

Medical Device

Disposal conditions

7. GENERAL DESCRIPTION

7.1 Mattresses in non-fire retardant polyurethane foam ST625/X

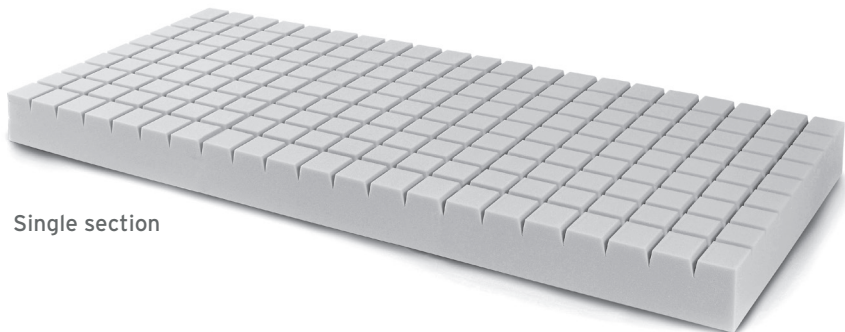
Mattress with a truncated pyramid-shaped surface

The ST625/X mattress is manufactured in standard open-cell micro-cellular polyurethane foam. Truncated pyramid shape, density 25 Kg/m³. Not covered.


- Produced at origin without the use of C.F.C (chlorofluorocarbons)
- Latex-free
- The mattress has been treated to prevent the formation of mites, bacteria, mould and microbes and to guarantee chemical protection against odours

HOW TO USE

Position the device on the bed with the shaped part turned towards the user. It is recommended to place a further lining between the device and the user.

Single section

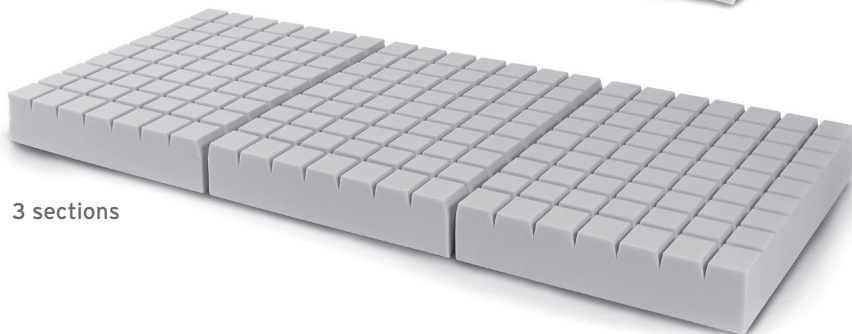3 sections

7.2 Mattresses in Class 1 IM fire-retardant polyurethane foam ST725/X

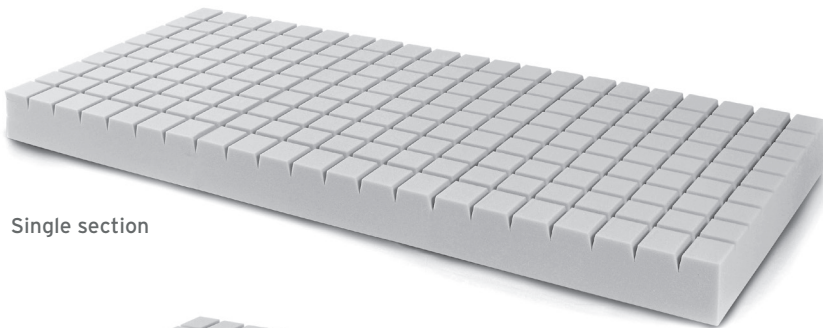
CLASS 1 IM fire-retardant mattress with a truncated pyramid-shaped surface

The ST725/X mattress is manufactured in standard open-cell micro-cellular polyurethane foam, specifically designed to meet the numerous international standards that require a high fire resistance and a low smoke opacity and toxicity level. Truncated pyramid shape, density 30 Kg/m³. Not covered.


- Produced at origin without the use of C.F.C (chlorofluorocarbons)
- Latex-free
- **Certified and compliant with standards UNI 9175 and UNI 9175/FA1 - FIRE REACTION CLASS 1 IM**
- Compliant with smoke optical density and analysis control standard: AIRBUS ATS 1000.001
- The mattress has been treated to prevent the formation of mites, bacteria, mould and microbes and to guarantee chemical protection against odours

HOW TO USE

Position the device on the bed with the shaped part turned towards the user. It is recommended to place a further lining between the device and the user.

Single section

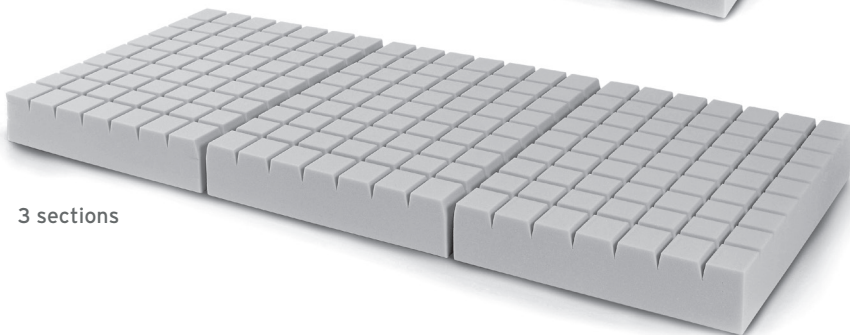3 sections

ST729/X

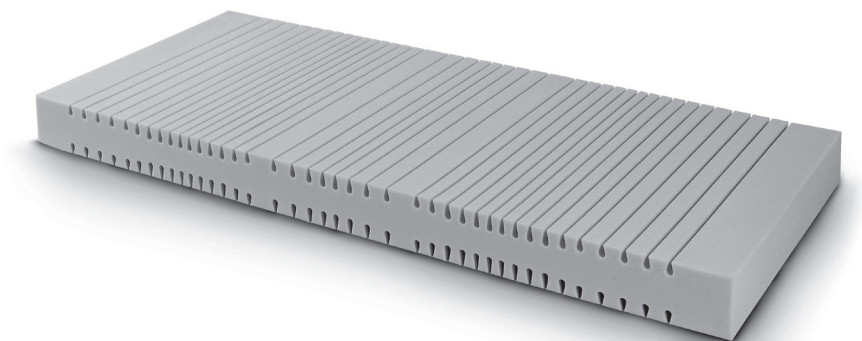
CLASS 1 IM fire-retardant mattress with a composite ovoid-shaped surface

The ST729/X mattress is manufactured in standard open-cell micro-cellular polyurethane foam, specifically designed to meet the numerous international standards that require a high fire resistance and a low smoke opacity and toxicity level. Ovoid shape, density 30 Kg/m³. Usable on both sides. Not covered.

- Produced at origin without the use of C.F.C (chlorofluorocarbons)
- Latex-free
- **Certified and compliant with standards UNI 9175 and UNI 9175/FA1 - FIRE REACTION CLASS 1 IM**
- Compliant with smoke optical density and analysis control standard: AIRBUS ATS 1000.001
- The mattress has been treated to prevent the formation of mites, bacteria, mould and microbes and to guarantee chemical protection against odours

HOW TO USE

Position the device on the bed. The mattress can be used on both sides. It is recommended to place a further lining between the device and the user.

ST728/X

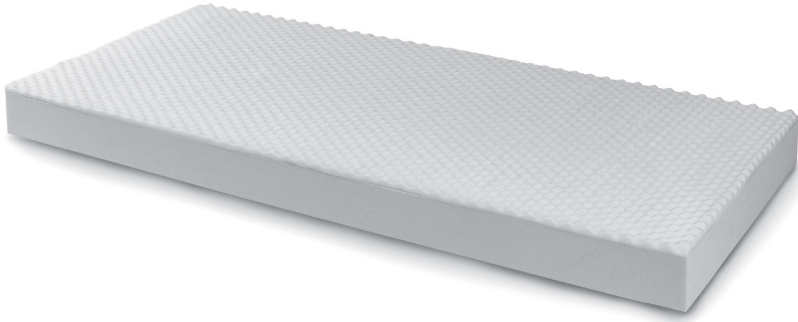
CLASS 1 IM fire-retardant mattress with a composite textured surface

The weight-bearing structure of the ST728/X mattress is manufactured in standard open-cell micro-cellular polyurethane foam, specifically designed to meet the numerous international standards that require a high fire resistance and a low smoke opacity and toxicity level, with density of 30 Kg/m³. The resting surface is made of slow-return, textured memory foam with an extremely low elastic yield point that adapts to the shape of the body, thus avoiding the pressure associated with normal foams. Not covered.

- Produced at origin without the use of C.F.C (chlorofluorocarbons)
- Latex-free
- **Certified and compliant with standards UNI 9175 and UNI 9175/FA1 - FIRE REACTION CLASS 1 IM**
- Compliant with smoke optical density and analysis control standard: AIRBUS ATS 1000.001
- The mattress has been treated to prevent the formation of mites, bacteria, mould and microbes and to guarantee chemical protection against odours

HOW TO USE

Position the device on the bed with the shaped part turned towards the user. It is recommended to place a further lining between the device and the user. Identify the smooth part, which is smaller than the other two: this is the area on which to place the pillow.

ST732/X

CLASS 1 IM fire-retardant mattress with a composite truncated pyramid-shaped surface

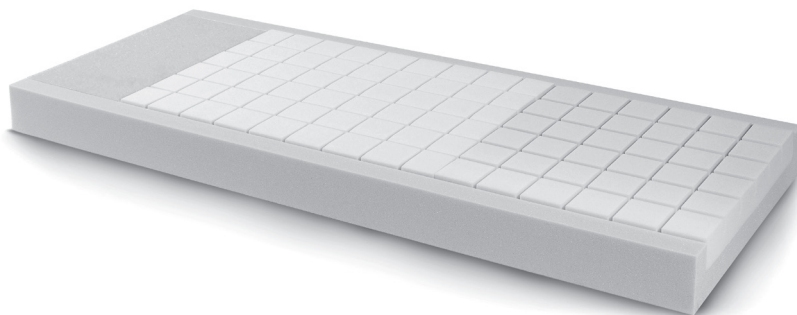
The ST732/X mattress is manufactured in standard open-cell micro-cellular polyurethane foam, specifically designed to meet the numerous international standards that require a high fire resistance and a low smoke opacity and toxicity level. Formed of a weight-bearing structure with a density of 40 Kg/m³. The detachable resting surface is made of slow-return, truncated pyramid-shaped memory foam with an extremely low elastic yield point that adapts to the shape of the body, thus avoiding the pressure associated with normal foams.

1. Head area: smooth surface, density 30 Kg/m³
2. Trunk and pelvis area: truncated pyramid shape, density 60 Kg/m³
3. Feet area: truncated pyramid shape, density 50 Kg/m³

- Produced at origin without the use of C.F.C (chlorofluorocarbons)
- Latex-free
- **Certified and compliant with standards UNI 9175 and UNI 9175/FA1 - FIRE REACTION CLASS 1 IM**
- Compliant with smoke optical density and analysis control standard: AIRBUS ATS 1000.001
- The mattress has been treated to prevent the formation of mites, bacteria, mould and microbes and to guarantee chemical protection against odours

HOW TO USE

Position the device on the bed with the shaped part turned towards the user. It is recommended to place a further lining between the device and the user. Identify the smooth part, which is smaller than the other two: this is the area on which to place the pillow.

8. CLEANING AND DISINFECTION

WARNING!

Considering the large number of variables present in these processes, always carry out a preliminary test before each treatment. The indications below are to be considered as indicative only.

8.1 Washing

Detergents containing oxidising and bleaching agents, such as hypochlorite and perborate, must be used diluted in water and in the concentrations normally used for cotton linen. The article can be washed with water and normal detergent, but always roll it up and tie it before placing in the washing machine or spin dryer.

8.2 Wringing

Use either a wrangle (1/4 gap between the rollers) or a low-speed spin cycle.

8.3 Drying

Use a forced-air heater (maximum temperature of 70°C). Alternatively, dry the mattress in a room, away from sunlight and in air currents, although the drying time will obviously be much longer in this case.

8.4 Disinfection

Follow the instructions of the supplier of the disinfecting system.

- **Spraying the surface with disinfectant solutions**

Indicatively, after being sprayed with common disinfectants, the foam can be dried with a normal forced-air heater, at a maximum temperature of 70°C.

- **On-site insufflation with gases, vapours or aerosols**

The high degree of permeability and the absence of thickening or felting means that the mattress is more suitable than any other type to efficient insufflation with disinfectants.

8.5 Immersion in solutions

If quaternary ammonium compounds are used, do not rinse or spin, but simply drip-dry or dry as described in the paragraph on "washing": this ensures that the residue of the quaternary ammonium compounds that remains is sufficient to maintain a lengthier bactericidal effect.

8.6 Treatment in an autoclave

Treatment in an autoclave with saturated steam (DIN 13014), at a maximum temperature of 105° C with saturated steam at 1.25 bar, for a maximum of ten minutes.

9. STORAGE

Keep in a cool, dry place, away from damp environments.

10. MAINTENANCE

LEVITAS devices manufactured by Moretti are CE marked and carefully checked before being commercialised.

For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original replacement parts and accessories for repairs.

11. DISPOSAL CONDITIONS

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

12. TECHNICAL SPECIFICATIONS

12.1 Mattresses in non-fire retardant polyurethane foam

SINGLE SECTION			
ST625/1		ST625/2	
Dimensions	190x85x14 cm	Dimensions	190x120x14 cm
Weight	5.7 kg	Weight	7.9 kg
Max. recommended weight	150 kg	Max. recommended weight	150 kg
Density	25 kg/m ³	Density	25 kg/m ³

SINGLE SECTION	
ST625/6	
Dimensions	190x85x10 cm
Weight	4 kg
Max. recommended weight	100 kg
Density	25 kg/m ³

3 SECTIONS			
ST625/3		ST625/4	
Dimensions	190x85x12.5 cm	Dimensions	190x85x14 cm
Weight	5.3 kg	Weight	5.7 kg
Max. recommended weight	120 kg	Max. recommended weight	150 kg
Density	25 kg/m ³	Density	25 kg/m ³
Section dimensions (3)	63x85 cm	Section dimensions (3)	63x85 cm
ST625/5		ST625/7	
Dimensions	190x120x14 cm	Dimensions	183x85x10 cm
Weight	7.9 kg	Weight	4 kg
Max. recommended weight	150 kg	Max. recommended weight	100 kg
Density	25 kg/m ³	Density	25 kg/m ³
Section dimensions (3)	63x85 cm	Section dimensions (3)	61x85 cm

12.2 Mattresses in Class 1 IM fire-retardant polyurethane foam

SINGLE SECTION			
ST725/1		ST725/2	
Dimensions	190x85x14 cm	Dimensions	190x120x14 cm
Weight	5.8 kg	Weight	8.1 kg
Max. recommended weight	150 kg	Max. recommended weight	150 kg
Density	30 kg/m ³	Density	30 kg/m ³
ST725/6			
Dimensions	190x85x10 cm		
Weight	4.5 kg		
Max. recommended weight	100 kg		
Density	30 kg/m ³		

3 SECTIONS			
ST725/3		ST725/4	
Dimensions	190x85x12.5 cm	Dimensions	190x85x14 cm
Weight	5.5 kg	Weight	5.8 kg
Max. recommended weight	120 kg	Max. recommended weight	150 kg
Density	30 kg/m ³	Density	30 kg/m ³
Section dimensions (3)	63x85 cm	Section dimensions (3)	63x85 cm
ST725/5			
Dimensions	190x120x14 cm		
Weight	8.1 kg		
Max. recommended weight	150 kg		
Density	30 kg/m ³		
Section dimensions (3)	63x120 cm		
ST729/1		ST729/2	
Dimensions	190x85x14 cm	Dimensions	190x120x14 cm
Weight	6.4 kg	Weight	9.6 kg
Max. recommended weight	150 kg	Max. recommended weight	150 kg
Density	30 kg/m ³	Density	30 kg/m ³
ST728/1		ST728/2	
Dimensions	190x85x14 cm	Dimensions	190x120x14 cm
Weight	6.6 kg	Weight	9.8 kg
Max. recommended weight	150 kg	Max. recommended weight	150 kg
Density of the weight-bearing part	30 kg/m ³	Density of the weight-bearing part	30 kg/m ³
Density of the upper part	50 kg/m ³	Density of the upper part	50 kg/m ³
ST732/1			
Dimensions	190x85x14 cm		
Weight	8.5 kg		
Max. recommended weight	150 kg		
Density of the weight-bearing part	40 kg/m ³		
Density of the upper part - Head	30 kg/m ³		
Density of the upper part - Pelvis	60 kg/m ³		
Density of the upper part - Feet	50 kg/m ³		

Mass and dimension of all codes may vary by around $\pm 1\%$

- Produced at origin without the use of C.F.C (chlorofluorocarbons)
- Latex-free
- **Certified and compliant with standards UNI 9175 and UNI 9175/FA1 - FIRE REACTION CLASS 1 IM**
- Compliant with smoke optical density and analysis control standard: AIRBUS ATS 1000.001
- The mattress has been treated to prevent the formation of mites, bacteria, mould and microbes and to guarantee chemical protection against odours

13. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject to the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual. Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

14. REPAIRS

14.1 Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

14.2 Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

14.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

15. SPARE PARTS

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

16. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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