ITALIANO

MA EM201 02 A_ITA_06-2021

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INDICE

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Venduto a _____

Via _____ **Località** _____

JIANGSU RIXIN MEDICAL EQUIPMENT CO.,LTD.

No.427 Yangjin Road, Jinfeng, Zhangjiagang, Jiangsu Province, CHINA

SUNGO EUROPE B.V.

Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands.

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MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

EASYRED®

MA EM201 02 A_ENG_06-2021

BODY SPLINT

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODE PAG.3

2. INTRODUCTION..... PAG.3

3. INTENDED USE..... PAG.3

4. GENERAL WARNINGS PAG.3

5. SYMBOLS PAG.4

6. BEFORE USE..... PAG.4

7. WARNINGS FOR USE PAG.4

8. HOW TO USE..... PAG.4

9. MAINTENANCE..... PAG.5

10. CLEANING AND DISINFECTION PAG.5

11. CONDITIONS OF DISPOSAL PAG.5

12. TECHNICAL FEATURES..... PAG.5

13. WARRANTY PAG.5

 13.1 Repairs..... pag. 5

 13.2 Spare parts pag. 6

 13.3 Exempt clauses pag. 6

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODE

BS-01/EM201 - Body splint

2. INTRODUCTION

Thank you for purchasing a EASYRED by Moretti S.p.A. device. Made of vinyl coated nylon. Water resistant. The built-in bamboo supports make the product even more durable. It is used for the spinal immobilization of patients in emergency situations. Complete with carrying bag. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

Device for the immobilization and transportation of patients in emergency.

WARNINGS!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

This product is intended for exclusive use by professional and qualified staff, duly trained for emergency care. It is always recommended to follow the procedures designed for emergency response. Improper use of the device can cause damage to the patient who is cared for in an emergency.

4. GENERAL WARNINGS

- READ carefully the entire manual before using the device .
- The product should be used only by qualified personnel
- Make sure that the product is delivered complete and keep it packed away from heat sources
- SERVICE LIFE- the device use limit is defined by the wear of the parts
- Store in a dry place
- DO NOT allow children to play or operate the device
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

5. SYMBOLS

	Product code
	CE mark
	Manufacturer
	Production dates
	Batch Lot
	Attention
	Keep dry
	Medical Device
	Name and address of the European Representative
	Expiration date
	Read the instruction manual
	Not sterile
	Humidity limit
	Conditions of disposal

6. BEFORE USE

- Always check the state of wear of the parts in such a way as to ensure it is used in complete safety for persons and property.

7. WARNINGS FOR USE

- Take special care in fractures.
- Periodically check its sturdiness, the stability and the rubber tips are in good conditions. Replace them once worn out .
- In case of doubts about use, adjustment or maintenance, contact your dealer
- Keep it away from children to avoid any possible risks.

8. HOW TO USE

The product should be handled by at least 3 players simultaneously. An operator must stand behind the patient , extend the immobilizer and bring the part without buckles in the back of the patient. The other two players must stand at the side of the patient and place the two immobilizer sides under his armpits , adjusting the position for adhere the product to the patient's back. Pull off straps (red, yellow , green) on both sides of the immobilizer, connect belts buckles of its color, pick the three handles (blacks) , leaving the upper edges of the left and right well located in the armpit of the patient, and adjust the length of the 3 belts. Place the lower straps (black) around the patient's legs and fasten. Put the neck cushion between the patient's head and the

immobilizer. Attach the two straps to the head on the forehead and on the chin of the patient separately, with the smooth side of the belt facing towards the patient's head

The device is used for fixed adult.

1. Put the splint at the back of the patient when the patient is sitting. Close up the fixing pieces for body and connect them through fixing belts and buckles. Fold and fix the rest belts. If the patient is injured in cervical, first fix the patient with a cervical collar, then wrap up patient's head by the fixing pieces (use together with cushion if needed, put the cushion at the back of the patient before use), fix patient's forehead and chin by fixing belts.
2. After use, clean, disinfect and dry the splint, fold it according to the way before folding, then put into storage.

9. MAINTENANCE

The EASYRED devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

10. CLEANING AND DISINFECTION

Use water and a mild disinfectant, dry away from direct sunlight.

Note Never use acids, alkalis or solvents such as acetone

11. CONDITIONS OF DISPOSAL

General conditions of disposal. In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

12. TECHNICAL FEATURES

Code	Dimensions	Weight
BS-01/EM201	82x71 cm	2.5 kg

13. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

13.1 Repairs

Warranty repair: In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

Repair not covered by warranty: A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non defective device: The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer .

13.2 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

13.3 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

WARRANTY CERTIFICATE

ENGLISH

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

JIANGSU RIXIN MEDICAL EQUIPMENT CO.,LTD.

No.427 Yangjin Road, Jinfeng, Zhangjiagang, Jiangsu Province, CHINA

SUNGO EUROPE B.V.

Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands.

Imported and distributed by:

MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MORETTI S.P.A.
Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com