ITALIANO

MA EM151 02 A_ITA_06-2021

BARELLA PIEGHEVOLE

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INDICE

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
Venduto a _____

Via _____ Località _____

**JIANGSU RIXIN MEDICAL EQUIPMENT CO.,LTD.**

No.427 Yangjin Road, Jinfeng, Zhangjiagang, Jiangsu Province, CHINA

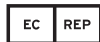**SUNGO EUROPE B.V.**

Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands.

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MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

EASYRED®

MA EM151 02 A_IENG_06-2021

BASKET STRETCHER

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODE	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DESCRIPTION OF THE PARTS	PAG.3
5. GENERAL WARNINGS	PAG.3
6. SYMBOLS	PAG.4
7. BEFORE USE	PAG.4
8. WARNINGS FOR USE.....	PAG.4
9. HOW TO USE	PAG.4
9.1 Stretcher joining	pag. 5
9.2 Dividing the stretcher.....	pag. 5
9.3 Belts positioning.....	pag. 6
9.4 Adjusting the belts	pag. 6
9.5 Mounting the foot rest.....	pag. 6
9.6 Lifting and transport	pag. 6
9.7 Use with harness from helicopter.....	pag. 7
9.8 Requirements for operators and their training	pag. 7
10. MAINTENANCE	PAG.8
10.1 Ordinary maintenance	pag. 8
11. CLEANING AND DISINFECTION	PAG.8
12. STORAGE	PAG.8
13. CONDITIONS OF DISPOSAL	PAG.8
14. WARRANTY	PAG.9
14.1 Repairs	pag. 9
14.2 Spare parts.....	pag. 9
14.3 Exempt clauses.....	pag. 9

 I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODE

YDC-8B1/EM151 Basket Stretcher

2. INTRODUCTION

Thank you for purchasing an EASYRED device by Moretti S.p.A. Widely used in various emergency situations. It is robust and flexible to use, it is a must for all types of aid in special situations. This manual contains some suggestions for proper use of the device of your choice and valuable tips for your safety. You should read carefully all of this manual before using the device. In case of doubt please contact your dealer, who will help and advise you properly.

3. INTENDED USE

Device for the transportation of patients in emergency.

WARNINGS!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

This product is intended for exclusive use by professional and qualified staff, duly trained for emergency care. It is always recommended to follow the procedures designed for emergency response. Improper use of the device can cause damage to the patient who is cared for in an emergency.

4. DESCRIPTION OF THE PARTS

The stretcher is divisible into two parts, which makes it compact and easy to carry and reliable fittings allow the rapid first-aid personnel to operate safely and quickly. Thanks to the harness the supplied stretcher is ideal for lifting and transport by helicopter. It is equipped with adjustable foot rest, safety belts and a mattress.

5. GENERAL WARNINGS

- READ carefully the entire manual before using the device;
- The product should be used only by qualified personnel;
- Make sure that the product is delivered complete and keep it packed away from heat sources;
- SERVICE LIFE- the device use limit is defined by the wear of the parts;
- Store in a dry place;
- DO NOT allow children to play or operate the device.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code
	CE mark
	Manufacturer
	Production dates
	Batch Lot
	Attention
	Keep dry
	Medical Device
	Name and address of the European Representative
	Expiration date
	Read the instruction manual
	Not sterile
	Humidity limit
	Conditions of disposal

7. BEFORE USE

- Always check the state of wear of the parts in such a way as to ensure it is used in complete safety for persons and property.

8. WARNINGS FOR USE

- Use belts when necessary;
- DO NOT use with patient who exceed the weight indicated in the product label and in the technical features chapter of the present manual;
- Periodically check its sturdiness, the stability and the rubber tips are in good conditions. Replace them once worn out;
- In case of doubts about use, adjustment or maintenance, contact your dealer;
- Keep it away from children to avoid any possible risks.

9. HOW TO USE

1. Operators must carefully read this manual, follow the instructions and familiarize yourself with the correct operating procedures and maintenance of the stretcher.
2. In particular circumstances or in case of excessive weight of the patient may need the intervention of additional staff to ensure comfort and safety to the patient. Always inform the patient when adjusting the stretcher. Never leave unattended stretcher with a patient inside.


ATTENTION!



When using the stretcher, follow always the emergency response protocol in force

9.1 Stretcher joining


Overturn the two sides of the stretcher: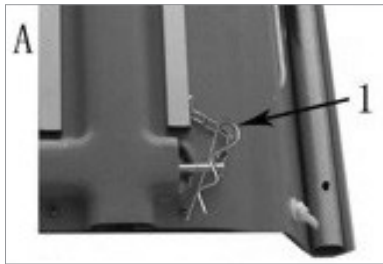

(A) Remove the safety pin (1);
(B) Remove the lower pin (2);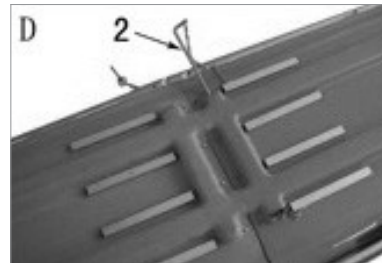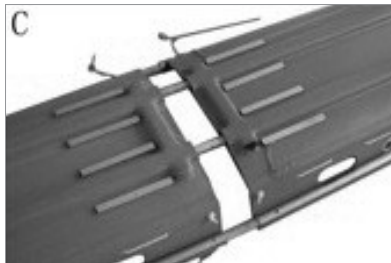

(C) Bring the two sides close to one another so that the pins are aligned with their respective holes.
(D) Fully snap the pins (2) in their holes

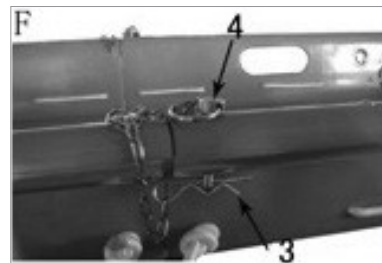(E) Rearrange the safety pin (1)
(F) Reverse the stretcher and mount the upper joint pins (3) (4)

WARNING!

Make sure that the spring pins are properly secured during use

9.2 Dividing the stretcher

- Remove the pins of the upper junction (3);
- Remove the pins (4);
- Turn over the stretcher;
- Remove the safety pins (1);
- Remove the pin junction bottom (2);
- Separate the two sides of the stretcher.

9.3 Belts positioning

WARNING!
Check the condition of belts

The stretcher YDC-8B1/EM151 has four quick release straps to immobilize the patient during transport.

Position of belts: a belt is located in the chest area to hold the chest area, another area of the basin to curb the pelvis and the other two to immobilize his legs.



The 4 safety belts must be fixed on the stretcher as photos G - H.

9.4 Adjusting the belts

Inform the patient whenever it is going to be adjusted.

procedure quick release: **(Photos I)**

Unfasten the seat belts, as shown in the photo sequence I-JK.

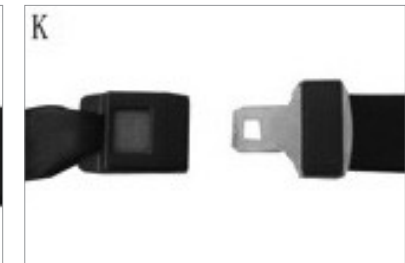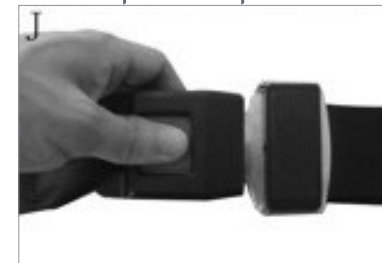9.5 Mounting the foot rest

Fix the foot rest clips to the stretcher as photos L

9.6 Lifting and transport

ATTENTION!
When using the stretcher, follow always the emergency response protocol in force

The movement of the stretcher must be carried out with the immobilizing straps fastened and safety devices inserted.



There are no further details specific to the lifting and transport of the patient on the stretcher.

9.7 Use with harness from helicopter

LOOK OUT!

There are four straps for helicopter transport. Two longer of yellow color to be used for the lower part of the body, 2 shorter of green color to be used for the upper part of the body. (Photo M and N).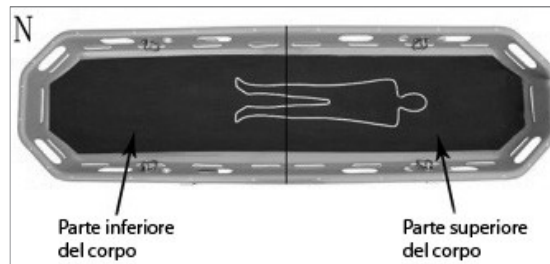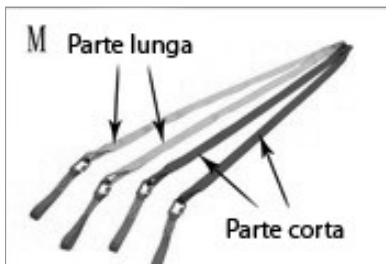

Connect lift belt and hook with stretcher together before lifting.

Photo O, fixed the safety belt through little hole, then (photo P) revolve the button open (photo Q), and then lock the belt and ring together by hook (photo R).

LOOK OUT!

The movement of the stretcher with the patient must be made with the immobilizing straps fastened and safety devices inserted.

9.8 Requirements for operators and their training

Operators must:

- Have a working knowledge of handling patients in emergency situations;
- Be physically able to assist a patient;
- Read the present manual and become familiar with the procedures;
- Be familiar with the stretcher YDC-8B1/EM151 before using it in real situations, with a real patient;
- Follow a training program approved by the head of the formation of the Emergency Medical;
- Do appropriate exercises and be careful to avoid the risk of damage to the spinal cord.

10. MAINTENANCE

The EASYRED devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

10.1 Ordinary maintenance

Check the basket stretcher after each use. The inspection is intended for research of anything that might compromise the integrity and proper functioning of the stretcher, bent, or damaged.

WARNING!

In case of damage, put the stretcher out of service until repairs are made or replace it. Keep a log of maintenance activities carried out.

Every 30 days:

- Clean the rubber parts with water and disinfectant, rinse and dry thoroughly;
- Check all removable parts including hooks;
- Perform a complete check of the stretcher.

11. CLEANING AND DISINFECTION

LOOK OUT! The sanitizing products are corrosive

For the use of disinfecting products scrupulously observe the instructions provided by the manufacturer of the detergent.

When possible, consult the manufacturer for guarantees concerning the degree of corrosivity of the solutions used.

Use water and a mild disinfectant, dry away from direct sunlight.

Note Never use acids, alkalis or solvents such as acetone.

12. STORAGE

In the event that the stretcher must remain unused for a long time, it is recommended to follow the following instructions:

- The stretcher must be packaged to protect it from dust, grease the parts that could be damaged by oxidation or excessive drying;
- The storage area should be dry and not exposed to direct sunlight;
- Do not stack more than six stretchers.

13. CONDITIONS OF DISPOSAL

General conditions of disposal. In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

14. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product , and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair , damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

14.1 Repairs

Warranty repair: In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

Repair not covered by warranty: A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non defective device: The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer .

14.2 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

14.3 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

WARRANTY CERTIFICATE

ENGLISH

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____


Sold to _____

Address _____ **Town/city** _____

JIANGSU RIXIN MEDICAL EQUIPMENT CO.,LTD.

No.427 Yangjin Road, Jinfeng, Zhangjiagang, Jiangsu Province, CHINA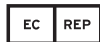

SUNGO EUROPE B.V.

Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands.

Imported and distributed by:

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MORETTI S.P.A.
Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com