KYARA®

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KYARA®

Compressor nebulizer INSTRUCTION MANUAL

INDICE

1. Introduction.....	2
2. Product Identification	2
3. Important Safeguards	3
4. Operating Your Compressor Nebulizer	4
5. Cleaning	4
6. Specifications	5
7. Symbol descriptions.....	5
8. Disposal.....	6
9. Warranty.....	7


1. INTRODUCTION

Thank you for purchasing our compressor nebulizer. The device is a compact medical device designed to effectively deliver doctor-prescribed medication along the bronchial cavities. With proper care and use, the device will provide many years of effective treatment.

This device is designed to treat asthma, allergies and other respiratory problems. It releases a stream of air through the clean aerosol tube, once inside this stream converts the medicine present into a cloud that can be easily inhaled. The nebulizer must be used under the supervision of a licensed physician or respiratory therapist.

The user is encouraged to read the contents of this manual carefully to learn how to operate the device. Any use of the product other than its intended use is always discouraged.

2. PRODUCT IDENTIFICATION

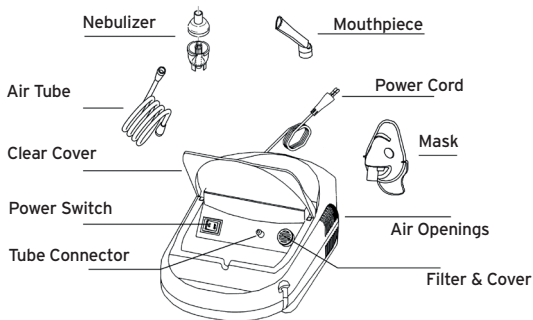3. IMPORTANT SAFETY NOTES

Note: Read the operating instructions carefully before starting treatment.

The following precautions are necessary when using an electronic device:

Warning: Failure to read and observe the warnings may result in personal injury or damage to equipment.

Device Warnings:

1. To avoid electric shock: Keep the unit away from water.
Do not immerse the power cord or the unit in liquids. Do not use the unit while bathing. If the unit falls into water, do not attempt to pick it up unplug it immediately.
2. Never operate the device if it has damaged parts (including the power cord), if it has been dropped or immersed in water. Promptly send the device to a service center for inspection and repair.
3. The unit must not be used in the presence of flammable gases, oxygen, or aerosol products.
4. Unplug the unit from the power outlet before cleaning, filling, or after each use.
5. Do not use any accessories other than those recommended by the manufacturer.

Operating Warnings:

1. Connect the product to a power outlet suitable for the model in use.
2. Do not operate the product without supervision.
3. Never operate the device if the cord or plug is damaged, if it has been dropped or immersed in water, or if it is not functioning properly. Return the device to a service center for repair.
4. If any malfunctions occur, stop using the unit immediately until it has been inspected and repaired.
5. Always unplug the product immediately after use.
6. Never block the air openings of the main unit and do not place it in a location where the air openings could be obstructed.

Storage Precautions:

1. Do not store the unit in direct sunlight, at high temperatures, or in humid environments.
2. Keep the unit out of the reach of children.
3. Always keep the unit unplugged when not in use.

Cleaning Precautions:

1. Do not immerse the unit in water. This could damage the unit.
2. Unplug the unit from the power outlet before cleaning.
3. Clean all necessary parts after each use as indicated in this guide.

4. OPERATION

Note: Before first use, the nebulizer must be thoroughly cleaned.

1. Place the compressor nebulizer on a flat, stable surface.
Make sure you can easily reach the controls when seated.
2. Open the transparent cover and remove the internal accessories.

Important: Before initial operation, the aerosol must be thoroughly cleaned referring to the "Cleaning Procedures" in the manual.

3. Gently rotate the top of the ampoule anti-clockwise to open.
4. Fill the lower part of the ampoule with the medication prescribed by your doctor. Ensure that the cone is inserted into the lower section.
5. Gently rotate the top section clockwise to refit the ampoule. Ensure that the two sections fit together well.
6. Connect one end of the air tube to the base of the bulb.
7. Connect the other end of the air tube to the connector located on the front of the device.
8. Connect the mouthpiece or mask of your choice to the top of the ampoule.
9. Connect the power cord to an appropriate electrical outlet. Ensure that the power switch is in the 'OFF' position at this stage.
10. Press the power switch to start the prescribed treatment.

Important: The compressor motor is equipped with a thermal protection that switches the unit off before it overheats, should it activate please:

- a. Switch off the unit.
 - b. Unplug the unit from the electrical outlet.
 - c. Wait 30 minutes for the motor to cool before performing another treatment.
11. When treatment is complete, switch the unit off and unplug it from the electrical outlet.

5. CLEANING

Cleaning and disinfection of aerosol accessories (including: ampoule, mouthpiece, mask for children, mask for adults, air tube).

After use of aerosol accessories:

1. Clean aerosol accessories with distilled water.
2. Completely immerse the aerosol accessories in approx. 75 % medical alcohol.
3. Soak for approximately 30 minutes.
4. Wash and rinse the aerosol accessories with warm (approx. 50°) distilled water.
5. Remove excess water and dry them with a clean, soft towel or leave them to air dry in a clean place.
6. Make sure the aerosol accessories and other clean parts are completely dry, then store all parts in a dry, sealed container.

Main unit

1. Clean daily with a damp cloth.
2. Do not use powdered detergents or soapsuds, which could damage the unit.

Changing the filter

1. Do not use cotton or other materials. Do not wash or clean the filter.
2. Change the filter every 30 days or when the filter turns grey.
3. Replacement procedure:
 - A. Remove the filter cover
 - B. Replace the used filter with a new one.
 - C. Replace the filter cover.

6. TECHNICAL DATA

Model	LTK215 (S600C)
Operating mode	30 mins ON, 30 mins OFF
Power	230V 50Hz
Power consumption	0.8A
Cup capacity	6 ml
Particle size	Respirable fraction > 73 % 0.5 µm
MMAD	2.52±0.89 µm
Nebulization rate	≥ 0.2ml/min
Noise	under 65dBA (DIN EN 13544-1)
Pressure range	30 ~ 50 Psi / 206Kpa ~ 345Kpa
Operating pressure range	9.5 ~ 16 Psi / 65.5Kpa ~ 110Kpa
Flow range	6~10 lpm
Operating temperature range	10°C ~ 40°C (50°F~104°F)
Operative humidity range	10% ~ 95% RH
Storage temperature range	-20°C ~ 70°C (-4°F~158°F)
Storage umidity range	10% ~ 95% RH
Atmospheric pressure	500 ~ 1060hPa
Dimension	Approx. L: 180 x W: 310 x H: 105 mm
Weight	1.2 kg
Standard accessories	Cup, Air Hose, Mouthpiece, Filters, Mask

7. SYMBOL CLASSIFICATION AND EXPLANATION

	Warnings/precautions/notes
	Class II equipment as per IEC 60601-1
	Applied part BF as per IEC 60601-1
No AP/APG	Degree of safety in the presence of flammable anaesthetics or oxygen
IP21	IP21 protection

	Check user manual
	Manufacturer
	Date of manufacture
	Batch number
	Disposal of Electrical & Electronic Equipment(WEEE): Do not treat the appliance as domestic waste
	CE marking: complies with the requirements of the Medical Device Directive 93/42/EEC
	Authorised European Representative

8. DISPOSAL CONDITIONS

When disposing of the device, never use regular municipal solid waste disposal systems. Instead, it is recommended to dispose of the device at local recycling centers for the proper recycling of the materials used.

Warnings for Proper Disposal of the Product in accordance with European Directive 2012/19/EU:

At the end of its useful life, the product must not be disposed of with municipal waste. It can be handed over to the appropriate recycling centers provided by local authorities or to retailers who offer this service. Separately disposing of the product helps avoid potential negative consequences for the environment and health due to improper disposal and allows the recovery of the materials it is made of, leading to significant savings in energy and resources. To emphasize the obligation of separately disposing of medical devices, the product features the crossed-out wheellie bin symbol.

9. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided)

use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

Repair

Warranty repairs

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty.

Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customee.

Spare parts

For spare parts only refer to MORETTI master catalogue.

Disclaimer

Beyond the specifications of this warranty and within the limits of law, Moretti makes no other express or implied representations, warranties or conditions with respect to merchantability, fitness for particular purposes, non-infringement and non-interference. Moretti does not warrant that the use of the Moretti product will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law is limited to the warranty period in accordance with the limits of the law. Some states or countries do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with consumer products. In such states and countries, some exclusions or limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE**

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

Dongguan Aidisy Machinery & Electronic Equipment Co, Ltd
Part B, 3rd F, Block A, Wentang Industrial Park, Longhua Road, Zhouwu Dongheng
District, Dongguan City, Guangdong Province, 523000, P.R. China

**MedPath GmbH****Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany****Imported and distributed by:**

Moretti S.p.a. Via Bruxelles, 3 - Melegnano 20122 Cavriaglia (Arezzo) Tel. +39 055 96 21 11
www.morettispa.com email: info@morettispa.com

** Please consult our website for the latest available version of the user manual*