**MOPEDIA®**

MA MI094\_MI095\_MI096\_MI097 02 A\_ITA\_07 2020

**ALZAMALATI**

---

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ITALIANO

## INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

## 1. CODES

**MI094** - Patient lift for beds with headboard (Tulipano type)

**MI095** - Patient lift for beds with headboard (Ibisco type)

**MI096** - Aluminium I.V. Poles with 2 Plastic Hooks for beds (Ibisco Type)

**MI097** - Universal patient lift with base

## 2. INTRODUCTION

Thank you for choosing a patient lift by MOPEDIA range, manufacturer by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. Patient lift by MOPEDIA by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the lift you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the patient lift. Should you have any queries, please contact your retailer for advice and assistance.

## 3. INTENDED USE

Mopedia Patient Lifts poles (MI094-MI095-MI097) are designed for people in hospital or at home who have difficulties to sit or lie down on a bed.

The I.V. Poles for hypodermoclysis MOPEDIA (MI096) is a support device medical and health activities during administration to the patient of intravenous fluids.

**NOTE: Check that the product has not been damaged during shipment. In case of damage do not use it and contact your dealer for further instructions.**

### WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

## 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ACCESSORIES FOR BEDS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation

6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

#### 4.1 Standards and guidelines reference

Safety tests required by UNI CEI 60601-2 -52

### 5. GENERAL WARNINGS

**WARNING!** Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The shelf-life of the accessory is determined by the wear of parts that cannot be repaired and / or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.


### 6. SYMBOLS

<b>REF</b>	Product code
<b>UDI</b>	Unique Device Identification
	CE mark
	Manufacturer
<b>LOT</b>	Batch Lot
	Read the instruction manual
<b>MD</b>	Medical Device
	Conditions of disposal

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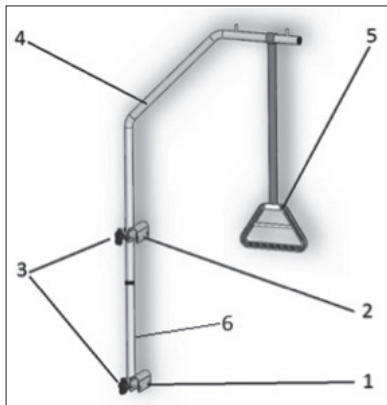
- 1. Extendible pole
- 2. Pole connection
- 3. Fixing screw with washer
- 4. Plastic Hook

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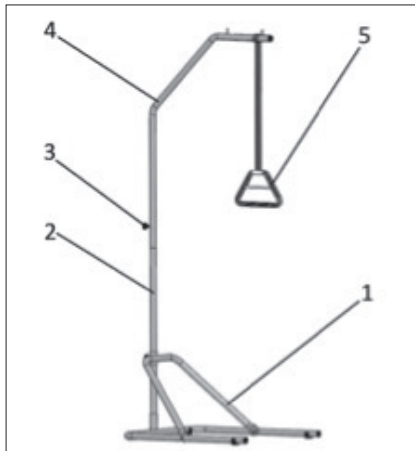- 1. "U" lower tube-shaped with tightening knob - 1 pcs;
- 2. "U" upper tube-shaped with tightening knob - 1 pcs;
- 3. Stright connection tube-shaped - 1 pcs;
- 4. "L" upper arm with tightening knob - 1 pcs;
- 5. Triangular patient handle - 1 pcs;
- 6. Assembly/dismantling kit (wrenches 2 pcs, screws 2 pcs, washers 4 pcs, nuts 2 pcs)

## 8. ASSEMBLING/DISASSEMBLING

### 8.1 Assembling

#### MI094

- a. Unpack the product and check that the content matches with the description of the parts in the manual.
- b. Fix the lower support (n.1) to the bottom of the headboard's tube.
- c. Attach the upper support (n.2) to the upper tube of the headboard. Do not tighten the knobs completely. (n.3).
- d. Insert the lower tube (n.6) in the lower support, until it stops.
- e. Insert the bent tube (4) in the lower tube and in the upper support, until it stops.
- f. Now tighten all the knobs completely and hang the belt. The rubber ring of the belt has to be placed between the two pegs on the bent tube.
- g. Adjust the belt length to allow the patient to grip the handle with both hands.
- h. Now the lift is ready for use.

#### MI095

- a. Unpack the product and check that the content matches with the description of the parts in the manual.
- b. Insert the lower tube (n.4) to the bottom of the bed frame. Until it stops by the pin.
- c. Insert the upper tube (n.2) onto the lower tube (n.3). Now tighten the knobs completely and hang the belt. The rubber ring of the belt has to be placed between the two pegs on the bent tube (figure n.2).
- d. Adjust the belt length to allow the patient to grip the handle with both hands.
- e. Now the patient lift is ready for use.

#### MI096

- a. Unpack the product and check that the content matches with the description of the parts in the manual.
- b. Fix the pole connection (n.2) to the bed tube, then insert the pole (1).
- c. Tighten the screw with washer (3) to fix the pole to the bed;
- d. Insert the plastic hook (4) in the pole;
- e. Now the pole is ready for use.

#### MI097

- a. Unpack the product and check the presence of every part written in this manual.
- b. Place the "U" lower tube-shaped on the ground and insert the straight connection tube-shaped.
- c. Insert the "U" upper tube-shaped in the straight connection tube until it touches the ground.
- d. Fix the "U" upper tube-shaped to the "U" lower tube-shaped using screws/washers/nuts, with wrenches of the kit.
- e. Insert the "L" upper arm onto the straight connection tube-shaped, placing on the upper edge the triangular patient handle (placing the belt between the 2 rods of the tube-shaped).
- f. Tighten knobs of "U" upper/lower tube-shaped and "L" upper arm tube-shaped, checking that the structure is well fixed and ready to be used.
- g. Regulate the belt in order to let patient take it comfortably.

### 8.2 Disassembling

- To disassemble do the exact opposite of the above instructions.

## 9. BEFORE USE

- Always check the status of the mechanical parts to guarantee they are used in total safety.
- Check the device is properly assembled. Pay particular attention to the locking of the knobs.

## 10. PRECAUTIONS FOR USE

- Do not exceed the maximum load capacity.
- Make sure that the floor surface supporting the patient lift is level.
- Make the adjustment operations without load.
- Check that the belt and the handle are secure.
- Adjust the length of the belt to facilitate the elevation of the patient.
- Always assist the patient during the elevation operation
- Be very careful when using the manual or electric backrest of the bed to help the patient to rise.

## 11. HOW TO USE

The assistant must place and adjust the lifting pole with the maximum care to avoid sudden movements or dangerous oscillations. An assistant has to check any movement of the patient. The patient lifts helps the patient to lift his chest. To help the patient to grip the handle, the assistant can raise the headrest of the bed or lengthen the belt.

## 12. MAINTENANCE

Check the structure periodically: the correct positioning of the tubes, the connections, the hooks, and the complete tightening of screws and/or terminals.

## 13. CLEANING AND DISINFECTION

### 13.1 Cleaning

Use a damp cloth or a solution of lukewarm water and neutral soap.

**ATTENTION!** Do not use abrasive substances, acids, alkali, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of the metal surfaces.

The manufacture company is not responsible for any damage caused by using materials that can damage the product surface or corrosive chemicals during cleaning.

### 13.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

## 14. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

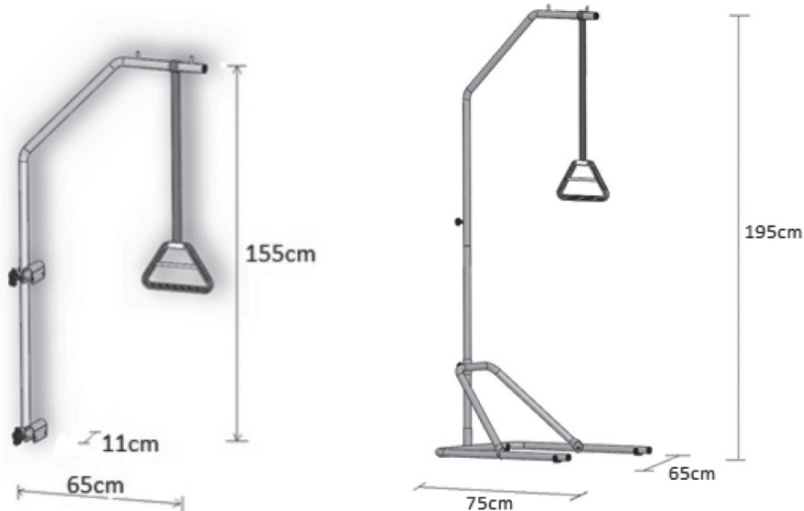
## 15. SPARE PARTS

For spare parts and accessories, please always refer to the MORETTI master catalogue.

ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

## 16. TECHNICAL FEATURES

### 16.1 Dimensions

#### MI094-MI095

Height	155 cm
Width	11 cm
Depth	65 cm
Weight about	6Kg
Pay load	75Kg

#### MI097

Height	195cm
Width	65cm
Depth	75cm
Weight about	12Kg
Pay load	75Kg

#### MI096

Height (min/max)	1330/2300mm
Width	200mm
Depth	30mm
Weight about	2 kg
Pay load (single hook)	5kg
Total pay load	10kg

## 17. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

## 18. REPAIRING

### Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the customer, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

### Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti customer service. Labour and delivery costs for an out-of-warranty reparation will be fully at the customer's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the customer receives the repaired product.

### Non defective device

The customer is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the customer and all the return delivery costs will be at his expense.

## 19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

## 20. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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