**MOPEDIA®**

MA M1190-M1195 02 B\_ITA\_03 2021

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ITALIANO

## INDICE

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 Class I medical device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 on medical devices

## 1. CODES

MI190 Hydraulic bed lifting base

MI195 Electric bed lifting base

## 2. INTRODUCTION

Thank you for choosing a hospital bed lifting device in the MOPEDIA range by Moretti. The MOPEDIA lifting bases by Moretti have been designed and constructed to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this manual in full before you use the lifting base you have purchased. Should you have any queries, please contact your dealer for suitable advice and assistance.

**NOTE:** Check to ensure that no parts of the product have been damaged during shipment. Do not use the product if it is damaged and contact the retailer for further instructions.

### WARNING!

- Do not use the product for any purpose other than that specified in this manual.
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual.
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features.

## 3. INTENDED USE

The MOPEDIA bed lifting bases are designed to support and adjust the height of the Moretti MI100 - MI110 - MI120 hospital beds, in hospital and/or healthcare structures and entities or at home.

## 4. EU DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products made and sold by MORETTI SpA in the BED ACCESSORIES product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES.

To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

**Note:** The complete product codes, the manufacturer's single registration number (SRN), the

basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

## 5. GENERAL PRECAUTIONS

### WARNING!

- Take extreme care in the presence of moving parts that could trap limbs and cause injury.

- Please consult this manual carefully for correct use of the device
- Always consult your doctor or therapist for correct use of the device
- Keep the packaged product away from sources of heat, as the packaging is cardboard
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts
- Always take care when children are present.
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

## 6. SYMBOLS USED

Product code

Unique device identification

CE marking

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Disposal conditions

Warning

Production date

Disposal of the product according to directive EC/19/2012

## IP54

Protection against dust and water spray

Type B applied part

Maximum permissible load

Maximun Load


## 7. GENERAL DESCRIPTION

The folding sides have been designed to be adapted to hospital beds, with a simple structure to be positioned on the headboard and footboard.

### 7.1 View and parts list

MI190 - MI195

Fig. 1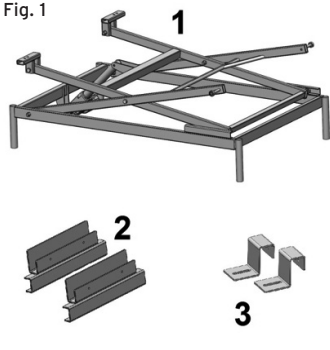

1. Lifting base frame with motor drive
2. Attachment bracket at foot of bed
3. Attachment bracket at head of bed
4. Plastic stop for electrical connections (only for MI195)
5. Power supply unit (only for MI195)
6. Remote control (only for MI195)
7. Hydraulic lifting pedal (solo MI190)
8. Screws for assembly of bracket at head of bed
9. Screws for attachment of the brackets onto the bed
10. Screws for attachment of the pedal (only for MI190)
11. Pedal attachment nuts

ENGLISH


## 8. ASSEMBLY

### 8.1 Assembly of the structure (MI190 - MI195)

**Note:** The illustrations for assembly and adjustments refer to the Moretti hospital beds MI100, MI110 and MI120. The illustrations refer to the hospital bed with two rotating handles. The assembly and adjustment operations shown also apply for the hospital bed with one rotating handle.


- Unpack the product and check that the contents correspond with the illustrative description of the parts
- Assemble the bracket (2) on the frame of the bed using the screws (9) - 2 pieces, then position and turn the bracket as shown in (Fig. 2). Repeat this operation for both sides of the bed

Fig. 2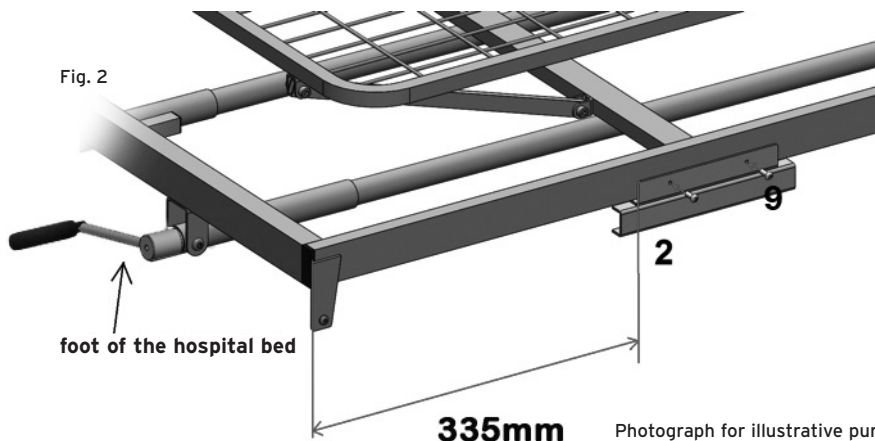


Photograph for illustrative purposes

Fig. 3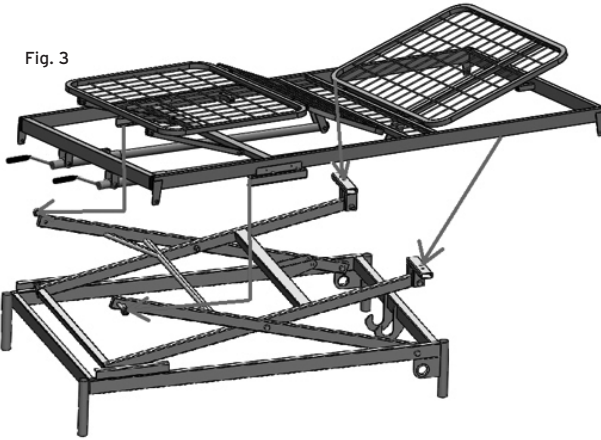


- With the assistance of a second operator, position the bed frame on the lifting base as shown in (Fig. 3), making sure to insert the bearings of the lifting base into the bracket just assembled at the foot of the hospital bed.

Fig. 4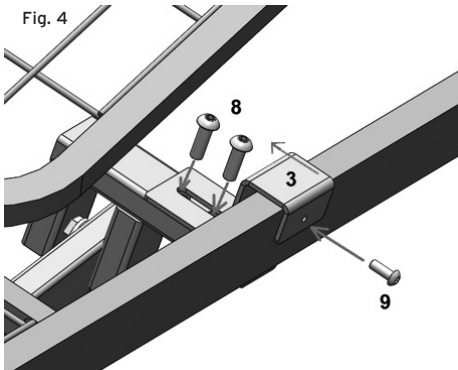

- Assemble the bracket (3) on the lifting base using the screws (8) - 2 pieces, positioning it as shown in (Fig 4), then insert the screw (9). Repeat this operation for both sides of the hospital bed.

Fig. 5

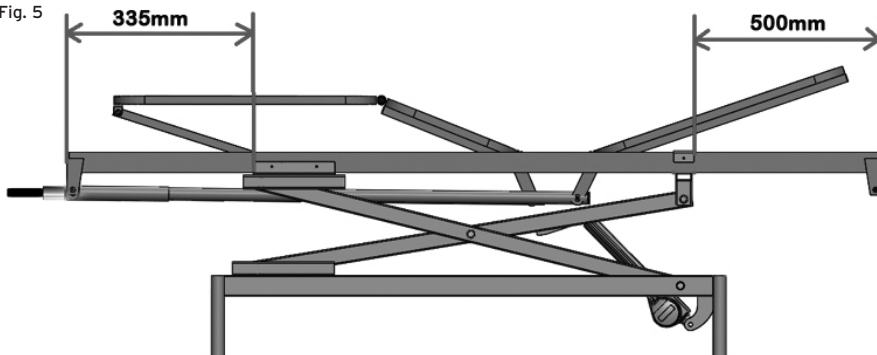- Before using the bed, check that the distance between the brackets and the ends of the hospital bed is as shown in Fig. 5, in order to ensure that the bed is correctly balanced.
- At this point, the lifting base is correctly assembled on the hospital bed.


### 8.2 Assembly of the electrical kit (only MI195)

Fig. 6

Photograph for illustrative purposes


Fig. 7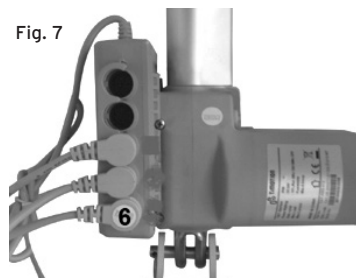

- Connect the end of the connector of the wall-mounted power supply unit extension cable (5), ensuring the position of the relative notch is correct, then lock the connection in place with the plastic cap. (Fig. 6)
- Connect the remote control (6) to the control unit and insert the plastic stop (4) onto the connectors, as shown in Fig. 7

### 8.3 Assembly of the pedal (only MI190)

Fig. 8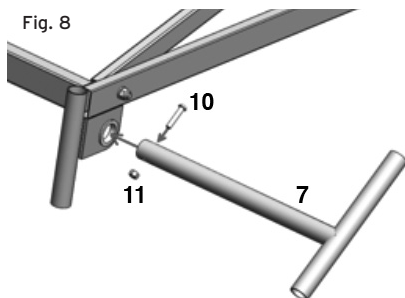

- Insert the pedal (7) into its housing, as shown in Fig. 8
- Use the screws (10) and nuts (11) to attach the pedal to the hydraulic actuator
- Repeat this operation for both sides of the lifting base

## 9. BEFORE EVERY USE

- Always check for wear and tear of moving mechanical parts, in order to ensure that the device can be used in total safety for both people and property.
- Check that the device is correctly assembled, taking particular care over correct insertion and locking of the bolts on the connection brackets between the lifting base and the hospital bed
- For correct balancing, check that the connection brackets (2) and (3) are positioned as shown in Fig. 5.

## 10. WARNINGS ON USE


- Do not exceed the maximum permitted weight under any circumstances
- When the lifting base is being lowered, make sure no one places their hands, legs or other parts of the body underneath the mesh of the hospital bed
- Ensure that the floor on which the lifting base is positioned has an even surface

- Always disconnect the power cable from the mains electricity before performing any cleaning or maintenance operations or simply moving the device
- Do not use additional mobile multiple plug sockets or extension cables
- Do not connect any electrical devices not intended for normal functioning to the bed or to the lifting base

## 11. HOW TO USE

### 11.1 Use of the remote control (only MI195)

Fig. 9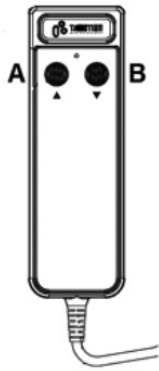


- Use the buttons on the remote control (Fig. 9) to raise or lower the lifting base

1. Button A to raise the lifting base
2. Button B to lower the lifting base

Photograph for illustrative purposes

### 11.2 Use of the pedal (only MI190)

Fig. 10

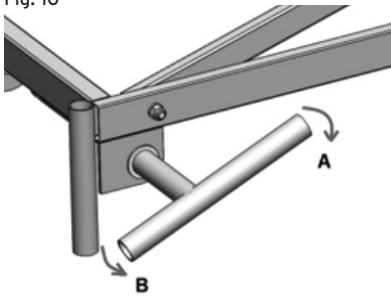- Press down several times on the pedal on side A to raise the height of the lifting base, as shown in Fig. 10
- Press down several times on the pedal on side B to lower the height of the lifting base, as shown in Fig. 10

## 12. MAINTENANCE

The products in the MOPEDIA range by Moretti have been carefully checked and CE marked before being placed on sale.

For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original replacement parts and accessories for repairs.

- It is extremely important to check the device before using it for the first time. In particular, check that all the screws and bolts are properly tightened.
- Check that the mechanical parts of the device can move correctly (headrest/legs, rotating handles, actuators, etc.).
- Check that the rear wheel brakes are working properly (optional).
- Check the mechanical parts (hinges) once a month and lubricate them with a drop of oil to ensure they slide easily and to avoid wear.

### 13. CLEANING AND DISINFECTION

#### 13.1 Cleaning

Use only a damp cloth and mild soap to clean the parts.

#### 13.2 Disinfection

If the device is used in hospitals or care homes where sanitisation is necessary, use only sanitising products and disinfectants for which the degree of corrosiveness is known, in order to avoid damaging certain parts of the device. Follow the specifications on concentration, temperature and reaction times of the products and all the manufacturer's instructions.

**WARNING!**

Sanitising products are corrosive. Avoid contact with the eyes and skin. Wear protective clothing, gloves, protective goggles and safety shoes. In the case of accidental contact, wash the part concerned with plenty of warm water and seek medical advice immediately.

### 14. DISPOSAL CONDITIONS

#### General conditions of disposal

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

**Instructions for correct disposal in accordance with European directive 2012/19/EC:**



At the end of its working life, the product must not be disposed of together with normal urban waste. It must be delivered to municipal separated waste collection facilities, or to appropriate dealers that provide this service. Separated waste disposal helps to reduce possible negative effects on the environment and health deriving from improper disposal and allows for recycling the materials comprising the product, which translates into significant energy and resource savings. The product bears the barred bin symbol to underline the obligation of disposing of electro-medical equipment.

### 15. TECHNICAL SPECIFICATIONS

#### 15.1 Dimensions

MI190 Hydraulic bed lifting base

MI195 Electric bed lifting base

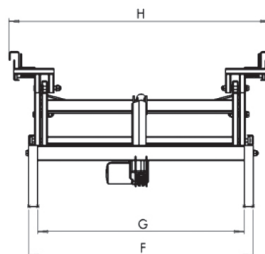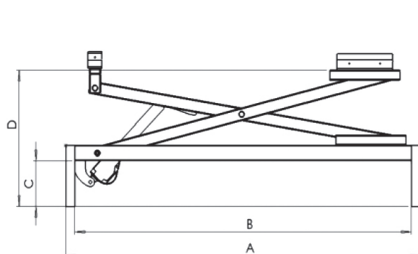- A. 126cm
- B. 120cm
- C. 15cm
- D. (cod. MI190): from 52cm to 80cm
- D. (cod. MI195): from 42cm to 80cm
- F. 80cm
- G. 74cm
- H. 93cm

### 15.2 Technical specifications

Base frame and movement: Powder-coated rectangular steel tube structure.

Adjustments: Height of the bed support surface adjustable from 42cm to 90cm

Total weight: 47 Kg

Maximum load: 135 Kg

### 15.3 Electrical kit (only MI195)

Units of force needed to control the commands on the remote control: 5N

Power supply: 220 VAC 50-60 HZ

Voltage output from the control unit: 24 VDC

Protection class: IP 54

Noise level: < 45 dB

Insulation class: Class 2

## 16. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual.

Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual.

Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual.

Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

## 17. REPAIRS

### Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

### Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipment.

### Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

**18. REPLACEMENT PARTS**

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

**19. NON-LIABILITY CLAUSE**

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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