**MOPEDIA®**

ITALIANO

MA RH150-RH720 02 A\_ITA\_09 2020

**BASTONE DA PASSEGGIO E QUADRIPODE**

---

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## INDICE

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 I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

### 1. CODES

**RH150** Height adjustable walking stick - WALKING STICKS

**RH720** Height adjustable 4-leg cane - TRIPOD AND QUADRIPOD

### 2. INTRODUCTION

Thank you for purchasing a MOPEDIA by Moretti walking stick and 4-leg cane. This instruction manual contains some useful tips for a correct use of the device you have bought as well as some important advises for your safety. In case of doubts, please contact your retailer who will be able to help you correctly.

### 3. INTENDED USE

The MOPEDIA by Moretti walking stick and 4-leg cane are aids manipulated by one arm adapted to support and/or help the mobility of people with mobility problems.

#### WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

### 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the groups of WALKING STICKS (RH150) and TRIPOD AND QUADRIPOD (RH720) complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

Read carefully the entire manual before using the device.

- Advise with physician or therapist to determinate the correct adjustment and the correct use of the product.
- Make sure that the product is delivered complete and keep it packed away from heat sources.
- The device use limit is defined by the wear of the parts.
- Do not allow children to play or operate the device.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

## 7. GENERAL DESCRIPTION

### RH150

The BRIO HD walking stick is made of steel and is characterized by an open handle open in PVC and a wide non-slip rubber tip. Adjustable in height every 2.5 cm. Is useful to guarantee security of support and confidence. The stick also facilitates the stay in an upright position and the path. It favors the discharge of lower body segments, stabilizes the person in balance problems and supports it when there are strength problems.

### RH720

The BRIO HD 4-leg cane is made of steel and is characterized by an open handle in PVC and non-slip rubber tips. Height adjustable in height every 2.5 cm. Is useful for facilitate the user's standing position. It favors the discharge of lower body segments, stabilizes the person in balance problems and supports him when there are strength problems.

## 8. BEFORE USE

### WARNING!

Check if the frame is damaged in order to guarantee a safe use for people and things.  
Check the correct insertion of the button in the hole as well as the assembly of the device.

### 9. WARNINGS FOR USE

- Use extra caution on wet and slippery surfaces.
- Avoid leaving the device in contact with water for long periods of time.
- Periodically check the stability and strength of the product as well as the condition of the support tip. Replace it when worn.
- In case of doubts about use, adjustment or maintenance, contact your dealer.
- Keep it away from children to avoid any possible risks.

**WARNING!**

DO NOT use with patient who exceed the weight indicated in the product label and in the "technical features" chapter of the present manual.

### 10. HOW TO USE

Note: Refer to the safety instructions in this manual

The use of the walking stick and 4-leg cane varies from case to case. For this reason, you need a minimum of training by qualified personnel with reference to information on the type of step or support type. The operator during patient care is always positioned on the side of the body more weak and sick.

#### 10.1 Height adjustment for RH150

To adjust the height of the walking stick:

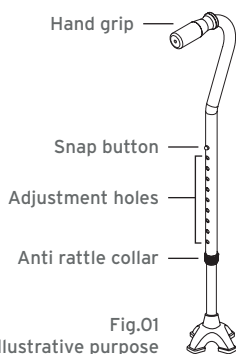
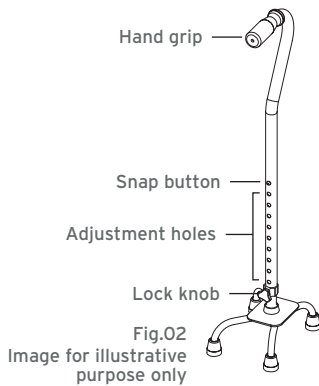
Fig.01  
Image for illustrative purpose only

- Turn the anti-rattle collar counterclockwise to loosen;
- Press snap button;
- Slide extensions up or down until snap button protrudes through an adjustment hole on handle end of cane (an audible "click" will be heard when snap button securely protrudes through the adjustment hole);
- Turn the anti-rattle collar clockwise to tighten.

### 10.2 Height adjustment for RH720

To adjust the height of the 4-leg cane follow the instructions below:

- Loosen lock knob by turning counterclockwise;
- Press snap button;
- Slide extensions up or down until snap button protrudes through an adjustment hole on handle end of cane (an audible "click" will be heard when snap button securely protrudes through the adjustment hole);
- Tighten lock knob by turning clockwise.

### 10.3 Adjust 4-leg cane for right or left hand use

To adjust the side of use of the cane to the right or left is necessary:

- Loosen lock knob;
- Depress the snap button and turn cane handle 180 degrees until snap button locks into same height adjustment hole on the other side of cane;
- When cane handle is firmly positioned at the desired height and snap button protrudes through height adjustment hole on cane handle, turn lock knob clockwise to secure.

## 11. MAINTENANCE

The Mopedia devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety, we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

For periodic inspection we mean an examination of the following parts:

- The product frame
- The rubber tip.

In case of repair are to be used only approved parts and accessories.

## 12. CLEANING AND DISINFECTION

### 12.1 Cleaning

Use only a damp cloth to remove dust, use a damp cloth with a mild soap to remove dirt that is more difficult. Dry thoroughly before use.

#### WARNING!

Never use water with temperature above 30°C. Never use jet water or steam washing machine.

### 12.2 Disinfection

If you need to disinfect the product, use a common disinfectant cleaner.

#### WARNING!

Absolutely avoid using acid, alkaline or solvent products such as acetone or diluent.

### 13. CONDITIONS OF DISPOSAL

In case of disposal, do not use the inserting container for municipal waste. We recommend disposing the bath chair in the appropriate disposal areas for recycling.

### 14. TECHNICAL FEATURES

#### RH150

 ADJUSTABLE HEIGHT: DA 74 A 97 CM

 MAX LOADING WEIGHT: 226 KG

Steel frame

Open handle with PVC handgrip

Anti-slip rubber tip, large base

#### RH720

 ADJUSTABLE HEIGHT: DA 74 A 97 CM

DIMENSIONS:  L: 15,5 CM  P: 20,5 CM

 MAX LOADING WEIGHT: 226 KG

Steel frame

Open handle with PVC handgrip

### 15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### 16. REPAIRING

#### 16.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

**16.2 Repair not covered by warranty**

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

**16.3 Non defective device**

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

**17. SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date. For spare parts and accessories please refer to Moretti master catalogue.

**18. EXEMPT CLAUSES**

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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