**MOPEDIA®**

MA DB100x\_DB105x\_DB110x\_DB115x\_DB200x\_DB205x\_DB210x\_DB230x\_DB25x\_DB300x\_DB305x\_DB31x\_DB33x\_DB35x\_DB360 01 B\_IT\*\_04 2026

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 I Class Medical Device  
 REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
 of 5 April 2017 concerning medical devices

## 1. CODES

### Walkers with four tips

- DB100x** Fixed walker adjustable height
- DB105x** Foldable walker adjustable height
- DB110** Foldable walker adjustable height double handgrip
- DB115x** Foldable walker with dual fixed/oscillating function height adjustable

### foldable and height-adjustable walkers, two castors and two tips

- DB200x** Walker with 2 fixed front castors and 2 rear tips
- DB205** Walker with 2 fixed front castors and 2 rear tips with self-locking system
- DB210** Walker with 2 swivel front castors and 2 rear tips
- DB230** Walker with 2 front castors with directional lock and 2 rear tips
- DB250** Walker with 2 fixed front castors and 2 rear tips
- DB255** Walker with 2 swivel front castors and 2 rear tips

### foldable and height-adjustable walkers, four castors

- DB300** Walker with 2 swivel front castors and 2 rear castors with self-locking system
- DB305x** Walker with 4 wheel swivel
- DB310** Walker with 4 swivel castors at the rear with brakes
- DB315** Demontable walker, 4 swivel castors, rear castors with brakes
- DB330** Walker with 4 swivel castors at the rear with brakes
- DB335** Walker with 2 swivel front castors and 2 twin rear castors with brakes
- DB350** Walker with 2 swivel front castors and 2 twin rear castors with brakes
- DB355** Walker with 2 swivel front castors and 2 fixed rear castors
- DB360** Walker with 4-wheel swivel

## 2. INTRODUCTION

Thank you for purchasing a MOPEDIA walker by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

## 3. INTENDED USE

The Mopedia by Moretti walkers are devices to be used with two arms and are intended for people with movement difficulties.

### WARNING!

- Do not use the product for a purpose not indicated in this manual;
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product;
- The manufacturer reserves the right to change the information contained in this document without previous notice.

## 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of WALKERS MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.

5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

EN

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

### 5.1 Warnings and instructions for the correct use of the device

To ensure safe and effective use of the device, it is recommended that the following instructions be strictly followed:

#### Consult the instruction manual

- Before each use, please read this manual carefully, paying particular attention to the sections on routine maintenance and safety warnings.

#### Medical advice

- The device should be used in consultation with a qualified physician or therapist to ensure full compatibility with the therapeutic needs of the user.

#### Container storage

- Keep the original packaging away from heat sources or open flames, as it is made of flammable material.

#### Lifetime of the appliance

- The useful life of the appliance is determined by the wear and tear of the components that cannot be repaired or replaced and, in any case, does not exceed 10 years from the date of manufacture indicated on the label.

#### Safety for children

- Pay attention to the presence of children near the appliance to avoid accidents.

#### Load limits

- Never exceed the max load specified by the manufacturer. Exceeding this limit may compromise the structural integrity of the appliance and the safety of the user.

#### Notification of adverse events


- The user or patient must promptly report any serious incident related to the use of the product to both the manufacturer and the competent authority of the Member State in which he/she resides.

## 6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Max loading weight
	Conditions of disposal

**7. GENERAL DESCRIPTION**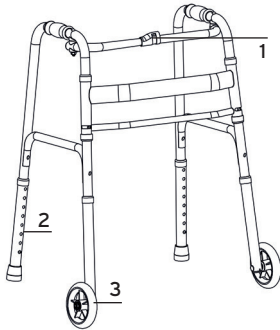

1. Closing pin
2. Adjustable height
3. Castors and/or tips depending on the configuration purchased

(image for illustrative purposes only)

EN

**8. BEFORE USE**

- Always check that the brakes work before using the walker, if available;
- Do not use the walker if it is damaged or malfunctioning please contact retailer for repairing;
- Check if the frame is damaged in order to guarantee a safe use for people and things;
- Users must travel along pedestrian paths and not on busy roads;
- Pay attention to the possibility of injury from entrapment; avoid placing your fingers between the moving parts of the walker. Pay attention closing and opening the walker.

**9. WARNINGS FOR USE**

This device provides greater stability, support, and safety when walking. However, it cannot be used by people with visibly weak hand and/or arm function, or by people with balance problems. The walker can be used both outdoors and indoors.


- It is recommended to use the device only on surfaces where all four legs rest firmly on the ground;
- Use on rocky terrain or slopes is not recommended, as it could compromise the stability of the walker;
- The device is not designed for transporting people or materials;
- Use with caution on wet and slippery surfaces;
- Use with caution on wet and slippery surfaces;
- Do not exceed the maximum patient weight indicated on the product label and in the technical specifications section of this manual;
- Use on slopes greater than 10° is not recommended; on sloping surfaces, the user must be assisted by an operator during turning maneuvers.
- Periodically check the stability and sturdiness of the product as well as the condition of the support tips and castors. Replace them when worn;
- If in doubt about the use, adjustment, or maintenance of the product, contact the dealer;
- To avoid any risks, keep the device out of the reach of children.

**10. ASSEMBLY INSTRUCTIONS**

**10.1 DB100x Fixed walker**


Parts list

1. Main Frame (1pc)
2. Legs (2pcs)
3. Lower frame (1 pc)
4. Assembly Kit (1pc)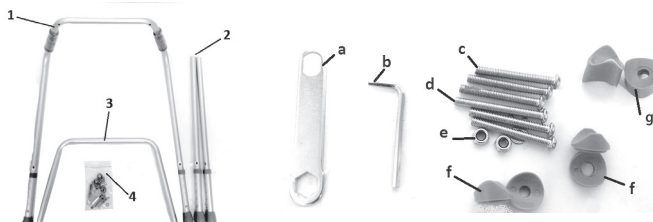

EN

- a. Open the assembly kit and start by taking the support frame (1).
- b. Secure the legs (2) to the support frame (1) using the saddle washers (f) and short screws (c).  
Do not tighten the screws completely at this stage.
- c. Assemble the lower frame (3) on the front of the supporting frame (1) using the saddle washers (g), long screws (d), and nuts with washers (e). Do not tighten the screws completely.
- d. Secure the lower frame (3) to the legs (2) using the saddle washers (f) and short screws (c). Again, do not tighten the screws completely at this stage.
- e. Tighten all screws securely, and the walker is now ready for use. (See photo below)

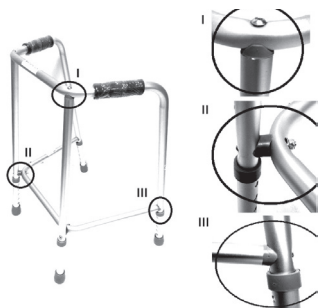## 10.2 DB115x Dual-function walker: Fixed/Oscillating

Fig.1

Fig.2

- (Fig 1) The lock is open; in this configuration, the walker is free to swing.
- (Fig. 2) The lock is closed; in this configuration, the walker is locked and cannot swing.

## 11. HOW TO USE

**Note:** Refer to the safety instructions in this manual. Use the walker as aid device for walking.

To do this use the two handles and push forward the aid thus obtaining a support convenient and reliable for walking.

With your arm extended down, handgrips should be at your wrist height. With your hand placed on the handgrip, your elbow should be flexed by 20° - 30°.

### WARNING!

Do not use the walker in the presence of electrical wires on the ground, wet floors, carpets, toys or other obstacles, furniture, steps or uneven surfaces. It is forbidden to use the walker to go up or down stairs, or on escalators.

1. To transfer the patient from the wheelchair to the walker, check that the castors of the wheelchair and walker (if provided) are correctly locked.

**WARNING!**

- For the transfer and the first steps get help by an operator.

2. a) Walker with tips: to walk with the rollator, lift it and position it comfortably in front of you, making sure that all four legs rest firmly on the ground. Proceed with caution, moving forward one step at a time.

b) Walker to walk with a rollator, push it in a controlled manner in front of you, making sure that all four legs, including those with castors, are firmly on the ground.

Walk carefully, taking one step at a time.

c) Walker with double handles: designed to make it easier to get up from a chair. The lower handles allow the user to start lifting themselves up without having to reach for the upper handles straight away, which are often more difficult to grasp.

**Note: The walker must always be held by the handles.**

Do not push the walker too far away from the body. This can cause falls and inadequate loads.

## 12. MAINTENANCE

The Mopedia devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. For periodic inspection we mean an examination of the following parts:

- The walker frame and the connecting parts;
- Check mounting and operation of the castors with its brake;
- Check all the welded points;
- Check the wear of the tips, where present;

In case of repair must be used only approved parts and accessories.

## 13. CLEANING AND DISINFECTION

### 13.1 Cleaning

To clean the device, use only a damp cloth to remove dust and surface dirt.

For stubborn dirt, use a damp cloth with mild soap.

Make sure the device is completely dry before use. The water used must not exceed 30°C.

The use of pressure washers or steam cleaners is prohibited.

### 13.2 Disinfection

If you need to disinfect the device use a common disinfectant cleaner.

**Note:** Never use acids, alkalis or solvents such as acetone.

## 14. CONDITION OF DISPOSAL

In case of disposal of the device don't use the common container for municipal waste. We recommend to dispose the device in the appropriate disposal areas for recycling.

## 15. TECHNICAL FEATURES

## Dimensions

Codes					Base* 	Maximum weight 
DB100M	76÷86 cm	4	5	-	A: 55/56 B: 43,5/46 C: 45/46	130 kg
DB100L	86÷96 cm		5	-	A: 56/57 B: 45/47 C: 45,5/47	
DB105M	77÷87 cm		5	-	A: 61/62 B: 48/50 C: 59/60	130 kg
DB105L	83÷101 cm		8	-	A: 63/67 B: 49/53 C: 60/62	
DB110	71÷89 cm		8	-	A: 60/62 B: 51/52 C: 58/59	130 kg
DB115M	77,5÷87,5 cm		5	-	A: 54/56 B: 47,5/49 C: 52/52	130 kg
DB115L	83,5÷101,5 cm		8	-	A: 54/56 B: 47,5/49 C: 52/52	
DB200S	68,5÷78,5 cm		2	5	2 x Ø 7.6 cm	A: 62/63 B: 46/48 C: 59/60
DB200M	76÷86 cm	5		2 x Ø 7.6 cm	A: 63/65 B: 48/50 C: 60/62	
DB200L	83÷101 cm	8		2 x Ø 7.6 cm	A: 63/65 B: 49,5/53,5 C: 60/62	
DB205	84÷102 cm	8		2 x Ø 7.6 cm	A: 61/64 B: 53/57 C: 62/65	
DB210	84÷102 cm	8		2 x Ø 7.6 cm	A: 61/63 B: 50/54 C: 60/61,5	
DB230	84÷102 cm	8		2 x Ø 10.2 cm	A: 65/67 B: 54/60 C: 59/60	
DB250	86÷103 cm	8		2 x Ø 12.7 cm	A: 65/67 B: 54/60 C: 59/60	
DB255	86÷103 cm	8		2 x Ø 12.7 cm	A: 65/67 B: 54/60 C: 59/60	
DB300	84÷102 cm	-	-	4 x Ø 7.6 cm	A: 66/69 B: 55/59 C: 62/64	130 kg
DB305S	70÷80 cm	-	-	4 x Ø 7.6 cm	A: 62/65 B: 55/56 C: 58/60	130 kg
DB305M	78÷88 cm	-	-	4 x Ø 7.6 cm	A: 63/65 B: 56/58 C: 60/61	
DB305L	84÷102 cm	-	-	4 x Ø 7.6 cm	A: 65/67 B: 58/60 C: 62/63	
DB310	86÷96 cm	-	-	4 x Ø 7.6 cm	A: 60/61 B: 54/56 C: 58/59	130 kg
DB315	77÷92 cm	-	-	4 x Ø 7.6 cm	A: 54/54 B: 54/54 C: 54/54	130 kg
DB330	86÷103 cm	-	-	4 x Ø 10.2 cm	A: 65/67 B: 54/60 C: 59/60	130 kg
DB335	86÷103 cm	-	-	4 x Ø 10.2 cm	A: 65/66 B: 57/59 C: 60/61	130 kg
DB350	76÷86 cm	-	-	4 x Ø 12.7 cm	A: 65/67 B: 54/60 C: 59/60	130 kg
DB355	86÷103 cm	-	-	4 x Ø 12.7 cm	A: 65/67 B: 54/60 C: 59/60	130 kg
DB360	86÷103 cm	-	-	4 x Ø 12.7 cm	A: 65/67 B: 54/60 C: 59/60	130 kg

\*Overall dimensions measured at the lowest setting and then at the highest setting.  
The measures may change  $\pm 1$  cm

## 16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

## 17. REPAIR

### 17.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

### 17.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

### 17.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer .

## 18. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

## 19. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and

limitations of this warranty may not apply to you. This warranty is subject to change without notice.

EN

**WARRANTY CERTIFICATE**

Product \_\_\_\_\_

Purchased on (date) \_\_\_\_\_

Retailer \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Sold to \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

**MADE IN P.R.C.**

*\* Please consult our website for the latest available version of the user manual*

**MORETTI S.p.A.**  
Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11  
Fax. +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)