MA RP729x-RP73x-RP74x 04 D_07-2018

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Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN P.R.C**

INDEX

1. INTRODUCTION	Pag.13
2. INTENDED USE.....	Pag.13
3. EC DECLARATION OF CONFORMITY	Pag.13
4. GENERAL WARNING	Pag.13
5. GENERAL DESCRIPTION.....	Pag.13
5.1 Walkers with 4 tips.....	Pag.13
5.2 Walkers with 2 swivel castors and 2 tips	Pag.14
5.3 Walkers with 4 swivel castors.....	Pag.14
5.4 Walkers with 4 castors and brake	Pag.14
5.5 To adjust height	Pag.14
6. BEFORE USE	Pag.14
7. WARNINGS FOR USE.....	Pag.14
8. ASSEMBLY INSTRUCTIONS FOR RP729X.....	Pag.15
9. HOW TO USE	Pag.15
10. MAINTENANCE	Pag.16
11. CLEANING AND DISINFECTION.....	Pag.16
11.1 Cleaning	Pag.16
11.2 Disinfection	Pag.16
12. CONDITIONS OF DISPOSAL	Pag.16
12.1 General condition of disposal	Pag.16
13. TECHNICAL FEATURES	Pag.17
13.1 Spare parts and accessories	Pag.19
14. WARRANTY	Pag.20
15. REPAIR.....	Pag.20
15.1 Warranty repair	Pag.20
15.2 Repair not covered by warranty.....	Pag.20
15.3 Non-defective device.....	Pag.20
16. SPARE PARTS	Pag.20
17. EXEMPT CLAUSES	Pag.20

CE I Class Medical Device

according to Directive 93/42/EEC and further modifications

1. INTRODUCTION

Thank you for purchasing a MOPEDIA walker by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

2. INTENDED USE

The Mopedia by Moretti walkers are devices to be used with two arms and are intended for people with movement difficulties.

- Do not use the product for a purpose not indicated in this manual;
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product;
- The manufacturer reserves the right to change the information contained in this document without previous notice.

3. EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the cane aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive;
2. The devices in question **ARE NOT MEASUREMENT DEVICES**;
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing;
5. The devices in question should be considered as belonging to class I;
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.;
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications.

4. GENERAL WARNINGS

- **READ** carefully the entire manual before using the device;
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the walker;
- Make sure that the product is delivered complete and keep it packed away from heat sources;
- **SERVICE LIFE**- the device use limit is defined by the wear of the parts;
- **DO NOT** allow children to play or operate the device.

5. GENERAL DESCRIPTION

5.1 Walkers with 4 tips

- RP729S-M-L** Fixed walker adjustable height
- RP730S-M-L** Foldable walker adjustable height
- RP731** Foldable walker adjustable height child model
- RP747** Foldable walker adjustable height double handgrip
- RP749S-M-L** Foldable walker adjustable height double function

5.2 Walkers with two swivel castors and two tips

- RP734** Foldable walker adjustable height, 2 swivel castors and 2 tips
RP735S-M-L Foldable walker adjustable height, 2 fixed castors and 2 tips
RP736 Foldable walker adjustable height 2 castors, child model
RP737 Foldable walker adjustable height, 2 fixed castors and 2 tips
RP744 Foldable walker adjustable height, 2 swivel castors and 2 tips
RP745 Foldable walker adjustable height, 2 fixed castors and 2 tips

5.3 Walkers with four castors

- RP733S-M-L** Foldable walker adjustable height, 4 swivel castors
RP738 Foldable walker adjustable height, 2 self locking castors and 2 swivel castors
RP741 Foldable walker adjustable height, 4 swivel castors, 2 with brakes
RP748 Demontable walker adjustable height, 2 swivel with brake castors and 2 swivel castors

5.4 Walkers with four castors and brake

- RP742** Foldable walker with hand brake

The walkers MOPEDIA are realized in lightweight anodized aluminum tubing with padded handles, non-slip rubber tips and/or swivel castors and/or fixed castors. They are height adjustable every 2,5 cm. These devices facilitate people movement with limited static equilibrium and / or dynamic, increasing the amplitude to the supporting base. They provide support in situations of weakness and/or paralysis of the lower limbs. Helps to download on the arms part of the weight of the person, where it is required a limitation of the load on one or both lower limbs.

5.5 To adjust height

To adjust the height, press the push button and slide the cane to the desired height. Be sure the push button is fully protruded through the desired hole when finished. Repeat described procedure for each leg, at the end, check all four legs have the same height. Maximum endurable weight 130kg, 110kg only model RP729x, 60 kg for child model (RP731-RP736)

6. BEFORE USE

- Always check that the brakes work before using the walker, if available;
- Do not use the walker if it is damaged or malfunctioning please contact retailer for repairing;
- Check if the frame is damaged in order to guarantee a safe use for people and things;
- Users must travel along pedestrian paths and not on busy roads;
- Pay attention to the possibility of injury from entrapment; avoid placing your fingers between the moving parts of the walker. Pay attention closing and opening the walker.

7. WARNINGS FOR USE

This device provides greater stability, support and safety while moving. In any case, it can not be used by people with a functionality of the hand and / or arm visibly weak and even by people with balance problems. The walker can be used both outside and inside. We recommend using only on surfaces on which all four legs touch the ground. It should not be used on rocky terrain or on inclined surfaces because the walker can become unstable. The device is not designed for the transport of persons or other material

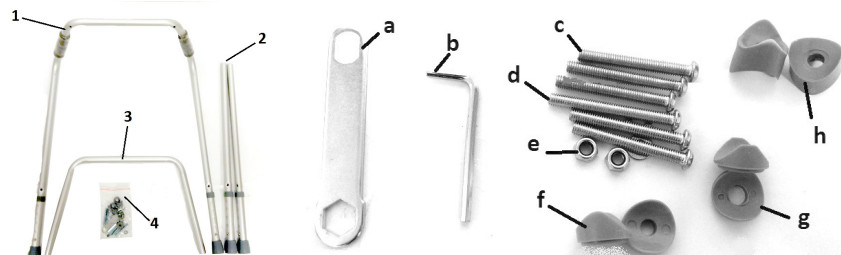
- Use extra caution on wet and slippery surfaces;
- DO NOT use with patient who exceed the weight indicated in the product label and in the technical features chapter of the present manual;
- Avoid use on slopes greater than 10 degrees and turn around on uphill or downhill the disabled must be assisted by an operator
- Avoid to leave the device in contact with water for long periods of time;
- Periodically check its sturdiness, the stability and the rubber tips are in good conditions. Replace them once worn out;
- In case of doubts about use, adjustment or maintenance, contact your dealer;

- Keep it away from children to avoid any possible risks;


8. ASSEMBLY INSTRUCTIONS FOR RP729X

Parts list

1. Main Frame (1pc)
2. Legs (2pcs)
3. Bent tube (1pc)
4. Assembly Kit (1pc)


- Open the assembly kit and start the assembly taking the main frame (1)
- Assemble on the main frame (1) : the legs (2), the saddle washers (g) and the SHORT screws (c). Do not tighten completely the screws.
- Assemble on the main frame (1): the bent tube (2) with saddle washers (h), the LONG screws (d), nuts and washers (s). Do not tighten completely the screws.
- Assemble on the main frame (1): the bent tube (2) with the saddle washers (f) and the SHORT screws (c). Do not tighten completely the screws.
- Tighten all screws, the walker is now ready to use. (Look the picture below)

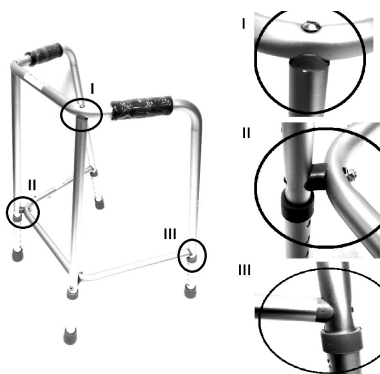9. HOW TO USE

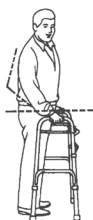
NOTE: Refer to the safety instructions in this manual. Use the walker as aid device for walking. To do this use the two handles and push forward the aid thus obtaining a support convenient and reliable for walking.

With your arm extended down, handgrips should be at your wrist height. With your hand placed on the handgrip, your elbow should be flexed by 20° - 30°.

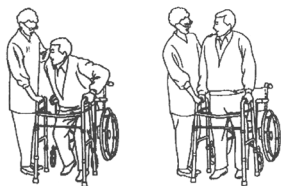- Pay attention on the presence of electrical wires, wet surfaces, rugs, toys or debris, furniture, little steps and gaps in floors. Do not use the walker on stairs or escalators.

1. To transfer the patient from the wheelchair to the walker, be sure castors of the wheelchair and walker if available are locked.

- For the transfer and the first steps get help by an operator.

2. a) Walkers with tips: to walk lift it up and place it comfortably in front of you, ensuring all four legs are on the floor. Walk carefully taking, one step at a time.

- b) Walkers with castors: to walk push it comfortably in front of you and ensuring all four legs even the one with castors are on the floor. Walk carefully taking, one step at a time.

NOTE: The walker must always be grasped from the handles.

Do not push the walker too far away from the body. This can cause falls and inadequate loads.

10. MAINTENANCE

The Mopedia devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. For periodic inspection we mean an examination of the following parts:

- The walker frame and the connecting parts;
- Check mounting and operation of the castors with its brake;
- Check all the welded points;
- Check the wear of the tips, where present;

In case of repair are to be used only approved parts and accessories.

11. CLEANING AND DISINFECTION

11.1 Cleaning

Use only a damp cloth to remove dust, use a damp cloth with a mild soap to remove more difficult dirt. Dry thoroughly before use. Never use water with temperature above 30°C. Never use jet water or steam washing machine.

11.2 Disinfection

If you need to disinfect the device use a common disinfectant cleaner.

NOTE: Never use acids, alkalis or solvents such as acetone

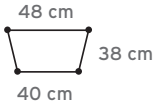
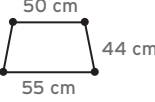
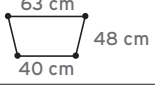
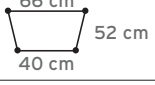
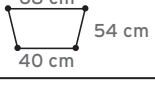
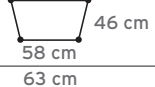
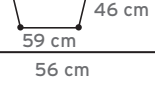
12. CONDITION OF DISPOSAL









12.1 General condition of disposal

In case of disposal of the device don't use the common container for municipal waste. We recommend to dispose the device in the appropriate disposal areas for recycling.

13. TECHNICAL FEATURES


Codes	Max user's weight	Adjustable height every 2,5 cm	Tips/castors	Base*
RP731 (childs)	60 kg	From 53 to 62,5 cm	4 tips	
RP736 (childs)	60 kg	From 41 to 60 cm	2 tips and 2 castors	
RP729S	110 kg	From 67 to 74,5 cm	4 tips	
RP729M		From 77 to 84,5 cm		
RP729L		From 87 to 94,5 cm		
RP730S	130 kg	From 65 to 75 cm	4 tips	
RP730M		From 73 to 83 cm		
RP730L		From 80 to 97,5 cm		
RP749S	130 kg	From 72 to 82 cm	4 tips	
RP749M		From 82 to 92 cm		
RP749L		From 85 to 103 cm		

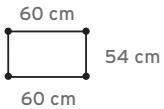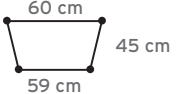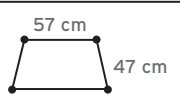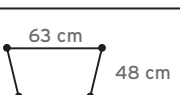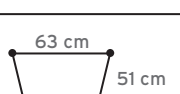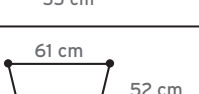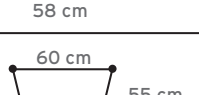Codes	Max user's weight	Adjustable height every 2,5 cm	Tips/castors	Base*
RP747	130 kg	From 50/70 to 79/96 cm	4 tips	
RP735S	130 kg	From 65 to 75 cm	2 tips and 2 castors	
RP735M	130 kg	From 72 to 82 cm	2 tips and 2 castors	
RP735L	130 kg	From 80 to 97,5 cm	2 tips and 2 castors	
RP745	130 kg	From 80 to 97,5 cm	2 tips and 2 castors	
RP737	130 kg	From 82 to 99 cm	2 tips and 2 castors	
RP734	130 kg	From 82 to 99 cm	2 tips and 2 castors	
RP744	130 kg	From 85 to 102,5 cm	2 tips and 2 castors	
RP738	130 kg	From 85 to 103 cm	4 castors	
RP733S	130 kg	From 68 to 78 cm	4 castors	

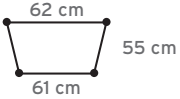
Codes	Max user's weight	Adjustable height every 2,5 cm	Tips/castors	Base*
RP733M	130 kg	From 75 to 85 cm	4 castors	
RP733L	130 kg	From 85 to 103 cm	4 castors	
RP741	100 kg	From 85 to 95 cm	4 castors 2 with brake	
RP742	130 kg	From 80 to 98 cm	4 castors with brake	
RP748	130 kg	From 76 to 91 cm	4 castors with brake	
RP748L	100 kg			

*Dimensions measured at the lower height adjustment.
The measures may change ± 1 cm

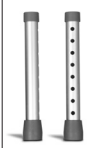13.1 Spare parts and accessories

RV7056
Spare legs pair

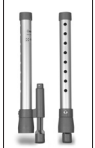
RV7052
Glide brake feet piece with attachments pair

RV7053
Glide brake wheels pair

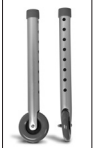RV7054-8
Swivel castors for walker pair (ø int. 2,6cm)

RV7055-8
Fixed castors pair - 8 holes

RV7055-5
Fixed castors pair - 5 holes

RV7154-8
Swivel castors pair - 8 holes Ø 12cm

RV7155-8
Fixed castors pair - 8 holes Ø 12cm

RV7035
Rubber tips for RP729 pair

RV7030
ø28mm Grey rubber tips - pair

RV7031
ø28mm Black rubber tips - pair

RV7045
Basket for one point foldable walkers

RV7046
Basket for one point foldable walkers

RV7114
Anti-brachium support - pair

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

15. REPAIR

15.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

15.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

15.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer .

16. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

17. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**MOPEDIA®****WARRANTY CERTIFICATE****Product** _____**Purchased on (date)** _____**Retailer** _____**Address** _____ **Town/city** _____**Sold to** _____**Address** _____ **Town/city** _____**MORETTI S.P.A.**

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C**

[illegible]

Note

[illegible]

MORETTI S.p.A.

Via Bruxelles, 3 - Meleto
52022 Caviglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com