MOPEDIA®

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MANUALE D'ISTRUZIONE

INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODE

RP779 4 in 1 function commode chair, foldable

2. INTRODUCTION

Thank you for purchasing a **MOPEDIA** multifunction chairs for bathroom Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The multifunction chairs for bathroom **MOPEDIA** by Moretti are intended to be used as an aid in shower and bathroom for people with walking difficulties. They include four functions in one: a stabilizer for water rise, support for water, commode chair and shower seat.

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SEAT AND SUPPORT FOR WC complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- READ carefully the entire manual before using the device
- For a correct use of the device always consult your physician or therapist
- Make sure that the product is delivered complete and keep it packed away from heat sources
- The useful life of the device is determined by the wear and tear of nonrepairable parts

and not more than 10 years

- DO NOT allow children to play or operate the device
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Max loading weight

Conditions of disposal

7. GENERAL DESCRIPTION

The multifunction chairs for bathroom **MOPEDIA** are made of anodized aluminum they include four functions in one: a stabilizer for water rise, support for water, commode chair and shower seat.

8. BEFORE USE

- Always check if there is any part worn out in order to ensure the device is used in complete safety for persons and property
- Check the correct assembly of the device

9. BEFORE ASSEMBLY

Unpack the product and check that no damage has occurred during transport. In the event that the product is damaged, please contact your nearest dealer.

10. ASSEMBLY INSTRUCTION

1. Insert the legs into the main structure
2. Insert the backrest
3. Insert the toilet stand, checking that it is properly fitted between the horizontal rails
4. Assemble the seat and seat cover of the toilet stand.

11. SEAT

Press the safety clip to adjust the height of the chairs by removing the inner part of the leg and bringing the holes to the desired height. Check that all four legs are adjusted to the same height.

12. USING WARNINGS

- Use caution on wet and slippery surfaces
- DO NOT use with patient who exceeds the weight indicated in the product label and in the technical features chapter of the present manual
- The chair can be used only on a hard, flat surface. Do not use the chair on stairs, inclines or declines
- Do not use the chair in a way other than that described in this manual
We disclaim any responsibility for improper use of the chair
- Avoid to leave the device in contact with water for long periods of time
- Periodically check the stability and hardness of the product as well as the status of the tips and castors. Replace worn once
- In case of doubts about use, adjustment or maintenance, contact your dealer
- Keep it away from children to avoid any possible risks

13. HOW TO USE

- Setting out to sit with your weight on the seat assembly. Do not sit by pushing on one side only
- Leaning always on both arms to get up and keep your balance
- In case of use of the chair by people with movement difficulties, help them always in the sitting or in getting up from the chair

CAUTION!

Do not move the aid with the patient seated

14. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

For periodic inspection means an examination of the following parts:

- The frame of the chair and connecting parts
- Check mounting and operation of the wheels with its brake
- Check all the welding points
- Check the wear of the tips where present

In case of repair, use only original spare parts and accessories.

15. CLEANING AND DISINFECTION

15.1 Cleaning

To clean the aid and the structure use neutral soap (in equal parts) mixed with water.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

15.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

16. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

17. TECHNICAL FEATURES

Codes	Max user's weight	Adjustable height armrest	Adjustable height seat	Base	Material used
RP779	130 kg	From 59 to 74 cm	From 42 to 58 cm		Aluminum frame

The measures have a tolerance ± 1 cm

18. SPARE PARTS AND ACCESSORIES

RPR783

Spare seat and cover

RPR784

Bucket, hook and cover

RV7024

\varnothing 28,5 mm
Tips - 4 pcs

(Picture with illustrative purpose only)

19. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below.

The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use.

Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

20. REPAIRS

20.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

20.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior

authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

20.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

21. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

22. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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