MOPEDIA®

MA RS825-RS826-RS827-RS829-RS831-RS833-RS838-RS839-RS908-RP770 02 A_ITA_0-2020

AUSILI DA BAGNO

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ITALIANO

INDICE

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REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RS825	Foldable shower chair with backrest and wheels
RS826	Foldable shower chair with backrest
RS827	Bath bench without backrest
RS829	Bath bench with backrest
RS831	Bath bench U-shape without backrest
RS833	Bath bench U-shape with backrest
RS838	Shower chair with armrests
RS839	Shower chair with armrests and backrest
RS908	Toilet safety rails height adjustable and removable
RP770	Commode frame height adjustable

2. INTRODUCTION

Thank you for purchasing a MOPEDIA bath products by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

3. INTENDED USE

The bath chair by Moretti is intended for disable patient assistance in bath tub.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SHOWER - BATHROOM CHAIRS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.

4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur.
- Keep the packed bath chair away from heat sources.
- The product use limit is defined by the wear of the parts.
- ALWAYS keep hands and fingers clear of moving parts to avoid injury.
- DO NOT allow children to play on or operate the bath chair.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

7. GENERAL DESCRIPTION

7.1 RS825-RS826

Foldable shower chair in painted steel and HDPE seat. Anti-slip rubber tips (RS826) or wheels (RS825).

7.2 RS827-RS829

Bath bench with or without backrest, aluminium frame, seat and backrest (if present) in HDPE. Height adjustable, anti-slip rubber tips.

7.3 RS831-RS833

Bath bench with or without backrest, aluminium frame, HDPE and U-shape seat and HDPE backrest (if present). Height adjustable, anti-slip rubber tips.

7.4 RS838-RS839

Shower chair with armrests and with or without backrest, aluminium frame, seat and backrest in HDPE. Height adjustable, anti-slip rubber tips.

7.5 RS908

Support stand. Support to get up from the toilet, made of painted steel, width adjustable for better adaptability. Padded armrests for better comfort, height adjustable legs, anti-slip rubber tips.

7.6 RP770

Safety handles for WC.

Realized in anodized aluminum tube, armrests in HDPE. Useful aid to sit and get up from the toilet. Adaptable to any type of toilet. Height-adjustable legs, antislip rubber tips.

8. ASSEMBLING

8.1 RS825-RS826

1. Open the frame;
2. Lock the frame of the seat on the two front supports;
3. Add the seat strengthening on the surface to lock it with the frame;
4. Insert the backrest inside the frame support tubes;
5. Add the 4 legs (RS826) or wheels (RS825).

8.2 RS827-RS829

1. Reverse the seat, take the two u-shape frames of the legs and tighten them to the center of the seat using one of the short screws provided;
2. For RS829, take the backrest supports and tighten them to the seat and to the leg supports using the long screws;
3. For RS829, tighten the backrest to their supports using the remaining short screws;
4. Add the 4 legs and fix them at the desired height.

8.3 RS831-RS833

1. Reverse the seat, take the two u-shape frames of the legs and tighten them to the center of the seat using one of the short screws provided;
2. For RS833, take the backrest supports and tighten them to the seat and to the leg supports using the long screws;
3. For RS833, tighten the backrest to their supports using the remaining short screws;
4. Add the 4 legs and fix them at the desired height.

8.4 RS838-RS839

1. Reverse the seat, take the two u-shape frames of the legs and tighten them to the center of the seat using one of the short screws provided;
2. For RS839, take the backrest supports and tighten them to the seat and to the leg supports using the long screws;
3. For RS839, tighten the backrest to their supports using the remaining short screws;
4. Add the 4 legs and fix them at the desired height.

8.5 RS908

1. Connect the two lateral frames with the connecting bars, for fixing, use the clips supplied;
2. Once the frame is fully assembled with all four clips, you can adjust the height of the legs (see section 11.3 How to use).

8.6 RP770

1. Position the handles in the same attachment point of the toilet seat;
2. Reposition the tablet and screw;
3. Adjust the height of the legs. Press the stop, slide the tube bringing it to the desired height. Release the stop, check the stability of the product and check if all legs are the same height.

9. BEFORE USE

- Check if the frame is damaged in order to guarantee a safe use for people and things.
- Check the correct assembly of the device.

10. CAUTIONS FOR USE

The manufacturer and distributor are not responsible for any responsibility for the consequences of improper use of the product.

- Always help the patient with walking difficulties to position themselves on the seat.
- Do not use with patients who exceed the maximum weight shown on the label placed on the product frame.
- Avoid sudden movements during positioning.
- Be careful on wet floors, on wet or slippery surfaces, it could make difficult the patient positioning.
- Do not store the product outside. Periodically check the stability of the product and the correct fastening screws and the status of protections rubber.
- In case of doubt for the use, the adjustment or the maintenance of the product, please contact your dealer.

WARNING!

Do not exceed the maximum load capacity for any reason!

11. HOW TO USE

11.1 RS825-RS826

Height adjustment

Remove the 4 clips. Adjust the height of the 4 legs (RS826) or wheels (RS825) according to your own need. Reposition the 4 clips in the corresponding holes to lock the legs (RS826) or wheels (RS825) at the selected height. Make sure the pin has completely penetrated the hole.

11.2 RS827-RS829-RS831-RS833-RS838-RS839

Height adjustment

Press the stop to adjust the seat by removing the barrel and bringing it into correspondence of the holes according to the desired height and the subjective manoeuvrability of the patient himself. Reposition the pin on the hole to lock the barrel. Check that all four legs have the same height. Place the seat on a flat and regular surface. To check seat stability before use.

11.3 RS908

Width adjustment

The frame is width adjustable, to adjust it, remove the connecting bars from the frame and secure it with the clip to the desired location

Height adjustment

Press the pin to remove the barrel and bring it into correspondence of the holes according to the desired height. Reposition the retainer on the hole to lock the barrel. Check that all four legs have the same height.

11.4 RP770

Height adjustment

Press the pin to remove the barrel and bring it into correspondence of the holes according to the desired height. Reposition the retainer on the hole to lock the barrel. Check that both legs have the same height.

12. CLEANING AND DISINFECTION

12.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water. Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

12.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

13. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

14. TECHNICAL FEATURES

RS825-RS826

*DIMENSIONS:  L: 53 CM  W: 54 CM  H: 90÷101 CM
SEAT: 40 X 40 X H49÷60 CM

HEIGHT OF THE ARMRESTS FROM THE SEAT: 15 CM

 MAX LOADING WEIGHT: 136 KG

RS827

*DIMENSIONS:  L: 46 CM  W: 46 CM  H: 45÷64 CM
SEAT: 51 X 31 X H45÷64 CM

 MAX LOADING WEIGHT: 136 KG

RS829

*DIMENSIONS:  L: 46 CM  W: 46 CM  H: 74÷93 CM
SEAT: 51 X 31 X H45÷64 CM

 MAX LOADING WEIGHT: 136 KG

RS831

*DIMENSIONS:  L: 45 CM  W: 45 CM  H: 41÷54 CM
SEAT: 48 X 33 X H41÷54 CM

 MAX LOADING WEIGHT: 136 KG

RS833

*DIMENSIONS:  L: 45 CM  W: 45 CM  H: 70÷83 CM
SEAT: 48 X 33 X H41÷54 CM

 MAX LOADING WEIGHT: 136 KG

RS838

*DIMENSIONS:  L: 46 CM  W: 40 CM  H: 47÷66 CM
SEAT: 41 X 34 X H47÷66 CM

 MAX LOADING WEIGHT: 136 KG

RS839

*DIMENSIONS:  L: 46 CM  W: 40 CM  H: 80÷99 CM
SEAT: 41 X 34 X H47÷66 CM

 MAX LOADING WEIGHT: 136 KG

RS908

*DIMENSIONS:  L: 62÷71 CM  W: 50 CM  H: 66÷77 CM
SEAT: 41 X 34 X H47÷66 CM

 MAX LOADING WEIGHT: 136 KG

RP770

* DIMENSIONS:  L: 54÷63 CM  W: 47 CM  H: 66÷77 CM
SEAT: 41 X 34 X H47÷66 CM

 MAX LOADING WEIGHT: 136 KG

* THE MEASUREMENTS REFER TO THE MEDIUM DIMENSIONS

15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

16. REPAIR

16.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

16.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

16.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

17. SPARE SPARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date. For spare parts and accessories, refer only to Moretti general catalog.

18. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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