MOPEDIA®

MA_RS890_RS891_02 A_ITA_01-2024

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ITALIANO

INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

- RS890X** - Tilting overbed table on wheels with two shelves, painted steel structure, automatic lift
- RS891X** - Single tier tilting overbed table on wheels, painted steel frame, automatic lifting

2. INTRODUCTION

Thank you for choosing an overbed table by MOPEDIA range, manufacturer by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. Overbed tables by MOPEDIA by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides some suggestions to correctly use the tables you have chosen and offers a lot of valuable advice for your safety. Please read carefully through the manual before using the overbed table. Should you have any queries, please contact your retailer for advice and assistance.

- Do not use the product for a purpose not indicated in this manual
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

3. INTENDED USE

The MOPEDIA bed trays are primarily intended for bedridden patient where it is expected the consumption of meals in bed or chair. However they are adaptable both in hospitals and home.

NOTE:

Check that the product has not been damaged during shipment. In case of damage do not use it and contact your dealer for further instructions.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the groups of BEDS TABLE complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

4.1 Standards and guidelines reference

Safety tests required by UNI EN 12182

5. GENERAL WARNINGS

Always pay close attention to the presence of moving parts that could cause trappings to the limbs and personal injury

- Always pay close attention to the presence of moving parts that could cause limb entrapment and injury
- For proper use of the device refer to this manual
- For proper use of the device consult your doctor or therapist
- Keep the packaged product away from any source of heat because the packaging is made of cardboard
- The shelf-life of the accessory is determined by the wear of parts that cannot be repaired and/or replaced
- Be careful when children are nearby
- Do not exceed the maximum payload

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

7. GENERAL DESCRIPTION

7.1 View and List of Components

RS890X

- A. Big screws - 2pcs
- B. Locking system (locking rod and ring, knob and cap) - 4pcs
- C. Small table - 1pc
- D. Perforated plate - 1pc
- E. Big table - 1pc
- F. Base - 1pc
- G. Castors - 4pcs
- H. Column - 1pc


RS891X

- A. Big screw - 2pcs
- B. Small screw - 2pcs
- C. Column - 1pc
- D. Perforated plate - 1pc
- E. Base - 1pc
- F. Castors - 4pcs

8. ASSEMBLING

Unpack the product and check that the contents correspond to the description of the parts of this manual.

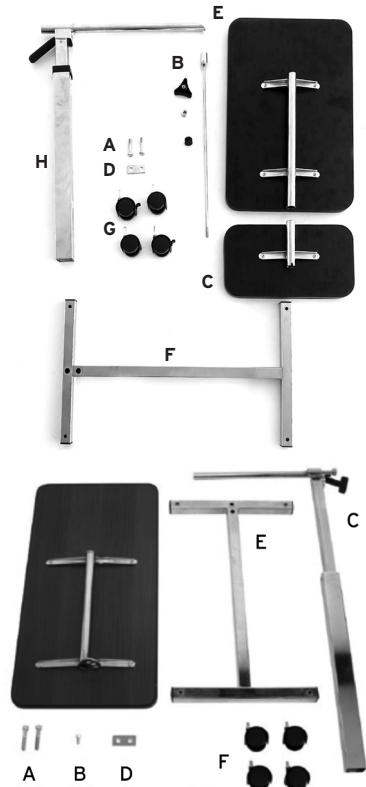Insert the height adjustment lever as shown in **Fig 1**, into the height adjustment slot, and screw it in as shown in **Fig 2**

ENGLISH

Fig 1*

Fig 2*

*Images for illustrative purposes

8.1 RS890 only:

Arrange the small shelf C and the large shelf E by aligning them on the floor and turning the metal components towards each other as shown in fig x, then insert the column tube H inside the aligned tubes of the shelves on the floor as shown in Fig 3

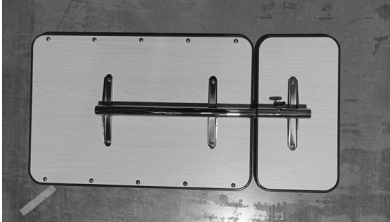
Fig 3*

8.2 RS891 only:

Fig 4*

Arrange the ground plane with the metal components facing each other as shown in Fig 4, then insert column tube H inside the aligned tubes of the ground planes as shown in Fig 4

Fig 5*

Insert the metal ring on the column side and screw in the knob until the column is firmly locked perpendicular to the plane Fig 5

Fig 6*

*Images for illustrative purposes

Fig 7*

Insert the cylindrical shims into the appropriate holes under the metal base as shown in Fig 6, then place the perforated plate at the holes on the base and insert both large screws as shown in Fig 7

Fig 8*

Fig 9*

Orient the base to the column as shown in Fig 8, then screw the previously prepared screws onto the column as shown in Fig 9 and tighten them firmly.

Fig 10*

Fig 11*

Insert the castors into the appropriate holes in the base, taking care to position the castors with the brake on the same side.

Fig 12*

Insert the castors into the appropriate holes in the base, taking care to position the castors with the brake on the same side
Fig 12

*Images for illustrative purposes

Fig 13*

Fig 14*

Remove the BLACK screw (and keep it) at the height of the black plastic support and replace it with the short screw as shown in figx and Fig 14

Note: If you wish to store or dismantle the table, be sure to replace the short screw with the original long black screw.

9. BEFORE USE

- Always check the status of the mechanical parts to guarantee they are used in total safety.
- Check the device is properly assembled. Pay particular attention to the locking of the knobs.

10. PRECAUTIONS FOR USE

- Do not exceed the maximum load capacity.
- Make sure that the floor surface supporting the overbed table is level.
- Make the adjustment operations without load.
- Check all components are well fixed.
- Always assist the patient during the elevation operation.
- Do not leave excessive loads on the table for extended periods of time.
- Placing excessive loads on the sides of the table can force it to tilt.

11. HOW TO USE

The assistant must place and adjust the overbed table with the maximum care to avoid sudden movements or dangerous oscillations. Remove the screw indicated on the column to unlock the lifting system.

RS890X

Tighten the handle to block at the required height. By turning the handle the big tray can rotate. Tighten the handle again to block it at the required inclination.

By pushing the handle the tray will lift up automatically. Pushing on the tray, it will come down. Tighten the lever to block it at the required height.

RS891X

By pushing the handle C1* the tray will lift up automatically.

*Images for illustrative purposes

Pushing on the tray, it will come down. Tighten the lever to block it at the required height.

By turning the handle **C2***, the tray can rotate. Tighten the handle again to block it at the required inclination. Images for illustrative purposes.

*Images for illustrative purposes

12. MAINTENANCE

Check the structure periodically: the correct positioning of the tubes, the connections, the hooks, and the complete tightening of screws and / or terminals.

13. CLEANING AND DISINFECTION

13.1 Cleaning

Use a damp cloth or a solution of lukewarm water and neutral soap.

Do not use abrasive substances, acids, alkali, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of the metal surfaces.

The manufacture company is not responsible for any damage caused by using materials that can damage the product surface or corrosive chemicals during cleaning.

13.2 Disinfection

If you need to disinfect the product, use a common delicate disinfectant neutral detergent.

14. CONDITIONS OF DISPOSAL

When disposing of the device, never use the normal return systems of municipal solid waste. It is recommended instead the disposal through the joint municipal recycling areas for the expected operations of recycling the materials used.

15. SPARE PARTS

For spare parts and accessories, please always refer to the MORETTI master catalogue. ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

16. TECHNICAL FEATURES

16.1 Dimension

RS890X

Height 74-119 cm

Small table Width 20,5 cm

Big table Width 56 cm

Depth 38,5 cm

Height from ground to base 9 cm

Wheel diameter 5 cm

Weight 11 Kg

Pay load 20 Kg

*Images for illustrative purposes

RS891X

Height 78-110 cm

Width 80 cm

Depth 50 cm

Height from ground to base 9 cm

Wheel diameter 5 cm

Weight 11 Kg

Pay load 20 Kg

16.2 Technical specifications**RS890X - RS891X**

- Painted and chromed steel frame
- Automatic adjustable height with lever
- Laminate scratch resistant table
- Containment edge made in polypropylene
- Antishock and cut resistant edge made in polyethylene
- Reclining table in more positions
- Small table(*only for RS890X)
- Available in natural or brown color

17. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

18. REPAIRING**Warranty repair**

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the customer, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti customer service. Labour and delivery costs for an out-of-warranty reparation will be fully at the customer's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the customer receives the repaired product.

Non defective device

The customer is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the customer and all the return delivery costs will be at his expense.

19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

20. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

**MOPEDIA®**
WARRANTY CERTIFICATE**Product** _____**Purchased on (date)** _____**Retailer** _____**Address** _____ **Town/city** _____**Sold to** _____**Address** _____ **Town/city** _____**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com** **MADE IN P.R.C.**

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