MOPEDIA®

MA_RS950_RS951_RS952_RS953_RS954_02_A_ITA_07 2020

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ITALIANO

INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

- RS950 Safety bath handle 30cm
- RS951 Safety bath handle 40cm
- RS952 Safety bath handle 45cm
- RS953 Safety bath handle 60cm
- RS954 Safety bath handle 80cm

2. INTRODUCTION

Thank you for choosing a Safety bath handle from the MOPEDIA by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. The Safety bath handles from the MOPEDIA by MORETTI S.p.A. are designed and built to meet all your demands for a practical and correct use. This user manual provides some suggestions as how to use the item you have chosen correctly and offers a lots of valuable advices for your safety. Please read carefully this manual before using the metal ward screen. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The Safety bath handle from the MOPEDIA is a device made to support people with movement difficulty and allow the patient to get up and sit down on the WC toilet easier.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SAFETY BATH HANDLES complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- For a proper use of the device refer to this manual
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The shelf-life of the accessory is determined by the wear of parts that cannot be repaired and / or replaced.
- Before using the device, always check that the mounting screws are properly tightened.
- Be careful when children are nearby.
- Paying close attention to prevent injury and / or property damage when moving the device
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

7. GENERAL DESCRIPTION

The Safety bath handle from the MOPEDIA is a device made in white powder coated steel with tubular \varnothing 25mm. fastening plates at wall with three holes. Can be installed it to your liking on a vertical or horizontal wall

List of the parts

1. Handle (Pz.1)
2. Fixing Kit (Pz.1)

8. ASSEMBLY

WARNING!

Perform these operations paying close attention to prevent injury and / or property damage when moving the device

8.1 Installation instructions

- Unpack the product.
- Drill the wall as the 6 holes in the handle.
- Insert the plastic plugs on the wall.
- Place the handle on the wall interposing the washer.
- Insert screws and tighten very well.
- At this point the Safety bath handle is mounted and ready for use.

9. PRECAUTIONS FOR USE

WARNING!

Perform these operations paying close attention to prevent injury and / or property damage when moving the device

- Before each use always check the state of wear of the mechanical parts to ensure a safe use of the device for persons and things.
- Before each use check the correct assembly of the device, with particular attention to the joints connecting screws.

10. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

- More specifically, please check that all screws and bolts are well tightened.

11. CLEANING AND DISINFECTION

11.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

11.2 Disinfection

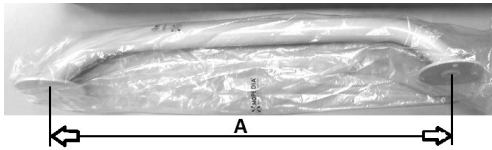
If you need to disinfect the product use a common disinfectant neutral detergent.

12. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

13. TECHNICAL FEATURES

- White painted steel tube
- 25mm diameter with wall mounting plates
- Can be used horizontally or vertically
- Gaskets for better adhesion to the wall

Measures:

RS950	A=30cm
RS951	A=40cm
RS952	A=45cm
RS953	A=60cm
RS954	A=80cm

14. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

14.1 Warranty repair

Warranty repair If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

Repair not covered by warranty A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

Non defective device The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

14.2 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

14.3 Exempt clauses

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exoneration or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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