**LEVITAS®**

MA ST840-ST841-ST850-ST851-ST852-ST853-ST855-ST856-ST857-ST858-ST860-ST861-ST862-ST863 01 A ITA\_01-2024

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---

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## INDICE

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**1. CODES**

|         |   |
|---------|---|
| ST840/1 | 1-section PVC air cell cushion, 35X40 h 6 cm                    |
| ST840/2 | 1-section PVC air cell cushion, 40X40 h 6 cm                    |
| ST840/3 | 1-section PVC air cell cushion, 45X40 h 6 cm                    |
| ST840/4 | 1-section PVC air cell cushion, 50X45 h 6 cm                    |
| ST840/6 | 1-section PVC air cell cushion, 35X40 h 9 cm                    |
| ST840/7 | 1-section PVC air cell cushion, 40X40 h 9 cm                    |
| ST840/8 | 1-section PVC air cell cushion, 45X40 h 9 cm                    |
| ST840/9 | 1-section PVC air cell cushion, 50X45 h 9 cm                    |
| ST841/1 | 2-sections PVC air cell cushion, 35X40 h 6 cm                   |
| ST841/2 | 2-sections PVC air cell cushion, 40X40 h 6 cm                   |
| ST841/3 | 2-sections PVC air cell cushion, 45X40 h 6 cm                   |
| ST841/4 | 2-sections PVC air cell cushion, 50X45 h 6 cm                   |
| ST841/6 | 2-sections PVC air cell cushion, 35X40 h 9 cm                   |
| ST841/7 | 2-sections PVC air cell cushion, 40X40 h 9 cm                   |
| ST841/8 | 2-sections PVC air cell cushion, 45X40 h 9 cm                   |
| ST841/9 | 2-sections PVC air cell cushion, 50X45 h 9 cm                   |
| ST850/1 | 1-section PVC air cell cushion, 36x36 cm - h 6 cm               |
| ST850/2 | 2-section PVC air cell cushion, 36x36 cm - h 6 cm               |
| ST851/1 | 1-section PVC air cell cushion, 40x40 cm - h 6 cm               |
| ST851/2 | 2-section PVC air cell cushion, 40x40 cm - h 6 cm               |
| ST852/1 | 1-section PVC air cell cushion, 46x40 cm - h 6 cm               |
| ST852/2 | 2-section PVC air cell cushion, 46x40 cm - h 6 cm               |
| ST853/1 | 1-section PVC air cell cushion, 50x46 cm - h 6 cm               |
| ST853/2 | 2-section PVC air cell cushion, 50x46 cm - h 6 cm               |
| ST855/1 | 1-section PVC air cell cushion, 36x36 cm - h 10 cm              |
| ST855/2 | 2-section PVC air cell cushion, 36x36 cm - h 10 cm              |
| ST856/1 | 1-section PVC air cell cushion, 40x40 cm - h 10 cm              |
| ST856/2 | 2-section PVC air cell cushion, 40x40 cm - h 10 cm              |
| ST857/1 | 1-section PVC air cell cushion, 46x40 cm - h 10 cm              |
| ST857/2 | 2-section PVC air cell cushion, 46x40 cm - h 10 cm              |
| ST858/1 | 1-section PVC air cell cushion, 50x46 cm - h 10 cm              |
| ST858/2 | 2-section PVC air cell cushion, 50x46 cm - h 10 cm              |
| ST860/1 | 1-section polyurethane (PU) air cell cushion, 35x41 cm - h 5 cm |
| ST860/2 | 2-section polyurethane (PU) air cell cushion, 35x41 cm - h 5 cm |
| ST861/1 | 1-section polyurethane (PU) air cell cushion, 41x41 cm - h 5 cm |
| ST861/2 | 2-section polyurethane (PU) air cell cushion, 41x41 cm - h 5 cm |
| ST862/1 | 1-section polyurethane (PU) air cell cushion, 47x41 cm - h 5 cm |
| ST862/2 | 2-section polyurethane (PU) air cell cushion, 47x41 cm - h 5 cm |
| ST863/1 | 1-section polyurethane (PU) air cell cushion, 47x47 cm - h 5 cm |
| ST863/2 | 2-section polyurethane (PU) air cell cushion, 47x47 cm - h 5 cm |

**2. INTRODUCTION**

Thank you for purchasing a LEVITAS air cell cushion by Moretti S.p.A. product range. Moretti cushions have been designed and manufactured to meet all your needs for practical, correct and safe use. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

### 3. INTENDED USE

The LEVITAS by Moretti air cell cushion is conceived for the patient balance and for the protection against bed sores and tissue injuries. It guarantees an undifferentiated weight distribution, thanks to the pressure support in every position. The cushion follows the body of the patient and the ischemic parts can sink into it.

Who can use the air cells cushion?

- Everyone who has problems being seated.
- Wheelchair users.
- Anyone that need to stay seated for a long period (e.g. the aftercare of an incident or an illness, elderly people, etc.).
- Anyone risking to develop pressure sores.
- Anyone has to adjust pelvic obliquity.

Why do we use air cells cushions?

- We use it to position properly the pelvic zone and the lower limbs. Pelvic zone and lower limbs are a solid base for trunk, head, neck and upper limbs.
- For pressure relief.
- For prevention of skin injuries.

#### **WARNING!**

We would like to inform you that the most effective way to prevent the development of bed sores is to change often position, moving the patient frequently.

### 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of AIR CELLS CUSHIONS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

### 5. WARNINGS

#### 5.1 General warnings

- For correct use of the device, refer to the following manual carefully.
- Always consult your doctor or therapist for correct use of the device.
- Keep the product packaged away from any heat source as the packaging is made of cardboard.
- The useful life of the device is determined by the wear of non-repairable and/or

non-replaceable parts.

- Always pay attention to the presence of children.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

### 5.2 Warnings for use

- Avoid exposure to the sun and heat sources.
- The maximum weight supported by the air cell cushion is 130 Kg. Do not overload.
- Do not use the cushion if it is damaged.
- For hygiene reasons, each pillow should be used by one patient and must be cleaned regularly.
- Keep the cushion away from sharp objects.
- Before sitting down, make sure that no unusual objects are left on the cushion.
- The cushion must be used individually, do not use with other heating or freezing equipment.
- Always use the cushion with a non-slip cover to prevent accidents. Please do not sit on the pillow without cover.

### 6. SYMBOLS

|  |                              |
|--|------------------------------|
|    | Product code                 |
|    | Unique Device Identification |
|    | CE mark                      |
|    | Manufacturer                 |
|    | Batch Lot                    |
|    | Read the instruction manual  |
|  | Medical Device               |
|  | Conditions of disposal       |
|  | Do not bleach                |
|  | Do not iron                  |
|  | Do not dry clean             |
|  | Do not tumble dry            |

## 7. GENERAL DESCRIPTION

The air cell cushion operates with an outward air micro-interchange, which maintains the patient dry, and guarantee him the best comfort. It guarantees a perfect weight distribution, exploiting to the utmost pressure relief in every position. It adapts itself to the individual shapes of the patient and allows the full immersion of the sciatic points.

The LEVITAS by Moretti air cell cushions are available in the following versions:

### PVC cushion

|         |  |
|---------|--|
| ST840/* | <b>1-section PVC air cell cushion</b>  |
| ST841/* | <b>2-sections PVC air cell cushion</b> |
| ST85*/1 | <b>1-section PVC air cell cushion</b>  |
| ST85*/2 | <b>2-section PVC air cell cushion</b>  |

### PU cushion

|         |                                      |
|---------|--------------------------------------|
| ST86*/1 | <b>1-section PU air cell cushion</b> |
| ST86*/2 | <b>2-section PU air cell cushion</b> |

The kit of the air cell cushion is composed by:

- 1 air cell cushion
- 1 cover for PVC models
- 1 cover for PU models
- 1 pump
- 1 repairing kit
- 1 instruction manual

## 8. HOW TO USE

### 8.1 Before use

- Ensure the cushion does not have any damages which can compromise the operation.
- Verify that the cushion is correctly inflated.
- Verify that any object is on the cushion before have seat.

### 8.2 Use of the cushion

All the cushions are equipped with inflated pump.

1. Place the cushion on the chair.
2. Open the valve counter clockwise and connect the pump to the cushion (the part that must rotate is the one provided with knurling for the handle).
3. Connect the pump tube to the valve.
4. Inflate the cushion slowly. Do not exceed with the inflation.
5. Close the cushion valve clockwise.
6. Remove the pump from the valve. If the cushion has more than 1 valve repeat the process with other valves.
7. Put the cushion into the cover.
8. Put the cushion on the wheelchair. Seat on the cushion and check that pelvic zone is not in contact with the base.
9. Check that the patient is sank in the cushion for no more than 1-2cm. We suggest to put an hand between the patient and the cushion.
10. Adjust the pressure in order to guarantee the best comfort. If necessary deflate by unscrewing the valve and let the air escape.

### 8.3 Recommendation

Adjust cushion pressure with the pump every day.

Working temperature: -5°;+50° C

### 8.4 Suggestion

Check cushion pressure if the environmental temperature or the altitude changed.

### 8.5 Use of the cover

The air cushion is provided with a cover that protects the surface. The base of the cover is in non-slip material that blocks it on the chair and improves the balance of the user.

To insert the cushion in the cover, follow the instructions below:

1. Lay the cover with the nons-lip black base downward and the zip upward;
2. Insert the cushion with the air cell side upward;
3. Be sure that valves are placed outward (valves have corresponding holes in the cover);
4. Press the cushion gently into the cover. It may be necessary to shake slightly a few times the cushion;
5. Close the zip carefully. The cushion is now ready for use.

## 9. MAINTENANCE

The LEVITAS devices by Moretti are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

## 10. HOW TO REPAIR THE CUSHION

1. Deflate the cushion.
2. Scratch the damaged part with sandpaper until it's smooth.
3. If the cushion is accidentally punctured and the diameter of the damage area is lower than 1 cm, use the patch provided with the cushion to prevent the damage from becoming larger.
4. If the diameter of the damaged area is more than 1 cm, stop using the cushion.

## 11. CLEANING AND DISINFECTION

Do not bleach

Do not iron

Do not dry clean

Do not tumble dry

### 11.1 Cushion

1. Clean the cushion with water and mild soap.
2. Wipe the cushion with a damp cloth and rinse out with cold water.
3. Dry it.

#### **WARNING!**

- Use mild soap.
- Do not clean in the washing-machine.
- Do not use an autoclave to clean the cushion.
- Keep the product far from sun rays.

### 11.2 Cover

- Washable by hand or washing machine at 30°C.
- Air dry only.

#### WARNING!

- Do not use drier, heater or bleacher.
- Do not iron.

### 12. CONDITION OF DISPOSAL

In case of disposal, do not use the inserting container for municipal waste. We recommend disposing the air cushion in the appropriate disposal areas for recycling.

### 13. TECHNICAL FEATURES

#### ST84\*/1

Dimensions:  L: 35 cm  P: 40 cm  H: 6 cm

Max weight: 130 kg

Cushion material: PVC

Cover material: PL e PVC

#### ST84\*/2

Dimensions:  L: 40 cm  P: 40 cm  H: 6 cm

Max weight: 130 kg

Cushion material: PVC

Cover material: PL e PVC

#### ST84\*/3

Dimensions:  L: 45 cm  P: 40 cm  H: 6 cm

Max weight: 130 kg

Cushion material: PVC

Cover material: PL e PVC

#### ST84\*/4

Dimensions:  L: 50 cm  P: 45 cm  H: 6 cm

Max weight: 130 kg

Cushion material: PVC

Cover material: PL e PVC

#### ST84\*/6

Dimensions:  L: 35 cm  P: 40 cm  H: 9 cm

Max weight: 130 kg

Cushion material: PVC

Cover material: PL e PVC

**ST84\*/7**

Dimensions:  L: 40 cm  P: 40 cm  H: 9 cm

 Max weight: 130 kg

Cushion material: PVC

Cover material: PL e PVC

**ST84\*/8**

Dimensions:  L: 45 cm  P: 40 cm  H: 9 cm

 Max weight: 130 kg

Cushion material: PVC

Cover material: PL e PVC

**ST84\*/9**

Dimensions:  L: 50 cm  P: 45 cm  H: 9 cm

 Max weight: 130 kg

Cushion material: PVC

Cover material: PL e PVC

**ST850/\***

Dimensions:  L: 36 cm  P: 36 cm  H: 6 cm

 Max weight: 130 kg

Cushion material: PVC

Cover material: Nylon and PU

**ST851/\***

Dimensions:  L: 40 cm  P: 40 cm  H: 6 cm

 Max weight: 130 kg

Cushion material: PVC

Cover material: Nylon and PU

**ST852/\***

Dimensions:  L: 46 cm  P: 40 cm  H: 6 cm

 Max weight: 130 kg

Cushion material: PVC

Cover material: Nylon and PU

**ST853/\***

Dimensions:  L: 50 cm  P: 46 cm  H: 6 cm

 Max weight: 130 kg

Cushion material: PVC

Cover material: Nylon and PU

**ST855/\***

Dimensions:

 L: 36 cm
  P: 36 cm
  H: 10 cm

Max weight:

130 kg

Cushion material:

PVC

Cover material:

Nylon and PU

**ST856/\***

Dimensions:

 L: 40 cm
  P: 40 cm
  H: 10 cm

Max weight::

130 kg

Cushion material:

PVC

Cover material:

Nylon and PU

**ST857/\***

Dimensions:

 L: 46 cm
  P: 40 cm
  H: 10 cm

Max weight:

130 kg

Cushion material:

PVC

Cover material:

Nylon and PU

**ST858/\***

Dimensions:

 L: 50 cm
  P: 46 cm
  H: 10 cm

Max weight:

130 kg

Cushion material:

PVC

Cover material:

Nylon and PU

**ST860/\***

Dimensions:

 L: 35 cm
  P: 41 cm
  H: 5 cm

Max weight:

130 kg

Cushion material:

PU

Cover material:

Nylon and PU

**ST861/\***

Dimensions:

 L: 41 cm
  P: 41 cm
  H: 5 cm

Max weight:

130 kg

Cushion material:

PU

Cover material:

Nylon and PU

**ST862/\***

Dimensions:

 L: 47 cm
  P: 41 cm
  H: 5 cm

Max weight:

130 kg

Cushion material:

PU

Cover material:

Nylon and PU

**ST863/\***

Dimensions:

 L: 47 cm
  P: 47 cm
  H: 5 cm

Max weight:

130 kg

Cushion material:

PU

Cover material:

Nylon and PU

**WARNING!**

Resistance to ignition: the materials used for cover have been treated with methods specified in Directive EN1021-1/2.

**14. WARRANTY**

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

**15. REPAIR**
**15.1 Warranty repair**

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

**15.2 Repair not covered by warranty**

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

**15.3 Non-defective device**

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

**16. SPARE SPARTS**

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

### 17. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free.

The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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