MOPEDIA®

MA AU120_121_122_123 02 A_ITA_08 2020

PINZE PRENSILI

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ITALIANO

INDICE

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Via _____ Località _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

NOTE

MOPEDIA®

MA AU120_121_122_123 02 A_ENG_08 2020

HANDY GRIP

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.4
8. BEFORE TO USE.....	PAG.5
9. CAUTION TO USE	PAG.5
10. HOW TO USE.....	PAG.6
11. MAINTENANCE	PAG.6
12. CLEANING AND DISINFECTION	PAG.6
12.1 Cleaning.....	pag. 6
12.2 Disinfection	pag. 6
13. CONDITIONS OF DISPOSAL	PAG.6
14. SPARE PARTS AND ACCESSORIES.....	PAG.6
15. SPECIFICATIONS	PAG.7
15.1 AU120 Handy grip with swivel head and magnet - 67 cm	pag. 7
15.2 AU121 Handy grip with swivel head and antibrachial rest - 55cm	pag. 7
15.3 AU122 Handy grip with swivel head - 71cm	pag. 8
15.4 AU123 Handy grip - 74cm	pag. 8
16. WARRANTY	PAG.9
17. REPAIRING.....	PAG.9
17.1 Warranty repair.....	pag. 9
17.2 Repair not cover by warranty	pag. 9
17.3 Non defective device	pag. 9
18. SPARE PARTS.....	PAG.9
19. EXEMPT CLAUSES	PAG.9

CE | Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

- AU120** - Handy grip with swivel head and magnet - 67 cm
- AU121** - Handy grip with swivel head and antibrachial rest - 55cm
- AU122** - Handy grip with swivel head - 71cm
- AU123** - Handy grip - 74cm

2. INTRODUCTION

Thank you for choosing an accessory MOPEDIA by Moretti S.P.A. The handy grip have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some suggestions for proper use of the accessory of your choice and valuable tips for your safety. You should read carefully all of this manual before using the Handy grip. If in doubt please contact your dealer, who will help and advise you properly.

3. INTENDED USE

The handy grip by Moretti S.P.A. are designed to help people with hands and fingers limited mobility to keep objects.

CAUTION!

- Do not use the following device for purposes other than those described in this manual.
- Moretti S.p.A. disclaims any liability for damage resulting from improper use of the device or use other than that described in this manual.
- We reserve the right to make changes to the device and this manual without notice in order to improve the characteristics.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of HANDY GRIP complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- For a proper use of the device refer to this manual.
- For a proper use of the device to consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the device is defined by the wear of parts.
- Always pay close attention to the presence of moving parts that could cause personal injury and limb entrapment.
- Always pay attention to the presence of children.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

7. GENERAL DESCRIPTION

The range of handy grip Moretti S.P.A. it consists of 4 models with aluminum frame, coated, plastic grab, anti-slip rubber grip and ergonomic handle.

AU120 Handy grip with swivel head and magnet - 67 cm

The head is adjustable horizontally, vertically or in the intermediate positions, with a saw-toothed extremity to grab small objects; provided with a small hook placed in the upper end to reach clothes or something else and a small magnet to attract small metal objects.

AU121 Handy grip with swivel head and antibrachial rest - 55cm

The head is adjustable horizontally, vertically or in the intermediate positions. Fork has got particular anti-slip suction cups that allow you to collect a wide variety of objects. The ergonomic handle allows to use at best the strength of the hand and to obtain a more effective grip in any situation. The anti-brachial rest is suitable for people with impaired hand function helps support the weight of the grasped object. Furthermore, the handle has got a button that allows to lock the fork in closed position once grabbed the object, this requiring less effort and less prolonged in time.

AU122 Handy grip with swivel head - 71cm

The head is adjustable horizontally, vertically or in the intermediate position, it is covered with anti-slip rubber for a secure grip for any object.

AU123 Handy grip - 74cm

The head is equipped with a prehensile saw-toothed extremity to grab small objects.

8. BEFORE TO USE

- Always check the wear of the mechanical parts in the game so as to ensure use of the device in total safety for persons and things.

9. CAUTION TO USE

- Don't try to grasp heavy objects (over the maximum loading weight **2Kg**) as it could be difficult to lift them in safety, due to the device length (the object lifted can fall and break causing personal injuries).

10. HOW TO USE

The use of the Moretti handy grip is easy. It is necessary to grasp the device and approach it with caution to an object, then press the lever on the handle to grab the object. Once you moved and positioned the object, release the lever to lay down the object.

The AU120 and AU121 models are equipped with grasping tool adjustable horizontally, vertically or in the intermediate positions. To rotate the grasping tool it is necessary to grab the stem, with the other hand grab the fork and pull it towards the top, rotate it appropriately and put it back on the stem until full locking.

The AU121 model is equipped the handle has got a button that allows to lock the fork in closed position once grabbed the object, this requiring less effort and less prolonged in time.

The AU122 model is equipped with grasping tool adjustable horizontally, vertically or in the intermediate positions To rotate the forks is necessary to grasp with one hand the handle of the gripper and with the other hand the fork. Pull and turn the handle to the desired position, making sure that everything stays fully assembled.

11. MAINTENANCE

The MOPEDIA devices by Moretti S.P.A. are checked carefully and supplied with CE mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

12. CLEANING AND DISINFECTION

12.1 Cleaning

Use only a damp cloth and mild soap. Afterwards dry thoroughly before using.

12.2 Disinfection

If you need to disinfect the product using a common disinfectant.

Note: Never use acids, alkalines or solvents such as acetone or thinner.

13. CONDITIONS OF DISPOSAL


When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

14. SPARE PARTS AND ACCESSORIES

For spare parts and accessories, refer to the MORETTI master catalogue.

15. SPECIFICATIONS

15.1 AU120 Handy grip with swivel head and magnet - 67 cm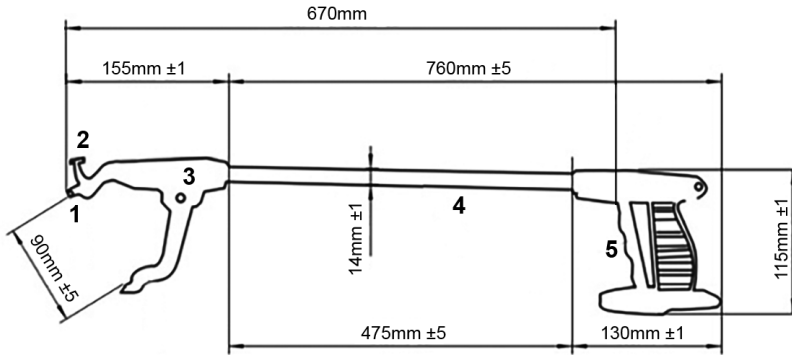


1. Magnet
2. Hook
3. Head
4. Main tube
5. Handle with trigger

Technical specifications

- Tube: anodized aluminum
- Head: anti-slip rubber grips to grasp objects
- Ergonomic handle: nylon
- Jaw width: 90mm
- Trigger: ABS

15.2 AU121 Handy grip with swivel head and antibrachial rest - 55cm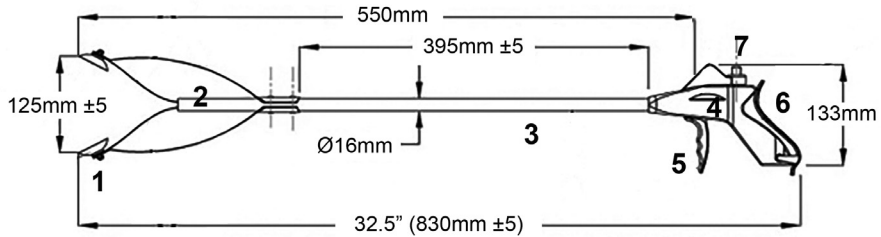


1. Sucker
2. Head
3. Main tube
4. Handle
5. Trigger
6. Antibrachium rest
7. Stopper

Technical specifications

- Tube: anodized aluminum
- Head: anti-slip rubber suckers
- Ergonomic handle: ABS
- Antibrachium rest: PC
- Jaw width: 125mm
- Trigger: ABS and TPR

15.3 AU122 Handy grip with swivel head - 71cm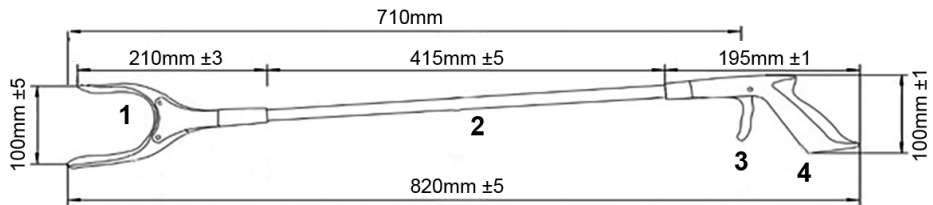


1. Head
2. Main tube
3. Trigger
4. Handle

Technical specifications

- Tube: anodized aluminum
- Head: anti-slip rubber grips to grasp objects
- Ergonomic handle: PP and TPR
- Jaw width: 100mm
- Trigger: PP

15.4 AU123 Handy grip - 74cm

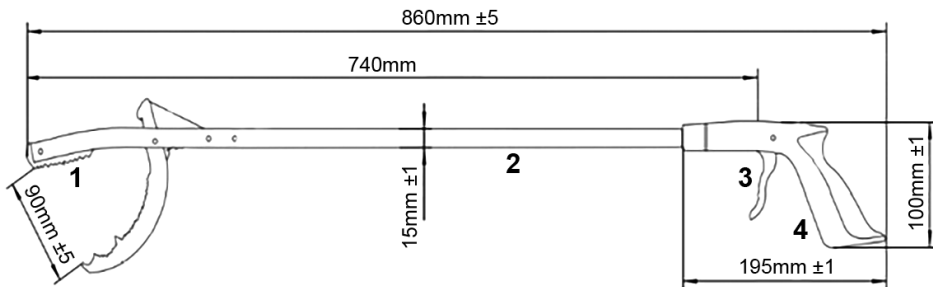1. Head
2. Main tube
3. Trigger
4. Handle

Technical specifications

- Tube: anodized aluminum
- Head: anti-slip rubber grips to grasp objects
- Ergonomic handle: PP and TPR
- Jaw width: 90mm
- Trigger: PP

16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17. REPAIRING

17.1 Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

17.2 Repair not cover by warranty

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

17.3 Non defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

18. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

19. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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