ITALIANO

MA CP762\_CP765\_CP770\_CP775\_CP780\_CP785\_CP787\_CP790 03 B\*\_ITA\_07-2023

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---

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## INDICE

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MA CP762\_CP765\_CP770\_CP775\_CP780\_CP785\_CP787\_CP790 03 B\*\_ENG\_07\*2023

# LIGHTWEIGHT WHEELCHAIRS HELIOS SERIES

---

## INSTRUCTION MANUAL

ENGLISH

## INDEX

<b>1. CODES</b> .....	<b>PAG.3</b>
<b>2. INTRODUCTION</b> .....	<b>PAG.3</b>
<b>3. INTENDED USE</b> .....	<b>PAG.4</b>
<b>4. DECLARATION OF CONFORMITY</b> .....	<b>PAG.4</b>
<b>5. GENERAL WARNINGS</b> .....	<b>PAG.4</b>
<b>6. SYMBOLS</b> .....	<b>PAG.5</b>
<b>7. GENERAL DESCRIPTION</b> .....	<b>PAG.5</b>
7.1 Parts view .....	pag. 5
<b>8. ASSEMBLING/DISASSEMBLING</b> .....	<b>PAG.5</b>
8.1 Opening.....	pag. 5
8.2 Closing.....	pag. 6
8.3 Armrest.....	pag. 6
8.4 Install/remove the swing away legrest.....	pag. 7
8.5 Adjust the swing away legrest .....	pag. 7
8.6 Assembly/Disassembly of CP762-xx backrest .....	pag. 7
8.7 Elastic adjustable backrest (CP790-xx) .....	pag. 7
8.8 Adjust the footplate (CP780-xx, CP785-xx, CP787-xx, CP790-xx) .....	pag. 8
8.9 Adjust the height of rear wheels (CP780-xx, CP785-xx, CP790-xx).....	pag. 8
8.10 Adjust the front wheels (CP780-xx, CP785-xx, CP790-xx) .....	pag. 8
8.11 Adjust the depth (CP790-xx).....	pag. 8
8.12 Adjust the height of the push handles (CP790-xx) .....	pag. 9
<b>9. BEFORE USE</b> .....	<b>PAG.9</b>
<b>10. USING WARNINGS</b> .....	<b>PAG.9</b>
<b>11. HOW TO USE</b> .....	<b>PAG.9</b>
11.1 Get in the wheelchair.....	pag. 10
11.2 Get out of the wheelchair .....	pag. 10
11.3 Stairs and height difference .....	pag. 10
11.4 Use the armrest.....	pag. 10
11.5 Use the 60cm wheels.....	pag. 10
11.6 Carrying .....	pag. 10
<b>12. MAINTENANCE</b> .....	<b>PAG.11</b>
<b>13. CLEANING AND DISINFECTION</b> .....	<b>PAG.11</b>
13.1 Cover and padding.....	pag. 11
13.2 Frame and metallic parts .....	pag. 11
13.3 Brakes and wheels .....	pag. 11
13.4 Disinfection.....	pag. 11
<b>14. CONDITIONS OF DISPOSAL</b> .....	<b>PAG.11</b>
<b>15. TECHNICAL FEATURES</b> .....	<b>PAG.11</b>
15.1 Dimensions .....	pag. 11
15.2 Technical specification.....	pag. 13
<b>16. ACCESSORIES</b> .....	<b>PAG.13</b>
<b>17. WARRANTY</b> .....	<b>PAG.13</b>
<b>18. REPAIRING</b> .....	<b>PAG.14</b>
18.1 Warranty repair .....	pag. 14
18.2 Repair not cover by warranty.....	pag. 14
18.3 Non defective device .....	pag. 14
<b>19. SPARE PARTS</b> .....	<b>PAG.14</b>
<b>20. EXEMPT CLAUSES</b> .....	<b>PAG.14</b>

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT  
AND OF THE COUNCIL of 5 April 2017 concerning medical devices**1. CODES**

<b>CP762-43</b>	Series Helios Go!2 transit model wheelchair - 43 cm
<b>CP762-46</b>	Series Helios Go!2 transit model wheelchair - 46 cm
<b>CP762-48</b>	Series Helios Go!2 transit model wheelchair - 48 cm
<b>CP765-40</b>	Series Helios Skinny self-propelled wheelchair - 40 cm
<b>CP765-43</b>	Series Helios Skinny self-propelled wheelchair - 43 cm
<b>CP770-40</b>	Series Helios Smart self-propelled wheelchair - 40 cm
<b>CP770-43</b>	Series Helios Smart self-propelled wheelchair - 43 cm
<b>CP770-46</b>	Series Helios Smart self-propelled wheelchair - 46 cm
<b>CP770-48</b>	Series Helios Smart self-propelled wheelchair - 48 cm
<b>CP770-50</b>	Series Helios Smart self-propelled wheelchair - 50 cm
<b>CP775-40</b>	Series Helios Smart Go! transit model wheelchair - 40 cm
<b>CP775-43</b>	Series Helios Smart Go! transit model wheelchair - 43 cm
<b>CP775-46</b>	Series Helios Smart Go! transit model wheelchair - 46 cm
<b>CP775-48</b>	Series Helios Smart Go! transit model wheelchair - 48 cm
<b>CP775-50</b>	Series Helios Smart Go! transit model wheelchair - 50 cm
<b>CP780-40</b>	Series Helios Act self-propelled model wheelchair - 40 cm
<b>CP780-43</b>	Series Helios Act self-propelled model wheelchair - 43 cm
<b>CP780-46</b>	Series Helios Act self-propelled model wheelchair - 46 cm
<b>CP780-48</b>	Series Helios Act self-propelled model wheelchair - 48 cm
<b>CP780-50</b>	Series Helios Act self-propelled model wheelchair - 50 cm
<b>CP785-40</b>	Series Helios Dyne self-propelled model wheelchair - 40 cm
<b>CP785-43</b>	Series Helios Dyne self-propelled model wheelchair - 43 cm
<b>CP785-46</b>	Series Helios Dyne self-propelled model wheelchair - 46 cm
<b>CP785-48</b>	Series Helios Dyne self-propelled model wheelchair - 48 cm
<b>CP785-50</b>	Series Helios Dyne self-propelled model wheelchair - 50 cm
<b>CP787-40</b>	Series Helios Dyne transit model wheelchair - 40 cm
<b>CP787-43</b>	Series Helios Dyne transit model wheelchair - 43 cm
<b>CP787-46</b>	Series Helios Dyne transit model wheelchair - 46 cm
<b>CP787-48</b>	Series Helios Dyne transit model wheelchair - 48 cm
<b>CP787-50</b>	Series Helios Dyne transit model wheelchair - 50 cm
<b>CP790-40</b>	Series Helios Pro self-propelled model wheelchair - 40 cm
<b>CP790-43</b>	Series Helios Pro self-propelled model wheelchair - 43 cm
<b>CP790-46</b>	Series Helios Pro self-propelled model wheelchair - 46 cm
<b>CP790-48</b>	Series Helios Pro self-propelled model wheelchair - 48 cm
<b>CP790-50</b>	Series Helios Pro self-propelled model wheelchair - 50 cm

**2. INTRODUCTION**

Thank you for purchasing a **ARDEA ONE** wheelchair by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

**Note:** check all parts for shipping damage and test before using. In case of damage, do not use the device. Contact your dealer for further instruction.

**WARNING!**

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without notice.

**3. INTENDED USE**

This wheelchair is designed for the mobility of people with mobility problems and can be used for self-propelling (if equipped with big rear wheels) or with the aid of an operator.

**4. DECLARATION OF CONFORMITY**

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of WHEELCHAIRS - HELIOS ARDEA ONE complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

**5. GENERAL WARNINGS**

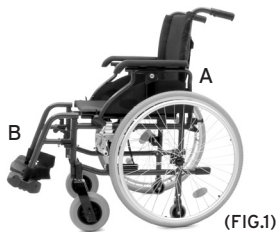
- **DO NOT** use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before using this equipment otherwise injury or damage may occur.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the wheelchair.
- Keep the packed wheelchair away from heat sources.
- The wheelchair use limit is defined by the wear of the parts.
- **ALWAYS** keep hands and fingers clear of moving parts to avoid injury.
- **DO NOT** allow children to play on or operate the wheelchair.
- Close the wheelchair before lifting it.
- **DO NOT** attempt to lift the wheelchair by any removable parts.
- The user and/or the patient will have to report any serious accident that have occurred • related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

**6. SYMBOLS**

<b>REF</b>	Product code
<b>UDI</b>	Unique Device Identification
<b>CE</b>	CE mark
	Manufacturer
<b>LOT</b>	Batch Lot
	Read the instruction manual
<b>MD</b>	Medical Device
	Conditions of disposal
<b>SN</b>	Serial Number

**7. GENERAL DESCRIPTION****7.1 Parts view**

The components can change depending on the model you choose:

(FIG.1)

- A - Frame
- B - Swing away legrest

Image for illustrative purposes only

**8. ASSEMBLING/DISASSEMBLING****8.1 Opening**

(FIG.2)

Note: make reference to safety cautions in this manual.

- Get the handles and pull outward, then press the seat until the wheelchair is completely open. (FIG.2)

Image for illustrative purposes only

## 8.2 Closing


- Lift the footplate;
- Lift the seat and close the wheelchair by pushing the handles close. (FIG.3)

Image for illustrative purposes only

## 8.3 Armrest

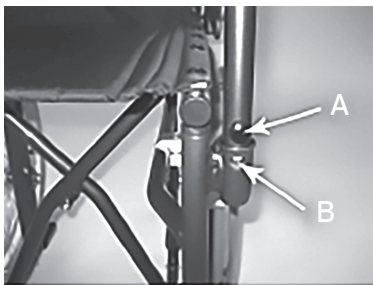According to the wheelchair model you have chosen, the armrests are:

Removable (CP765-xx):

- Insert the armrests into the rear mount, then fold them forward and also engage them in the front bracket
- Insert the tubes into their holders by snapping the locking clip correctly
- To remove, press the front release lever, then fold back the armrests and pull the release knob. The armrests and pull the rear release knob to remove them
- For removal press the release and then lift the upwards

A. Locking system - B. Locking clip

- Flip-up, detachable, height adjustable and sliding (CP790-xx);
- Flip-up, detachable and height adjustable (CP785-xx);
- Flip-up and detachable (CP780-xx, CP775-xx, CP770-xx).

To flip-up the armrests, unlock the armrest by pushing the front release lever and then overturn them. (FIG.4)

**WARNING!**  
Never lift the wheelchair by the armrest.

(FIG.4) Image for illustrative purposes only

#### 8.4 Install/remove the swing away legrest

- Assemble the swing away or elevating legrest by installing the hinge plates on the footrest into the hinge pins on the wheelchair frame. Push the legrest toward inside of the wheelchair until it locks into place (FIG.5)
- Remove the legrest by pushing the legrest release lever in ward, rotate outward, lift the legrest off the hinge pins

(FIG.5) Image for illustrative purposes only

#### 8.5 Adjust the swing away legrest

- To adjust the length of the footrest, unscrew the nut lever at the bottom on the left side, slide the tube until the desired position. Then screw the nut to fix it. (FIG.6)

(FIG.6) Image for illustrative purposes only

#### 8.6 Assembly/Disassembly of CP762-xx backrest

- Manually raise both sides of the backrest until both frame clips shown in (FIG.7) snap into place; press the clips in and manually lower both sides of the backrest to close it.


(FIG.7) Image for illustrative purposes only

#### 8.7 Elastic adjustable backrest (CP790-xx)

The backrest for model CP790-xx is elastic and adjustable to facilitate the containment of the trunk. To adjust the tension of the backrest:

- Remove the cushion from the backrest;
- Loosen the Velcro straps;
- Position the patient to allow more winding trunk;
- Close bands;
- Reposition the cushion back.
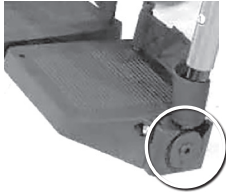
### 8.8 Adjust the footplate (CP780-xx, CP785-xx, CP787-xx, CP790-xx)

To adjust the angle of the footplate:

- Loosen the screw located on the outside of footplate (FIG.8)
- Adjust the angle of the footplate;
- Tighten the screw.

(FIG.8) Image for illustrative purposes only

### 8.9 Adjust the height of rear wheels (CP780-xx, CP785-xx, CP790-xx)

To adjust the height of the rear wheels:

- Remove the wheels through the quick release;
- Loosen the screws of the wheels adapter (FIG.9)
- Place the wheels adapter at the desired height;
- Tighten the screws and the nut;
- Reposition the wheels.

#### **WARNING!**

The adjustment of the wheelchair must be performed by qualified personnel to assess its suitability and correct configuration

(FIG.9) Image for illustrative purposes only

### 8.10 Adjust the front wheels (CP780-xx, CP785-xx, CP790-xx)

To change the height of the front wheels:

- Loosen the screw which locks the wheel to the fork (FIG.10);
- Remove the wheels and the screw;
- Choose the desired hole;
- Insert the wheels and tighten the screw and the nut.

(FIG.10) Image for illustrative purposes only

### 8.11 Adjust the depth (CP790-xx)

To adjust the depth of the seat:

- Unscrew the 4 screws that lock the vertical support to the seat and to the lower frame (FIG.11);
- Move the seat support forward or back as needed;
- Screw the screws in the right position;
- Adjust the seat by increasing or reducing the seat available through the Velcro strips.

(FIG.11) Image for illustrative purposes only

**WARNING!**

The adjustment of the wheelchair must be performed by qualified personnel to assess its suitability and correct configuration.

**8.12 Adjust the height of the push handles (CP790-xx)**

To adjust the height of the push handle, loosen the knobs at the side ends of the backrest, adjust the handles to the desired height and tighten the knobs. (FIG.12)

(FIG.12) Image for illustrative purposes only

**9. BEFORE USE**

- Inspect the frame for any damage. Check the tubing for cracks and rust, in order to guarantee the safe for people ant thing.
- Check the correct assembly, in order to guarantee a safe use.
- Check the seat is completely open. If not push the seat down from the lateral frame.
- Check the brake correctly works.
- Make sure the armrests have been positioned correctly.
- Remember that the center of gravity of the wheelchair has been studied with footrests in the right position, therefore do not use the wheelchair without the footrests because it could overturn.

**10. USING WARNINGS**

- The wheelchair cannot be used as a seat on motor vehicles.
- Always make sure that the brake control is properly adjusted and functioning.
- It is recommended to be careful of trapping fingers in some parts like wheels or brakes when using the wheelchair.
- Remember that the wheelchair surface temperature may increase when exposed to external heat sources, such as sunlight.
- WARNING! There is a risk of slipping in the presence of gravel or uneven ground or in the wet. Always adapt the driving method to the conditions of use.
- WARNING! Additional loads hanging from

**11. HOW TO USE**

**Note:** make reference to "GENERAL WARNINGS" chapter.

**WARNING!**

- Getting in and out of the wheelchair may be performed independently, provided you have a adequate mobility and upper body strength.
- Do not stand on removable parts.

**DO NOT STAND ON THE FOOTRESTS WHEN GETTING IN OR OUT OF THE WHEELCHAIR, OVERTURNING RISK!!!**

**11.1 Get in the wheelchair**

1. Position the wheelchair as close as possible to you.
2. Engage wheel locks. We suggest to remove the armrest to ease the patient getting in the wheelchair.
3. Lean on the frame with arms and slowly go down towards the seat.

**11.2 Get out of the wheelchair**

1. Position the wheelchair as close as possible along side the seat to which you are transferring.
2. Engage wheel locks. We suggest to remove the armrest to ease the patient getting out of the wheelchair.
3. Shift body weight into seat with transfer.

**11.3 Stairs and height difference****WARNING!**

The following methods must be used only from experienced patients. If you are not experienced, let you help from an assistant!

**GET ON**

Position the wheelchair at the curb to be crossed. Balance the wheelchair on the rear wheels, in order to lift the front wheels, enough to lean them on the obstacle. Bend your trunk forward and vigorously push the handrail. Continue pushing until the rear wheels are climbing over the obstacle.

**GET OFF**

To get off the stair follow the reverse procedure, positioning the rear side of the wheelchair at the curb, keep the handrails vigorously and get off the obstacle slowly; then balance the wheelchair on the rear wheels in order to complete the descent.

**WARNING!**

We recommend to use this method only in presence of obstacle lower than 10cm!

**11.4 Use the armrest**

The armrest can be flip-up and detachable, flip-up, detachable and height adjustable, flip-up, detachable, height adjustable and sliding.

**Flip-up armrest:** press the front release lever and turn it towards the back.

**Detach the armrest:** press the front release lever, the rear button and pull upwards.

**Adjust the armrest in height:** press the button on the outside of the armrest and raise or lower the armrest as needed. Release the button once the desired position is found.

**Adjust the armrest in depth:** press the button on the outside of the cover soft armrest and slide the armrest forward or back as needed. Release the button once the desired position is found.

**11.5 Use the 60cm wheels**

The rear 60cm wheels are used when self-propelled.

Grasp the handrail and wheel through the desired direction.

**11.6 Carrying**

The wheelchair has been designed to be easily carried. Once the armrest and the footrest are removed, can be closed and easily positioned in the car boot, for example. If this can't be possible, position the wheelchair in front of the car rear seat so that it is stable and cannot slip or slide. Lock the wheelchair with the seatbelt.

**12. MAINTENANCE**

The **ARDEA ONE** devices by Moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair use only original spare parts. Before using follow the "BEFORE USE" chapter.

Moretti **ARDEA ONE** wheelchairs require few elementary and periodic maintenance, below we list the checks to be carried out periodically:

- WEEKLY: frame and metallic parts.
- MONTHLY: braking capacity.
- AS NEEDED: seat cleaning.

**13. CLEANING AND DISINFECTION**

**13.1 Cover and padding**

The cover and padding are in Nylon. Clean with warm water and mild soap.

**13.2 Frame and metallic parts**

Clean with warm water and mild soap. Dry with a soft cloth in order to not scratch the painting.

**13.3 Brakes and wheels**

Clean the wheels with warm water and mild soap. Pay attention in drying the wheels, it's dangerous use the wheelchair with wet wheels.

**13.4 Disinfection**

if you need to disinfect the device use a common disinfectant cleaner.

**WARNING!**  
Do not use solvent and abrasive product.

**14. CONDITIONS OF DISPOSAL** 

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycle.

**15. TECHNICAL FEATURES**

**15.1 Dimensions**

**DIMENSION AND WEIGHT**

Measures in CM*																	
<b>CP762-43</b>								43		56							
<b>CP762-46</b>	93	62	41	100	73	49	41+51	46	46	59	25	12 kg	24	20	-	-	130 kg
<b>CP762-48</b>								48		62							
Measures in CM*																	
<b>CP765-40</b>								40		51							
<b>CP765-43</b>	100	73	41	92	80	53	45+57	43	37	54	26	14,5 kg	50	18	-	-	120 kg

## 15.2 Technical specification

- Aluminium frame
- Max user weight (CP770-xx, CP775-xx, CP780-xx) 120 kg
- Max user weight (CP785-xx, CP790-xx) 130 kg
- Front wheels (CP780-xx, CP785-xx, CP790-xx) 20 cm PU
- Front wheels (CP770-xx, CP775-xx, CP787-xx) 20 cm PVC
- Rear wheels (CP770-xx, CP780-xx, CP785-xx, CP787-xx, CP790-xx) 60 cm PU
- Rear wheels (CP775-xx, CP787-xx) 30 cm PU
- Fabric: Nylon
- Reference standard: EN12183 - EN1021-1, EN1021-2 for upholstery and padding of seat and backrest
- Environmental temperature: -20°C/+ 40°C
- Storage: store in a cool place, away from heat sources, such as sunlight (suggested temperature between +10°C and +40°C).

## 16. ACCESSORIES

On the Helios wheelchair line the user can install the following accessories:

- CPA100 wheelchair cushion
- CAR900 headrest
- CPA501 linear table (only for CP765, CP770, CP775, CP780)
- CPA506 shaped table (only for CP765, CP770, CP775, CP780)
- CAR605 anti-theft bar (only for CP770, CP775, CP780)
- CPA605 oxygen bottle holder (only for CP770, CP775)
- CPA130 single footrest
- RP245 safety belt
- CPA234 one hand drive kit (only for CP770, CP775, CP780)
- CPA772 safety wheels for CP770
- CPA771 elevating footrest in aluminum for CP770 and CP775
- CPA775 armrest adjustable in height for CP770 and CP775
- CPA776 armrest adjustable in height for CP780
- CPA777 anti-tipper wheels for CP770, CP775, CP780
- CPA778 anti-tipper wheels for CP785, CP790

**Note:** The list of accessories and compatibility with wheelchairs is in constant update. We suggest to contact the retailers to be informed about new accessories, availability and more information.

## 17. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

## 18. REPAIRING

### 18.1 Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

### 18.2 Repair not covered by warranty

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for goods out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the goods received.

### 18.3 Non defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

## 19. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date. For spare parts only refer to MORETTI master catalogue.

## 20. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

ENGLISH

**WARRANTY CERTIFICATE**

**Product** \_\_\_\_\_

**Purchased on (date)** \_\_\_\_\_

**Retailer** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**Sold to** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

**MADE IN P.R.C.**

*\* Please consult our website for the latest available version of the user manual*

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