ITALIANO

MA CPA5xx 06 A\_ITA\_08 2020

**TAVOLINI PER CARROZINE**  

---

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## INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

## 1. CODES

**CPA501** Linear table 55x32cm**CPA506** Table for Wheelchair 58x38cm**CPA517** Table for Wheelchair 68x57cm**CAR525** Wheelchair table 58x38cm only for CP9xx series**CBR500** Linear table 55x32cm only for CP810 series

## 2. INTRODUCTION

Thank you for purchasing a **ARDEA ONE product** by **Moretti** product range. Its design and quality guarantee comfort, safety and reliability to the user. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

**Note: check all parts for any shipping damage, and test before using. In case of damage, do not use the device. Contact your dealer for further instruction.**

## 3. INTENDED USE

This product is an accessory for medical devices (wheelchairs) for people with mobility problems. Adaptable for wheelchairs of measures from 38 to 48cm (**CPA501, CPA506, CAR525**) and > of 50cm (**CPA517, CBR500**).

### WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without notice

## 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of TABLES FOR WHEELCHAIRS AND COMMODOE CHAIRS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation

6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNING

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Do not smoke and / or contact with heat sources.
- Keep the packaged product away from any heat source.
- The useful life of the device is determined by wear of non-repairable parts and / or replaceable.
- Always pay attention to the presence of children.
- Do not put on the tray weights exceeding the payload of the table.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

### WARNING!

Danger of tipping and/or deformation of the supports

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

## 7. GENERAL DESCRIPTION

### 7.1 Parts view

**CPA501** Linear table 55x32cm

- Tray - 1 pc
- Support for fixing - 2 pcs
- Screws M5 - 4 pcs
- Screws Withworth - 4 pcs
- Knobs - 2 pcs
- Spacer - 4 pcs

**CPA506** Table for Wheelchair 58x38cm

- Tray - 1 pc
- Support for fixing - 2 pcs
- Screws M5 - 4 pcs
- Screws Withworth - 4 pcs
- Knobs - 2 pcs
- Spacer - 4 pcs

**CPA517** Table for Wheelchair 68x57cm

- Tray - 1 pc
- Support for fixing - 2 pcs
- Screws M5 - 4 pcs
- Screws Withworth - 4 pcs
- Knobs - 2 pcs
- Spacer - 4 pcs

**CAR525** Wheelchair table 58x38cm only for CP9xx series

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**CBR500** Linear table 55x32cm only for CP810 series

- Tray - 1 pc
- Support for fixing - 2 pcs
- Screws M5 - 4 pcs
- Screws Withworth - 4 pcs
- Knobs - 2 pcs
- Spacer - 4 pcs

**8. ASSEMBLING AND USE**

The product can be easily installed, just following a few manual operations.

**Note:** make reference to safety warnings in this manual.

**8.1 Assembling**

**CPA501** Linear table 55x32cm

**CPA506** Table for Wheelchair 58x38cm

**CPA517** Table for Wheelchair 68x57cm

**CBR500** Linear table 55x32cm only for CP810 series

1. Unscrew the screws and put the support between armrest and the handrail tube, fix the whole set with the screws provided or reuse the existing ones.
2. Repeat for the other armrest.
3. Insert the tube into the supports of the tray and fix with the knobs.
4. The wheelchair tray is now ready for use.

Only illustrative purpose images

**NOTE** If you need, cut the table's tubes to use the accessory

For the wheelchairs CP200-xx e CP205-xx use the spacers and the screws M5x25

- A. PU padding
- B. Spacers (2 pcs)
- C. Tray support
- D. Screws M5x25
- E. Armrest frame


Turn over the armrest

1. As showed in the picture, remove the PU padding (A), from the armrest frame (E)
2. Place the PU padding on a flat surface and insert the spacers (B)
3. Then set the tray support (C), the armrest frame (E) and insert the screws (D).
4. Tighten the screws (D).

**CAR525** Wheelchair table 58x38cm only for CP9xx series


1. Remove or tip over the armrest to facilitate the operation;
2. Unscrew the screws under the armrest in order to assembly the support bars;

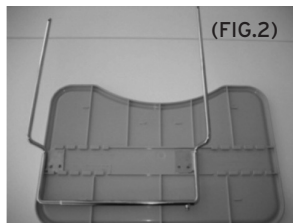
(FIG.2)

(FIG.3)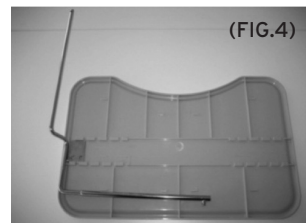

(FIG.4)

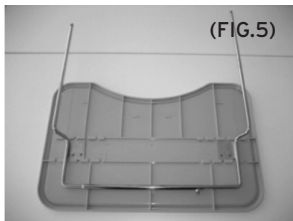(FIG.5)

(FIG.6)

(FIG.7)

3. Screw the support bar under the armrest, paying attention to maintain the knob on the front side (FIG.2)
4. Assembly the armrest on the wheelchair.
5. Put the shorter bar into the guidelines under the desk. (FIG.3)
6. Insert the longer bar into the shorter one following the guide lines. (FIG.4)
7. Adjust the bars depending on the distance between the armrests. (FIG.5)
8. Tighten the screw for bars locking.
9. The supporting bars should be inserted into the tubes under the armrest and locked with knobs. (FIG.6)

### 8.2 Disassembling

- Follow the previous instruction in reversed order

## 9. BEFORE USE

- Always check the wear of the parts where the accessories have to be fixed to ensure the safe use of the device for persons and things.
- Check the correct assembly of the product. Ensure that all parts are properly assembled.
- After completing the assembly, verify the correct operation of the product.
- You can use the self-propelling wheelchairs, being very careful to entrapment due to the reduction of visibility.
- Adapt products to the user and the wheelchair in use.

## 10. CLEANING AND DISINFECTION

### 10.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

#### WARNING!

Do not use abrasive substances, acids, alkaline, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

### 10.2 Disinfection

If you need to disinfect the product use a common disinfectant cleaner.

**Note: absolutely avoid the use of acids, alkaline or solvents such as acetone or diluent.**

## 11. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycle.

## 12. TECHNICAL FEATURES

### 12.1 Dimensions and weights

**CPA501** Linear table 55x32cm

- Height **3cm (folded)**
- Width **55cm**
- Depth **32cm**
- Pay load **5Kg**

**CPA506** Table for Wheelchair 58x38cm

- Height **7cm (folded)**
- Width **58cm**
- Depth **38cm**
- Pay load **5Kg**

**CPA517** Table for Wheelchair 68x57cm

- Height **7cm (folded)**
- Width **68cm**
- Depth **57cm**
- Pay load **5Kg**

**CAR525** Wheelchair table 58x38cm only for CP9xx series

- Height **2cm (folded)**
- Width **58cm**
- Depth **38cm**
- Pay load **5Kg**

**CBR500** Linear table 55x32cm only for CP810 series

- Height **3cm (folded)**
- Width **55cm**
- Depth **32cm**
- Pay load **5Kg**

**12.2 Technical specifications****CPA501** Linear table 55x32cm

- Adaptable for wheelchair <50cm
- Steel frame
- Tray by 100% PP
- rectangular shape with rounded corners
- Supports adaptable for any type of armrest, made of black painted steel
- Recommended storage: Store in a cool and dry place and keep away from direct heat sources such as sunlight. (Temp. Recommended between +10 ° C / + 40 ° C)

**CPA506** Table for Wheelchair 58x38cm

- Adaptable for wheelchair <50cm
- Steel frame
- Tray by 100% PP
- rectangular shape with cut, rounded corners
- Supports adaptable for any type of armrest, made of black painted steel

**CPA517** Table for Wheelchair 68x57cm

- Adaptable for wheelchair >50cm
- Steel frame
- Tray by 100% PP
- rectangular shape with cut, rounded corners
- Supports adaptable for any type of armrest, made of black painted steel
- Recommended storage: Store in a cool and dry place and keep away from direct heat sources such as sunlight. (Temp. Recommended between +10 ° C / + 40 ° C)

**CAR525** Wheelchair table 58x38cm only for CP9xx series

- Easygoing on polyfunctional Relief from 40 to 45 cm
- Steel frame
- 100% PP table
- Rectangular hollow smooth edges shape
- Black painted steel supports adaptable on every type of armrest
- Recommended storage: Store in a cool and dry place and keep away from direct heat sources such as sunlight. (Temp. Recommended between +10 ° C / + 40 ° C)

**CBR500** Linear table 55x32cm only for CP810 series

- Adaptable for wheelchair <50cm
- Steel frame
- Tray by 100% PP
- rectangular shape with rounded corners
- Supports adaptable for any type of armrest, made of black painted steel
- Recommended storage: Store in a cool and dry place and keep away from direct heat sources such as sunlight. (Temp. Recommended between +10 ° C / + 40 ° C)

**13. WARRANTY**

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

**14. REPAIRING****14.1 Warranty repair**

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

**14.2 Repair not covered by warranty**

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

**14.3 Non-defective device**

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

**15. SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

**16. EXEMPT CLAUSES**

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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