MOPEDIA®

MA_RP159_RP160-x_RP162-x_RP163-x_RP167-x_RP168-1_RP169-x 04 A*_ITA_03 2024

BASTONI IN ALLUMINIO

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ITALIANO

INDICE

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NOTE

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

NOTE

MORETTI S.p.A.
Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com

MOPEDIA®

MA RPI59_RPI60-x_RPI62-x_RPI63-x_RPI67-x_RPI68-1_RPI69-x 04 A* ENG_03 2024

ALUMINUM WALKING STICKS

INSTRUCTION MANUAL

ENGLISH

INDICE

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.5
7.1 Height adjustment	pag. 5
8. BEFORE USE	PAG.5
9. WARNINGS FOR USE	PAG.5
10. HOW TO USE.....	PAG.5
11. MAINTENANCE.....	PAG.5
12. CLEANING AND DISINFECTION.....	PAG.5
12.1 Cleaning.....	pag. 5
12.2 Disinfection	pag. 6
13. CONDITIONS OF DISPOSAL	PAG.6
14. TECHNICAL FEATURES.....	PAG.6
15. WARRANTY	PAG.8
16. REPAIRING	PAG.8
16.1 Warranty repair	pag. 8
16.2 Repair not covered by warranty	pag. 8
16.3 Non defective device	pag. 8
17. SPARE PARTS.....	PAG.8
18. EXEMPT CLAUSES.....	PAG.8

Class Medical Device
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND
OF THE COUNCIL of 5 April 2017 concerning medical devices

1. CODES

- RP159** Foldable aluminum walking stick, brown wooden T handle, for woman
- RP160-1** Foldable aluminium walking stick derby handle, in ABS and TPR Tartan pattern
RP160-2 Foldable aluminium walking stick derby handle, in ABS and TPR Black-white pattern
RP160-3 Foldable aluminium walking stick derby handle, in ABS and TPR Flowers pattern
- RP162A** Monocromatic aluminium walking stick derby handle, in ABS and TPR Metallic
RP162B Monocromatic aluminium walking stick derby handle, in ABS and TPR Blue
RP162V Monocromatic aluminium walking stick derby handle, in ABS and TPR Green
RP162R Monocromatic aluminium walking stick derby handle, in ABS and TPR Red
RP162P Monocromatic aluminium walking stick derby handle, in ABS and TPR Purple
RP162N Monocromatic aluminium walking stick derby handle, in ABS and TPR Black
- RP163-1** Aluminium stick T handle Squares pattern
RP163-2 Aluminium stick T handle Flowers pattern
- RP167-2** Aluminium stick derby handle Cashmere pattern
RP167-3 Aluminium stick derby handle Violet flowers pattern
RP167-4 Aluminium stick derby handle Circles pattern
RP167-5 Aluminium stick derby handle Black and white pattern
- RP168-1** Non foldable aluminum walking stick, wooden T handle, flowers pattern
- RP169-1** Non foldable aluminum walking stick, ABS and TPR derby handle, Flowers pattern
RP169-2 Non foldable aluminum walking stick, ABS and TPR derby handle, Zigzag pattern
RP169-3 Non foldable aluminum walking stick, ABS and TPR derby handle, Optical blue pattern
RP169-5 Non foldable aluminum walking stick, ABS and TPR derby handle, Optical carbon pattern
RP169-6 Non foldable aluminum walking stick, ABS and TPR derby handle, Tartan pattern
RP169-7 Non foldable aluminum walking stick, ABS and TPR derby handle, Squares pattern

2. INTRODUCTION

Thank you for purchasing a MOPEDIA by Moretti walking stick. This instruction manual contains some useful tips for a correct use of the device you have bought as well as some important advises for your safety. In case of doubts, please contact your retailer who will be able to help you correctly.

3. INTENDED USE

The MOPEDIA by Moretti walking stick is aid manipulated by one arm adapted to support and/or help the mobility of people with mobility problems.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of WALKING STICKS complies with the provisions of the

regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

Read carefully the entire manual before using the device.

- Advise with physician or therapist to determinate the correct adjustment and the correct use of the walking canes.
- Make sure that the product is delivered complete and keep it packed away from heat sources.
- The device use limit is defined by the wear of the parts.
- Do not allow children to play or operate the device.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Max loading weight

Conditions of disposal

7. GENERAL DESCRIPTION

MOPEDIA walking stick are made of aluminum and no-slip rubber tip. Adjustable height every 2.5 cm. These devices are useful to ensure a safety of support and confidence. The walking sticks also make it easy to stay upright and walk. Facilitate the discharge of the lower body segments, stabilize the person in the problems of balance and supports when there are force problems.

7.1 Height adjustment

Unscrew the locking ring, press the pin and slide the cane to match the hole at the desired height. tighten the locking ring.

8. BEFORE USE

WARNING!

Check if the frame is damaged in order to guarantee a safe usage for people and things. Check the correct assembly of the device.

9. WARNINGS FOR USE

- Use extra caution on wet and slippery surfaces.
- Avoid leaving the device in contact with water for long periods of time.
- Periodically check its sturdiness, the stability and rubber tips conditions.
Replace them once worn out.
- In case of doubts about usage, adjustment or maintenance, contact your dealer.
- Keep it away from children to avoid any possible risks.

WARNING!

DO NOT use with patient who exceed the weight indicated on the product label and in the "technical features" chapter of this user manual.

10. HOW TO USE

Note: Refer to the safety instructions in this manual

The usage of the walking stick varies from case to case. For this reason, you need a minimum of training by qualified personnel with reference to information on the type of step or support type. The operator during patient care is always positioned on the side of the body more weak and sick.

11. MAINTENANCE

The Mopedia devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety, we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. For periodic inspection we mean an examination of the following parts:

- The walking cane frame
 - The rubber tip.
- In case of repair must be used only approved parts and accessories.

12. CLEANING AND DISINFECTION

12.1 Cleaning

Use only a damp cloth to remove dust, use a damp cloth with a mild soap to remove dirt that is more difficult. Dry thoroughly before use.

**WARNING!**

Never use water with temperature above 30°C. Never use jet water or steam washing machine.

12.2 Disinfection

If you need to disinfect the product, use a common disinfectant cleaner.

13. CONDITIONS OF DISPOSAL

In case of disposal, do not use the inserting container for municipal waste. We recommend disposing the walking stick in the appropriate disposal areas for recycling.

14. TECHNICAL FEATURES**RP159**

ADJUSTABLE HEIGHT: FROM 83 TO 93 CM

MAX LOADING WEIGHT: 100 KG

Diameters:

INNER 16 MM / OUTER 19 MM

Neck ring:

METALLIC

T handle:

BROWN WOODEN

Frame:

ALUMINUM

Rubber:

ANTI-SLIP TIP

RP160-1, RP160-2, RP160-3

ADJUSTABLE HEIGHT: FROM 83 TO 95 CM

MAX LOADING WEIGHT: 100 KG

Diameters:

INNER 19 MM/ OUTER 22 MM

Neck ring:

METALLIC

Handle:

DERBY IN ABS + TPR BLACK

Frame:

ALUMINIUM

Rubber:

ANTI SLIP-TIP

RP163-1, RP163-2, RP167-2, RP167-3, RP167-4, RP167-5

ADJUSTABLE HEIGHT: FROM 76 TO 98,5 CM

MAX LOADING WEIGHT: 100 KG

Diameters:

INNER 19 MM/ OUTER 22 MM

Neck ring:

METALLIC

Handle:

DERBY IN ABS + TPR BLACK (for RP163-x)

HANDLE SOFT BLACK (for RP167-x)

Frame:

ALUMINIUM

Rubber:

ANTI SLIP-TIP

RP168-1

ADJUSTABLE HEIGHT: FROM 76 TO 98,5 CM

MAX LOADING WEIGHT: 100 KG

Diameters:

INNER 19 MM / OUTER 22 MM

Neck ring:

METALLIC

T handle: BLACK WOODEN
 Frame: ALUMINUM
 Rubber: ANTI-SLIP TIP

RP162A, RP162B, RP162V, RP162R, RP162P, RP162N,
 RP169-1, RP169-2, RP169-3, RP169-5, RP169-6, RP169-7

 ADJUSTABLE HEIGHT: FROM 76 TO 98,5 CM

 MAX LOADING WEIGHT: 100 KG

Diameters: INNER 19 MM / OUTER 22 MM

Neck ring: METALLIC

Derby handle: BLACK ABS AND TPR

Frame: ALUMINUM

Rubber: ANTI-SLIP TIP

15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

16. REPAIRING

16.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

16.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

16.3 Non defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

17. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date. For spare parts and accessories please refer to Moretti master catalogue.

18. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE**

ENGLISH

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 20022 Cavriaglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN P.R.C.**** Please consult our website for the latest available version of the user manual*

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