**MOPEDIA®**

ITALIANO

MA XMP165 02 A\_ITA\_08 2020

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices**1. CODE**

MP165 - Aluminum bedside rails with support on the ground

**2. INTRODUCTION**

Thank you for choosing an accessory MOPEDIA by Moretti. The bedside rails have been designed and built to meet all your needs for a practical use, correctly and safely. This manual contains some suggestions for proper use of the accessory of your choice and valuable tips for your safety. You should read carefully all of this manual before using the bedside rails.

**CAUTION!**

- Do not use the following device for purposes other than those described in this manual.
- Moretti S.p.A. disclaims any liability for damage resulting from improper use of the device or use other than that described in this manual.
- We reserve the right to make changes to the device and this manual without notice in order to improve the characteristics.
- Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.
- Always pay attention to the presence of children.

**3. INTENDED USE**

The bedside rail is a universal accessory designed to prevent an accidental fall of a person from a any bed. It may provide a useful support to facilitate the raising and the seat from the bed and can be placed, optionally on both sides of the bed.

**4. DECLARATION OF CONFORMITY**

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the groups of BEDS - TULIPANO, IBISCO complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

- For proper use of the device refer to this manual.
- For proper use of the device to consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts can not be repaired and/or replaced.
- Always pay attention to the presence of children.
- The bedside rail is not intended to carry the full weight of an individual. This product is only intended to provide balance and support while sitting and standing
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

## 6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

## 7. GENERAL DESCRIPTION

The universal bedside rails are designed to fit any bed thanks to the telescopic tubes to be placed under the mattress.

### 7.1 Parts list

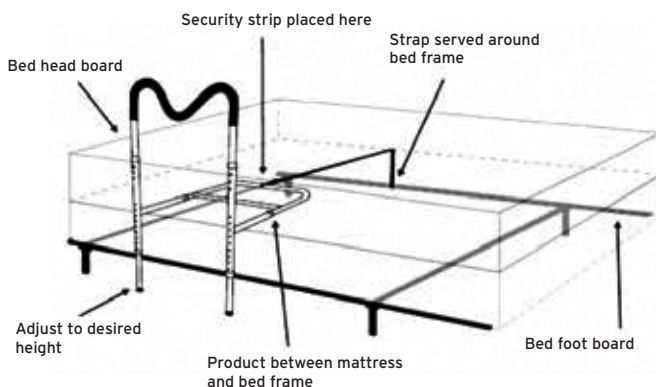
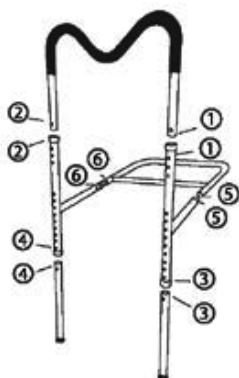
1. Handle upper (pc.1)
2. Frame (pc.1)
3. Cross-member connection (pc.1)
4. Legs (pcs.2)
5. Utility pocket (pc.1)
6. Security strap (pc.1)

## 8. INSTALLATION/REMOVAL

### 8.1 Assembly

Follow the below instructions:

1. Insert the handle-M on the frame, adjust the height by manipulating clips and fit the utility pocket. (1-1 and 2-2).
2. Place the legs on the frame by adjusting the length of the legs depending on the height of the bed by manipulating clips. (3-3 and 4-4) Use the adjustment screw on the bottom for a perfect contact with the floor.
3. Insert the Cross-member connection connecting the main frame. (5-5 and 6-6).
4. Place the assembled structure between the mattress and the bed frame.
5. Tighten the belt to the bed frame on the opposite side on the bed.

### 8.2 Dismantling

Follow the procedure described above in reverse.

## 9. BEFORE TO USE

- Always check the wear of the mechanical parts in the game so as to ensure use of the device in total safety for persons and things.
- Double check for proper assembly of the device and pay attention fixing the bedside rails on the bed frame.

## 10. HOW TO USE

Assembly the device according to instruction manual, adjust the height bedside rail to suit your needs by clips. Place the bedside rail between the mattress and the bed frame.

**NOTE:** Always check the bedside rail is in the locking position.

## 11. MAINTENANCE

The MOPEDIA devices by Moretti S.P.A. are checked carefully and supplied with CE mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

## 12. CLEANING AND DISINFECTION

### 12.1 Cleaning

Use only a damp cloth and mild soap. Afterwards dry thoroughly before using.

### 12.2 Disinfection

If you need to disinfect the product using a common disinfectant.

**NOTE:** Never use acids, alkalines or solvents such as acetone or thinner

## 13. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

## 14. TECHNICAL FEATURES

### 14.1 Dimension

- Leg height adjustable from ground from 34 to 52 cm
- Bedside rail height adjustment from bed from 47 to 52 cm
- Width 45 cm
- Length 64 cm

### 14.2 Technical specifications

- Frame anodized aluminium
- Handle support polyurethane adjustable in 3 levels of height, useful support to facilitate the get up and the seat from the bed
- Height adjustable leg in 9 positions
- The stabilizing frame is to be inserted under the mattress and is provided with thicknesses soft polyurethane anti-slip of fixed length

## 15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device

## 16. REPAIRING

### 16.1 Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

### 16.2 Repair not cover by warranty

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

### 16.3 Non defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

## 17. SPARE PARTS AND ACCESSORIES

For spare parts and accessories, refer to the MORETTI master catalogue. Only use original MORETTI accessories. Failure to do so will entail our refusing all liability for any incident caused by improper use. The original Moretti spare parts are guaranteed for 6 months from the receiving date.

## 18. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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