MOPEDIA®

ITALIANO

MA RP553_RP562A_RP562R_01 C* IT_12 2024

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Via _____ Località _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

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 8.1 Before assembling the product..... pag. 5

 8.2 Assembling model RP562 pag. 5

 8.2.1 Opening and closing the RP562 rollator..... pag. 6

 8.3 Assembling model RP553..... pag. 6

 8.3.1 Installation of the handles and adjustable handles pag. 7

 8.3.2 Bag installation pag. 7

 8.3.3 Seat pag. 7

 8.3.4 Tyre handling and maintenance pag. 7

 8.4 Overcoming small obstacles..... pag. 8

9. HOW TO USE PAG.8

10. BRAKES PAG.8

 10.1 To brake pag. 8

 10.2 Lock the brake (handbrake) pag. 8

11. CARE AND MAINTANANCE..... PAG.8

12. CLEANING PAG.8

13. WHEEL CARE PAG.9

14. TECHNICAL FEATURES..... PAG.9

5. CONDITIONS OF DISPOSAL PAG.9

16. SPARE PARTS / ACCESSORIES PAG.9

17. WARRANTY..... PAG.9

 17.1 Repairing..... pag. 10

 17.2 Spare parts pag. 10

 17.3 Exempt clauses pag. 10

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices**1. CODES**

RP562A	Folding rollator with pneumatic wheels Ermes Light Blue
RP562R	Folding rollator with pneumatic wheels Ermes Red
RP553	Folding rollator Artemis

2. INTRODUCTION

Thank you for choosing a MOPEDIA range rollators by Moretti S.p.A. MOPEDIA range rollators manufactured by Moretti S.p.A. are designed to satisfy all your requirements for practical, correct and safe use. This manual provides some useful suggestions for correctly using the device you have selected and offers valuable advice for your safety. We recommend reading the entire manual carefully before using the rollator you have purchased. Should you have any queries, please contact your retailer for proper advice or assistance.

3. INTENDED USE

This rollator fit for people with movement problems. Can be used as assistance to facilitate ambulation or even servo-assisted ambulatory. Anyway, it can't be used by people with evident malfunction of the hand or the arm, even by people with balance problems. The rollator can be used inward and outward. We recommend using on flat surface where the wheels can reach the land. It can't be used on rock ground or on slanted area as the rollator can be unstable. The device is not intended to carry people or things.

**WARNING!**

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ROLLATORS - MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. WARNINGS

5.1 General warnings

- Do NOT install or use the device before reading and understanding this manual; If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional dealer or technical personnel before attempting to install this equipment to avoid injure or damage;
- The rollator should only be used by patients with a suitable motor capacity which ensures their use without risks;
- Advise with physician or therapist to determinate the correct adjustment and the correct use;
- Physician or therapist should contribute to height adjustment for the maximum support, for a correct brake activation and for a general assembly;
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

5.2 Warnings for use

- Insure the brake cable to be correctly set;
- Verify the correct operation of the brake after every height adjustment to avoid a free flexion of the cable;
- Breaking this simple rules can cause personal damages;
- The rollator can be used only on a hard, flat surface. Do not use the rollator to climb the stairs
- Leaning always on both dumbbells to maintain balance;
- Pay particular attention to wet, icy or snowy;
- Do not overload the rollator;
- Do not use the rollator seat (where present) to carry persons or property, the seat is for the user's session with the parking brake engaged;
- All wheels must be in contact with the floor at any time during use to ensure the proper balancing of the rollator and maximum safety;
- The brake must be engaged before using the seat.

Note When the rollator is used in steady position, the handbrake must be locked

5.3 Stability warnings

- The rollator can provide ambulatory assistance to an individual weighing up to 136 kg, INCLUDING the loading bag;
- The rollator bag has a weight limitation of 5 kg.

5.4 Assembling & folding warnings

- Verify that both handle support tubes are correctly inserted in the frame;
- Both handle tubes are securely locked in the main support part with wing nuts and knobs without any movements;
- Do not hanging anything on the rollator frame to avoid distortions of the tubes when the rollator is open and damage the stability. Injury may also be caused by distortions;
- Take care to entrapment of fingers and / or clothing during the opening and closing.

6. SYMBOLS

-  Product code
-  Unique Device Identification
-  CE mark
-  Manufacturer
-  Batch Lot
-  Read the instruction manual
-  Medical Device
-  Conditions of disposal
-  Maximun loading weight

7. BEFORE USE

Before using the device, always check:

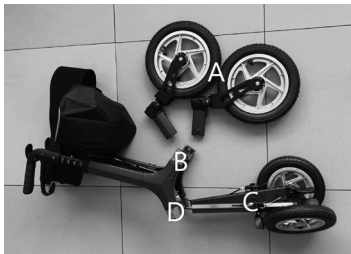
- That all parts of the rollator are locked in a secure manner;
- Check that all screws are tight;
- Make sure that the seat is properly locked (if that 'not press further with the power seat bottom frame);
- Check that the locking of the seat is correctly inserted;
- Check the correct operation of the brake and the regular brake;
- For RP562 check that the tyres are correctly inflated and that there is no damage on them.

8. ASSEMBLING INSTRUCTION

8.1 Before assembling the product

Unpack the product and check that no damage has occurred during shipping. In the event when the rollator is damaged, please contact your nearest dealer.

8.2 Assembling model RP562

- Carefully unpack the product, taking care not to damage it, and arrange the parts on the floor
- Insert the front wheels A into the front housings B until the metal clip clicks into place and tighten the locking knob. Insert rear wheels C into rear housing D and tighten locking knob.

 **WARNING!** Make sure that the knobs is located into the holes

8.2.1 Opening and closing the RP562 rollator

After positioning the rollator so that it is stable on the floor, press down on the sides of the seat until it opens fully and you hear a click from below.

To close it again, vigorously lift upward on the belt on the seat.

8.2.2 Inflating tires RP562

To inflate tires use the supplied pump and proceed according to the following steps:

- Unscrew the valve lock cap.
- Snap the pump head onto the valve in place, with the lever in the open position, (as shown in fig.1) and push firmly all the way down.
- Lower the pump lever to the closed position (as shown in fig.2).
- After placing the pump on the ground, lower the support base and lock it with your foot (fig.3).
- Start pumping air until the desired tension is reached.
- Remove the pump head from the valve and close it with the cap.

Fig.1

Fig.2

Fig.3

8.2.3 Bag installation

To install the bag, hang the ears of the bag on the seat.

WARNING! Use the basket for small items only. Do not overload the bag to prevent imbalances and the risk of overturning

8.2.4 Seat

Before sitting, place the rollator on a flat and solid surface and then lock the brakes.

Make sure the latch of the lock is secured properly and that the rollator is in a stable position.

WARNING! Questions to sit down with your weight on the seat assembly. Do not sit by pushing only on one side.

8.3 Assembling model RP553

- Carefully unpack the product taking care not to damage it.
 - Insert the front wheel pivot, into the wheel wells, gently lift the handles of the rollator to a fully upright position, then open the rollator sideways until the mechanism under the seat snaps into place, and insert the backrest into the front of the handles (fig.4,5,6).
- To close, pull up on the handle on the seat until the rollator is fully closed on the side, and gently push the handles down.

Fig.4

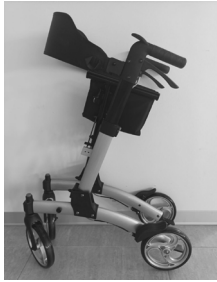
Fig.5

Fig.6

WARNING!

When opening and closing the roll, keep your hands on the handles to avoid crushing and trapping your fingers between the metal parts.

8.3.1 Installation of the handles and adjustable handles

Simply depress the red button and set to the desired height. Repeat the opposite side and make sure that both handles are secure and set at the same height before use. The rollator is equipped with height adjustable handles through the pins to meet all user needs.

WARNING!

Adjust the height of the rollator so that a user who is standing with his arms at his sides have the pulse height handles. This will ensure the positioning of the arms bent at about 20 ° -30 ° when using the rollator.

8.3.2 Bag installation

To install the bag, hang the ears of the bag on the seat, stop the tape with Velcro to the structure.

WARNING! Maximum capacity for bag 5 kg. Use the bag for small items only. Do not overload the bag to prevent imbalances and the risk of overturning

8.3.3 Seat

Before sitting, place the rollator on a flat and solid surface and then lock the brakes. Make sure the latch of the lock is secured properly and that the rollator is in a stable position.

WARNING! Questions to sit down with your weight on the seat assembly. Do not sit by pushing only on one side.

8.3.4 Tyre handling and maintenance

- Do not use motorised equipment to inflate tyres, use only the hand pump provided;
- Do not use the rollator if the tyres are visibly deflated;
- Periodically check the inflation and general condition of the tyres before and after each use, especially outdoors;
- Do not over-inflate tyres;
- Do not allow the tyres to come into contact with direct sources of puncture heat;
- Store the rollator in a place out of direct sunlight;
- Do not use acidic solvents or aggressive products to clean the tyres, use suitable or neutral products.

8.4 Overcoming small obstacles

WARNING! Small obstacles such as curbs and steps should be approached with both front wheels aligned parallel to the obstacle

To overcome small obstacles get as close as possible to the obstacle until both front wheels touch the obstacle

1. Make sure you are stable in the upright position by firmly grasping both handles.
2. Press down with one foot on the tilt to raise the wheels front wheels.
3. Begin pushing the rollator forward by balancing the force on the foot and the thrust, making the wheels stay in contact with the obstacle
4. Move forward together with the rollator making sure not to trip over the obstacle

9. HOW TO USE

- Start pushing when you are in an upright position with both hands on the handle;
- Make sure your hands are dry before using the rollator to avoid the risk of slipping;
- Walking maintaining an upright position with his eyes facing forward and not down;
- Do not push the rollator too far away from the body, this may cause falls and loads inadequate;
- Do not use the rollator to overcome steps higher than 20mm;
- When using outdoors, prefer the sidewalks properly prepared in accordance regulations for the removal of architectural barriers;
- To overcome small obstacles to leverage the rear footrests to raise the front of the rollator

10. BRAKES

10.1 To brake

To brake the rollator pull the brake lever with your fingers upwards until the rollator slows down or stops. To continue to move the rollator, release the brake handles.

10.2 Lock the brake (handbrake)

Press down on the brake lever with the palm of your hand, when you hear a "click" wheels are locked.

WARNING! When the user is stationary, the brakes must be locked

11. CARE AND MAINTANANCE

The devices MOPEDIA by Moretti SpA on the market are carefully checked and carry the CE mark. For the safety of the patient and the doctor is recommended to be checked by the manufacturer or by an approved laboratory, at least every 2 years, the suitability for use of your rollator. If repairs are to be used only original spare parts and accessories.

The useful life of the product depends on the use, maintenance, storage and cleaning.

12. CLEANING

To clean parts, use only a damp cloth with a neutral detergent, not use cleaners containing solvents and metal brushes or abrasive or sharp objects, may damage the product.

13. WHEEL CARE

1. Clean the wheels with warm water and mild detergent. Wipe dry with a clean cloth.
2. Periodically lubricate the wheels.

14. TECHNICAL FEATURES

Series ERMES										
Product codes	RP562									
Seat	46x24 cm									
Frame	Aluminum									
Weight and dimensions										
										
RP562A	80	100	56	62	45	24	11,45 kg	30	24	136 kg
RP562R		85								

Series ARTEMIS										
Product codes	RP553									
Seat	45x25 cm									
Frame	Aluminum									
Weight and dimensions										
										
RP553	70	91	52	63	45	27	8,8	20	20	136 kg

The dimensions are in cm and could be changed ± 0,5 cm

5. CONDITIONS OF DISPOSAL 

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

16. SPARE PARTS / ACCESSORIES

For spare parts and accessories please refer to the Moretti S.p.A. master catalogue

17. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use.

Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized,

shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

Repair not cover from warranty: A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

Non-defective device: The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

17.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months as from the receiving date.

17.3 Exempt clauses

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be fixed by according to the law is limited by the warranty period, in the limits of law. Some states or countries don't allow to limit the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user.

The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

ENGLISH

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

 **MORETTI S.P.A.**

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com**

MADE IN P.R.C.

** Please consult our website for the latest available version of the user manual*

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MORETTI S.p.A.