ITALIANO

MA_STA100_STA110_ST680_ST682_02 A_ITA_10-2020

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LEVITAS®

MA_STA100_STA110_ST680_ST682_02 A_ENG_10-2020

**HOLLOW FIBRE MATTRESS PROTECTORS
AND MATTRESS COVERS**

INSTRUCTION MANUAL

ENGLISH

CONTENTS

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. EU DECLARATION OF CONFORMITY	PAG.3
5. GENERAL PRECAUTIONS.....	PAG.4
6. SYMBOLS USED.....	PAG.4
7. GENERAL DESCRIPTION.....	PAG.5
7.1 Mattress covers	pag. 5
7.2 Hollow fibre mattress protectors.....	pag. 5
8. HOW TO USE	PAG.6
8.1 How to us Covers.....	pag. 6
8.2 How to use hollow fibre mattresses	pag. 6
9. CLEANING AND DISINFECTION.....	PAG.6
9.1 Cleaning and disinfecting the covers	pag. 6
9.2 Cleaning and disinfecting hollow fibre mattresses	pag. 7
10. STORAGE	PAG.7
11. MAINTENANCE	PAG.7
12. DISPOSAL CONDITIONS	PAG.8
13. TECHNICAL SPECIFICATIONS.....	PAG.8
13.1 Mattress covers.....	pag. 8
13.2 Hollow fibre mattresses.....	pag. 8
14. WARRANTY	PAG.9
15. REPAIRS	PAG.9
15.1 Repairs under warranty.....	pag. 9
15.2 Repairing a product not covered by the warranty	pag. 9
15.3 Non-defective products.....	pag. 10
16. SPARE PARTS	PAG.10
17. NON-LIABILITY CLAUSE.....	PAG.10

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 on medical devices

1. CODES

- STA100/x** Mattress cover
STA100/7 FIRE-RETARDANT mattress cover
STA110/x FIRE-RETARDANT mattress cover
ST680 FIRE-RETARDANT silicone hollow fibre mattress protector
ST682 FIRE-RETARDANT silicone hollow fibre mattress protector with removable cylinders

2. INTRODUCTION

Thank you for choosing an anti-decubitus device in the LEVITAS range by Moretti. Moretti anti-decubitus devices have been designed and manufactured to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this entire manual carefully before you use the device purchased. Should you have any queries, please contact your retailer for suitable advice and assistance.

NOTE: indicate several recommendations.

CAUTION: indicate the correct operating or maintenance procedures, in order to avoid damage to the device, the equipment or other items.

WARNING: draw attention to a potential hazard that requires correct procedures or practices to avoid injury.

3. INTENDED USE

The LEVITAS anti-decubitus mattresses by Moretti are intended to prevent the bedsores that can occur in any situation requiring lengthy hospitalisation and/or being laid up at home.

WARNING!

- Do not use the product for any purpose other than that specified in this manual
- The device must be installed by personnel qualified to perform general nursing procedures and who have received appropriate training in the prevention and treatment of bedsores
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features

4. EU DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products made and sold by Moretti SpA in the HOLLOW FIBRE ANTI-DECUBITUS product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES.

To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATIONS.
4. The devices in question are sold in NON STERILE packing.

5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

5. GENERAL PRECAUTIONS

- Please consult this manual carefully for correct use of the device
- Always consult your doctor or therapist for correct use of the device
- Use by one user only is recommended
- Use and store in the horizontal position
- Always sanitise the device before any reuse
- The devices are extremely sensitive to air and light
- Any fading of the colour does not alter the efficacy of the mattress
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts
- Store and use the device away from open flames and sources of heat
- Always take care when children are present
- Do not place the patient in direct contact with the mattress. Always use a full cover

6. SYMBOLS USED

Product code

Unique device identification

CE marking

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Disposal conditions

Maximum washing temperature of 60°C; delicate cycle

Maximum washing temperature of 95°C; delicate cycle

Do not bleach

Do not iron

Can be tumble dried; low drying temperature

Hanging

Do not dry clean

Dry on a flat surface

7. GENERAL DESCRIPTION

7.1 Mattress covers

STA100/1	190x85x14 cm polyester cover
STA100/2	190x120x14 cm polyester cover
STA100/3	63x85x12.5 cm polyester cover
STA100/4	63x85x14 cm polyester cover
STA100/5	63x120x14 cm polyester cover
STA100/6	190x85x12.5 cm polyester cover
STA100/7	190x85x10 cm FIRE-RETARDANT polyester cover (not shown)

The mattress covers are completely removable, via a zip around the edge. Breathable.

STA110/1	190x85x14 cm FIRE-RETARDANT polyester/PU cover
STA110/2	190x120x14 cm FIRE-RETARDANT polyester/PU cover
STA110/3	63x85x12.5 cm FIRE-RETARDANT polyester/PU cover
STA110/4	63x85x14 cm FIRE-RETARDANT polyester/PU cover
STA110/5	63x120x14 cm FIRE-RETARDANT polyester/PU cover
STA110/6	190x85x12.5 cm FIRE-RETARDANT polyester/PU cover

The polyester and polyurethane mattress covers are completely removable, via a zip around the edge. Breathable, bi-elastic, antibacterial and antifungal.

7.2 Hollow fibre mattress protectors

ST680 - Silicone hollow fibre mattress protector

Manufactured in silicone hollow fibre contained in a fire-retardant polyester fabric cover. The main property of silicone hollow fibre is its resilience and thus its ability to return to the original shape even after heavy stress or pressure, thereby maintaining its softness and elasticity over time. It insulates against heat and cold, as hollow fibre has a high power of thermal insulation.

In addition to being highly breathable, it allows uniform distribution of weight and thus helps to avoid the vein occlusions that are the main cause of bedsores. The quantity of hollow fibre allows an excellent load response, due to the high denier, allowing resting of areas of the body with bony prominences and thus avoiding bottom out.


COVER CLASS 1 FIRE RESISTANCE

ST682 - Silicone hollow fibre mattress protector with removable cylinders

Manufactured in silicone hollow fibre, contained in a fire-retardant polyester fabric cover, removable via a zip. The main property of silicone hollow fibre is its resilience and thus its ability to return to the original shape even after heavy stress or pressure, thereby maintaining its softness and elasticity over time. It insulates against heat and cold, as hollow fibre has a high power of thermal insulation. In addition to being highly breathable, it allows uniform distribution of weight and thus helps to avoid the vein occlusions that are the main cause of bedsores. The quantity of hollow fibre allows an excellent load response, due to the high denier, allowing resting of areas of the body with bony prominences and thus avoiding bottom out.

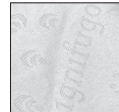COVER CLASS 1 FIRE RESISTANCE

8. HOW TO USE

8.1 How to use Covers

Position the cover on the bed, insert the mattress, then close on three sides using the zip.

8.2 How to use hollow fibre mattresses

Position the mattress protector on the normal mattress. After positioning, before placing the patient on the bed, check that the overall height is not excessive. If this is the case, it is advisable to equip the bed with protection systems. It is advisable to interpose another cover or protective sheet in exclusively microporous, air-permeable and waterproof material. An accessory that does not satisfy these recommendations could undermine the therapeutic effects.

9. CLEANING AND DISINFECTION
9.1 Cleaning and disinfecting the covers

Wash on delicate cycle at 60° (**STA100/x**) or 95° (**STA110/x**) with mild soap. Rinse thoroughly with plenty of water; do not tumble dry, do not wring, do not use solvents, dry away from sources of heat, do not iron, do not bleach.

Maximum washing temperature of 60°C; delicate cycle

Maximum washing temperature of 60°C; delicate cycle

Do not bleach

Do not iron

Can be tumble dried; low drying temperature

Hanging

Do not dry clean

Dry on a flat surface

9.2 Cleaning and disinfecting hollow fibre mattresses

ST680: wash in specific structures

ST682: wash in the washing machine at 60° with mild soap, separating all the components

Maximum washing temperature of 60°C; delicate cycle

Do not bleach

Do not iron

Can be tumble dried; low drying temperature

Hanging

Do not dry clean

Dry on a flat surface

10. STORAGE

Keep in a cool, dry place, away from damp environments.

11. MAINTENANCE

LEVITAS devices manufactured by Moretti are CE marked and carefully checked before being commercialised.

For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original replacement parts and accessories for repairs.

12. DISPOSAL CONDITIONS

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

13. TECHNICAL SPECIFICATIONS

13.1 Mattress covers

Dimensions and weight							
Product codes	STA100/1	STA100/2	STA100/3	STA100/4	STA100/5	STA100/6	STA100/7
 L:	1900 mm	1900 mm	630 mm	630 mm	630 mm	1900 mm	1900 mm
 D:	850 mm	1200 mm	850 mm	850 mm	1200 mm	850 mm	850 mm
 H:	140 mm	140 mm	125 mm	140 mm	140 mm	125 mm	100 mm
 kg	575 ± 10 g	770 ± 10 g	230 ± 10 g	230 ± 10 g	300 ± 10 g	565 ± 10 g	530 ± 10 g

- Zip on three sides
- Mass and dimensions may vary by around ±1%
- Breathable
- 100% Polyester

Dimensions and weight						
Product codes	STA110/1	STA110/2	STA110/3	STA110/4	STA110/5	STA110/6
 L:	1900 mm	1900 mm	630 mm	630 mm	630 mm	1900 mm
 D:	850 mm	1200 mm	850 mm	850 mm	1200 mm	850 mm
 H:	140 mm	140 mm	125 mm	140 mm	140 mm	125 mm
 kg	555 ± 10 g	745 ± 10 g	225 ± 10 g	425 ± 10 g	555 ± 10 g	545 ± 10 g

- Class 1 fire-retardant
- Zip on three sides
- Mass and dimensions may vary by around ±1%
- Breathable
- Antibacterial
- Bi-elastic

- Waterproof
- Washable at 95°
- 42% Polyester - 58% Polyurethane

13.2 Hollow fibre mattresses

Dimensions and weight					
Product codes	 L:	 D:	 H:		
ST680	1950 mm	850 mm	120 mm	450 g per cylinder (12 sections) 6 kg	kg*
ST682	1960 mm	880 mm	140 mm	500 g per cylinder (14 sections) 8.5 kg	kg*

* 100 kg without support mattress - 120 kg with levelling unit

- Breathable
- Washable at 60°
- External cover: 64% polyester, 36% polypropylene
Class 1 fire-retardant
- Silicone hollow fibre 100% polyester - 17 denier

* Mass and dimensions may vary by around $\pm 1\%$

14. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual. Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

15. REPAIRS

15.1 Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

15.2 Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

15.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

16. SPARE PARTS

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

17. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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