# MOPEdia®

---

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MA BI100\_BI101\_BI110\_BI111\_BI180\_BI200 01 A\_ENG\_06 2024

---

**Shower Trolleys Nefti**  
**INSTRUCTION MANUAL**

## INDICE

<b>1. CODES</b> .....	<b>PAG.3</b>
<b>2. INTRODUCTION</b> .....	<b>PAG.3</b>
<b>3. INTENDED USE</b> .....	<b>PAG.3</b>
<b>4. DECLARATION OF CONFORMITY EU</b> .....	<b>pag.3</b>
4.1 Reference Standards and Directives.....	pag. 3
<b>WARNING!</b> .....	<b>PAG.3</b>
<b>5. AVVERTENZE GENERALI</b> .....	<b>PAG.4</b>
<b>6. SYMBOLS</b> .....	<b>PAG.4</b>
<b>7. GENERAL DESCRIPTION</b> .....	<b>PAG.5</b>
<b>8. BEFORE EACH USE</b> .....	<b>PAG.6</b>
<b>9. USE MODE</b> .....	<b>PAG.6</b>
9.1 Mode of use for <b>HYDRAULIC</b> models BI100-BI101-BI180.....	pag. 6
9.2 Mode of use for <b>ELECTRIC</b> models BI110, BI111.....	pag. 6
9.3 Mode of use for <b>ELECTRIC</b> model BI200.....	pag. 7
9.4 How recharge the batteries BI110-BI111.....	pag. 7
9.5 How recharge the batteries BI200.....	pag. 8
9.6 Use of folding siderails.....	pag. 8
9.7 Use of wheels.....	pag. 9
9.7.1 Use of wheels with directional/total lock for BI100-BI110-BI180-BI200.....	pag.9
9.7.2 Wheels with integrated total lock for BI101-BI111.....	pag.9
<b>10. MAINTENANCE</b> .....	<b>PAG.9</b>
<b>11. CLEANING AND DISINFECTION</b> .....	<b>PAG.9</b>
11.1 Cleaning.....	pag. 9
11.2 Disinfection.....	pag. 9
<b>12. DISPOSAL TERMS</b> .....	<b>PAG.10</b>
12.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE.....	pag. 10
<b>13. DECLARATION OF ELECTROMAGNETIC COMPATIBILITY</b> .....	<b>pag.10</b>
<b>14. SPARE PARTS</b> .....	<b>PAG.13</b>
<b>15. TECHNICAL SPECIFICATIONS</b> .....	<b>PAG.13</b>
15.1 General specifications.....	pag. 13
15.2 General specifications for hydraulic trolley NEFTI B100.....	pag. 14
15.3 General specifications for hydraulic trolley NEFTI B1101.....	pag. 14
15.5 General specifications for electric trolley NEFTI BI111.....	pag. 15
15.6 General specifications for pediatric hydraulic trolley NEFTI SMALL BI180.....	pag. 15
15.7 General specifications for bariatric electric trolley NEFTI BIG BI200.....	pag. 15
<b>16. WARRANTY</b> .....	<b>PAG.16</b>
16.1 Warranty of recyclable batteries.....	pag. 16
<b>17. REPAIRS</b> .....	<b>PAG.16</b>
<b>18. SPARE PARTS</b> .....	<b>PAG.16</b>
<b>19. NON-LIABILITY CLAUSE</b> .....	<b>PAG.16</b>

I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

EN

## 1. CODES

- B1100** Shower trolley **NEFTI HYDRAULIC**, max weight (water included) 180 kg
- B1101** Shower trolley **NEFTI SLIM HYDRAULIC**, fixed-tilt table, max weight (water included) 180 kg
- B1110** Shower trolley **NEFTI ELECTRIC**, max weight (water included) 200 kg
- B1111** Shower trolley **NEFTI SLIM ELECTRIC**, fixed-tilt table, max weight (water included)) 200 kg
- B1180** Shower trolley pediatric **NEFTI SMALL HYDRAULIC**, max weight (water included) 180 kg
- B1200** Shower trolley pediatric **NEFTI BIG ELECTRIC**, max weight (water included) 325 kg

## 2. INTRODUCTION

Their design and quality ensure the maximum comfort, safety and reliability. **MOPEDIA** shower trolleys **Nefti by Moretti S.p.A** are designed and manufactured to meet all your needs for a practical, correct and safe use. This manual contains small suggestions for proper use of the device you have chosen and valuable advice for your security. You may read the complete manual before using the shower trolley. In case of doubts please contact your dealer, who will help and advice you properly.

## 3. INTENDED USE

**MOPEDIA** shower trolleys **Nefti by Moretti S.p.A** are intended for patients care during washing/showering with table in horizontal or inclined position, with the help of an assistant that operates commands and executes the washing operations. It cannot be used by a single person.

### WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

## 4. DECLARATION OF CONFORMITY EU

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of **SHOWER TROLLEYS** complies with the provisions of the regulation 2017/745 on **MEDICAL DEVICES** of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices **ARE NOT MEASURING INSTRUMENTS**.
3. The devices **ARE NOT MADE FOR CLINICAL TESTS**.
4. The devices are packed in **NON-STERILE BOX**.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

### 4.1 Reference Standards and Directives

To guarantee safety standards for users, Moretti S.P.A. tests and complies with the standards:

- EN 12182 (Products intended to assist people with disabilities)
- EN ISO 21856 (Technical aids for the disabled - General requirements and test methods)

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### WARNING!

- Always pay close attention of moving parts that could cause arms entrapment limbs and injuries
- If there are injuries on the skin, it is recommended to protect the exposed part

- Always check the presence of damage to the structure that could compromise the proper safe operation of the product
- For the correct use of the device, please refer to the present manual, giving particular attention to the maintenance and warnings before each use
- For the correct use of the device, consult your doctor or therapist
- Maintain the product packed away from heat sources because the packaging is made of carton
- The service life of the device is determined by wear of parts not repairable and/or replaceable
- Always pay attention to the presence of children
- Do not exceed the weight limit, washing water included
- Do not open the sides, neither during the transport nor during the patient washing
- Do not clean with aggressive products such as ammonia, bleach, acids and abrasive detergents
- Do not load the patient weight on the sides of the bathing plane
- MOPEDIA shower trolleys by Nefti must be used only by properly informed medical staff
- Set the trolley in horizontal position before let a patient sit on/go down
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Serial Number

Read the instruction manual

Medical Device

Conditions of disposal

Attention, should read the instructions

Production dates

Product disposal according to directive CE/19/2012

Protection against dust and water splashes

B Type

Max loading weight

7. GENERAL DESCRIPTION

EN

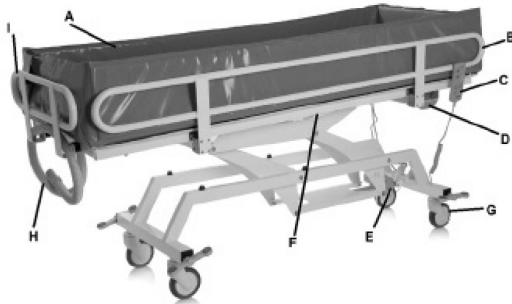
Fig. 1 (photo for illustration purposes for Code BI100-BI110-BI180)

Fig. 2 (photo for illustration purposes for Code BI200)

Fig. 3 (photo for illustration purposes for Code BI101-BI111)

- A. Washing sheet
- B. 180° foldable side rail
- C. Motor unit remote control (only for BI110-BI111-BI200)
- D. Control unit with battery (only for BI110-BI111-BI200)
- E. Lifting motor unit of bathing plan (hydraulic BI100-BI101-BI180, electric BI110-BI111)
- F. Wash table inclination actuator (gas spring BI100-BI180, electric BI110)
- G. Wheels (with brake and directional lock controlled via side pedals for BI100-BI110-BI180-BI200; with wheel brake for BI101-BI111)
- H. Drainage pipe
- I. Head/foot side locked rail
- J. Column actuators for elevation and inclination of the bathing plan (BI200)

## 8. BEFORE EACH USE

- Always check the correct closing of the lateral containment sides. In case it is not possible to close the sides the product must not be used.
- Before washing or transferring the patient, check that the wheels are braked and the side rails raised.
- During the transfer, avoid passing obstacles that could cause an overturning.

### WARNING!

Always check that the washcloth and drain hose are in good condition; damage due to wear and tear can cause water leakage during washing.

## 9. USE MODE

### 9.1 Mode of use for HYDRAULIC models BI100-BI101-BI180



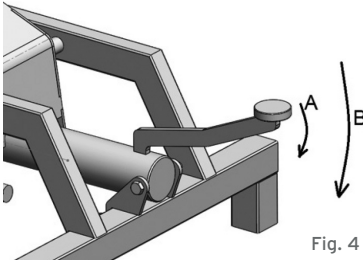
Fig. 4

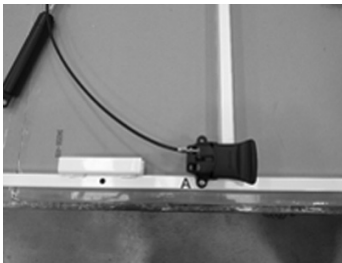For all hydraulic models: To raise the stretcher bed, press the elevation pedal repeatedly as in Fig. 4 (ref A), to lower the bed press the elevation pedal fully down (ref B).


For BI100-BI180 models only: To adjust the inclination of the wash table, release the gas spring via the lever in the centre of the wash table and perform the inclination (do this operation without the load on the trolley).

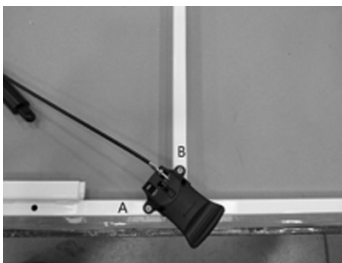Unscrew the screw (A) visible in the photo, turn the handle clockwise and remove the screw (initially covered by the handle) from the corresponding hole.

Position the handle so that the holes (on the handle and on the plane) match and then tighten the screws (A and B) moderately (so as not to compromise the handle structure).

### 9.2 Mode of use for ELECTRIC models BI110, BI111

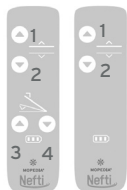(BI110) (BI111)

Fig. 5

Before each use, check that the battery is correctly inserted in the battery compartment of the control unit and that it is charged; batteries cannot be recharged during use.

- To switch on the control unit, check that the emergency button is not pressed, and if it is, turn it clockwise to release it and activate the control unit.
- Press button 1 to increase the height of the washing surface;
- Press button 2 to decrease the height of the wash table;
- Press button 3 to increase the inclination of the wash table (only BI110);
- Press button 4 to decrease the inclination of the wash table (only BI110).

Emergency stop: There is a red "STOP" button on the control unit which, if pressed, immediately deactivates all controls.

To restore operation, turn the red button clockwise and out again.

EN

### 9.3 Mode of use for ELECTRIC model BI200

Before each use, check that the battery is correctly inserted in the battery compartment of the control unit and that it is charged; batteries cannot be recharged during use. To turn the stretcher on press the 'ON' button on the control unit for approximately two seconds, to turn it off press the 'OFF' button for approximately 4 seconds.

Fig. 8

- Press key 1 to increase the height of the wash table;
- Press key 2 to decrease the height of the wash table;
- Press key 3 to increase the inclination of the wash table;
- Press key 4 to decrease the inclination of the wash table.

Fig. 9

### 9.4 How recharge the batteries BI110-BI111

**Notes:** Make sure that the red emergency "STOP" button is not pressed and that the control unit is therefore active. To do this, turn and pull the RED button towards yourself.

#### Indication of the battery state of charge

There is a 3-Led indicator for the battery state of charge. This indicator is placed on the remote control. 3 Leds on indicate a complete charge of the battery, whereas one just Led on indicates the battery is out of power and needs to be charged.

#### Recharging Batteries

- Connect the charging cable to the main socket and then to the socket on the control unit
- On the remote control there is a battery charge indicator which flashes during the charging phase; once all three LEDs are lit steadily, the battery is fully charged
- Before activating the shower trolley again, make sure to disconnect the battery charger cable
- Charge the batteries for at least 6 hours before using the device

Fig. 10a

### 9.5 How recharge the batteries BI200

- Connect the adaptor to the mains socket and then to the input of the control unit (UNTIL FULLY) after having removed its protective cap.
- On the charger adaptor there is a LED which is orange during the charging phase. Charge the battery until the led turns green.
- Before using the shower stretcher always remember to disconnect the adaptor to recharge the batteries.
- Charge the batteries for at least 6 hours before using the shower stretcher.

Fig. 10b

### 9.6 Use of folding siderails

The folding side rails are equipped with a 3-position support with a safety guide against accidental release, which supports the folding sides and allows them to be securely and correctly fixed in the 3 foreseen positions. Description of sides positions and instructions for their handling:

- Side panels in raised position, with pins inserted in the "internal" notch (Fig.11 e 11a). Slightly lift the side rails using both hands to unlock.
- Side rails in an inclined position, reached by slightly lifting the side rails and pulling them slightly backwards, inserting the pins in the "external" notch (Fig.12). Slightly lift the side rails using both hands to unlock.

Fig. 11

Fig. 11a

Fig. 12

- Side rails in lowered position, reached by slightly lifting the side rails and making them rotate forward until, when passing the safety guide, it is possible to extract the side rails by further lifting by rotating them backwards and accompanying the rotation movement out of the support until reaching the complete lowering avoiding shocks and sudden movements (Fig. 13, 14)

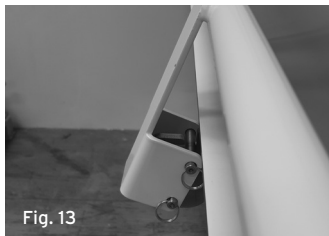
Fig. 13

Fig. 14

- To close the side panel, carry out the operations indicated above in reverse order.

#### WARNING!

- Always check the correct closing of the sides through the locking pin.
- The operator shall not lean with his weight on the sides.
- Do not use the sides as a support for the patient's movement.
- Always move the side rail with 2 hands, and always make sure that pins of the 2 sides proceed in parallel, and maintain the same position both during movement and at rest, (for left and right brackets) and that if raised or tilted both pins are positioned in the same notch.

**9.7 Use of wheels**

**9.7.1 Use of wheels with directional/total lock for BI100-BI110-BI180-BI200**

- Press the pedal until you hear the first “click” to obtain the directional lock (for long and straight transfers)
- Press the pedal until you hear the second “click” to obtain the total lock of wheels
- For a reliable braking, totally lock the wheels on both the axes of the trolley.

**9.7.2 Wheels with integrated total lock for BI101-BI111**

- Press the pedal on the wheel until you hear the “click” to obtain full wheel braking
- Lower the raised lever during wheel braking to release the brake and release the wheel

Braked wheel

Free wheel

**10. MAINTENANCE**

When Mopedia by Moretti accessories are placed in commerce they are carefully checked and provided of CE brand. For patient and doctor safety we recommend that the eligibility of your device must be checked, at least every 2 years ,by the producer or by an authorized laboratory. In case of repair must be used only original spare part and original accessories.

- Is very important that the shower trolley is checked before every use. In particular we please to check that all screws and bolts are well tightened.
- Is important that the shower trolley can move freely with the wheels brakes deactivated
- For a correct use verify the wheels brakes.
- Is very important to verify periodically that the connecting pins/mechanical joints and relative blocking rings are well placed.
- The mechanic parts must be lubricated with oil to help the fluidity and wearing resistance. Keep close attention to the lifting/tilting mechanic device of the washing plan. Lubricate as indicated above every 2 or 3 months.

**11. CLEANING AND DISINFECTION**

**11.1 Cleaning**

To clean the product, only use a cloth soaked in water and dry with a clean cloth and dry, or with warm water and neutral soap.

**WARNING!**

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone to clean the trolley, because these can cause abrasions of plastic parts and rusting of metallic areas.

The manufacturer won't be responsible for damages caused by usage of materials that can corrode the areas of the product or chemical corrosive during cleaning.

**11.2 Disinfection**

If needed, perform the sanitization with a mild disinfectant detergent.

## 12. DISPOSAL TERMS

In case of device disposal never use an urban rubbish container. Is recommended to dispose the device through the common municipal ecological islands.

### 12.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE

At the end of its service life, the product must not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted waste collection centre or to the resellers, in case they provide for this service. Sorted waste disposal reduces the environmental and health consequences of an improper disposal. Furthermore, the materials which make up the product may be recycled, thus resulting in energy and resource savings. In order to highlight the compulsory sorted disposal of the electro-medical equipment, the product itself bears the barred bin symbol.

### 12.2 Treatment of dead batteries (Directive 2006/66/CE)

The symbol on the product indicates that the batteries must not be treated like common household waste. Make sure the batteries are properly disposed; this prevents the possible environmental and health consequences of an improper disposal. Recycling helps to preserve natural resources. Deliver dead batteries to a proper recycling centre. For more detailed information regarding the disposal of dead batteries or of the product itself, please contact the municipality, the local waste disposal centre or the reseller from whom you purchased the device.

## 13. DECLARATION OF ELECTROMAGNETIC COMPATIBILITY

The NEFTI shower trolley device is designed for use in the electromagnetic conditions specified in the following tables. The NEFTI device user must be sure that these conditions, during the usage, are real.

Table 1

Manufacturer's guide and declaration - electromagnetic emissions		
The NEFTI device is intended to be used in the following electromagnetic conditions. The client or user must ensure that it is used in these conditions		
Emission tests	Conformity	Electromagnetic environment - guide
RF emissions CISPR 11	Group 1	The NEFTI device uses RF energy only for charging the battery. Its RF emissions are therefore very low and in all likelihood do not interfere with nearby electronic equipment.
RF emissions CISPR 11	Class B	The NEFTI device is intended to be used in all buildings, including the home and buildings connected directly to the low voltage public power grid for residential applications.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuation/flicker emissions IEC 61000-3-3	Conforming	

Table 2

Manufacturer's guide and declaration - electromagnetic emissions			
The NEFTI device is intended to be used in the following electromagnetic conditions. The client or user must ensure that it is used in these conditions.			
Immunity tests	Test level IEC 60601	Conformity level	Electromagnetic environment - guide
Electrostatic discharge (ESD) IEC 61000-4-2	$\pm 6$ kV contact $\pm 8$ kV air	$\pm 6$ kV contact $\pm 8$ kV air	Information for using the device and charging the battery. The floor must be wood, concrete or ceramic. If the floor is covered with a synthetic material, the relative humidity must be at least 30%
Electrical fast transient/burst IEC 61000-4-4	$\pm 2$ kV for power lines $\pm 1$ kV for I/O lines	2kV for power lines $\pm 1$ kV for I/O lines	The quality of the mains supply must be that of a typical commercial hospital when charging the battery
Surge IEC 61000-4-5	1 kV differential mode $\pm 2$ kV common mode	1 kV differential mode $\pm 2$ kV common mode	The quality of the mains supply must be that of a typical commercial or hospital when charging the battery
Voltage drop outs, brief interruptions and variations of voltage on the power supply lines. IEC 61000-4-11	<5% UT (>95% UT drop out) for 0.5 cycles 40% UT (60% UT drop out) for 5 cycles 70% UT (30% UT drop out) for 25 cycles <5% UT (>95% UT drop out) for 5 s	<5% UT (>95% UT drop out) for 0.5 cycles 40% UT (60% UT drop out) for 5 cycles 70% UT (30% UT drop out) for 25 cycles <5% UT (>95% UT drop out) for 5 s	The quality of the mains supply must be that of a typical commercial hospital when charging the battery.
Mains frequency magnetic field (50/60 Hz) IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. (Applies both to using the device and charging the battery).
Note: UT is the AC mains voltage prior to application of the test level.			

Table 3

Manufacturer's guide and declaration - electromagnetic immunity			
The NEFTI device is intended to be used in the following electromagnetic conditions. The client or user must ensure that it is used in these conditions. Make sure it is used in these conditions.			
Immunity tests	Test level IEC 60601	Conformity level	Electromagnetic environment - guide
conducted RF IEC 61000-4-6	3 V eff. From 150 KHz to 80 MHz	3 V eff.	Portable and mobile RF communications equipment should be used no closer to any part of the [equipment or system], including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  Recommended separation distance: $d = 1,2\sqrt{P}$ from 80 MHz to 800 Mhz $d = 2,3\sqrt{P}$ from 800 MHz to 2,5 GHz $d = 7,7$ m during recharge operation  where "P" is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and "d" is the recommended separation distance in metres (m). Fields strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 
radiated RF IEC 61000-4-3	3 V/m From 80 MHz to 2.5 GHz	3 V/m	
<p><b>NOTE 1:</b> At 80 MHz and 800 MHz, apply the separation distance for the higher frequency band.</p> <p><b>NOTE 2:</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			
<p>The intensity of the field for fixed transmitters, such as base stations for radio-telephones (cellulars and cordless) and terrestrial mobile radios, amateur radio equipment, AM and FM radio transmitters and TV transmitters may not be known precisely in theory. To establish an electromagnetic environment caused by fixed RF transmitters, one must run an on-site electromagnetic inspection. If the field intensity measured in the place in which the Nefti is used exceeds the RF level of conformity indicated above, the function of the Nefti must be monitored. If abnormalities are observed, supplementary measures may be necessary, such as re-orientation or repositioning.</p> <p><sup>b</sup> The field intensity over the frequency range from 150 kHz to 80 MHz should be less than 3 V/m.</p>			

Table 4

Recommended separation distances between portable and mobile radio communications equipment and NEFTI						
The NEFTI device is intended to operate in an electromagnetic environment in which RF disturbances are under control. The client or user of NEFTI can contribute to preventing EM interference by ensuring a minimum distance between the mobile and portable RF communications devices (transmitters) and the NEFTI battery charger during charging and the internal NEFTI device during its operation as indicated below, in relation to the maximum output power of the transmitters						
Tests	Separation distance as a function of the frequency of the transmitter m					
Maximum rated output power of transmitter W	During battery charging	During the use of NEFTI	During battery charging	During the use of NEFTI	During battery charging	During the use of NEFTI
	from 150 kHz to 80 MHz d = 1,2√P	from 150 kHz to 80 MHz d = 1,2√P	from 80 kHz to 800 MHz d = 1,2√P	from 80 kHz to 800 MHz d = 1,2√P	from 800 kHz to 2.5 GHz d = 1,2√P	from 800 kHz to 2.5 GHz d = 1,2√P
0,01	0,12	0,12	0,12	0,12	0,23	0,23
0,1	0,38	0,38	0,38	0,38	0,73	0,73
1	1,2	1,2	1,2	1,2	2,3	2,3
10	3,8	3,8	3,8	3,8	7,3	7,3
100	12	12	12	12	23	23
For transmitters whose maximum rated output power is not listed above, the recommended separation distance d, in metres (m), can be calculated using the equation applicable to the transmitter frequency, where P is the maximum rated output power of the transmitter, in Watt (W), as given by the transmitter's manufacturer. NOTE 1: At 80 MHz and 800 MHz, apply the separation distance for the higher frequency band. NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.						

#### 14. SPARE PARTS

For spare parts and accessories, please always refer to the MORETTI S.P.A. master catalogue. TO ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

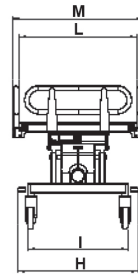
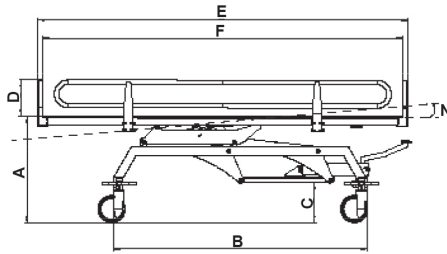
#### 15. TECHNICAL SPECIFICATIONS

##### 15.1 General specifications

- Powder coated steel structure, with specific antirust treatment
- Swivel castors with brake and rotational lock controlled with side pedals on each axle for BI100-BI110-BI180-BI200
- Swivel castors with total brake on each wheel for BI101-BI111
- Mechanism for raising the washing table
- Wash table tilting mechanism (only BI100-BI110-BI180-BI200)
- Three-position 180° tilting sides
- Waterproof, padded pvc wash cloth with side rails h. 25cm
- Flexible discharge hose 1,5m long
- Compliance with EN ISO 12182 and UNI EN ISO 21856
- MAXIMUM Load of 180 kg (including water) BI100-BI101-BI180
- MAXIMUM Load OF 200 kg (water included) BI110-BI111
- MAXIMUM Load OF 325 kg (water included) BI200

### 15.2 General specifications for hydraulic trolley NEFTI BI00

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

Misure: in CM:

A	B	C
59-93	141	23
D	E:	F
20	206	200*
H	I	L
67	56	66*
M	N	
73	0°-5°	180 Kg

### 15.3 General specifications for hydraulic trolley NEFTI BI101

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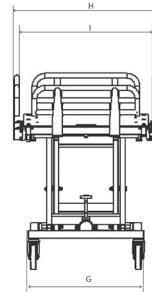
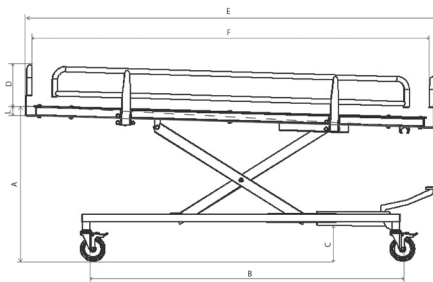
Dimensions in CM

A	B	C
60-90	160	20
D	E	F
20	206	200*
G	H	I
59	73	66*
L	-	
2,5°	-	180 Kg

### 15.4 General specifications for electric trolley NEFTI BI110

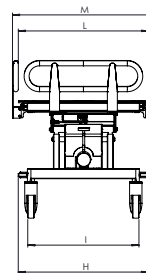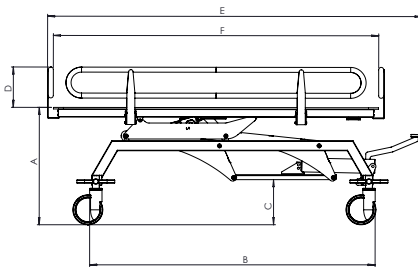Dimensions and weight



Dimensions in CM

A	B	C
59-99	141	16
D	E	F
20	206	200*
H	I	L
67	56	73
M	N	
66*	0°-5°	200 Kg

\* Internal measurement of side rail structure with sheet not installed

15.5 General specifications for electric trolley NEFTI BI111

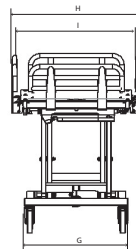
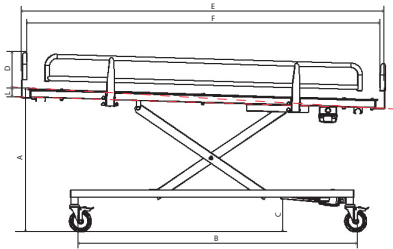
Dimensions and weight

Dimensions in CM		
A	B	C
60-95	160	15
D	E	F
20	206	200*
G	H	I
59	73	66*
L	-	
2,5°	-	200 Kg

15.6 General specifications for pediatric hydraulic trolley NEFTI SMALL BI180

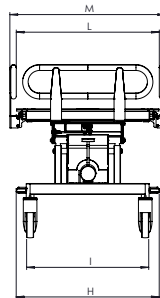
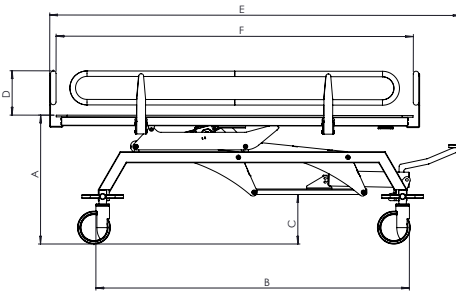
Dimensions and weight

Dimensions in CM		
A	B	C
65-91	141	23
D	E	F
20	170	165*
H	I	J
67	56	66*
M	N	
73	0°-5°	180 Kg

15.7 General specifications for bariatric electric trolley NEFTI BIG BI200

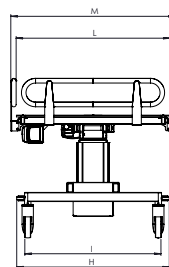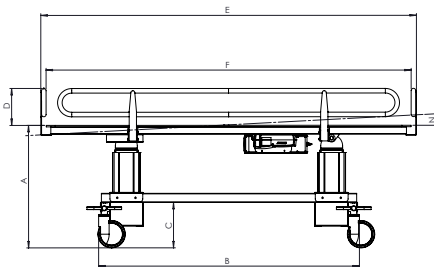Dimensions and weight

Dimensions in CM		
A	B	C
65-94	143,5	23
D	E	F
20	206	200*
H	I	L
84,5	75	86*
M	N	
91	0°-7°	325 Kg

\* Internal measurement of side rail structure with sheet not installed

## 16. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. Components which wear out are not covered by this warranty if the damage is due to the normal use of the product.

### 16.1 Warranty of recyclable batteries

Original and spare batteries are covered by a warranty that lasts 90 (ninety) days in relation with their performances and a 6 (six) months warranty in relation with factory defects or law requirement. If you leave them unused completely charged more than 3 (three) consecutive months, the warranty is automatically voided. If completely uncharged batteries are left more than 3 (three) consecutive days, the warranty is automatically voided.

## 17. REPAIRS

### Repair under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

### Repairing a product not covered by warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

### Non-defective products

Moretti will notify the customer if concludes that the product is not defective after reception and review. The product will be sent back to the customer at his cost.

## 18. SPARE PARTS

Moretti original spare parts are guaranteed for 6 (six) months from the day of delivery.

## 19. NON-LIABILITY CLAUSE

Except as expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, warranty or condition, express or implicit, including any eventual future declarations, warranties or sale conditions, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of Moretti product will be uninterrupted or problem-free. The duration of any implicit warranties imposed by legislations is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations of the duration of an implicit guarantee or the exclusion/limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not be applied to the user. This warranty can be upgraded without notification.

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