MA DMV120-DMV122-DMV124 02 A_05-2020

MARTELLO NEUROLOGICO

MANUALE D'ISTRUZIONE

ITALIANO

INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT
AND OF THE COUNCIL of 5 April 2017 concerning medical devices

1. CODES

DMV120	Buck hammer with needle and brush
DMV122	Babinsky neurological hammer
DMV124	Taylor neurological hammer

ENGLISH

2. INTRODUCTION

Thank you for purchasing an Neurological Hammer **Dimed by Moretti S.p.A.** Its design and quality guarantees comfort, safety and reliability to the user. Neurological Hammers **Dimed by Moretti S.p.A.** have been designed to satisfy all your demands. This instructions manual contains advices for your safety and for a correct use of the device. In case of doubts, contact your retailer.

3. INTENDED USE

The Neurological Hammer are devices realized to verify neurological reflexes. These devices can be used by professional and trained personnel only.

4. DECLARATION OF CONFOMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of NEUROLOGICAL HAMMER complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. SYMBOLS

 Product code

 Medical Device

 Unique Device Identification

 CE mark

 Read the instruction manual

 Manufacturer

 Conditions of disposal

 Batch Lot

6. WARNING

These devices can be used by professional and trained personnel only. Pay attention to the presence of children.

7. HOW TO USE

The professional who utilizes hammers knows the difference way to use.

8. CLEANING

Use warm water and neutral soap to clean the product.

9. GENERAL WARNINGS

Do not use the products for purposes not indicated in this manual. Moretti S.p.A. declines all responsibility for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product. The manufacturer can modify the device or the instruction manual without prior notification.

10. TECHNICAL FEATURES

DMV120 Buck hammer with needle and brush

- Zinc alloy handle, PVC head
- Length 18,6 cm
- Height width 5,6 cm
- Weight 90 g
- Diagnostic pin screwed on the head
- Diagnostic brush screwed on the handle

DMV122 Babinsky neurological hammer

- Zinc alloy handle, PVC head
- Length 18,6 cm
- Height width 4,4 cm
- Weight 97 g

DMV124 Taylor neurological hammer

- Zinc alloy handle, PVC head
- Length 18,8 cm
- Height width 2,1 cm
- Weight 56 g

11. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

12. REPAIRING

Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

13. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date. For spare parts and accessories please refer to Moretti master catalogue.

14. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

NOTE

NOTE**ENGLISH**

**WARRANTY CERTIFICATE****ENGLISH**

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

 MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C

MORETTI S.p.A.
Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com