**SKEMA<sup>®</sup>FORM**

MA MI440 02 A\_1TA\_09 2020

**PIANTANA PER IPODERMOCLISI**

---

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ITALIANO

## INDICE

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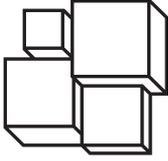
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

## 1. CODE

**MI440** Professional stainless steel I.V. Poles, hydraulic pump for elevation

## 2. INTRODUCTION

Thank you for choosing a **I.V. Poles stainless steel** from the **SKEMAFORM by MORETTI S.p.A.** Their design and quality are a guarantee of comfort, safety and reliability. I.V. Poles stainless steel from the **SKEMAFORM by MORETTI S.p.A.** are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the item you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the I.V. Poles stainless steel. Should you have any queries, please contact your retailer for advice and assistance.

## 3. INTENDED USE

The stainless steel I.V. Poles for hypodermoclysis SKEMAFORM is a support device in the administration of intravenous fluids to the patient.

### ATTENTION!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

## 4. DECLARATION OF CONFORMITY

Moretti S.p.a. declares under its sole responsibility that the product made and traded by Moretti S.p.a. and belonging to the group of I.V. STAND complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.a. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.a. releases and makes available through its channels.

## 5. GENERAL WARNINGS

- For proper use of the device refer to this manual.
- Keep the packaged product away from any source of heat because the packaging is made of paper.
- The shelf-life of the accessory is determined by wear of parts that cannot be repaired and/or replaced.
- Be careful when children are nearby.
- Do not exceed the max load capacity indicated in this manual for the hooks to prevent deformation and breakage.
- Before using the device, always check that the assembly screws are well tightened.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

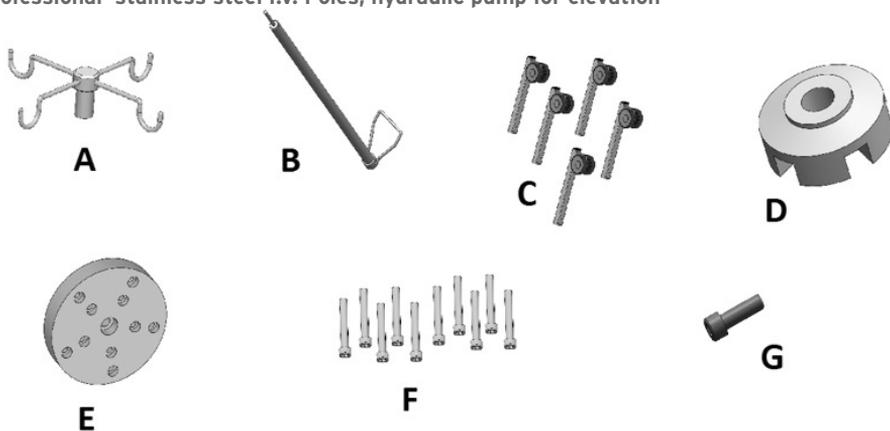
## 6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

## 7. GENERAL DESCRIPTION

### 7.1 Parts view and description stainless steel I.V. poles - MI440

#### Professional stainless steel I.V. Poles, hydraulic pump for elevation



**Parts list**

- A. Hook holder made of stainless steel (1 pc);
- B. Pole with pedal hydraulic pump (1 pc);
- C. Stainless steel legs with castors (5 pcs);
- D. Base for professional I.V. Pole (1pc);
- E. Base mounting plate for professional I.V. Pole (1pc);
- F. Screws to fix the legs to the base (10 pcs);
- G. Screw to fix hydraulic pole on the base (1 pc).



**8. ASSEMBLY**

**ATTENTION!**

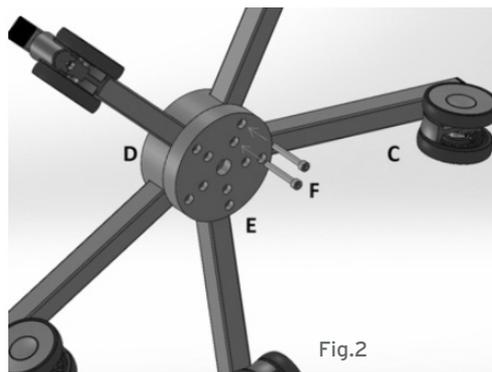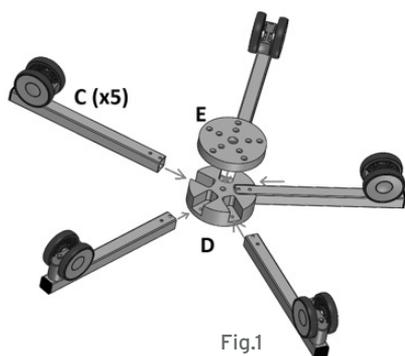During the assembling pay attention to avoid any injury to persons and/or damage to objects.

**8.1 Assembling instructions - MI440**
**Professional stainless steel I.V. Poles, hydraulic pump for elevation**

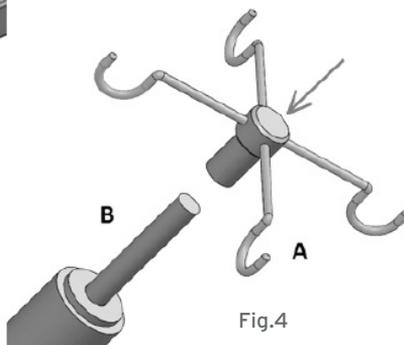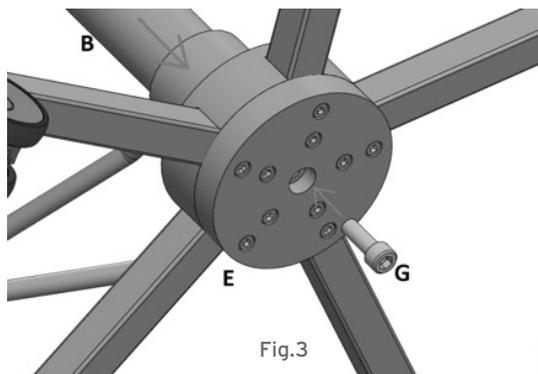1. Insert the castors (ref.C) in the lower part of the base (ref.D). (Fig.1)
2. Put the fixing plate (ref.E) on the base with castors paying attention to the positioning of the holes;
3. Tighten the screws (ref.F - 10pcs) to firmly attach the legs with castors at the base, using the fixing plate; (Fig.2)

4. Connect the hydraulic pole (ref.B) to the fixing plate at the base (ref.E), by tightening the screw (ref.G); (Fig 3)
5. Insert and screw the hook support (ref.A) onto the upper part of the hydraulic pole (ref.B); (Fig 4)
6. Now the I.V. pole is ready to use.

## 9. HOW TO USE

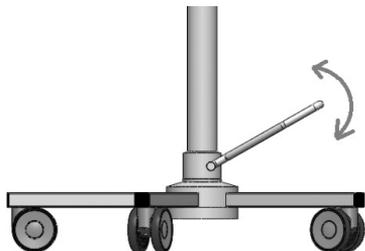
Fig.5

To increase the height of the hydraulic pole, apply a repeated intermittent force on the elevation pedal, as shown in the figure. To fully extend the IV pole are needed about 30 activations of the pedal (Fig. 5).

To decrease the height of the hydraulic pole, press the pedal and maintain the pressure until the desired height is reached.

### WARNINGS:

- Check the pole before every use. Make sure it hasn't any damage or signs of wear that could compromise its working. If it should be damaged, do not use the pole;
- Make sure the position of the pole is comfortable and safe for the patient, pay attention to the castors movement;
- Do not overload the hooks, check the maximum weight on the technical features.

## 10. CLEANING AND DISINFECTION

### 10.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

#### ATTENTION!

Do not use abrasive substances, acids, alkaline, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

### 10.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

## 11. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

## 12. ACCESSORIES

**MIA441** Stainless steel handle for MI 440

**MIA442** Handle with stainless steel basket for MI 440

**MIA443** Handle with stainless steel tray for MI 440

**MIA444** Basket and stainless steel tray for MI 440

### 13. TECHNICAL FEATURES

- I.V. Poles hypodermoclysis for operating room
- Stainless steel 18/10 (AISI 304)
- Elevation via hydraulic pump using pedal
- 4 hooks with anti-detachment
- Max payload 60Kg uniformly distributed, Max payload 15Kg for hook
- Anticapsize Ballasted Base
- 5 antistatic castors Ø75mm, 2 with brake antistatic "electrically conductive"

#### 13.1 Dimension and weight

Code	<b>MI440</b>
Height (min/max)	<b>1540mm / 2550mm</b>
Base encumbrance (mm)	<b>650mm</b>
Max Payload for hook	<b>15kg</b>
Max Payload for the I.V. Pole	<b>60kg</b>

### 14. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

### 15. REPAIRING

#### 15.1 Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

#### 15.2 Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

#### 15.3 Non defective device

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

**16. SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

**17. EXEMPT CLAUSES**

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period.

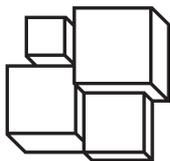
Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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